MOPEDIA®

MA RP14x_RP15x_01 A_07-2014

**BASTONI
MANUALE D'ISTRUZIONE**

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I Class Medical Device according
to Directive 93/42/EEC and further modifications

1. Introduction

Thank you for purchasing a **MOPEDIA** walking canes by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

2. Intended use

The MOPEDIA by Moretti walking sticks are aids manipulated by one arm adapted to support and / or help the mobility of people with mobility problems.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

3. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the cane aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

4. General warnings

- Read carefully the entire manual before using the device.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the walking canes.
- Make sure that the product is delivered complete and keep it packed away from heat sources.
- Service life- the device use limit is defined by the wear of the parts.
- Do not allow children to play or operate the device.

5. General description

5.1 Folding walking cane codes

RP145BC Aluminum folding walking cane with T-handle - Bronze color

- RP145NC Aluminum folding walking cane with T-handle - Black color
RP145NL Aluminum folding walking cane with T-handle - Black color 82-92 cm
RP159 Aluminum folding walking cane with T-handle - Woman model

5.2 Aluminum walking canes height adjustable codes

- RP146 Aluminum height adjustable walking cane T-handle - Black color
RP147 Aluminum height adjustable walking cane T-handle - Black COLOR
RP148 Aluminum height adjustable walking cane T-handle - Black color
RP149 Aluminum height adjustable walking cane T-plastic handle - Black color
RP150 Aluminum height adjustable walking cane ergonomic handle
RP151 Aluminum height adjustable walking cane curved
RP152 Aluminum height adjustable walking cane sponge handle
RP153D Aluminum height adjustable walking cane anatomical right handle
RP153S Aluminum height adjustable walking cane anatomical left handle
RP154 Aluminum height adjustable walking cane t-handle silver check pattern
RP156 Aluminum height adjustable walking cane T-handle wooden pattern
RP157 Aluminum height adjustable walking cane T-handle walnut pattern
RP158 Aluminum height adjustable walking cane woman model

MOPEDIA walking canes are made of lightweight anodized aluminum, wooden, plastic or sponge handle and non-slip rubber tip. Adjustable height every 2.5 cm.

These devices are useful to ensure a safety of support and confidence. The walking canes sticks also make it easy to stay upright and walk. Facilitate the discharge of the lower body segments, stabilize the person in the problems of balance and support when there are problems force.

5.3 Height adjustment

Press the button, slide the cane and bring it to match the holes in the height required and the maneuverability of the patient. Replace the stopper on the hole to lock the cane and make sure that the button is fully penetrated into the hole. Maximum patient's weight 100kg for folding canes, 110 kg for all the others.

6. Before use

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

7. Warnings for use

- Use extra caution on wet and slippery surfaces.
- DO NOT use with patient who exceed the weight indicated in the product label and in the technical features chapter of the present manual.
- Avoid to leave the device in contact with water for long periods of time.
- Periodically check its sturdiness, the stability and the rubber tips are in good conditions. Replace them once worn out.
- In case of doubts about use, adjustment or maintenance, contact your dealer.
- Keep it away from children to avoid any possible risks.

8. How to use

Note: Refer to the safety instructions in this manual

The use of the walking canes varies from case to case. For this you need a minimum of training

by qualified personnel with reference to information on the type of step or support type. The operator during patient care is always positioned on the side of the body more weak and sick.

9. Maintenance

The Mopedia devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

For periodic inspection we mean an examination of the following parts:

- The walking cane frame
- Check the wear of rubber tip.

In case of repair are to be used only approved parts and accessories

10. Cleaning and disinfection

10.1 Cleaning

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove more difficult dirt. Dry thoroughly before use. Never use water with temperature above 30°C. Never use jet water or steam washing machine.

10.2 Disinfection

if you need to disinfect the device use a common disinfectant cleaner.

Note: Never use acids, alkalis or solvents such as acetone

11. Conditions of disposal

11.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

12. Technical features

12.1 Aluminum folding walking cane

Codes	Max weight	Height adjustable	Handle	Frame
RP145BC	100 kg	From 77,5 to 87,5 cm	T-wood	In anodized aluminum
RP145NC	100 kg	From 77,5 to 87,5 cm	T-wood	In anodized aluminum
RP145NL	100 kg	From 82,5 to 92,5 cm	T-wood	In anodized aluminum
RP159	100 kg	From 77,5 to 87,5 cm	T-wood woman model	In anodized aluminum

12.2 Aluminum height adjustable walking cane

Codes	Max weight	Height adjustable	Handle	Frame
RP146	110 kg	Da 77,5 a 100 cm	T-wood	In anodized aluminum
RP147	110 kg	Da 77,5 a 100 cm	T-wood	In anodized aluminum

RP148	110 kg	Da 77,5 a 100 cm	T-wood	In anodized aluminum
RP149	110 kg	Da 77,5 a 100 cm	T-plastic	In anodized aluminum
RP150	110 kg	Da 77,5 a 100 cm	Ergonomic	In anodized aluminum
RP151	110 kg	Da 77,5 a 100 cm	Curved	In anodized aluminum
RP152	110 kg	Da 77,5 a 100 cm	Sponge	In anodized aluminum
RP153D	110 kg	Da 77,5 a 100 cm	Anatomical right	In anodized aluminum
RP153S	110 kg	Da 77,5 a 100 cm	Anatomical left	In anodized aluminum
RP154	110 kg	Da 77,5 a 100 cm	T-handle silver check pattern	In anodized aluminum
RP156	110 kg	Da 77,5 a 100 cm	T-handle walnut pattern	In anodized aluminum
RP157	110 kg	Da 77,5 a 100 cm	T-handle wooden pattern	In anodized aluminum
RP158	110 kg	Da 77,5 a 100 cm	T-wood woman model	In anodized aluminum

12.3 Spare parts

RV7029 (2pz)

Brown rubber tip
Ø 18 mm

RV7028 (2pz)

Black rubber tip
Ø 18 mm

13. Warranty

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

13.1 Repairs

Warranty repairs In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

Repair not covered by warranty A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non-defective device The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer .

13.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

13.3 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

 MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com