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I Class Medical Device according to Directive 93/42/EEC and further modifications

1. Codes

CP800-40	Reclining wheelchair COMFY STEEL 40 cm - Self propelling model
CP800-43	Reclining wheelchair COMFY STEEL 43 cm - Self propelling model
CP800-46	Reclining wheelchair COMFY STEEL 46 cm - Self propelling model
CP805-40	Reclining wheelchair COMFY STEEL GO! 40 cm - Transit model
CP805-43	Reclining wheelchair COMFY STEEL GO! 43 cm - Transit model
CP805-46	Reclining wheelchair COMFY STEEL GO! 46 cm - Transit model

2. Introduction

Thank you for purchasing comfy steel reclining steel reclining wheelchair **ARDEA ONE** by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: check all parts for shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. Intended use

This wheelchair is intended for people with mobility problems and can be used for self-propelling or with the aid of an operator. In all cases however it requires the intervention of an operator for the various adjustments you require.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the wheelchair category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the said directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

5. General warnings

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment otherwise injury or damage may occur
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the wheelchair
- Keep the packed wheelchair away from heat sources
- SERVICE LIFE- the wheelchair use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the wheelchair
- Do Not use the wheelchair as seat on motor vehicle

6. General description

6.1 Parts view

Depending on the model of wheelchair you choose, the wheelchair will be composed by the following parts

1. Frame*
 2. Metal pole to stop backrest
 3. Headrest
 4. Elevating footrest
- * with big rear wheels (60 cm \emptyset)
or small rear wheels (30 cm \emptyset)
according to the model you have chosen

(FIG.1)

7. Assembling/disassembling

7.1 Open the wheelchair

Note: Refer to the safety warnings in this instruction manual

- Get the handles and pull outwards, then press the seat until the wheelchair is completely open (FIG.2)

(FIG.2)

- Insert the metal pole inside its supports, then lock the knobs. The frame is duly fixed (FIG.3)

(FIG.3)

7.2 Footrest assembling

- Insert the hinges on the frame holes, then rotate inwards until the lever is locked (FIG.4)

(FIG.4)

7.3 Headrest assembling

(FIG.5)

- Insert the headrest in the backrest support, then lock the knobs once the desired height is found (FIG.5)

8. Before use

- Check the mechanical parts usage, in order to guarantee a safe use of the device for people and things
- Assure all the parts are correctly assembled. Checking carefully the correct assembling of the rear wheels
- Assure the brake handles are well regulated

9. Warning for a correct use

- Do not use the wheelchair as seat for vehicle
- Assure the brake handles is duly adjusted and correctly works
- Avoid entrapment of finger into wheels or brakes during the wheelchair use
- The wheelchair temperature increase if exposed to the sun for long period

10. How to use

Note: Read general warnings before proceeding

WARNING!

- Getting in and out of the wheelchair may be performed independently, be sure you have an adequate mobility and upper body strength
- Do not stand on removable parts

DO NOT STAND ON THE FOOTREST WHEN GETTING IN OR OUT OF THE WHEELCHAIR, OVERTURNING RISK!!!

10.1 Get in the wheelchair

1. Position the wheelchair as close as possible to you
2. Engage wheel locks . We suggest to remove the armrest to ease the patient getting in the wheelchair
3. Lean on the frame with arms and slowly go down towards the seat

10.2 Get out of the wheelchair

1. Position the wheelchair as close as possible alongside the seat to which you are transferring
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting out of the wheelchair
3. Shift body weight into seat with transfer from the chair to seat you want transfer

10.3 Stairs and height difference (how to face them)

WARNING!

The following method must be used only from experienced patients. If you are not experienced, let you help from an assistant.

GET ON: Position the wheelchair at the curb to be crossed. Balance the wheelchair on the rear wheels, in order to lift the front wheels, enough to lean them on the obstacle. Bend your trunk forward and vigorously push the handrail. Continue pushing until the rear wheels are climbing over the obstacle.

GET OFF: To get off the stair follow the reverse procedure, positioning the rear side of the wheelchair at the curb, keep the handrails vigorously and get off the obstacle slowly; then balance the wheelchair on the rear wheels in order to complete the descent.

WARNING!

We recommend to use this method only in presence of obstacle lower than 10cm

10.4 Use the footrest (Fig.7)

Length adjustment- loosen the screw A using a 5mm Allen wrench. Adjust the footrest length then tighten the screw.

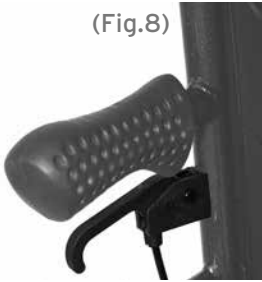
(Fig.7)

10.5 Use the reclining backrest

Press the release lever (Fig.8), push slightly the backrest downwards to adjust the inclination of the backrest. Gently push upwards to bring the angle of the backrest back to right-angled

(Fig.8)

**10.6 Use the 60 cm rear wheels**

Use rear wheels for self-propelling. Grasp the handrail and wheel through the desired direction.

10.7 Use the 30cm rear wheels

The rear 30cm wheels are used when wheel chair is driven by an assistant through the handles.

10.8 Use the rear handles

The handle is used to wheel a wheelchair with an operator aid, and to adjust the backrest inclination.

10.9 Use the anti-tip castors

The ant-tip castors are situated on the wheelchair backside.

To adjust the height pull the ring shown in Figure 11 and release once you have the correct height

(Fig.9)

10.10 Use the brakes

On the frame on the rear wheels, there is a mechanical brake operable by the patient or by the nurse. To block the wheels push the lever. To release the brake, pull the lever toward the rear. To lock the wheels push forward the brake lever. To unlock pull back the same lever. (Fig.11)

10.11 Carrying

The wheelchair has been designed to be easily carried. Once the armrest and the footrest are removed, can be closed and easily positioned in the car boot, for example. If this can't be possible, position the wheelchair in front of the car rear seat so that it is stable and cannot slip or slide. Lock the wheelchair with the seatbelt.

11. Maintenance

The ARDEA ONE devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair use only original spare parts. Before using follow the "BEFORE USE" chapter. Moretti ARDEA ONE wheelchairs require few elementary and periodic maintenance, below we list the checks to be carried out periodically:

- Braking capacity **Monthly**
- Seat cleaning **As needed**

12. Cleaning and disinfection

12.1 Cover and padded parts

The cover and padding are in Nylon. Clean with warm water and mild soap.

Note Do not use solvent and abrasive product

12.2 Brakes and wheels

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous using the wheelchair with wet wheels.

12.3 Frame and metal parts

Clean with warm water and mild soap. Dry with a soft cloth in order not to scratch the painting.

12.4 Disinfection

If you need to disinfect the device use a common disinfectant cleaner

Note: Never use acids, alkalis or solvents such as acetone

13. Conditions of disposal**13.1 General conditions of disposal**

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycling.

14. Spare parts/accessories

For spare parts and accessories refer to Moretti main catalogue

15. Technical features**15.1 Dimensions**

Dimensions and weight																
Measurement in cm																
CP800-40							40			62						
CP800-43	123	90	44	100/126	75	50	50/64	43	83	65	30	26	60	20	5	100
CP800-46								46		68						
Measurement in cm																
CP805-40							40			62						
CP805-43	123	90	44	100/126	75	50	50/64	43	83	65	25	30	20	5	100	
CP805-46								46		68						

15.2 Technical Specifications

- **Frame** Varnish steel frame
- **Coat** Nylon
- **Armrest** PU
- **Legrest** Padded
- **Headrest** Nylon
- **Front wheels** PU 20cm diameter
- **Rear wheels (CP800-xx models)** PU 60cm diameter
- **Rear wheels (CP805-xx models)** PU 30cm diameter
- **Backrest inclination angle** From 0° to 40°
- **Weight (CP800-xx models)** 26 kg
- **Weight (CP805-xx models)** 25 kg
- **Max weight** 100kg
- **Resistance to ignition** The materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2 EN12183
- **Reference regulation** -20°C/+ 40°C
- **Environmental temperature** Store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)
- **Storage**

16. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not covered by warrant: yA product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

Non defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

16.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

16.3 Exempt clauses

Moretti does not offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Adress _____ **Town/city** _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com