**MOPEDIA®**

MA ST3xx 02 A\_09-2016

**CUSCINI**  
**MANUALE D'ISTRUZIONE**  

---

**CUSHIONS**  
**INSTRUCTION MANUAL**

**Indice**

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I Class Medical Device  
according to Directive 93/42/EEC and further modifications

### 1. Codes

|                 |  |
|-----------------|--|
| <b>ST330-40</b> | inflating PVC ring cushion ø 40 cm                   |
| <b>ST330-42</b> | inflating PVC ring cushion ø 42,5cm                  |
| <b>ST330-45</b> | inflating PVC ring cushion ø 45 cm                   |
| <b>ST332-44</b> | round memory foam cushion ø 44 cm                    |
| <b>ST335-41</b> | memory foam cushion 41 x 32 x 8h cm                  |
| <b>ST335-48</b> | memory foam cushion 48 x 38 x 8h cm                  |
| <b>ST336</b>    | U-shape memory foam cushion 41 x 51 x 6,5h cm        |
| <b>ST335</b>    | No-prostatic memory foam cushion 44 x 37,5 x 7,5h cm |

### 2. Introduction

Thank you for purchasing a **MOPEDIA** cushion by Moretti S.p.A. the inflating ring cushions and the memory foam cushions are designed and realized to meet all your needs for a practical, proper and safe use. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

### 3. Intended use

The Mopedia by Moretti cushions are used to give relief to the patient during the session in the event of post-surgery or post-childbirth pain, pressure sores hemorrhoids and inflamed prostate.

#### WARNINGS!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

### 4. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the cane aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC

as prescribed by Annex VII of the above directive

2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

### 5. General warnings

- READ carefully the entire manual before using the device .
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- DO NOT allow children to play or operate the device

### 6. General description

#### 6.1 Inflating ring cushions

Inflatable ring cushions complete with pump for inflating are useful to give relief in case of post-operative and post-childbirth pain, pressure sores and hemorrhoids. It can be used on chairs, wheelchairs and any other flat surface.

#### 6.2 Memory foam cushions

Cushion made of high density molded foam polyuretane fire-resistant, designed to relieve pain related to long-term care, post-childbirth, post-operative and hemorrhoids. Also useful for reducing fatigue during long car trips. Can be used on any flat surface to ease the strain on the rectal area. Completely removable cover with polyester lining.

### 7. Before use

Always check the state of wear of the parts in such a way as to ensure it is used in complete safety for persons and property

### 8. Warnings for use

- Avoid to leave the device in contact with water for long periods of time
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks.

### 9. Maintenance

The MOPEDIA devices by MORETTI are checked carefully and supplied with CE mark, once launched on the market. For patient and physician's

safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

## 10. Cleaning

Washing indication for cover, where present:

Maximum temperature : 30°C

Do not bleach

Do not iron (do not use steam)

Do not dry clean, do not use stain removers with solvents

Do not tumble dry

**Note:** The foam can not be washed

- Inflating Cushion

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove dirt more difficult. Dry thoroughly before use.

## 11. Condition of disposal

### 11.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

## 12. Technical features

| CODES    | MATERIAL                  | SIZE                |
|----------|---------------------------|---------------------|
| ST330-40 | PVC                       | ø 40 cm             |
| ST330-42 | PVC                       | ø 42.5 cm           |
| ST330-45 | PVC                       | ø 45 cm             |
| ST332-44 | Memory foam (polyuretane) | ø 44 cm             |
| ST335-41 | Memory foam (polyuretane) | 41 x 32 x 8h cm     |
| ST335-48 | Memory foam (polyuretane) | 48 x 38 x 8h cm     |
| ST336    | Memory foam (polyuretane) | 41 x 51 x 6,5h cm   |
| ST355    | Memory foam (polyuretane) | 44 x 37,5 x 7,5h cm |

**Nota:** the memory foam used is fire-resistant and respect the EN1021 standard

### 13. Warranty

Moretti products are guaranteed from material or manufacturing faults for a period of ( two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product , and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual. Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair , damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti ), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

#### 13.1 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE**

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

**MADE IN P.R.C**

---

## Note

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)