GIMA Spa - Via Marconi, 1 - 20060 Gessate (MI) - Italia

ITALIA: Tel. 199 400 401 (8 linee r.a.) - Fax 199 400 403

E-mail: gima@gimaitaly.com - www.gimaitaly.com

INTERNATIONAL: Tel. ++39 02 953854209 - Fax ++39 02 95380056

E-mail: export@gimaitaly.com - www.gimaitaly.com

**Fonendoscopi GIMA - GIMA Stethoscopes -
Stéthoscopes biauriculaires GIMA - Fonendoscopios
GIMA - Estetoscópio GIMA - ΦΩΝΟΣΚΟΠΙΑ GIMA
- GIMA مسماع الأصوات الباطنية**

FONENDOSCOPI GIMA

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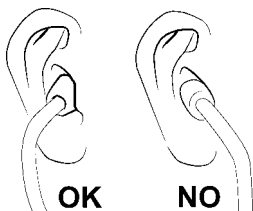
STETHOSCOPES GIMA

Congratulations for choosing a stethoscope of the series GIMA, as a guarantee of quality which has always characterized our products.

The product, according to the model and relevant features, is suitable to be used by various professional figures: cardiologist, general doctors, paediatricians, nurses and paramedics.

Bow adjustment

Position the bow so that its inclination is about 15° and the eartips are in line with the nose bridge (see picture). This way the sound will be sharp and high-pitched.

Adjustment of the bow tension

To reduce the tension of the bow, firmly hold the bow with your hands in the middle of the “Y” (tubes branching) and your thumbs on the sides. Bend the bow towards the outside until the desired tension is reached.

To increase the tension, seize the eartips and cross the tubes of the bow until the desired tension is reached.

Attention: An excessive tension could weaken the bow.

Replacement of the diaphragm

Although the diaphragm is solid and planned to last long, it could be necessary to replace it.

In this case proceed as follows:

- Remove the backing ring, paying attention not to damage it (for screw models, counter clockwise rotate the ring), then pull the diaphragm out.
- Fit the new diaphragm and reposition the backing ring making sure that it is well fixed (for screw models clockwise rotate the ring).

Selection of frequency / cushion – for double-head models only

180° rotate the head to change the listen modes: Select diaphragm to listen to high frequency sounds or open bell to listen to low frequency sounds.

To know which side is active, position the eartips on your ears and exert a light pressure on the diaphragm or on the bell hole.

With regular and accurate maintenance your stethoscope will give you excellent long-term performance:

- Avoid the contact with heat, cold sources, solvents and oils.
- Avoid exposing the stethoscope to direct sunlight for a long time and do not heat sterilize (autoclave).
- Do not dip the product in liquids. Clean it with alcohol or suitable detergent and a soft cloth.
- Avoid damaging the diaphragm and the lyre with sharpened objects.
- Periodically unscrew the eartips to clean them. Always check that they are correctly screwed down. Do not use the stethoscope without eartips and diaphragm.

GUARANTEE

Thank you for having acquired our product.

This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc.

Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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