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Materasso ad Aria K20 Plus.

**Destinazione d'uso**

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Air Mattress K20 Plus.

**Use**

The mattress K20 Plus has to be combined with Devicare R7E pump (code 05.03.R7E). The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Features	
Code	05.03.K20PLUS
Interchangeable Elements Material	PU • Polyurethane
Number of Elements	20
CELL in CELL Elements	The cell is divided in two independent compartments, one of which is the base always inflated and the other does the alternating part
Cells Height	21 cm
Dimensions Inflated	200 x 80 x 21 cm
Therapeutic Capacity	250 Kg
Air Mode	Two Ways Static and Dynamic (A-b-A-b) – Base Always Inflated
AUTODETECT SYSTEM: digital system of automatic weight and patient position detection	
TRANSPORT connector: allows to keep the air in the mattress for 12 hours without the pump working	
Quick release connectors CPC ONE WAY which allows to change a cell without deflating the mattress	
Straps at the ends for attachment to the bed	
External cover in fireproof Polyurethane with 360° zip on 4 sides	
CPR valve for quick deflation	
First three elements always inflated	
Optional: width 90 cm	
Cover Maintenance and Cleaning	
 Water temperature not above 95° C	 Do not iron
Sterilization is possible in autoclave up to 134° C, rubber cycle	
Mattress Maintenance and Cleaning	
Clean the mattress with a damp cloth and mild soap	
Disinfection is possible with 5% diluted disinfectants even with alcohol base	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	04.33.06.018
CND Code	Y033306
RDM code	664548/R