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Materasso ad Aria Supercare T01.

Destinazione d'uso

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Air Mattress Supercare T01.

Use

The mattress Supercare T01 has to be combined with one of this pumps: Vircare (code 05.03.6600A1), Butterfly (code 05.03.6600A2), Prestige (code 05.03.6600A4) e Supercare P08 (code 05.03.P08). The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Features	
Code	05.03.T01
Interchangeable Elements Material	Nylon / PVC
Number of Cells	18
Cells Height	12 cm
Dimensions Inflated	200 x 90 x 12 cm
Therapeutic Capacity	160 Kg
Air Mode	Two Ways Static and Dynamic (A-b-A-b)
Flaps and straps at the ends for attachment to the bed	
External cover in fireproof Nylon with press studs	
Optional: CPR valve for quick deflation and first three elements always inflated	
Cover Maintenance and Cleaning	
Disinfection is possible with 5% diluted disinfectants even with alcohol	
Mattress Maintenance and Cleaning	
Clean the mattress with a damp cloth and mild soap	
The mattress has to be dry before use, do not expose it to direct sunlight	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	04.33.06.018
CND Code	Y033306
RDM code	663507/R