IntenseWaves

PRESSOMASSAGE

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ENGLISH

Index

| | |
|--|-----------|
| Section 1 | 44 |
| Declaration of conformity | 45 |
| Manufacturer information | 47 |
| Manual presentation | 48 |
| Conventions..... | 49 |
| Section 2 | 50 |
| Warranty | 51 |
| General safety rules | 53 |
| Caution | 53 |
| Safety rules | 53 |
| Contraindications | 54 |
| Description | 55 |
| Technical characteristics | 55 |
| Technical information | 55 |
| Terminology..... | 55 |
| Program | 56 |
| Label..... | 56 |
| Direction of inflation | 56 |
| Configuration of applicators | 57 |
| Intended and unintended use | 58 |
| Intended use | 58 |
| Unintended use..... | 58 |
| Residual Risks..... | 58 |
| Pictograms | 59 |
| Caution | 60 |
| Safety devices..... | 61 |
| Section 3 | 62 |
| Unpacking instructions | 63 |
| Handling..... | 63 |
| Locations..... | 63 |
| Handling procedures | 64 |
| Connection and disconnection..... | 65 |
| Preliminary checks | 65 |
| Caution before, during and after use | 66 |
| Caution | 66 |
| Configuration..... | 68 |
| Operating instructions of the device..... | 69 |
| Pressure..... | 70 |

| | |
|---|-----------|
| Time | 70 |
| “Off-On” button | 70 |
| Applicators and accessories..... | 71 |
| Legs | 71 |
| Abdominal/glutes band | 71 |
| Armband | 71 |
| Single connector | 71 |
| Splitter | 71 |
| Operating instructions of the applicators | 73 |
| Section 4..... | 76 |
| Maintenance | 77 |
| Safety | 77 |
| Periodical maintenance..... | 78 |
| Extraordinary maintenance | 78 |
| Decommissioning | 78 |
| Disposal | 79 |

SECTION 1

Declaration of conformity

Manufacturer information

Manual presentation

Declaration of conformity

The pressomassage device described in this documentation is accompanied, upon sale, with the declaration of conformity drawn up in accordance with the European legislation.

NOTE

BEFORE USING THE DEVICE WHATSOEVER, CHECK THE PRESENCE OF THE DECLARATION OF CONFORMITY.

NOTE

IF THE DEVICE IS TRANSFERRED TO THIRD PARTIES, ALL THE DOCUMENTATION MUST BE HANDED OVER WITH IT.

Declaration of conformity

| | |
|--|--|
| The undersigned Walter Bottega | |
| As legal representative of Vima S.r.l. | |
| Having its headquarters in: Via Pigafetta, 65 – 10129 – Torino (TO) | |
| VAT Number: 11824140013 | |
| <i>Hereby declares</i> | |
| That the product: Device for presso-massaggio | |
| Model and code: IntenseWaves Pressomassage - PSG-IW | |
| Date of Manufacturing: see label | Batch Number or Serial Number: see label |
| Was manufactured in compliance with the following directives and standards: | |
| <ul style="list-style-type: none"> • Directive 2014/35/EU known as "Low voltage Directive" • Directive 2014/30/EU known as " Directive on electromagnetic compatibility " • Directive 2011/65/EC known as "RoHS" • Directive 2015/863 amending Directive 2011/65/EC • Directive 2012/19/EU known as "WEEE" • Directive 2001/95/EC known as "General product safety" • Directive 2009/125/EC establishing a framework for the setting of eco-design requirements for energy-related products • Italian Legislative Decree No. 206 of 15 October 2015 • IEC 61882:2016 standard on the method of analysis of risks according to the HAZOP procedure • IEC 61511-1:2016 standard on Functional safety - Safety instrumented systems for the process industry sector - Part 1: Framework, definitions, system, hardware and application programming requirements • UNI EN ISO 7010:2017 Title: Graphic signs – Safety colours and signs – Recorded safety signs • IEC 60335-2-32:2002 (Fourth edition) + A1:2008 + A2:2013 used in conjunction with IEC 60335-1:2010 (Fifth Edition) incl. Corr. 1:2010 and Corr. 2:2011 + A1:2013 • EN 55014-1:2017, EN 55014-2:2015 • EN 61000-3-2:2014, EN 61000-3-3:2013 • EN 62233 2008, EN 50564:2011 | |
| And therefore complies with the directives and regulations in force. | |
| This declaration of conformity is issued under the exclusive responsibility of the manufacturer. | |
| In: Turin | Signature: |
| On: 24/11/2020 | |

Via Lauro, 95 Cadoneghe Padova Italy
 Advisors and Experts for: Guardia di Finanza (Italian Finance Police), Tributaria (Italian Tax Police), Customs Authorities, Carabinieri, State Police, Unioncamere (Italian Union of Chambers of Commerce, Industry, Handicraft and Agriculture) Courthouses.

The correct drafting of the technical report, as well as the documents prepared by C&C s.a.s., have been verified by Ing. Renato Carraro.

Manufacturer information

| | |
|---|---|
| Company name | Vima S.r.l. |
| Company headquarters | Via Pigafetta, 65 – 10129 – Turin (Italy) |
| VAT Number | 11824140013 |
| R.E.A. Number | Turin 1243754 |
| Italy AEE Register (Register of electric and electronic products) | No. IT18040000010309 |
| PEC address | vimasrlpec@legalmail.it |
| Telephone and Fax number | Tel. (+39) 011.0447816 Fax. (+39) 011.0432817 |
| E-mail address | info@mesis.eu |
| Website | https://www.mesis.eu/ |

AUTHORISED SERVICE CENTRE

The manufacturer carries out the service on the product. In case of need, it can be contacted by using the details above.

Manual presentation

NOTE

UPON RECEIVING THE DEVICE, CAREFULLY READ THIS MANUAL BEFORE CARRYING OUT ANY OPERATION.

This manual includes the instructions for the connection, use and maintenance of the IntenseWaves Pressomassage mod. PSG-IW pressomassage device for beauty treatments. The manual consists of various sections, each of which deals with a number of topics, divided by chapters and paragraphs.

The general index provides a list of all the examined topics of the whole manual. The number of pages is progressive and each page includes its number. This manual relates to the technical life after production and sale of the device, it provides instructions for the connection, use and maintenance of the IntenseWaves Pressomassage device and is intended for private user for personal use and operators of beauty centres.

Should it be handed over to third parties for any reason whatsoever (sale, free loan, or any other reason), the device must be delivered with all its documentation.

Before starting any operation you must have read at least the whole manual and then go into detail about the topic of the operations you intend to perform.

This manual contains proprietary information that is confidential and may not be provided to third parties for any use and in any form whatsoever without the prior written consent of Vima S.r.l..

Vima S.r.l. declares that the information included in this manual is consistent with the technical and safety specifications of the IntenseWaves Pressomassage device whose material refers to. Photos and characteristics of the product refer to the time in which the manual was printed out, minor discrepancies may be possible during production. Any discrepancy does not affect the safety and performance of the product.

The certified true copy of this manual is stored in the technical file of the device, filed at Vima S.r.l..

Vima S.r.l. does not recognize any documentation that is not produced, issued or handed out by itself or its authorised representative.

This manual and its technical file shall be kept by the manufacturer for the period provided by law.

During such period, copy of the documentation accompanying the product at the time of its sale may be requested.

The entire technical file remains available for such period solely for the supervisory authorities which can request a copy of it.

After this period, it will be the obligation and care of those who manages the product to make sure that the product and its documentation comply with the laws in force at the time of the inspection.

Conventions

For the purposes of better understanding the topics, the manual uses the graphic and printing symbols and conventions described below.

Graphic caution conventions

NOTE

NOTES CONTAIN IMPORTANT INFORMATION HIGHLIGHTED OUTSIDE THE TEXT WHICH THEY REFER TO

WARNING

WARNING INDICATIONS REFER TO THE PROCEDURES WHOSE COMPLETE OR PARTIAL NON-COMPLIANCE MAY CAUSE DAMAGE TO THE DEVICE, ITS COMPONENTS AND MAY EXPOSE AN OPERATOR/USER TO HAZARDS.

HAZARD

HAZARD INDICATIONS REFER TO THE PROCEDURES WHOSE COMPLETE OR PARTIAL NON-COMPLIANCE MAY CAUSE DAMAGE OR INJURIES TO THE HEALTH OF THE OPERATOR/USER OR OTHER PEOPLE NEARBY, SEVERE DAMAGE OR LOSS OF THE DEVICE.

SECTION 2

Warranty

General safety rules

Description

Technical characteristics

Intended and unintended use

Residual Risks

Pictograms

Caution

Safety devices

Warranty

The warranty standards, which are entirely listed in the purchasing contract, apply only if the IntenseWaves Pressomassage device is used according to the conditions of intended use.

Barring the ordinary maintenance interventions described in section MAINTENANCE and carried out with the procedures indicated, any repair or modification to the device made by the user or unauthorised companies shall result in the voidance of the warranty.

The warranty does not cover the damage caused by inexperience or negligence in the use of the device or by poor or omitted maintenance.

Our products are covered by warranty. The warranty of the device is subject to the following conditions:

| | |
|---|---|
| 1 | The device is guaranteed for twenty-four (24) months for the first user, three (3) months for applicators and accessories subject to wear and deterioration, from the date of purchase against material or manufacturing defects, as long as it is used properly and maintained under normal conditions of use. No warranty for "consumable" accessories and materials is foreseen. |
| 2 | The Manufacturer undertakes to replace, at its own discretion, parts that are malfunctioning or incorrectly manufactured only after careful inspection and detection of poor manufacturing. |
| 3 | The transport and/or shipping costs shall always be borne by the buyer in case of incorrect use of the terms of warranty. |
| 4 | During the period of warranty, the replaced products become property of the manufacturer. |
| 5 | Only the original buyer who have complied with the instructions of standard maintenance described herein can benefit from this warranty. Our responsibility over the warranty expires when: the original owner transfers the property of the product or changes to it have been made. |
| 6 | The warranty does not cover damage resulting from excessive stress such as the use of the product after identifying an anomaly, from the use of unsuitable methods of use as well as failure to comply with the user and maintenance instructions. It does not cover issues due to fall or incorrect use of voltage either. If the device has been opened or modified by unauthorised staff. If the device has been used in unsuitable environments. If the device has been used without complying with the instructions of this manual. |
| 7 | The manufacturer assumes no responsibility for any difficulty that may arise while reselling or using the product outside Italy due to the provisions in force in the Country where the product is sold. |
| 8 | The product or one of its faulty components must be returned to the Manufacturer for the replacement. If this is not the case, the product or the replaced part shall be charged to the buyer. |
| 9 | The warranty is valid only if the intact seal of warranty is present. |

Our products are covered by warranty. The warranty of the device is subject to the following conditions:

| | |
|----|--|
| 10 | <p>In order to benefit from the warranty service:</p> <ul style="list-style-type: none"> • Products must be returned for repair in their original packaging. • The warranty of the product shall be subjected to the submission of a fiscal document (receipt, invoice or document of sale made by correspondence via an online store or catalogue), attesting the date of purchase of the product. • The repair shall have no effect on the original expiry date of the warranty and shall not result in the renewal or extension thereof. • If, at the time of the repair, no faults of the product are found, the costs for the technical inspection shall be charged anyway. • The warranty is voided if the fault is caused by: shocks, fall, incorrect or improper use of the product, use of non-original power supply or charger, accidental events, product alteration, replacement/detachment of the seals of warranty and/or tampering of product. • In addition, the warranty does not cover damage occurred during transport due to the use of unsuitable packaging. Any damage caused by the use of the device under exceptional circumstances (outdoor use, use in wet and damp areas and direct exposition to sunlight etc.) is not covered by warranty. • The warranty does not cover the impossibility of use of the product, other incidental or consequential costs or other costs incurred by the buyer. |
| 11 | For any controversy is the exclusive jurisdiction of the court of Turin. |

Notice: should the use of the warranty due to fault or malfunctioning be necessary, we invite you to:

| | |
|---|--|
| 1 | Turn to your reseller. |
| 2 | Indicate the type, brand, name, model and serial number of the device. |
| 3 | Indicate the date of purchase by submitting the purchasing document. |
| 4 | Provide detailed description of the problem. |

NOTE

FAILURE TO COMPLY WITH THE METHODS OF INTERVENTION AND USE OF THE DEVICE DESCRIBED HEREIN SHALL RESULT IN THE VOIDANCE OF THE TERMS OF WARRANTY.

General safety rules

Caution

In case of doubts contact your GP before using the device.

If part of the documentation was even partially illegible, refer to Vima S.r.l. before carrying out any further operation with the IntenseWaves Pressomassage device.

This chapter provides a description of the general safety rules to observe during any type of operation performed with the device. The procedures of intervention described in the following chapters must be carried out by complying with both the methods of execution provided and the general safety rules of this chapter.

The general safety rules and the procedure for connection, use and maintenance provided herein are an addition to the general safety rules at work that must be complied with.

Different Nations can have different regulations on safety. Therefore, it should be noted that, in any case whatsoever in which the rules of documentation are in conflict with or reductive than the rules of the Country where the device is used, the rules of the Country shall in any case have priority over those of the documentation.

NOTE

THE MANUFACTURER CANNOT BE HELD IN ANY CASE RESPONSIBLE FOR ACCIDENTS OR DAMAGE RESULTING FROM THE INAPPROPRIATE USE OF THE DEVICE, AS WELL AS FOR NON-COMPLIANCE, EVEN PARTIAL, WITH THE SAFETY RULES AND PROCEDURES OF INTERVENTION DESCRIBED HEREIN.

Failure to comply with the operating rules and methods of intervention, connection, use and maintenance of the IntenseWaves Pressomassage device described herein shall also result in the voidance of the terms of warranty.

Safety rules

During the connection and the subsequent use of the device, incorrect or unforeseen situations of functioning may occur. These situations, which are completely unusual, may be sometimes caused by environmental factors or unpredictable accidental faults. Should problems of any nature be found, contact the manufacturer to receive technical support.

The manual must be kept by the staff in charge of the management, maintenance and use of the IntenseWaves Pressomassage device.

In case of deterioration or loss, customer can request a true copy from the manufacturer. We recommend to keep a back-up copy in a place where it cannot be damaged or lost.

WARNING

SINCE IT WOULD BE IMPOSSIBLE TO OUTLINE ALL THE OPERATIONS THAT MUST NOT OR CANNOT BE CARRIED OUT, PLEASE CONSIDER THAT ALL THE OPERATIONS (DIFFERENT FROM STANDARD USE) THAT ARE NOT CLEARLY DESCRIBED IN THE MANUAL PROVIDED WITH THE DEVICE SHALL BE CONSIDERED UNFEASIBLE.

Contraindications

Refer to your GP to check any diseases that are not compatible with the massage of Mesis IntenseWaves Pressomassage. Below is a non-comprehensive list of any incompatible diseases.

- People suffering from inflammations or wounds with risk of bleeding either internal or external must not use the product, including those suffering from purulent inflammation
- Do not use in case of general ill feeling, flu or illness
- Pulmonary oedema
- Thrombophlebitis
- Heart failure
- Infections
- Deep vein thrombosis
- Episodes of pulmonary embolism
- Injuries, lesions or tumours in the area of application or in the vicinity where an increased venous and lymphatic return could be harmful
- Carriers of active implants such as: pacemaker, insulin pump or others
- Lymphangitis
- Plexopathy and neuropathy
- Carriers of metal joint prostheses
- People with inflammatory processes in progress, with skin lesions, neoplasms
- Pregnant women
- People with sensitive skin, possibly limit to very low spraying
- Apply on intact skin only

Do not use in the following cases:

- People with capillary fragility, arterial hypertension or heart diseases
- People with internal defibrillator
- People with sensitive skin, possibly limit to very low spraying
- Do not use the product directly on swollen or inflamed areas or skin rashes

In case of doubts, refer to your GP before use.

Description

The device described in this document is intended for beauty centres and end customers for private use for the execution of pressure massages.

The device is driven by an electric pump which lets air flowing into pads of various forms and dimensions which, in turn, are applied as such or are introduced into specific containers made of fabric, plastic or other suitable material.

The adjustment of the massage pressure is performed by a potentiometer and is controlled by a measuring tool and a safety device.

In addition, the device is equipped with a device that regulates the duration of pressure emission, pause, as well as any program sequence on the different pads.

It complies with the requirements indicated in technical sheet No. 18 of Italian Decree No. 206 of 15 October 2015 which defines it as: "Device for pressomassage".

Technical characteristics

- La Mesis IntenseWaves Pressomassage is sequential and intermittent
- Adjustable and continuous pressure
- Optimum massage through 4 chambers
- Integrated timer for the adjustment of the duration of massage
- Control valve to ensure constant pressure
- Quick accessory connection and easy machine use

Technical information

| | |
|--------------------------|---|
| Power input | 220/240 Vac \pm 10% |
| Frequency | 50/60 Hz |
| Power consumption | 30 W |
| Treatment time | Adjustable range between 0 and 30 minutes |
| Weight | 2 kg |
| Dimensions | 240 X 200 X 115 mm |
| Programs | 1 program |
| Applicators | 4 chambers |

Before connecting the device, check that the voltage data indicated on the ID plate correspond to that of the power supply available. The ID label is placed on the device.

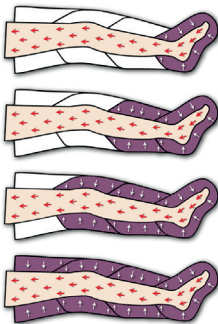
Terminology

- **Program:** chamber inflation mode
- **Chambers or Sectors:** elements which the applicator is made up of and which fill up with air. In Mesis IntenseWaves Pressomassage there are 4 applicators chambers.
- **Cycle:** 1 cycle corresponds to the inflation of all chambers, from the first one to the last one, regardless of the program used. After inflating the last chamber, there is a pause and then a new cycle starts
- **Applicator:** Pneumo-distension element made up of several chambers that inflate on a

sequential basis according to the chosen program (legs, armband and abdominal/glutes band)

- **Time:** duration of session (adjustable from 1 to 30 minutes)
- **Pressure:** amount of air injected into the chambers. Pressure is adjustable and continuous
- **Connector:** it is an accessory made up of joined rubber tubes and allows to connect the applicators to the machine or to the splitter
- **Splitter:** it is an accessory made up of joined rubber tubes. The use of 1 splitter allows to use the two legs while the use of 2 splitters allows to use three accessories at a time, e.g. the two legs and the abdominal/glutes band
- **Spout:** end (air outlet) of each tube of the connector which has to be inserted into the grafts located on the applicators
- **Graft:** seat located on the applicators where the connector spouts have to be inserted

Program

The chambers are filled with air one at a time, keeping the previous ones inflated until fully inflated.

Label

| | |
|---|---|
| VIMA s.r.l. - Via Pigafetta 65, 10129 Turin, Italia | |
| info@mesis.eu www.mesis.eu fax: +39 011.0432817 | |
| Item: device for presso-massage | |
| Model: PSG-IW | |
| Trade name: Intense Waves PressoMassage | |
| S/N | |
| Year 2020 | |
| Made in Cina | |
| Input: 220 - 240 Vac, 50/60Hz | |
| Power: 30 W | |
| Weight: 2 kg | |
|  | |
|  |  |
|  |  |
|  |  |

Label under device

Direction of inflation

Armband: direction of inflation

Band: direction of inflation

Legs: direction of inflation

The arrows of the pictures above show the correct direction of inflation of the applicators:

- If you use the armband, the inflation starts from the hand and ends at the shoulder.
- If you use the abdominal/glutes band, the inflation starts from the upper part of the abdomen and goes on downwards.
- If you use the legs, the inflation starts from the foot and ends at the groin.

Configuration of applicators

| | | |
|--|--|--|
| <p>Cod. PSG-IW-1B</p>  <p><i>Applicators:</i> 1 armband <i>Accessories:</i> 1 single connector</p> | <p>Cod. PSG-IW-F</p>  <p><i>Applicators:</i> 1 band <i>Accessories:</i> 1 single connector</p> | <p>Cod. PSG-IW-1G</p>  <p><i>Applicators:</i> 1 leg <i>Accessories:</i> 1 single connector</p> |
| <p>Cod. PSG-IW-2G</p>  <p><i>Applicators:</i> 2 legs <i>Accessories:</i> 2 single connectors and 1 splitter</p> | <p>Cod. PSG-IW-2GK</p>  <p><i>Applicators:</i> 2 legs and 1 band <i>Accessories:</i> 3 single connectors and 2 splitters</p> | <p>Cod. PSG-IW-2GKB</p>  <p><i>Applicators:</i> 2 legs, 1 band and 1 armband <i>Accessories:</i> 4 single connectors and 2 splitters</p> |

Intended and unintended use

Intended use

Device intended to be used in households and beauty centres for the performance of pressure massages.

Unintended use

No use other than those described in paragraph DESCRIPTION AND INTENDED USE is foreseen.

It is also strictly prohibited to:

- Use the device with unsuitable supply lines.
- Use the device for applications other than those described.
- Use the device in beauty centres by unauthorised operators.
- Make changes to the device.
- Use the device in environments other than those foreseen. Install it in bathrooms.

Reusing any unit after decommissioning of the device relieves the manufacturer of any responsibility resulting from its use.

NOTE

THE MANUFACTURER CANNOT BE HELD IN ANY CASE RESPONSIBLE FOR ACCIDENTS OR DAMAGE RESULTING FROM UNINTENDED USE OF THE DEVICE. IN ADDITION, ANY UNINTENDED USE OF THE DEVICE SHALL RESULT IN THE VOIDANCE OF THE TERMS OF WARRANTY.

Residual Risks

During the design stage, Vima S.r.l. carried out a detailed risk analysis on the inspected device. Such analysis identified unavoidable risks by their nature. Such risks were then examined individually and this manual emphasizes the instructions on how to avoid them. It is therefore important for any user and/or operator in charge of the connection, use and maintenance of the IntenseWaves Pressomassage device to read the manual beforehand.

NOTE

THE MANUFACTURER CANNOT BE HELD IN ANY CASE RESPONSIBLE FOR ACCIDENTS OR DAMAGE RESULTING FROM UNINTENDED USE OF THE DEVICE DUE TO NEGLIGENCE OF ONE OF THE OPERATORS.

In particular:

- It is strictly prohibited to make any changes to the device. Any damage to people, animals or objects resulting from the use of the device that has been improperly modified by the user and/or an unauthorised operator, relieves the manufacturer of any responsibility.
- Keep this manual carefully, which is necessary for a correct and safe use of the device.
- Check the conditions of the label installed on the product on a periodical basis and restore it in case of damage (if this is the case, contact the authorised service).
- If the external framing of the device and/or one of its components has sharp edges or borders following accidental shocks such as to make them dangerous, it is necessary to contact the authorised service and follow their instructions.

HAZARD

ACCESS TO CLOSED, PROTECTED OR INTERNAL PARTS OF THE DEVICE MUST BE CARRIED OUT ONLY FOR EXTRAORDINARY MAINTENANCE OPERATIONS AND BY AUTHORISED STAFF OF VIMA S.R.L. ONLY.

Important:

- Do not wet electrical wiring with water or other liquids.
- It is necessary to carry out the maintenance works described in the manual.
- The maintenance operations must be carried out by following the manual's instructions.
- The Section "Maintenance" indicates the operations in which it is necessary to unplug the IntenseWaves Pressomassage device from the power outlet.

Pictograms

The following pictograms are illustrated on the label under the IntenseWaves Pressomassage device:

Pictograms key

Read the instructions

Indoor use only

CE mark

Electrical insulation: Class II

Disposal as WEEE

Danger: Risk of electric shock

Compliant with RoHS Directives

WARNING

Safety warning signs must not be removed, covered or damaged.

Caution

- In beauty centres, only adequately trained staff can operate with the beauty device.
- Do not start the treatment if the leg, armband and/or band are not correctly positioned.
- Do not leave the machine unattended when it is ready for use.
- Turn the switch to 0 (zero) or OFF after use.
- Keep away from children. The device can be used by children of at least 8 years of age and by people with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, as long as they are supervised or trained on how to use the product and have understood the hazards thereof. Children must not play with the device. Cleaning and maintenance intended to be performed by the user must not be carried out by children without supervision. Keep the device and its cable away from children under 8 years of age.
- Do not perform any intervention within the device if this is plugged to the electrical system.
- After switching off, wait at least one minute before restarting the device.
- Do not open the device for any reason whatsoever. Due to high current intensity used, electrical discharges within the device may occur even when it is unplugged from the electrical system.
- Interventions within the device can be performed by authorised staff only.
- Before starting a treatment, position the device in such a way as it may not hinder the transit and the electrical cable does not represent an obstacle with subsequent risk of stumbling.
- Avoid positioning the device close to sources of heat and where it runs the risk of being hit by steam jets or liquids.
- Place the device in order to make sure that the equipment, cables and tools can cover the necessary distance by keeping the device in the position of maximum stability.
- Do not use the device if even slight anomalies of functioning are detected.
- Make sure the connection, work, and maintenance area of the device has lighting of at least 200 lux.
- In order to ensure better protection, it is recommended to install a differential current device in the electrical circuit whose rated operating current does not exceed 30mA (ideal 10mA). Ask your installer for advice.
- Before connecting the device, check that the voltage data indicated on the ID plate correspond to that of the power supply available.
- The ID label is placed on the device.
- DO NOT use the device with wet hands, wet feet or barefoot.
- DO NOT pull the power cord or the device itself to disconnect the plug from the power outlet.
- DO NOT expose the device to moisture or the influence of atmospheric agents (rain, sunlight).
- DO NOT block the air slots of the device and do not lean the device on a soft surface such as a bed or sofa, where you can block the slots.
- In case of fault or malfunctioning of the device, switch it off and do not tamper it.
- The device must be used exclusively indoor and in compliance with the user instructions.
- Do not place the device close to sources of heat.
- The packaging material is not a baby toy! Keep plastic bags away from children. Choking hazard!

Safety devices

- Power switch: use the switch to switch on and switch off the device.
- Automatic timer: the automatic timer automatically interrupts the treatment once completed.

Pressure adjustment

Time adjustment

Power on and power off button

SECTION 3

Unpacking instructions

Handling

Connection and Disconnection

Caution before, during and after use

Operating instructions of the device

Applicators and accessories

Operating instructions of the applicators

Unpacking instructions

The IntenseWaves Pressomassage device is provided into a rigid box.

According to the purchased configuration, into the box you can find:

- Compressor
- 4-chamber legs
- 4-chamber armband
- 4-chamber abdominal/glutes band
- Slim Body Kit, abdominal/glutes band + single connector + splitter for simultaneous user of the two legs 2 with the band
- Single connector (for band, leg or armband)
- Splitters
- Multi-language manual

The optional and spare accessories can be purchased at Vima S.r.l..

1. Unpack the device carefully.
2. Check that no damage due to transport is present.
3. Check that the standard equipment is complete.
4. If a component is missing or damaged, we invite you to refer to your reseller within and no later than 5 days from delivery.
5. The packaging material is not a baby toy! Keep plastic bags away from children. Choking hazard!

Handling

Locations

It is important to bear some aspects in mind before proceeding with the handling and positioning of the product (either for the use or the storage). In particular, it is necessary to check some factors:

- The IntenseWaves Pressomassage device must be positioned in non-corrosive atmosphere. Corrosives such as acids can ruin the surface of cables, electronic and optical components.
- The position of the IntenseWaves Pressomassage device must be chosen so that it may not be hit by steam jets, water or other liquids.
- Position the IntenseWaves Pressomassage device in a ventilated place and DO NOT obstruct its aeration grids. Dust particles in the air must be reduced to a minimum as they can damage the surface of the optical components. The metal particles, if any in the air, can corrode the electronic components.
- The minimum distance between the device and the wall must be at least 50 cm (19.68 in) during operation.
- The temperature of the environment where the device is stored must be between 10°C (50°F) and +55°C (131°F).
- The temperature of the environment where the device is used must be between 10°C (50°F) and +40°C (104°F).
- Moisture conditions between 30% and 85%.

- Atmospheric pressure: 86kPa ~ 106kPa.
- The support against which the IntenseWaves Pressomassage device has to be leant must bear its weight and have physical characteristics such as to ensure its safe positioning, without the risk of falling.
- Do not lean the device on a soft surface such as a bed or sofa, where you can block the slots.
- Do not position the device with its accessories in particularly cold areas or in areas with temperature below zero. It can be damaged seriously.
- If you do not use the device for a long period, store it in its box.

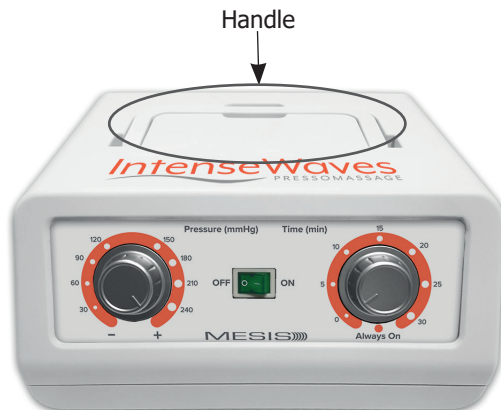
The IntenseWaves Pressomassage device can be used only indoor by complying with the factors mentioned above.

Do not position the IntenseWaves Pressomassage device during use close to: televisions, monitors, computers, mobile phones, magnetic tapes/discs, medical appliances/prosthesis etc.

The device must be leant against a suitable horizontal support, close to a power source which allows it to be connected safely (see "**CONNECTION**").

Handling procedures

Given its dimensions and weight, the IntenseWaves Pressomassage device can be moved manually. On the upper part of the device there is an ergonomic handle that allows it to be grasped easily.

Accessories must be grasped carefully. DO NOT grasp them by cables/pipes. While handling the accessories, avoid that cables and pipes crawl on the ground.

Electrical connection cables must never be twisted and pressed in the space between the device and the supporting surface.

Connection and disconnection

All beauty devices are tested and developed by the manufacturer before shipping and delivery to the customer.

The product is designed, manufactured and tested to meet all the specific standards (see declaration of conformity), when correctly connected.

If the connection, use and/or maintenance of the IntenseWaves Pressomassage device are not implemented perfectly, anomalies during use may occur. In addition, safety problems may take place. Inappropriate connection and maintenance shall void the terms of warranty.

WARNING

THE DEVICE CAN BE ACTIVATED ONLY AFTER BEING CONNECTED PROPERLY.

Before connecting and activating the IntenseWaves Pressomassage device, remove any protection against dust, moisture and any packaging material.

Preliminary checks

Check of any damage

Check that the different parts of the device do not have physical damage due to shocks, tears or abrasion.

In particular, check that:

- There are no scratches or dents, indicating shocks occurred during transport.
- The electrical power cable and its plug are not damaged.
- No water, moisture entered during transport and/or condensation formed. If this is the case, wait at least 30 minutes before carrying out any further operation.

The lighting system (natural and/or artificial) of the area of connection must guarantee the following minimum lighting value: **200 lux**.

Damage claim procedure

If any damage is identified, interrupt the procedure underway and report the nature of damage found to the customer service office of the reseller or manufacturer.

Connection, disconnection

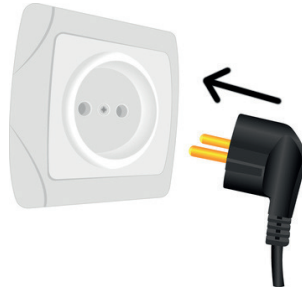
NOTE

THE MANUFACTURER CANNOT BE HELD IN ANY CASE RESPONSIBLE FOR ACCIDENTS OR DAMAGE RESULTING FROM THE USE OF THE DEVICE IN AN ELECTRICAL SYSTEM THAT DOES NOT COMPLY WITH THE LEGAL REQUIREMENTS.

Before connecting/disconnecting the machine from the power supply, make sure to switch it off and turn the general switch to OFF.

Now you can connect/disconnect the machine from the power socket.

To connect the device, just insert the power jack into the input of the device and insert the plug into a power socket.

For the preliminary actions to be performed on the device, its connection to the accessories (applicators), electrical outlet and its use, see chapter "OPERATING INSTRUCTIONS OF THE DEVICE" and "OPERATING INSTRUCTIONS OF THE APPLICATORS"

Caution before, during and after use

Caution

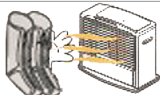
In the event of any unforeseen and hazardous anomaly during configuration and use of the IntenseWaves Pressomassage device, we recommend following the procedure below:

- Switch it off (see Section 3).
- Unplug it from the power socket by pulling out the plug from the power outlet into which it was connected.
- Contact technical service immediately.

In the event of unforeseen electromagnetic anomalies after connecting or powering up the IntenseWaves Pressomassage device, e.g. impossibility to communicate by telephone or smartphone, we recommend following the procedure below:

- Switch it off (see Section 3).
- Unplug it from the power socket by pulling out the plug from the power outlet into which it was connected.
- Contact technical service immediately.

Accessories cannot be repaired. Take care of them

| | | |
|---|---|--|
| Do not position legs, armbands, abdominal band or other accessories close to direct or indirect sources of heat. |  |  |
| Do not position the device in damp or wet areas. Do not wet the device under any circumstances. Do not pour any liquid on the device. |  |  |
| Do not clean the device or accessories with water or detergents. Simply wipe with a damp cloth. |  |  |
| Do not store the device with its accessories in particularly cold areas or in areas with temperature below zero. It can be damaged seriously. |  |  |
| If you do not use the device for a long period, store it in its box. | |  |

Usage environment and case

- Use and keep the device and its accessories in a clean environment with a temperature between 10 – 40°C (50 – 104°F), moisture 30 – 85%.
- Do not use outdoors.

| Problem found | Possible solution |
|---|---|
| The device does not power up | <ul style="list-style-type: none"> • Check correct connection to electrical outlet. • Make sure you have turned the switch (off-on (0 - I) button) to position ON (I). • Check the status of the power cord which must be intact. |
| A strange hiss similar to that of an air leak is perceived | <ul style="list-style-type: none"> • Check that all pipes, accessories and connectors are intact. • Make sure that all connections (to the machine and the applicators) are correctly inserted. • Make sure that the air connection pipes are not bent, pinched or crushed. |
| The air does not arrive at the applicators, there is not enough pressure | <ul style="list-style-type: none"> • Check the proper connection of connectors to the machine and the applicators. • Make sure that the air connection pipes are not bent, pinched or crushed. • Make sure that the inflation of the chambers occurs progressively and according to the chosen program. • Make sure that you have set the pressure control correctly. • Make sure there is actual air outlet from the machine. |
| The air continues to enter the chambers and an air leak in applicators is perceived | <ul style="list-style-type: none"> • Check the presence of holes or damage in the air chambers of the applicators. If no anomalies are found, switch the device off, restart it and check correct functioning again. |

Configuration

WARNING

THE CONFIGURATION PROCEDURE MUST BE CARRIED OUT WITH THE DEVICE SWITCHED OFF AND THE POWER PLUG DISCONNECTED FROM THE POWER OUTLET

WARNING

DO NOT OBSTRUCT THE AERATION GRIDS OF THE DEVICE DURING USE.

Use

Caution

- We recommend adequate sterilisation and/or disinfection of all parts that come into contact with the person that undergoes the massage. (See next paragraph).
- Always disconnect the plug from the power supply before cleaning or maintenance and if the device is not used.

Pre-usage cleaning

- Before each use, it is necessary to sanitise the parts that come into contact with the person that undergoes the massage by wiping them gently with a soft damp cloth and, possibly, with a neutral detergent. Do not use other cleaning methods, such as corrosive products and/or abrasive materials.
- The parts that come into contact with the skin must be cleaned between one massage and the other with standard cleaning methods: gentle disinfectant on a soft cloth.

Other cautions

The treatment time of one single anatomical area must not exceed 30 minutes.

Operating instructions of the device

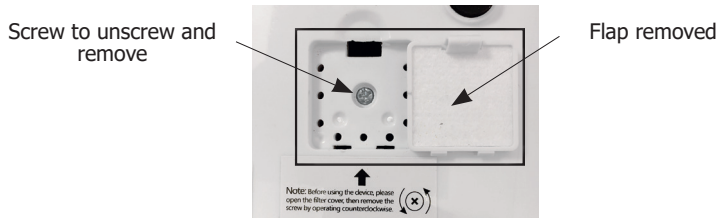
IMPORTANT: to avoid that the compressor (core of the device) suffers from sudden movements and shocks during transport, it was blocked. This solution is adopted to ensure integrity, functionality and quality of the device. Therefore, the compressor must be unlocked as follows.

BEFORE CONNECTING THE POWER CORD AND CARRYING OUT ANY OPERATION ON THE MACHINE, IT IS ABSOLUTELY NECESSARY TO PERFORM THE FOLLOWING OPERATIONS:


- Turn the device upside down and locate the square flap on the lower part;

- Remove the flap temporarily;

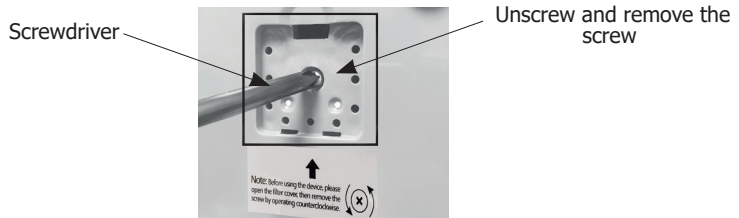- In the little space just opened, locate the screw that must be unscrewed and removed by means of a screwdriver;

- After removing the screw, reposition the flap.
- Turn the device upside down again by leaning it on its own base.

Proceed as described in the following paragraphs of this chapter and in chapter "Operating instructions of the applicators".

Pressure

Pressure can be adjusted between 30 mmHg and 240 mm Hg (+- 10%)

To **reduce pressure**, rotate the handle anticlockwise

To **increase pressure**, rotate the handle clockwise

- Perform 2 inflating cycles to minimum and then adjust pressure.
- The inflation of each chamber lasts approximately 4 seconds and then passes on to the next one.

Start massage at minimum pressure. Then increase it according to your needs.

Time

Time can be adjusted between 0 and 30 minutes. After 30 minutes of machine operation, it is necessary to wait at least 30 minutes (with the machine off) before starting a new session.

Caution: if you turn the handle to "Always On" automatic power off is disabled. Therefore, the machine must be switched off manually.

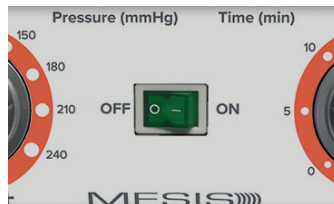
To **reduce time**, rotate the handle anticlockwise

To **increase time**, rotate the handle clockwise

"Off-On" button

Between the pressure handle and the time handle, there is a green (0 - I) OFF-ON button to start the device.

Applicators and accessories

IMPORTANT NOTE: applicators and accessories supplied with the machine vary according to the purchased configuration.

Legs

4-chamber legs

The legs are the specific accessories (“leg-shaped”) made up of **4 air chambers** to wear in order to perform the massage on lower limbs.

To avoid knee pain, do not bend the leg during functioning.
For the operating instructions see next chapters.

Abdominal/glutes band

The abdominal band is the specific accessory made up of **4 air chambers** to wear in order to perform the massage on the abdomen and glutes.

Do not position the band on the breast.
For the operating instructions see next chapter.

4-chamber abdominal/glutes band

Armband

4-chamber armband

The armband is the specific accessory made up of **4 air chambers** to wear in order to perform the massage on upper limbs.

To prevent it from sliding downwards, secure the armband by using the specific braces with Velcro.
For the operating instructions see next chapters.

Single connector

The single connector is an accessory made up of joined rubber tubes ending with a connector that must be connected to the machine or to the splitter on one extremity, and with spouts (air outlets) that must be inserted into the grafts located on the applicator on the other.

The single connector allows the use of Mesis IntenseWaves Pressomassage with 1 armband or 1 leg or the abdominal/glutes band or the 2 legs. In the latter case, the configuration provides for 2 single connectors and one splitter.

For the operating instructions see next chapters.

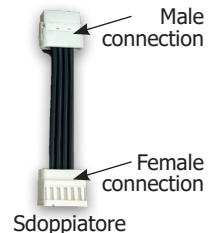
Single connector

Splitter

The splitter is an accessory made up of joined rubber tubes with two extremities, one of which is a male connection with jacks and the other is a twin female connection with holes. The splitter is combined with the configurations with two or three applicators.

1. In the configuration with two applicators there is 1 splitter. It allows to use two applicators at the same time, such as 2 legs;


Scoppiatore

2. In the configuration with three applicators there are 2 splitters. They allow to use three applicators at the same time, such as 2 legs and the abdominal/glutes band.

In **case 1**, the male connection of the splitter must be connected to the machine and the twin female one to the two single connectors of the two applicators.

In **case 2**, the two splitters must be connected to each other. More specifically, the 1st splitter must be connected to the machine as in case 1 while the male connection of the 2nd splitter must be inserted into one of the two seats of the female connection of the 1st splitter. A single connector must be inserted into the second free seat of the female connection of the 1st splitter.

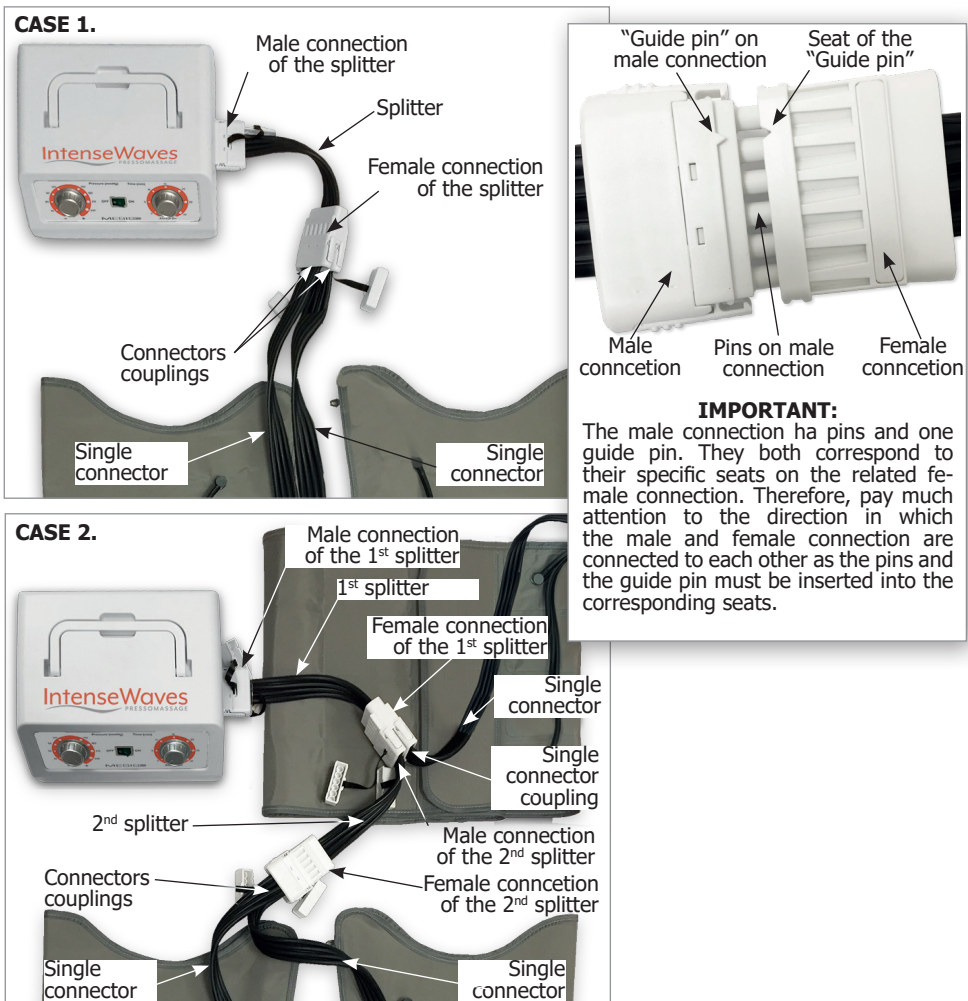The two remaining single connectors must be connected into both seats of the female connection of the 2nd splitter.

Operating instructions of the applicators

IMPORTANT NOTE: applicators and accessories supplied with the machine vary according to the purchased configuration.

1. Connect the connector to the applicator you wish to use.

Choose the connector and/or splitter to connect according to the applicators you wish to use:

- with the armband, use the single connector (Fig. A).
- with the abdominal/glutes band, use the single connector (Fig. B);
- with one leg, use the single connector (Fig. C)
- with two legs, use two single connectors and the splitter (Fig. D);
- with two legs and the abdominal/glutes band, use three single connectors and two splitters (Fig. E).

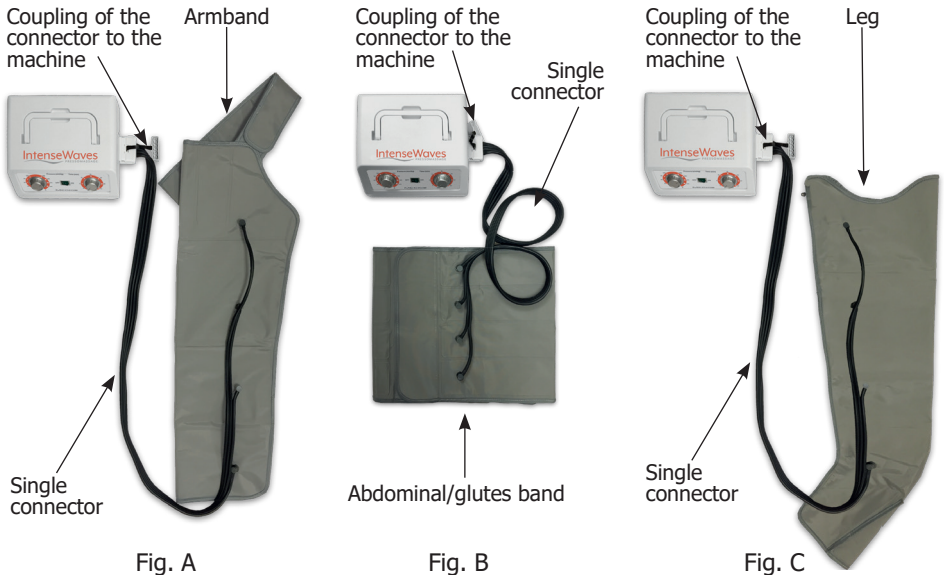
Connect each spout (air outlet) located on the extremity of the rubber tubes of the single connector to their grafts located on the applicator or applicators you wish to use.

CAUTION: perform the connection by combining the spout and the graft according to the colour. Spout and graft must be of the same colour.

Armband, Abdominal /glutes band and Leg used individually

- Connect the spouts located on the extremities of the rubber tubes of the single connector to the grafts located on the applicator you wish to use.

Leg

- If you wish to use the two legs, connect the spouts of the two single connectors to the grafts on the legs. After this, connect the single connectors to the female connection of the splitter.

NOTE: more detailed information on the connection between the connectors and the splitter at page 72 "Case 1."

Fig. D

Legs and abdominal/glutes band

- If you wish to use the abdominal/glutes band and the legs, connect the spouts of the single connectors to the grafts located on the band and on the legs. After this, connect the single connectors of the legs to the female connection of the 2nd splitter and the male connection of the latter to the 1st splitter. Connect the single connector of the band to the female connection of the 1st splitter.

NOTE: more detailed information on the connection between the connectors and the splitter at page 72 "Case 2."

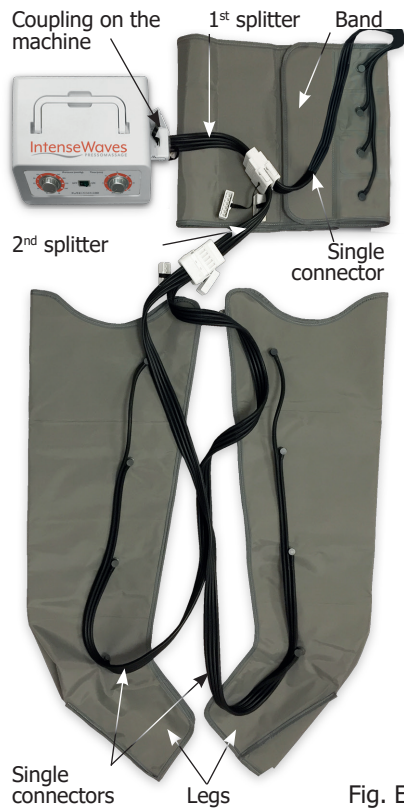
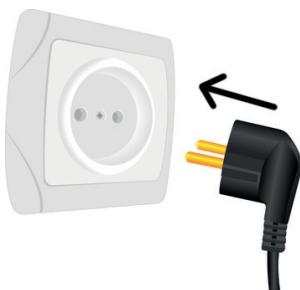
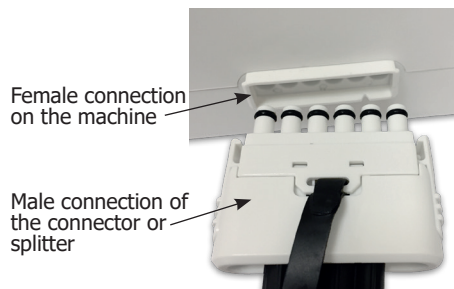
Fig. E

2. Plug the power cord.**3. Connect the coupling of the single connector or the splitter to the machine.**

In IntenseWaves Pressomassage, the „machine-connector or splitter“ coupling is located on the side of the machine.

4. Wear the applicators you wish to use.*Legs*

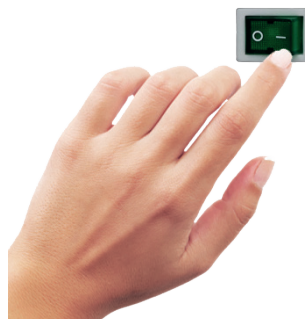
- Wear the legs and secure them with the zipper, position the legs with the couplings facing inwards.
- **The inflation must start from the foot up to the groin.**

Abdominal/glutes band

- Wear the abdominal band on glutes-hips-abdomen and secure it with the Velcro.
- **The inflation must start from the upper part of the abdomen and move downwards (groin).**

Armband

- Wear the armband.
- Fasten the “braces” with the Velcro on the shoulder opposite to that on which the armband is positioned.
- **The inflation must start from the side of the hand and move in sequence towards the shoulder.**

5. Press the ON button (-) to start the machine

In IntenseWaves Pressomassage devices, the power button is located on the front part of the machine.

IMPORTANT: for the operating instructions of the purchased machine, see the related chapter.

IMPORTANT:

Make sure the inflation is made according to the direction indicated at the paragraph “Direction of inflation” of chapter “Technical characteristics”.

SECTION 4

Maintenance

Disposal

Maintenance

Safety

WARNING

When performing the maintenance of the device, provide for the use of personal protective equipment (PPE) such as:

- **Clothing:**

which must comply with the essential safety requirements in force.

- **Gloves:**

when operating with parts that may reach high temperatures and live parts, it is necessary to wear CE marked gloves as PPE which protect against the risks that have just been analysed.

The maintenance operations must be carried out by staff that have read the manual beforehand.

Any type of physical cleaning or maintenance operation must be performed always with the device off and disconnected from the power supply (plug disconnected from the outlet). When performing these operations, thoroughly comply with the instructions of this manual.

The lighting system (natural and/or artificial) of the area of the maintenance operations must guarantee the following minimum lighting value: 200 lux.

HAZARD

DISCONNECT THE DEVICE FROM ITS POWER SUPPLY BEFORE PERFORMING ANY PHYSICAL CLEANING OR MAINTENANCE OPERATION.

For any maintenance and cleaning operation, in addition to the instructions provided in this manual, the general safety rules and the general rules on safety at work in force at the place where such operations are carried out must be complied with.

WARNING

IT IS PROHIBITED TO ACCESS THE INTERNAL PARTS OF THE DEVICE. NO MAINTENANCE THAT PROVIDES FOR THE DISASSEMBLY OF THE INTENSEWAVES PRESSOMASSAGE DEVICE IS ENVISAGED. MAINTENANCE OF THE INTERNAL PARTS CAN BE PERFORMED ONLY BY A TECHNICIAN AUTHORISED BY THE MANUFACTURER.

Periodical maintenance

- It is necessary to clean the device on a periodical basis from possible accumulation of dust that may form on the external frame. Possibly use a compressor to remove dust. It is possible to use also an alcohol-free non-abrasive damp cloth or without aggressive solvents. Do not use abrasive sponges, chemicals or detergents. Prevent water from touching internal electrical parts during cleaning.
- After each use clean the parts of the device that come into direct contact with the user (see Pre-usage cleaning).
- At the end of the useful life of the accessories, contact the manufacturer for spare parts.
- We recommend to perform a complete check test of the device every 12 months.

Extraordinary maintenance

Extraordinary maintenance is requested in the event of faults or breaks, unforeseeable accidents or inappropriate use of the device.

The situations that may occur from time to time are completely unforeseeable and therefore it is not possible to outline appropriate procedures of intervention.

In case of need, refer to the technical service of the manufacturer to receive instructions appropriate to the situation.

In any case, all ordinary and extraordinary mechanical, electrical or electronic interventions must be carried out by specialised and authorised staff.

Decommissioning

Device deactivation

The device is manufactured and built according to robustness, duration and flexibility criteria that allow it to be used for several years. After its technical and operating life cycle, the device must be decommissioned so that it can no longer be used for the purposes which it had been once designed and manufactured, while in any case making it possible to reuse the raw materials which it consists of.

The same deactivation procedures must be observed in all the following cases:

- Device decommissioning and storage in warehouse.
- Final disassembly and subsequent disposal.

NOTE

THE MANUFACTURER ASSUMES NO RESPONSIBILITY FOR DAMAGE TO PEOPLE OR OBJECTS RESULTING FROM THE REUSE OF SINGLE PARTS OF THE DEVICE FOR FUNCTIONS OR IN CONFIGURATIONS OF ASSEMBLY OTHER THAN THE ORIGINAL ONES. THE MANUFACTURER REFUSES ANY APPROVAL, EITHER IMPLICIT OR EXPLICIT, FOR SPECIFIC PURPOSES OF PARTS OF THE DEVICE THAT ARE REUSED AFTER FINAL DEACTIVATION PRIOR TO ITS DISPOSAL.

Disposal

The possibility to reuse some parts of the device is subject to the complete responsibility of the user.

The label shows the symbol indicated in Italian Legislative Decree No. 49 of 14 March 2014 which implements Directive 2012/19/EU on waste electrical and electronic equipment, which states the need to dispose the product in separate waste, more specifically in electrical and electronic waste. Contact your local waste collection centre. The crossed-out symbol dustbin symbol applied on the device indicates that the end-of-life product must be treated separately from other waste.

Therefore, the device with all its essential components that have reached the end of their useful life must be given to waste disposal centres specializing in separate collection of waste electrical and electronic equipment, or given back to the retailer at the time of purchasing new similar equipment, on a one for one basis or on a one for zero basis for the equipment having the larger side less than 25 cm.

The adequate separate collection for the subsequent start-up of the equipment sent to be recycled, treated and disposed of in an environmentally compatible way contributes to preventing possible negative effects on the environment and health and optimises the recycling and reuse of components making up the device. Abusive disposal of the product by the user shall result in the application of the administration sanctions in accordance with Italian Legislative Decree No. 49 of 14 March 2014.

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Co., Ltd 401, Jishan Road 19 #, Siming District, Xiamen, China 361000