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## INDEX

1. Introduction
3. Shipping
  - Assembling
  - Use specifications
  - Safety information
4. Components
5. Check before use
  - High chair use
  - Brake - Push brake
6. Armrests
7. Adjusting the armrest and the lip
8. Adjusting the seat tilt
  - Adjusting the seat depth
9. Adjusting backrest height
10. Footboard
11. Push handle
  - Anti-tip wheels
12. Transport by car
13. High chair maintenance
14. Measures
15. Optionals
16. General safety rules
17. Portable weight
  - Transfer from the high chair
18. Crossing narrow passages
  - Using elevators and lift bridges
  - Transportation by car or other means of transport
19. Declaration of compliance
20. Warranty
21. Plate and CE label
22. Maintenance log
24. Data sheet
25. Section for retailers

## Instructions for use and maintenance

### Introduction

We have been working for more than 20 years to guarantee maximum comfort, convenience and functionality to those who need wheelchairs and walking devices. This commitment of ours made us leaders in the field of research and technological innovation.

Our range of products is constantly evolving and some models have been studied to improve their features and performance levels.

For this reason, the BIRBA AL model was designed.

This manual provides instructions for this particular models.

Please read carefully the following instructions before using the device for the first time.

The special care taken in design and manufacture of these products is a guarantee of top-level safety and comfort. Use your wheelchair according to the instructions provided in this manual.

**Birba Model:** multifunctional high-chair - painted steel frame.

**Main clinical indications:** suitable for neurological patients in their early and late childhood. The frame is designed to accommodate rather complex postural setups.

**Basic equipment:** available in two sizes, equipped with a tilting device and mechanically reclining and height adjustable backrest. Adjustable seat width and depth, and anatomically shaped to accommodate situations of instability and deviations of the trunk and pelvis, lined with washable and breathable fabrics. Armrests with adjustable width and height, footboard with adjustable height, flexion and extension, anti-tip wheels, handle for carer, pushed trunk and adjustable headrest, pelvic girdle.

**Optionals:** it may be customized with accessories which will meet your needs of comfort and safety: orbital adjustable headrest, occipital headrest, plexiglass table, abductor pad.

## Introduction

### Symbols

While reading the manual you might come across these symbols of particular importance or interest.

symbolizes danger in general. Particular attention must be paid.

6 mm spanner or wrench.

Allen wrench for 6mm hollow hex-screws.

### BIRBA AL is equipped with:

- Tilting system
- Flexion-extension height adjustable footrest equipped with a legstrap
- Width adjustable armrest
- Attack table under the armrest
- Gas spring lever for reclining seat
- ø 175 soft cast front wheels
- ø 300 pneumatic or soft rear wheels
- Anti-tip wheels

In this way allowing a greater comfort

Moreover: folding backrest, removable armrests and rear wheels with quick release axles, minimize the space-footprint for an easy transport.

## Shipping

All Nuova Blandino products are tested in-house to verify compliance with all specifications and then are shipped in extra cardboard boxes.

Please, check the device soon after receipt, in order to verify if any damage occurred during transport, and if possible, in the presence of the carrier.

If the purchased product was damaged during the transport, the damage must be reported immediately, by attaching an official written claim to the delivery note.

All damaged parts (if removable) or the whole wheelchair must be delivered carriage free to Nuova Blandino Srl, within 8 days from the date of receipt, together with a copy of the receipt.

No damage will be acknowledged unless the aforementioned procedure is followed, or if the damage is reported after the receipt note is issued.

## Assembling

The orthopedic specialist will deliver your multifunctional highchair ready for use, adjusted according to your individual necessity.

## Use specifications

Your multifunctional highchair is designed to be used indoors and outdoors. It is intended exclusively to transport a person in the seat with the leg and the sides mounted, and not as a means of traction, transport, or otherwise.

## Safety information

All information for a safe use of the multifunctional highchair are shown in the second section of this handbook.

## Components

- 1 Reclining backrest
- 2 Removable armrest
- 3 Attack table under the armrest
- 4 Padded seat
- 5 Footrest staff
- 6 Aluminum singles footrest
- 7 Caster wheel
- 8 Brake lever
- 9 Ø 300 rear wheel
- 10 Gas spring
- 11 Pushed trunk
- 12 Headrest

## Check before use

Before using the multifunctional highchair, make sure that all parts are properly assembled and tightened, in particular the armrests and footrests.

Check that the push brakes works properly. While activating the brakes on both wheels, push the chair forward and backward; braked wheels are expected not to rotate.

## Using the highchair

The multifunctional highchair is ideal for indoor and outdoor leisure activities, provided that the ground is firm and not slippery.

At the beginning, use the multi-functional high-chair with great caution.  
Read the second part of the manual ("Safety information") carefully.

## Brake - push brake

The parking brake is one of the most important safety devices featured on this highchair. It is a push brake that operates directly on the tires.

The brakes must always be fully functional. The braking effect of the push brake may be impaired if:

- The sides of the tires are worn out
- The tires are wet
- Tire pressure is low

Check the wheels on a regular basis and before leaving your pressure!!

Check regularly and always before leaning the wheels pressure.

### **Recommendation:**

Ø 300 standard wheel (2.5 bar).

## Armrest

The armrests are equipped with removable padded armrest:

- The removable armrests make it easy to move from highchair to a bed, a chair, etc..
- act as lateral support of the body
- the height-adjustable armrests improve the adaptation of highchair to the user's body
- if you use a pillow you can adjust the height of the armrests to the new level seat.


**Caution: never lift the wheelchair by the armrests. The wheelchair must be used only when the armrests are mounted**

## Adjustment

### Adjusting the armrest and the side pad


#### Height adjustment:

- loosen the screw (a)
- adjust the height of the armrest
- find the best position
- tighten again the screw (a) to lock the armrest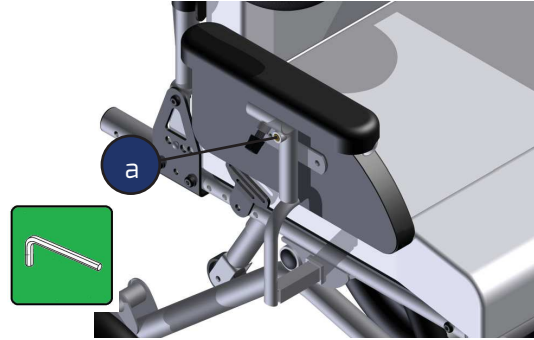

#### Width Adjustment:

- loosen the screw (b) shown in figure
- adjust the width of the flank
- retighten the screw (b)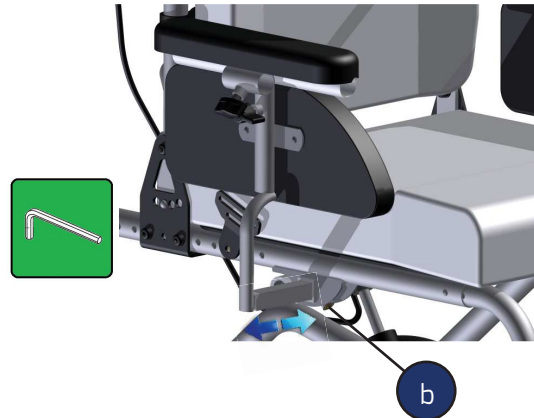

## Adjusting the seat tilt

The gas spring is operated by the lever on the push handle, allowing a smooth and gradual adjustment of the seat tilt.

## Seat depth adjustment plate

### Seat length adjustment:


- remove the pillow
- loosen the screws (a)
- move forward or backward the seat plate to find the best position of the seat.
- tighten the 3 screws (a)
- replace the pillow

### Seat depth adjustment:

The adjustment should be performed preferably by an orthopedist or a Nuova Blandino Srl technician. It allows you to vary the ratio between the seat and back, according to the user's needs.


Loosen the 4 allen screws (b) placed on each seat bar and replace all of the elements according to your needs. Then tighten the allen screws to secure everything.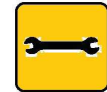

## Adjusting backrest height

### Height adjustment:

- remove the padded backrest
- loosen the 4 screws (a) and move the backrest to the desired height.
- tighten the 4 screws (a) to secure the backrest
- replace the padded backrest

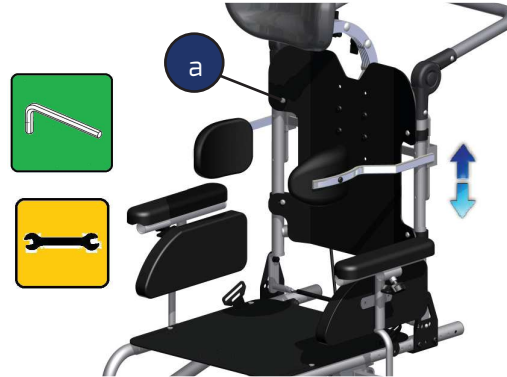## Footboard

The single footboard allows you to change the flexion-regulation. It is also adjustable in depth and height.

### 1. Flexion-regulation:

- loosen the bolts (a) placed under the platform and tilt it up to bring it to the desired location
- tighten the bolts / allen

### 2. Depth adjustment

- Loosen the screws on both sides (b) and move the two tubes in the desired position until the one of the holes which allows the adjustment, tighten the screws (b)

### 3. Height adjustment

- Loosen the bolts (c) and move the footplate support to the desired position till one of the holes that allow the adjustment, tighten the screws (c).

## Push handle

It is height adjustable to facilitate the helper's assistance according to his or her height.

### Height adjustment:

- press both handle joints (a) and rotate them to set them into the desired position and release.


## Anti-tip wheels

Retractable anti-tip wheels. They are mounted especially on tilting higchairs to avoid the risk of tipping backwards.

### Arming/disarming anti-tip wheels:

- **ARMING:** press the pin (b) and turn the small wheel in position rollover (see Fig. 1) until they snap pin (b) inside the hole on the top tube.
- **DISARMING:** press the pin (b) and turn the pinch upwards until it is in the position shown in figure 2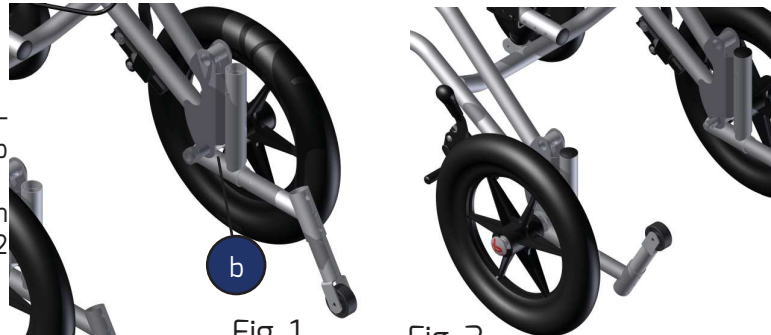

## Transport by car

To carry the highchair in the car:

- remove the seat and padded backrest (Fig. 1)
  - pull the cable to release the backrest (Fig. 2)
  - break down the handle and the back (Fig. 3)
  - In case, possibly remove the rear wheels acting on the quick release axles (Fig. 3)
- To reassemble the highchair you must follow all the steps in reverse order.

(Fig. 1)

(Fig. 2)

(Fig. 3)

## Highchair maintenance

### Yearly checks

Maintenance should be performed once a year. It is highly recommended to check the state of the bearings, since sand, dust and water may damage them. Lightly lubricate with bicycle resin-free oil.

The frame may be cleaned with hot water and soap (or household detergent) only. Plastic parts should be cleaned with detergents specific for this material.

#### Caution!

Do not use high pressure cleaning equipment  
Mechanical damage may be caused by water jets.

The highchairs should not be cleaned with aggressive cleaning products and tools.

#### Caution!

All components which come in direct contact with the skin must be replaced if the wheelchair is given to another user.

### Repairs

In case any repair is needed, contact your Nuova Blandino S.r.l. dealer, who knows how repairs should be performed according to our specifications and will be able to offer advice, support and provide repair services.

### Spare parts

Order through your dealer. In order to ensure the correct identification of the spare parts which must be replaced, indicate the highchair frame registration number. For safety reasons, use spare parts and accessories authorized by Nuova Blandino S.r.l. More information on how to retrieve the frame registration number are provided on page 21.

### Disposal

The highchair packaging materials can be recycled. The metal parts must be collected and recycled in the “old metal” skip. The fabric and plastic parts can also be recycled.

Disposal must be performed in compliance with specific national legislation.

## Measures


cm 36/28 - 28/23

cm 49

cm 43

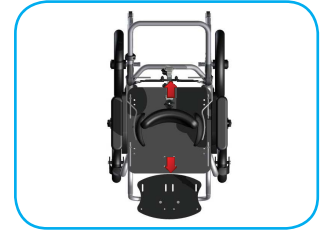cm 35 - 43

cm 42

cm 49

Backrest -11° .... +22°

Seat 40°

Kg 19

Max Load. Kg 80

## OPTIONALS

Multifunctional headrest  
(padded comfort)

Multifunctional headrest  
(occipital)

Abductor pad

Plexiglas desk

## General safety rules

Use the highchair only according to the instructions of this handbook of use.

To station, get on and off the wheelchair, make sure the ground is flat and stable and secure the wheelchair to prevent it from moving by activating both brakes.

If exposed to direct sunlight, the covering and the upholstery of the seats, the armrests, the legrests and the handles may reach temperatures over 41° C. Direct contact with unprotected skin may cause injury. In order to avoid the over-heating of the said parts, exposing the parked wheelchair to direct sunlight is not recommended.

Please ask your dealer about all available setups, in order to avoid any danger or any change and adjustment which may affect wheelchair operation and safety of navigation.

The dealer will be happy to adjust your multifunctional highchair to suit your individual needs, by maximum respect for your safety.

When you turn the adjusters (eg. Footrest), be careful and do not ever put your hands in the orientation of the components. There is a risk of injury.

When carrying objects, make sure that these do not affect or restrict the handling of highchair.

Check the pressure and the wear of the tires and check the brakes.

## Portable weight

Do not overload the highchair and do not use it for carrying more than one person.

The recommended maximum load is reported in the handbook and on the highchair plate.

## Transfer from multifunctional highchair

Getting off the highchair may be dangerous, if not assistance is available.

It requires:

- considerable strength
- good coordination abilities
- an anti-slip back surface (i.e. set the palms of your hands on the seat surface)
- much training

When the highchair is not load, for example when moving from bed to highchair or car, do not ever force on the platform.

**- Danger of tipping!**

## Crossing narrow passages

When crossing narrow passages, such as the entrance of a building, a door, etc., pay attention to your elbows. **Danger of injury!**

## Using elevators and lift bridges

Lifts and elevators pose a particular danger to the user.

**Caution!**

**Automated doors may entail a risk of injury for your hands!**

## Transportation by car or other means of transport

The wheelchair must not be used as a seat when using a car or another means of transport. Dismantle the highchair (as specified in the user's manual) and put it in the luggage compartment, or secure it in order to avoid accidents, in the event of dangerous maneuvers.

Take a proper seat inside the car or the means of transport and fasten your seatbelt.

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Responsible for designing, manufacturing, packaging, and labeling of this product, that falls within the category of  
**MULTIFUNCTIONAL HIGH CHAIRS**

### **DOES GUARANTEE AND DECLARE**

The afore mentioned device, launched and marketed as of the date certified in this certificate, fully complies with the provisions stated in annexes I and VII to EEC Directive 2007/47 and L.D. No 37 of 25 January 2010, that confirmed EU provisions at Italian level.

The products listed above are classified as Class 1 non-invasive medical devices as provided in Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product, the complete "EC" declaration of conformity is available on the website [www.nuovablandino.com](http://www.nuovablandino.com).

*The afore-mentioned products:*

- *are not measuring instruments*
- *are marketed in non-sterile packaging*

This is to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

Settimo, 26 Novembre 2015

## WARRANTY

Our folding multifunctional highchair are guaranteed for two years as of the date reported on the certificate of purchase (see page 25).

The warranty covers any manufacturing flaw or malfunction on guaranteed products.

Damages caused by carelessness or improper use are excluded from the warranty. Authorized resellers will provide additional information and quotations if any repair is needed. For safety reasons, only spare parts or accessories authorized by Nuova Blandino S.r.l. should be used. The use of unauthorized accessories is a reason of immediate decline of warranty.

In order to guarantee it works properly, the highchair must be kept clean, in order and maintained as described in the handbook of use.


Caution:

The company will not be liable for damage caused by the combined use of the highchair with parts supplied by other manufacturers (eg. trailer units, electrical equipment) that may result in significant danger - unless such parts have been expressly authorized in writing by Nuova Blandino S.r.l.

*Nuova Blandino S.r.l. will not be liable for any damage to persons, things, animals, and any third party due to modification or misuse of the highchair.*

## Plate and EC label

The EC label (see picture) is printed on the highchair on the front side of the crossbar under the seat.

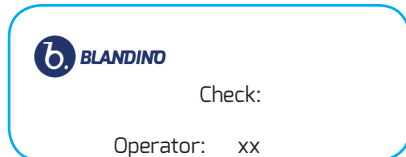## Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

# Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

## Data sheet

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive.

- **Intended use**

The device is supplied ready for use by technical orthopedics and adjusted according to the user's needs.

The multifunctional highchair is designed for both indoor and outdoor use. It is intended to accommodate one single person on the seat when the legrests and the lateral pads are mounted, and should not be used as a means of traction, transport or otherwise. Carefully read the instruction manual for a correct use of the device.

- **Check before use**

Before using this multifunctional high chair, make sure that all parts are properly assembled and tightened, in particular the footrests. For safety reasons, if visible flaws are detected on the product, all damaged parts must be replaced.

- **How to clean**

*Do not use high pressure cleaning devices! Mechanical damage may be caused by water jets.  
Chairs must not be cleaned with aggressive cleaning products.*

- **Lifespan of the device**

This multifunctional highchair may be used for a variable period of time, depending on the time and the frequency of use. A correct use and a proper maintenance of the wheelchair may extend its lifespan; on average, it is expected to be fully functional for 5-7 years.

- **Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product.

Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 12:00 HOURS AND FROM 13:30 TO 7:00 HOURS. CALL +390112625140.

**RETAILER STAMP**

--

**DATE OF PURCHASE**

--	--	--

**REGISTRATION DATE**

--	--	--

**N.B.: THIS MANUAL MUST BE PRODUCED WHENEVER ASSISTANCE IS NEEDED**

Nuova Blandino S.r.l.  
Strada Cebrosa, 24 - 10036 Settimo Torinese  
Torino - Italy  
Tel. +39 011 2625140 - fax +39 011 2625141  
[www.nuovablandino.com](http://www.nuovablandino.com) - [inforb@nuovablandino.com](mailto:inforb@nuovablandino.com)

Commitment and technology for people with special needs