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Table of contents

1. Introduction
3. Shipping
 - Assembling
 - Use specifications
 - Safety information
4. FOLLETTO components
5. Check before use
 - Using the wheelchair
 - Push brake
7. Adjustment
 - Tensioning backrest adjustment
 - Fender adjustment
 - Folding and unfolding the wheelchair
 - Footrest height adjustment
 - Rear wheel adjustment
 - Fork and the front wheels adjustment
 - Removable pins and narrow passage wheels
13. Transport by car
14. Wheelchair maintenance
15. Measures
16. Optional
17. General safety rules
18. Portable weight
 - Transfer from wheelchair
19. Training
 - Taking objects
20. Sidewalks
- Up and down the stairs
21. Crossing narrow passages
 - Using elevators and lift bridge
 - Transportation by car or other means of transport
22. Conformity declaration
23. Warranty
24. Label and CE mark
25. Maintenance log
28. Data sheet
29. Section of retailers

Use and maintenance handbook

Introduction

We have been working for more than 20 years to guarantee maximum comfort, convenience and functionality to those who need wheelchairs and walking devices. This commitment of ours made us leaders in the field of research and technological innovation.

Our range of products is constantly evolving and some models are being developed to improve their features and performance levels.

For this reason the FOLLETO model were created.

This manual provides instructions for these particular models. Please read the following instructions before using the device for the first time.

The special care taken in designing and manufacturing these products is a guarantee of top-level safety and comfort.

Use your wheelchair according to the instructions provided in this manual.

FOLLETO model: foldable lightweight wheelchair for children.

Main clinical indications: suitable for children and young people able to independently or half-independently operate it.

Basic layout: available in different sizes, its technical features may be customized ergonomically in order to optimize its performance: tensioning backrest adjustment for trunk comfort, variable positioning of the rear wheel axles, wheel quick release, front forks height and recline adjustment, clothes protection fenders, fixed height adjustable singlefootrest.

Optionals: it may be customized with a range of accessories to meet your comfort and safety needs: removable armrests, table, belts, longer handle for attendant..

Symbols

While reading the manual you might come across these symbols of particular importance or interest.

Symbolizes danger in general.
Particular attention must be paid.

6 mm wrenches for nuts or bolts 6 mm.

Allen wrench for screws with hexagon socket
6.

The FOLLETTO model is equipped with:

- Folding alluminum frame.
- Removable backrest.
- Clothes protection fenders.
- Variable positioning of the large wheel axles.
- Variable positioning of the small wheel forks with tilt adjustment.
- Pneumatic rear wheels 22"x1 Ø550.
- Front wheels Ø100x32.
- Height-adjustable single footrest.
- Quick release axle.

Shipping

All Nuova Blandino products are tested in-house to verify compliance with all specifications and are then shipped in extra cardboard boxes..

Therefore, check the device immediately after receipt, and in the presence of the carrier, to verify any damage that might have occurred during transport.

In the purchased product was damaged during transport, the damage must be reported immediately, by attaching an official written claim to the delivery note.

All damaged parts (if removable) or the whole wheelchair must be delivered carriage free to Nuova Blandino Srl, within 8 days from the date of receipt, together with a copy of the receipt.

No damage will be acknowledged unless the afore-mentioned procedure is followed, or if the damage is reported after the receipt note is issued.

Assembling

The orthopedist will deliver your wheelchair ready for use, adjusted according to your individual needs.

Use specifications


This wheelchair is designed for both indoor and outdoor use. Its sole purpose is carrying a single person once the

footrest and the lateral fenders are mounted, and not as a means of traction, transportation or otherwise.

Safety information

All safety information on a wheelchair will be presented in the second part of this manual.

FOLLETTO components

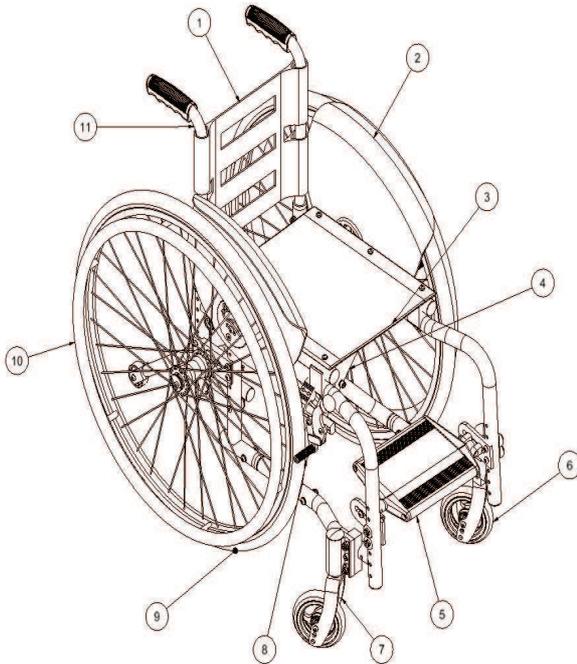- 1 Backrest
- 2 Clothes protection fenders
- 3 Seat + padded cushion
- 4 Crossbar
- 5 Single footrest
- 6 Castor wheel
- 7 Fork
- 8 Alluminum brake
- 9 Pneumatic rear wheel
- 10 Handrim
- 11 Push handle

Check before use

Before using a wheelchair, make sure that all parts are properly assembled and tightened, in particular the footrest. If the footrests are adjustable in height, make sure they allow to sit comfortably with a correct leg angle.

Make sure that brakes function properly. While activating the brakes on both wheels, push the chair forward and backward; braked wheels are expected to not rotate.

Using the wheelchair

This lightweight wheelchair is designed suitable for children and young people able to independently or half-independently operate it. It is ideal for both indoor (home, school) and outdoor use, provided that the ground is solid and not slippery.

Initially, use the wheelchair with caution. Read the second part of the manual (“Safety information”) carefully.

Brake - push brake

The parking brake is one of the most important safety devices featured on this wheelchair. It is a brake that operates directly on the tires.

The brakes must be fully functional. The braking effect of the push brake may be impaired if:

- the sides of the tires are worn out
- tires are wet
- tire pressure is low

Check tire pressure on a regular basis and before leaving your premises!

Recommended:

Ø 550 light wheel (max 7.0 bar) in case of doubt look at the sidewall of tyre.

Make sure that the wheel spokes are tightened up.

Caution:

Use your wheelchair brakes with great caution when going backwards. Particular caution must be used when going backward on steep terrain. A sudden stop due to handrail or brake lever use may result in the wheelchair tipping over.


Adjustments

Tensioning backrest adjustment

The backrest stretch may be adjusted to facilitate the containment of the trunk.



Backrest tension adjustment:

- detach the cover cushion
- loosen the Velcro straps (b)
- accommodate the user to provide greater stability to the trunk
- fasten the belts
- place the backrest cushions back to their original position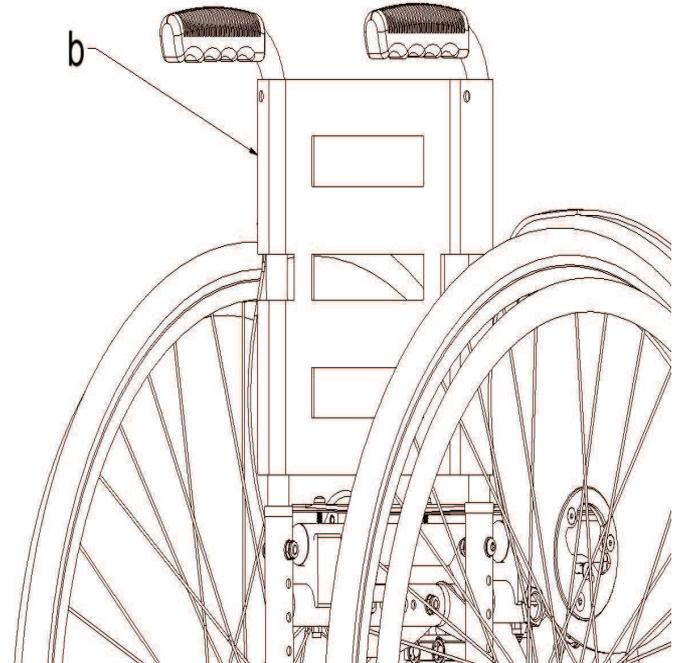

Fender adjustment

Adjust the fender according to the position of the wheel. A 1/2 cm distance is required between the tire and the side pads, along its whole radius. Otherwise, loosen the bolts (a), (b) and (c) and slide the side pad inside the slots to adjust to the correct position. Once the position is adjusted, tighten the bolts to secure it.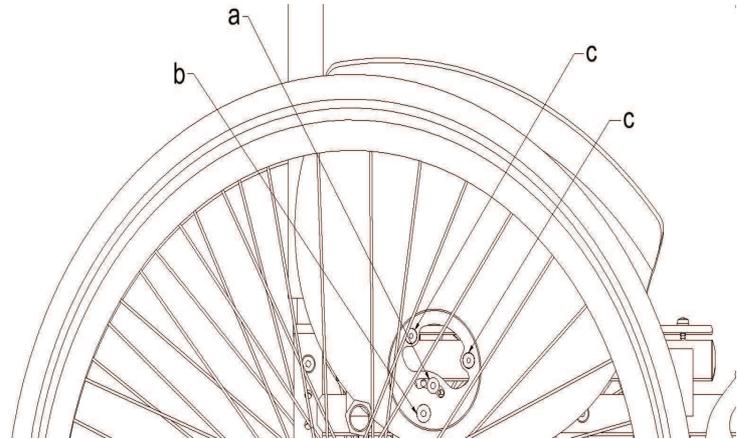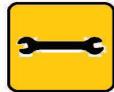

Folding and unfolding the wheelchair


The wheelchair may be folded easily, without any need for tools, by carrying out a few manual operations.

Unfolding:

- push down the upper part of crossbar to a stop
- fold the single footrest down
- place the cushion

Folding:

- remove the cushion
- pull the single footrest up
- pull the central part of the seat cloth up

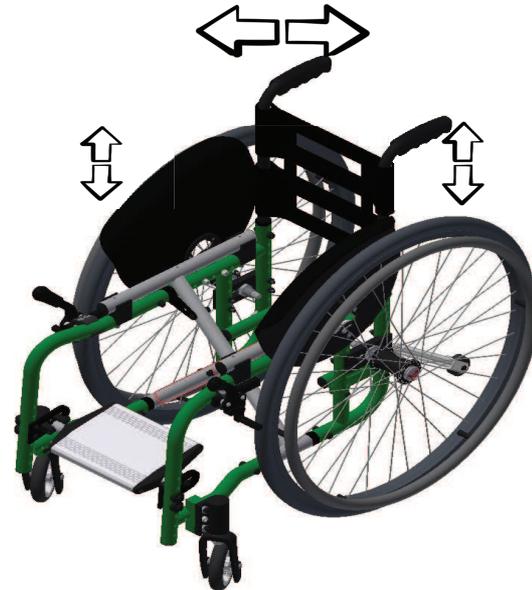Adjusting footrest height

Adjusting footrest height:

- loosen the bolt (g) and adjust according to the desired height
- tighten the bolt (g) to lock the footrest staff

Caution: adjust the footrest staff so that the distance between the footrest and the ground is about 6 cm.

Adjusting the rear wheel

Important: in order to set the optimum for your wheelchair, both its height and depth may be adjusted.

Height adjustment:

the rear wheel mounting plate may be adjusted at different heights:

- remove the rear wheel (see page 13) “Remove th rear wheels“
- remove the screws
- move the plate up or down (2 cm pitch)
- insert the screws and tighteen the nuts

Dept adjustment:

the plate allows to fix the (h) clamp in different positions:

- remove the rear wheel (see page 13) “Remove th rear wheels“
- remove the axle and its nut and washer
- once the right position is found, insert the screws in their respective pitches and srews all the parts back to the initial position.


Caution:

The rear wheel adjustment may require a variation of the seat tilt; in this case it is absolutely necessary to adjust the front wheel support (see page 12) “Adjusting the front wheels“.

Moving the seat forward will set the wheelchair to an “active“ position, thus resulting in the wheelchair rearing up more easily.

Moving the seat backward, sets the wheelchair to a more “cautions“ position.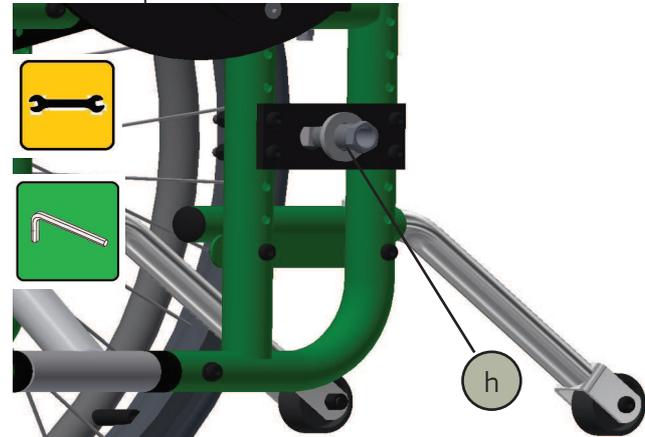

Adjusting the fork and the front wheels

Tilt adjustment:

Keeping the axle of the fork perpendicular to the ground is required.

- remove the fixing screw (i) and change the position of the cam nut (l) inside the fork support
- once at the correct position, reinsert the cam nut and tighten firmly into the slot

Whenever an adjustment to the rear wheel is required (see page 11), the front fork inclination must also be adjusted.

Front wheel height adjustment:

it is required in order to set the wheelchair at a correct height


- remove the screws (m), move the wheel so that it matches the holes on the fork
- insert the screws and tighten firmly

Caution:

The wheelchair front height and tilt must be adjusted whenever the front wheel is replaced with a wheel of different size, when the rear wheel adjustment changes or simply if you want to change the height of the wheelchair.

Whenever the wheelchair height is modified, the inclination of the wheelchair must also be modified and verified accordingly (perpendicular to ground).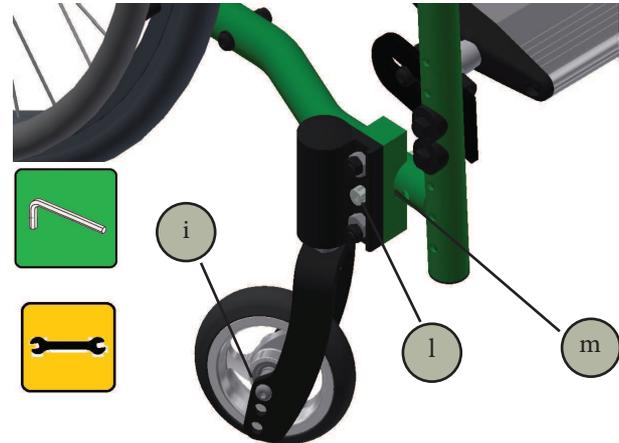

Removable axles

A slight pressure on the pin button (n) will result in the wheel being released, thus reducing the wheelchair width. To replace the wheel simply push the pin until it clicks into its locked position.

Transport by car

Load the wheelchair on a car:

- remove seat upholstery
- lift the footrests
- remove the quick-release pins (page 13)
- lift the central part of the seat cloth and fold the wheelchair (folding/unfolding the wheelchair, on page 9)

To re-assemble the wheelchair, follow the instructions in reverse order.

Wheelchair maintenance

Yearly check

The maintenance must be done once a year . We recommend a thorough check of the bearings , sand, water and dust damage them . Lubricate lightly with bicycle resin-free oil.

The frame may be cleaned with hot water and soap , or a household cleaner. The plastic parts must be cleaned with common detergents specific for this material .

Caution!

Don't use high pressure cleaning equipment
Mechanical damage may be caused by water jets.

Caution!

All components which come in direct contact with the skin must be replaced if the wheelchair is given to another user.

Repairs

In case any repair is needed, contact your Nuova Blandino s.r.l. dealer, who knows how repairs should be performed according to in-house specifications and will be able to offer advice, support and provide repair services.

Spare parts

Order through your dealer. In order to ensure the correct identification of the spare parts which must be replaced, indicate the wheelchair frame registration number. For safety reasons, use spare parts and accessories authorized by Nuova Blandino s.r.l.

More information on how to retrieve the frame registration number are provided on page 24.


Disposal

The wheelchair packaging materials may be recycled. The metal parts are collected and recycled in the "old metal" skip.


The fabric and plastic parts are recyclable.

Disposal must be performed in compliance with specific national legislation.


Measures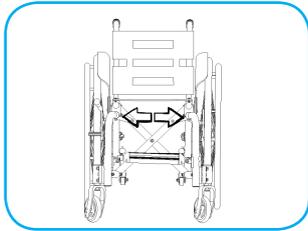


cm 28 30 33 36

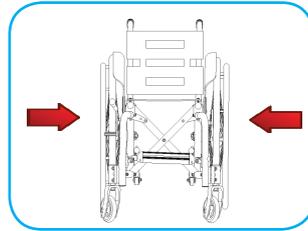
cm 46 48 51 54

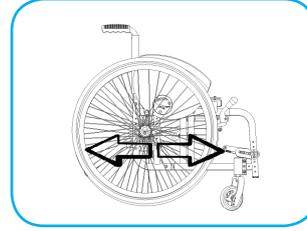
cm 75

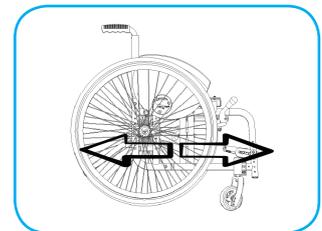
cm 102

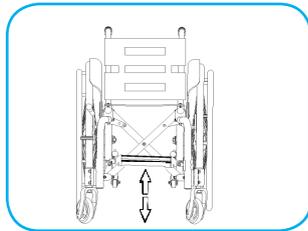cm 48

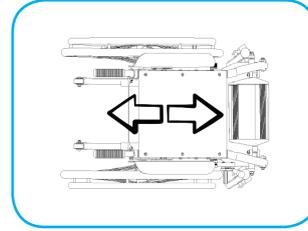cm 31

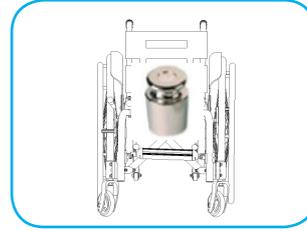Kg 11,5

Max. load Kg 60

OPTIONALS

“Spider” spoke guard

“Spider”

“Tigro” spoke cover

“Tigro”

Separated footrest

Plexiglas table

Multifunctional headrest

Lateral pad

Small pelvic girdle

General safety rules

Use the wheelchair only according to the instructions provided in this use handbook.

To station, get on and off the wheelchair, make sure the ground is flat and stable and secure the wheelchair to prevent it from moving by activating both brakes.

If exposed to direct sunlight, the covering and the handles may reach temperatures over 41°C. Direct contact with unprotected skin may cause injury. So as to avoid the over-heating of said parts, exposing the parked wheelchair to direct sunlight is not recommended.

Please ask your dealer about all the available layouts, in order to exclude any dangers or change and adjustment that may affect on the operation and safety of navigation.

The dealer will be happy to adjust your wheelchair to suit your individual needs with the utmost respect for your safety.

When you activate the adjustment device (e.g. the legrests), pay attention care and do not put your hands on the moving parts. Possible risk of injury.

When objects are carried, make sure that they do not affect or impair wheelchair control. Check the pressure and the tire wear, and check the brakes.

Portable weight

Do not overload the wheelchair and do not use it for carrying more than one person.

The recommended maximum load is reported in the handbook and on the wheelchair plate.

Transfer from the wheelchair

Getting off the wheelchair to move to another place may be dangerous, if no assistance is available.

It requires:

- remarkable physical strength
- good coordination abilities
- an anti-slip back surface (i.e. set the palms of your hands on the seat surface)
- much training

Be aware of all possible dangers.

Ask for assistance. Agree on how assistance should be provided, and be aware that there may be unforeseen hazards, i.e. stumbling; also make sure that the assistant is acquainted with the kind of disability you suffer from. When the wheelchair does not support any weight, for example when you move from the wheelchair to the bed, to the toilet or to the car, do not exert pressure on the footrest.

Danger of tipping over.

Training

In order to use the wheelchair safely, both indoors and outdoors, it is highly recommended to train for different situations, such as: being pushed forward, braking, going forward, turning, going backward on flat surfaces, possibly with the assistance of a helper. Initially, make sure to avoid steep gradients, bumpy roads or other similar situations.

Taking objects

Do not bend too much forward, sideways or backward, to pick up or place objects: you might tip over or lose your balance particularly if the wheelchair is equipped with a small and high seat (cushion).

Caution:

- tipping over may occur when leaning forward to grab an object
- loss of balance may occur when leaning sideways to grab an object
- tipping over may occur when leaning backward to grab an object


Sidewalks

Caution:

Before going off a sidewalk on your own, please check its height. If it is too high, tipping over may occur. Danger of tipping over in case of steep gradient.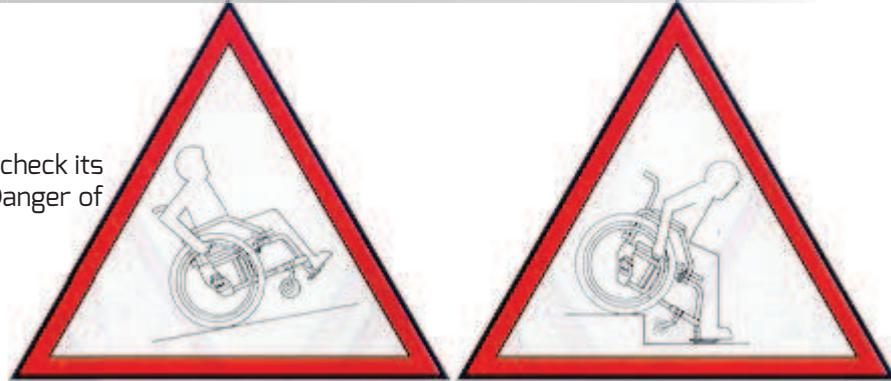

Up and down the stairs

Lean forward when driving uphill; when going downhill, keep you back to the backrest while holding the brake. For +1% gradients, seeking the assistance of a helper is recommended.

Caution:

When going uphill carrying heavy bags or backpacks may be dangerous. Danger of tipping. For your own safety, seek the assistance of two people to climb and go down a steep staircase.

While going down the stairs, the wheelchair should be tilted on the large rear wheels and carried step by step. While climbing the stairs, the helper should turn his or her back to the staircase, and pull the wheelchair one step at a time, keeping it tilted on the rear wheels.

Caution:

If the wheelchair is lifted from the floor, the person sitting on it might lose his or her balance and fall to the ground.

Crossing narrow passages

When crossing narrow passages, such as the entrance of a building, a door, etc. stay attention to your elbows.
Danger of injury.

Using elevator and lift bridge

Caution:

Lifts and elevators pose a particular danger to the user.
Automated doors may entail risk of injury for your hands!!

Transportation by car or other means of transport

The wheelchair must be not used as a seat when using a car or another means of transport.
Fold the wheelchair (as specified in user manual) and put it inside the car, or secure it in order to avoid accidents, in the event of dangerous maneuvers.

Take a proper seat inside the vehicle and fasten the seat belt.

Declaration of compliance

NUOVA BLANDINO Sr.L.

registered office: Strada Cebrosa, 24
10036 Settimo Torinese Fraz. - Torino

Responsible for designing, manufacturing, packaging, and labeling of this product, that falls within the category of
MANUAL WHEELCHAIR

DOES GUARANTEE AND DECLARE

The aforementioned device, launched and marketed as of the date certified in this certificate, fully complies with the provisions stated in annexes I and VII to EEC Directive 2007/47 and L.D. No 37 of 25 January 2010, that confirmed EU provisions at Italian level.

The products listed above are classified as Class 1 non-invasive medical devices as provided in Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product, the complete "EC" declaration of conformity is available on the website www.nuovablandino.com

The afore-mentioned products:

- *are not measuring instruments*
- *are marketed in non-sterile packaging*

This is to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

Settimo T.se, 26th February 2016

Warranty

On the aids from our factory, we provide a warranty period of two years from the correct completion of the certificate of purchase (see page 29).

The warranty is valid for products that have defects in workmanship or operation.

The warranty excludes products that exhibit defects due to negligence or improper use.

Through our dealers, you can still get information on estimates for repairs.

For safety reasons, use spare parts and accessories approved by Nuova Blandino S.r.l.

The use of spare parts and accessories not approved by Nuova Blandino S.r.l. are reason of the immediate invalidation of the warranty.

To ensure the efficiency of the electric wheelchair, it must be kept clean, organized and maintained as specified in the instructions for use.

Attenzione:


Warning:

We accept no responsibility for damage caused by the combined use of the electric wheelchair with parts of all kinds from other manufacturers (for example trailers units, electrical devices) that may involve significant dangers - except that a given product has not been expressly authorized in writing by the Nuova Blandino S.r.l.

Nuova Blandino S.r.l. declines all liability for damage to persons, property, animals, and any third party due to modifications or improper use of the wheelchair.

Label and CE mark

The CE label (see image) on the wheelchair is printed in the middle of the crossbar.

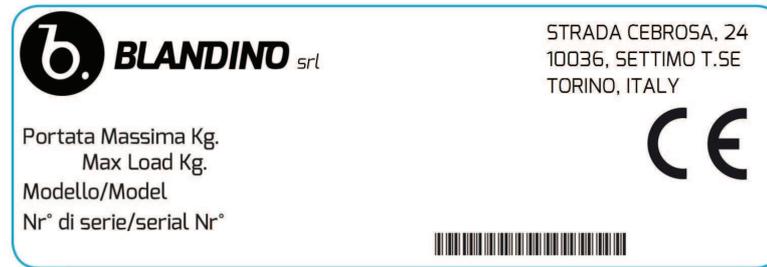Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

Note Informative

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive.

- **Intended use**

The device is supplied ready for use by technical orthopedics and adjusted according to the user's needs.

The multifunctional chair is designed for both indoor and outdoor use. It is intended to accommodate one single person on the seat when the legrests and the lateral pads are mounted, and should not be used as a means of traction, transport or otherwise. Carefully read the instruction manual for a correct use of the device.

- **Check before use**

Before using the wheelchair make sure that all parts are properly assembled and tightened, especially the platforms. If the product shows visible damage, damaged parts must be replaced to ensure your safety.

- **How to clean**

Do not use high-pressure cleaning equipment!

The chairs should not be cleaned with harsh detergents.

- **Lifespan of the device**

The duration of the multifunctional chair of our manufacturing, depends on the intensity of use to which it is subject. Proper use and proper maintenance allows you to maintain and prolong the life of the device, on average, an estimated duration of 5 to 7 years.

- **Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product.

Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 12:00 HOURS AND FROM 13:30 TO 17:00 HOURS. CALL +390112625140

RETAILER STAMP

--

DATE OF PURCHASE

--	--	--

REGISTRATION DATE

--	--	--

N.B.: THIS MANUAL MUST BE PRODUCED WHENEVER ASSISTANCE IS NEEDED

Nuova Blandino S.r.l.
Strada Cebrosa, 24 - 10036 Settimo Torinese
Torino Italy
Tel. +39 011 2625140 - fax +39 011 2625141
www.nuovablandino.com - infonb@nuovablandino.com

Commitment and technology for people with special needs