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INDEX

1. Introduction
3. Shipping
 - Assembling
 - Use specifications
 - Safety information
4. Components
5. Check before use
 - Aid use
 - Brake - Drum brake
6. Armrests
7. Settings
 - Adjusting the armrest and the side pad
8. Recline the backrest and the seat
12. Adjusting the seat depth
 - Adjusting backrest height
13. Adjusting multipurpose elevating platforms
 - Adjusting calf pad
14. Adjusting footboard height
15. Transport by car
16. Wheelchair maintenance
15. Repairs
18. Measures
19. Optionals
21. General safety rules
22. General safety rules
 - Portable weight
 - Transfer from the wheelchair
23. Crossing narrow passages
 - Using elevators and lift bridges
 - Transportation by car or other means
24. Declaration of compliance
25. Warranty
26. Plate and EC label
27. Maintenance log
30. Advisory notice
31. Section for retailers

Instructions for use and maintenance

Introduction

The range of products that will be proposed below, Optima E is the result of twenty years of experience that has seen our company engaged in a process of continuous research to improve and refine techniques for the design and production of products. This has been possible using Technical Orthopedists and advices from experts of Rehabilitation, Education and Support.

This manual provides instructions for the use of these models. Please read the following instructions before using the aid for the first time. The special care taken in the design and manufacture of these products, ensure you a high degree of safety and comfort.

Use your wheelchair by following the instructions contained in this manual.

Model Optima E: multifunctional highchair.

Main clinical indications: suitable for adults / elderly with reduced mobility, able to perform internal paths in tight spaces and external in autonomy (550 mm rear wheels model) and that, spending a long time in a sitting position, need to change the posture of the trunk; frame suitable to accommodate complex postural systems.

Base equipment: available in different sizes and two different rear wheels size (400 mm and 550 mm), made of washable and breathable fabrics, with tilting system from 0° to 32° reclining from 0° to 20° with adjustable angles, adjustable back height, adjustable seat depth and width, in molded polyurethane padding and anatomically shaped to accommodate situations of instability and deviation of the trunk-pelvis. The backrest stabilization base is made of aluminium and has got lateral supporting pads that favour the control action of the side of the trunk, armrests made of polyurethane material, removable and with adjustable height.

Optionals: customization possibilities through the choice of accessories to meet your needs for comfort and safety: range of headrests equipped with a new system of regulation, multifunctional platforms in various positions with adjustable fastening system strengthened, tension adjustable backrest, armrest for hemiplegic, tables of various characteristics, pushes for the trunk, retractor, footboard, pelvic girdle 45 ° and push bar for the partner, with adjustable height.

Introduction

Symbols

While reading the manual you might come across these symbols of particular importance or interest.

symbolizes danger in general. Particular attention must be paid.

key for nuts or bolts from 6/19 mm.

Allen wrench for 6mm hollow hex-screws.

The Optima E are equipped with:

- seat and back tilting system
- drum brakes
- footrest with adjustable height, tilt and flexion-extension
- height and width adjusting armrests for Optima 400

allowing in this way a greater comfort.

Moreover: a turning up backrest, removable footrests and armrests, minimize the space-footprint for easy transport.

Delivery

Every product of Nuova Blandino is checked out in the factory in order to avoid any blemish, then they are packed in special boxes and placed in wooden platforms.

For this reason we ask you to check your wheelchair immediately after receiving it and verify any possible damages happened during its transport. If it is possible do it with the presence of the deliver.

In case you find any damage caused by the transport, it must be immediately reported in the delivery seal.

Then you need to send it Carriage Free to Nuova Blandino S.r.l - in 8 days after the delivery date – the damaged parts if they are removable, or the whole chair, with a copy of the delivery seal.

The identification of the damage can not happen if you do not follow this process or do not signal the damage after received the wheelchair.

Assembling

The orthopaedic specialized technician will deliver your wheelchair ready for being used and suitable for your personal needs.

Using details

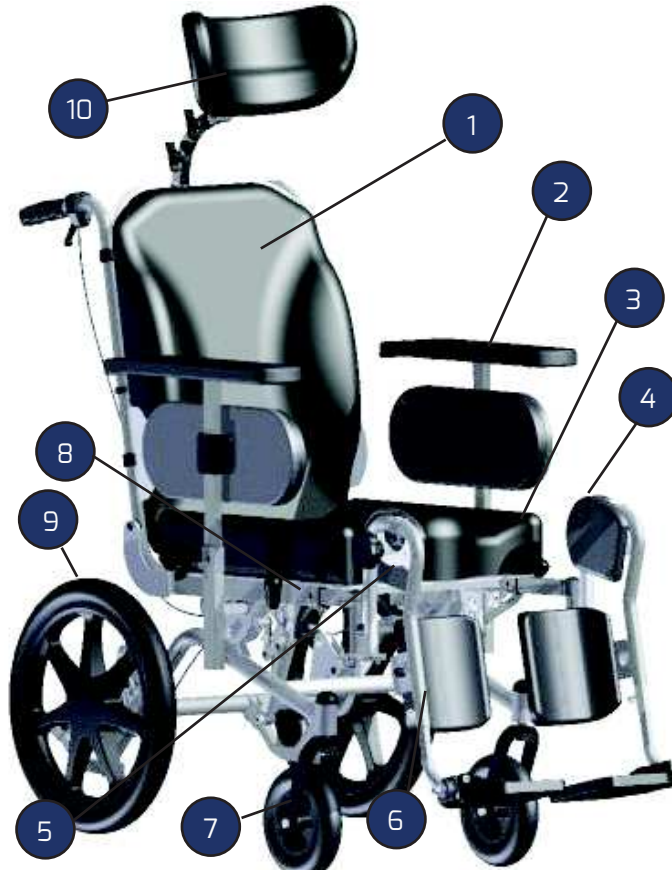
The electronic wheelchair is designed to be used indoor and outdoor. It has the only purpose to transport a person on the seat with the footrest and the side parts assembled.

It does not work as means of traction, transportation or others.

Information and security

Every information for a secure use of this wheelchair are reported in the second part of this booklet.

Components OPTIMA E

1. Reclining backrest
2. Removable armrest
3. Adjustable seat
4. Multifunction platform
5. Knob nut for platform adjustment
6. Screw nut for calf pad adjustment
7. Front wheel
8. Drum brake lever
9. Rear wheel Ø 400
10. Multifunctional headrest

Check before use

Before using the multifunctional highchair, make sure that all parts are properly assembled and tightened, in particular the armrests and footrests.

If there are elevating platforms, verify that they allow a comfortable seat with a proper angle of the legs.

Check that the push brakes work properly. While activating the brakes on both wheels, push the chair forward and backward; braked wheels are expected not to rotate.

Aid use

The aid is ideal for indoor, outdoor leisure pool, as long as the ground support is solid and not slippery.

At the beginning, use the multi-functional high-chair with great caution.

Read the second part of the manual ("Safety information") carefully.

Brake - drum brake

Advantages: braking action regardless of the profile of the tire.

Braking is independent of the pressure of the tire.

Braking action even with wet tire.

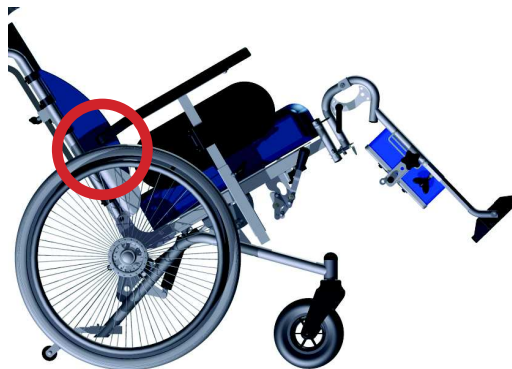
Armrests

- the armrests are removable and facilitate the transfer from the wheelchair to the bed, chair, etc. ...
- lateral support of the body
- protective clothing saves against the spokes of the wheels

Warning: The wheelchair must only be used when the armrests are mounted

Warning: In the model with 550 mm rear wheels (see figure) pay considerable attention: DO NOT insert the hands between the armrest and the rear wheel during the tilt of the seat. Risk of crushing and flattening of the phalanges.


Settings

Adjusting the armrest and the side pad

Height adjustment:

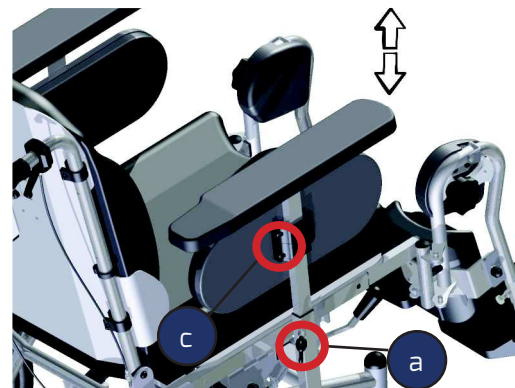- loosen the wing nut (a)
- adjust the height of the armrest
- find the best position
- tighten the wing nut (a)


Width adjustment:

- loosen the allen screw (b) (located on the cross under the seat)
- adjusting the width of the support arm
- find the best position
- tighten the allen screw (b)

Side pad height adjustment

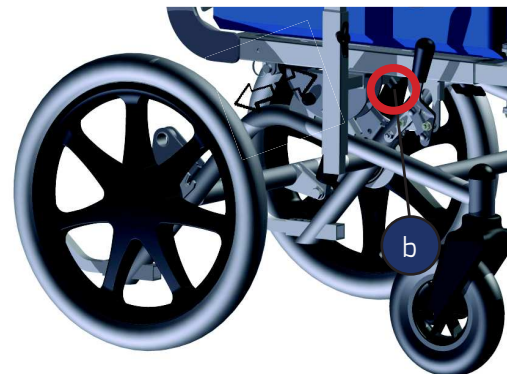- loosen the cutting cross screws (c)
- adjust the width of the little edge
- tighten the cutting cross screws (c)


Recline the backrest and the seat

The two actuators allow a soft and gradual adjustment of the backrest and the tilting of the seat. They are actuated by a control system comprising a hand control and a control unit , placed under the seat, which also contains the rechargeable batteries .

HAND CONTROL

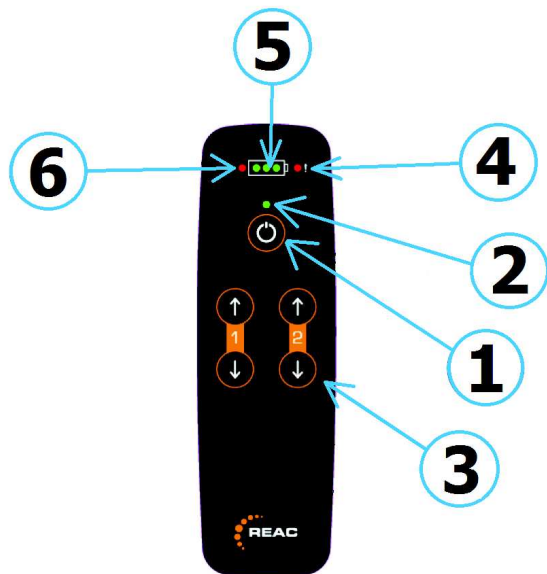In the figure below you will find an explanation of the buttons and the lights on the hand control

1. On/Off button
2. On /Off indicator light
3. Seat and backrest regulation button
4. Alert indicator light (internal errors/key lock)
5. Battery level/Status while charging indicator light
6. Charging is needed indicator light

Hand control: button and light

ON/OFF BUTTON AND ON/OFF INDICATOR

The on / off button (1) is used to activate / deactivate the system; the light above it indicates whether the system is on or off.

In order to minimize energy consumption, there is a timeout which will automatically turn the system off after 10 minutes inactivity.

RECLINING BACKREST AND SEAT

To adjust the seat and the backrest inclinations press the buttons 1 and 2 (indicated with the number 3 in the figure). It is only allowed to operate one actuator at a time. If two actuators are operated simultaneously, all movement will stop.

ALERT INDICATOR LIGHT

On hand control, there is an alert light (4) marked with the symbol “! “. If lit, it indicates that there was an error of the control unit or the ‘ keypad lock ‘ has been activated.

Fault handling and key lock will be explained below.

CHARGE INDICATOR LIGHT

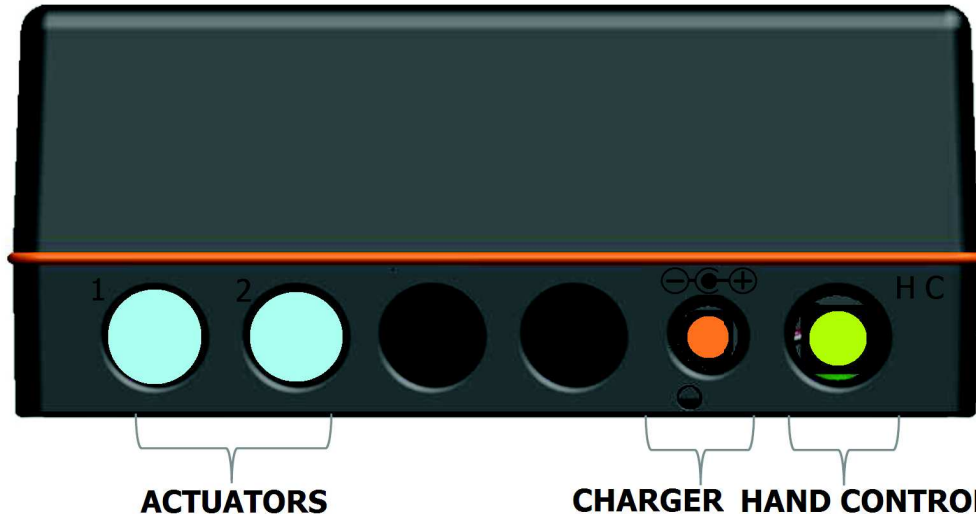
On the hand control there are the indicator lights (5) that show the battery charge level and its state while charging. If the battery level is very low, a warning light (6) shows that it is necessary to recharge the batteries as soon as possible.

Note: when the level is critically low all operation will be aborted to protect the batteries.

Control unit: connectors and charging

CONNECTOR PLACEMENT

The figure below shows the placement of the connectors on the control unit.

Note: actuators connectors and hand control connector are of the same type . Make sure that they are correctly plugged. A failure will not harm but it will not operate properly .

CHARGING

The batteries are charged through an external charger . Charging is only allowed with the charger supplied with the control system. The time required to fully recharge the batteries as they are discharged is about 6 hours.

Note : in order to protect the system, the movements of the actuators are not allowed during charging .

Fault handling and key lock

INTERNAL ERRORS

If any internal error is detected to the control system, the operation on the actuator/s is interrupted and the alert light on hand control is lit.

Internal errors can be deleted by pressing the On / Off button and activating the actuator.

If an actuator does not stop when released its actuating button, the pressure on any other button will lead to the arrest of all movements.

KEY LOCK

It is available for the hand control a key lock to inhibit the functions of the buttons. To lock the buttons of an actuator, press both buttons for 3 seconds until the alert light is lit.

If the system is turned off with the key lock activated and then is started again, the alert light will lit for a short while to indicate that a key lock is active.

The same procedure is used to unlock the keypad: press both operating buttons simultaneously for 3 seconds until the alarm light is switched off.

CLEANING AND DISINFECTION

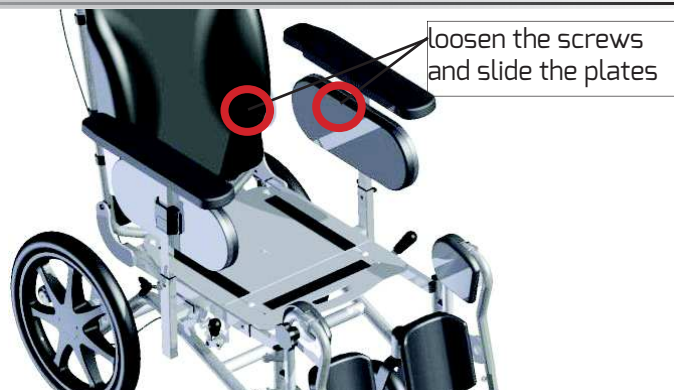
The control system and the hand control are adequately protected and can be cleaned with water. For their disinfection using mild soap.

Note: the connector for charging on the control unit is protected by a small cap; please be aware that this cap can be removed and swallowed by a child (choking hazard).

Note: do not try in any way to open the unit, the hand control or the charger. Such action will void the warranty. Contact Nuova Blandino for any kind of information.

Adjusting the seat depth

- remove the pillow
- loosen the screws indicated in the figure
- move forward or behind the seat plate and find the correct position
- tighten the screws
- replace the pillow

Adjusting the backrest

Height adjustment:

- remove the padded backrest
- loosen the nuts and move the seat back to the desired height
- tighten the nuts
- reposition the padded backrest

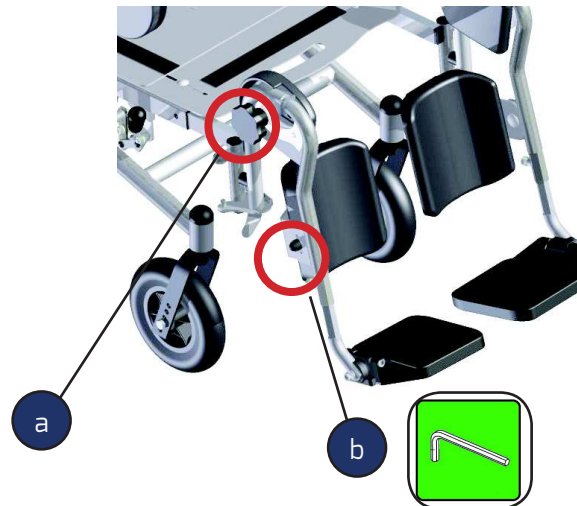
Adjusting multifunctional elevating platforms

The multifunctional elevating platform allows: to change the angle of the extension of the leg and the adjustment of the flexion-extension of the two platform.

The footrest can be folded down to facilitate the approach of the person on the wheelchair.

Adjusting the height of the elevating platform:

- loosen the handle (a) and accompany the platform until it is in the desired position
- tighten the handle (a)

Adjusting calf pad

The calf pad prevents slipping of the legs from the platform.

calf pad adjusting:

- loosen the knob (b), turn the calf pad until it is in the desired position and tighten the knob (b).

Attention!

Never lift the wheelchair by calfpad as they are mobile elements.

Do not insert your fingers into the gap when adjusting the calf pad. Risk of crushing.

Adjusting footrest height

Adjusting footrest height:

- loosen the wing screw (a) and adjust it to the desired height
- tighten the wing screw (a) in order to lock the rod of the platform.

WARNING!

Make sure that the pin of the platform is inserted at least 5 cm inside the support platform.

Adjust the pin of the platform in order to always have a gap between the plastic platform and the ground about 6 inches

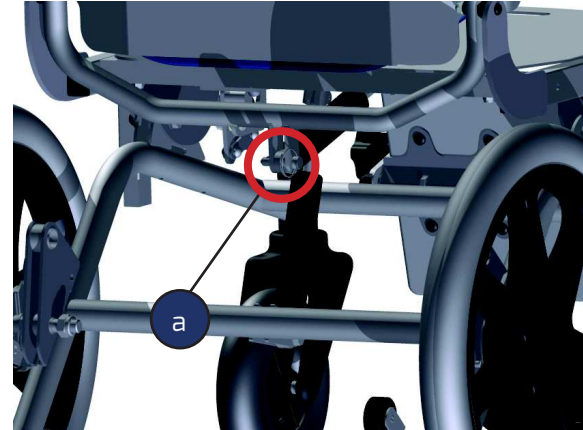
Transport by car

Carrying the wheelchair in the car:

- remove the footrests
- pull the armrests
- remove seat and backrest padded
- remove the pin with ring (fixing piston) (a)
- break down the backrest

for the model Optima 550 remove the wheels by the quick release axle.

To reassemble the wheelchair just follow all the steps in reverse order.

Maintenance of the wheelchair

Yearly checks

Maintenance should be performed once a year. It is highly recommended to check the state of the bearings, since sand, dust and water may damage them. Lightly lubricate with bicycle resin-free oil.

The frame may be cleaned with hot water and soap (or household detergent) only. Plastic parts should be cleaned with detergents specific for this material.

Caution!

Do not use high-pressure cleaning equipment!

The chairs should not be cleaned with harsh detergents.

Caution!

All components which come in direct contact with the skin must be replaced if the wheelchair is given to another user.

Repairs

Repairs

In case any repair is needed, contact your Nuova Blandino S.r.l. dealer, who knows how repairs should be performed according to our specifications and will be able to offer advice, support and provide repair services.

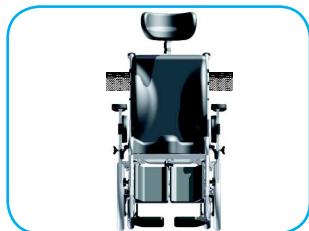
Spare parts

Order through your dealer. In order to ensure the correct identification of the spare parts which must be replaced, indicate the highchair frame registration number. For safety reasons, use spare parts and accessories authorized by Nuova Blandino S.r.l.


Disposal

Disposal must be performed in compliance with specific national legislation.

Measures

cm 36 40 45

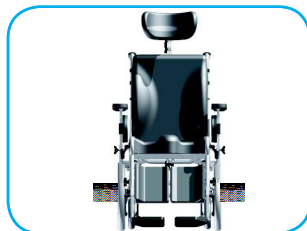
cm 57 61 66


cm 86

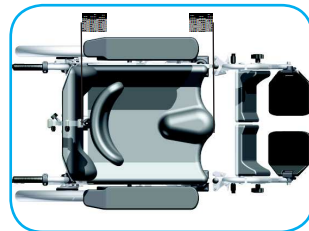
cm 111


cm da 41 a 56

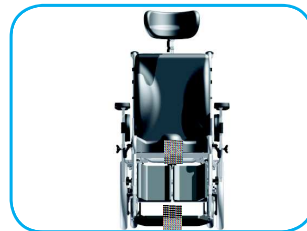
cm 52

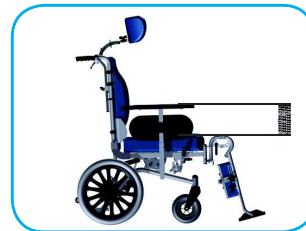cm from 20 to 30

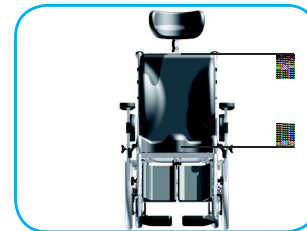
cm from 54 to 60

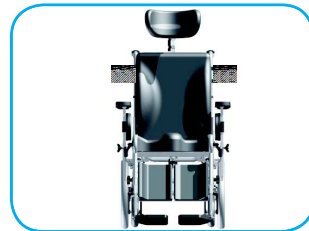optional 50 cm

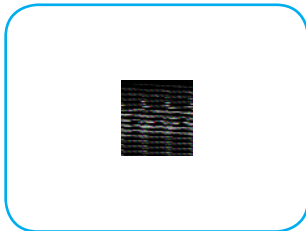Kg 29

Max. Load Kg 120

Optionals

Hemiplegic armrest

Anatomic tray table with
fixing brackets

Rectangular table with fixing
brackets

Padded anatomical table with
brackets

Multifunctional headrest

Tension adjustable backrest

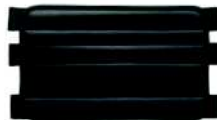
Multifunction platform

Footboard

Strap cm 15

Strap cm 25

Abduction

Lateral pad

Optionals

Single belt

Safety belt

Great pelvic belt

Retractor band

Single harness

“Y” shaped belt

General safety rules

Use OPTIMA only as described in the instructions for use. To stop going up or down always choose a leveled and stable ground. Ensure the wheelchair to prevent it from moving by using both brakes.

The cover and the filling of the seats and armrests, leg rests and the handles can reach temperatures of over 41 ° C. If exposed to direct sunlight, if you touch these parts with unprotected skin there is a risk of burns. To avoid the heat of these parts of your wheelchair, park it in the shade.

Your dealer can adjust the OPTIMA in according to your individual needs, in the utmost respect for your safety.

When carrying objects, make sure that these do not affect or restrict the maneuverability of the wheelchair. Check the state of wear of the tires and check the brakes.

Portable weight

The maximum permissible load is 120 kg and can be found in the instructions for use and on the type label of the aid.

The aid can transport only one person.

Transfer from the wheelchair

Disembarking from a wheelchair to move to another place, if done independently, it is dangerous.

Require:

- relevant physical forces
- good coordination skills
- a safe place
- much exercise

knowledge of every possible danger situations.

Let yourself be helped by a man, discuss with him the exercise and keep in mind that there may be unforeseen dangers such as: possible tripping obstacles.

Crossing narrow passages

When you travel through narrow-tracks, for example, building entrances, doors, etc., Attention to elbows. - **Risk of injury.**

Use the elevator and lifts

Lifts and elevators are a particular danger for the user.

Caution!

Pay attention to the size of the lift and pack of AT to avoid getting stuck.

Transport by car or other means

The wheelchair should never be used as a seat when you use a car or other means of transport. Provide the disassembly of the wheelchair (as specified in the instructions for use) and store it in the trunk, or at least secure it to prevent accidents in case of dangerous maneuvers.

Occupy a suitable place inside the car or the means of transport and fasten your seatbelts.

Declaration of conformity

NUOVA BLANDINO Sr.l.

registered office: Strada Cebrosa, 24
10036 Settimo Torinese Fraz. - Torino

Responsible for designing, manufacturing, packaging, and labeling of this product, that falls within the category of
MULTIFUNCTIONAL CHAIRS

DOES GUARANTEE AND DECLARE

The aforementioned device, launched and marketed as of the date certified in this certificate, fully complies with the provisions stated in annexes I and VII to EEC Directive 2007/47 and L.D. No 37 of 25 January 2010, that confirmed EU provisions at Italian level.

The products listed above are classified as Class 1 non-invasive medical devices as provided in Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product, the complete "EC" declaration of conformity is available on the website www.nuovablandino.com

The afore-mentioned products:

- *are not measuring instruments*
- *are marketed in non-sterile packaging*

This is to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

Settimo, 25th November 2015

WARRANTY

On the aids from our factory, we provide a warranty period of two years from the correct completion of the certificate of purchase (see page 29).

The warranty is valid for products that have defects in workmanship or operation.

The warranty excludes products that exhibit defects due to negligence or improper use.

Through our dealers, you can still get information on estimates for repairs.

For safety reasons, use spare parts and accessories approved by Nuova Blandino S.r.l.

The use of spare parts and accessories not approved by Nuova Blandino S.r.l. are reason of the immediate invalidation of the warranty.

To ensure the efficiency of the electric wheelchair, it must be kept clean, organized and maintained as specified in the instructions for use.

Attenzione:


Warning:

We accept no responsibility for damage caused by the combined use of the electric wheelchair with parts of all kinds from other manufacturers (for example trailers units, electrical devices) that may involve significant dangers - except that a given product has not been expressly authorized in writing by the Nuova Blandino S.r.l.

Nuova Blandino S.r.l. declines all liability for damage to persons, property, animals, and any third party due to modifications or improper use of the wheelchair.

Plate and EC label

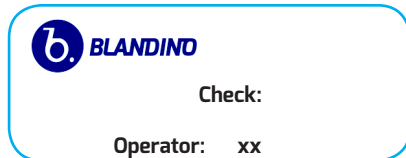The 'EC label (see picture) on the wheelchair found on the front crossmember under the seat.

Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

Advisory notice

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive.

- **Intended use**

The device is supplied ready for use by technical orthopedics and adjusted according to the user's needs.

The multifunctional chair is designed for both indoor and outdoor use. It is intended to accommodate one single person on the seat when the legrests and the lateral pads are mounted, and should not be used as a means of traction, transport or otherwise. Carefully read the instruction manual for a correct use of the device.

- **Check before use**

Before using OPTIMA, make sure that all parts are properly assembled and tightened, especially the platforms. If the product shows visible damage, damaged parts must be replaced to ensure your safety.

- **How to clean**

Do not use high-pressure cleaning equipment!

The chairs should not be cleaned with harsh detergents.

- **Lifespan of the device**

The duration of the multifunctional chair of our manufacturing, depends on the intensity of use to which it is subject. Proper use and proper maintenance allows you to maintain and prolong the life of the device, on average, an estimated duration of 5 to 7 years.

- **Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product.

Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 2:00 HOURS AND FROM 13:30 TO 7:00 HOURS. CALL +390112625140.

RETAILER STAMP

--

DATE OF PURCHASE

--	--	--

REGISTRATION DATE

--	--	--

N.B.: THIS MANUAL MUST BE PRODUCED WHENEVER ASSISTANCE IS NEEDED

Nuova Blandino S.r.l.
Strada Cebrosa, 24 - 10036 Settimo Torinese
Torino Italy
Tel. +39 011 2625140 - fax +39 011 2625141
www.nuovablandino.com - inforb@nuovablandino.com

Commitment and technology for people with special needs