MOPEDIA®

SOLLEVATORI ARKIMED

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INDICE

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INSTRUCTION MANUAL

INDEX

1. CODES	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.4
8. DESCRIPTION OF THE PARTS	PAG.5
8.1 Patient lift structure	pag. 5
8.2 Hydraulic actuator - RI700C (135 KG), RI701, RI701C (180 KG).....	pag. 5
8.3 TiMOTION electrical TiMOTION RI703C (135 KG) - RI705 (150 KG) - RI710C (180 KG).....	pag. 5
8.4 TiMOTION electrical KIT - RI710 (180 KG)	pag. 6
9. FRAME ASSEMBLY INSTRUCTIONS	PAG.6
10. ACTUATOR ASSEMBLY ISTRUCTION	PAG.7
10.1 Assembly HYDRAULIC actuator - RI700C, RI701, RI701C.....	pag. 7
10.2 Assembly TiMOTION electrical KIT (for RI703C, RI705, RI710, RI710C).....	pag. 7
11. BEFORE USE	PAG.8
12. INSTRUCTIONS FOR USE.....	PAG.9
12.1 Instructions for using HYDRAULIC model RI700C, RI701, RI701C	pag. 9
12.2 Instructions for use ELECTRICAL Timotion model RI703C, RI705, RI710, RI710C.....	pag. 9
13. MAINTAINANCE.....	PAG.12
14. CLEANING AND DISINFECTION	PAG.12
14.1 Cleaning.....	pag. 12
14.2 Disinfection	pag. 12
15. SPARE PARTS	PAG.12
16. CONDITIONS OF DISPOSAL	PAG.13
16.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE	pag. 13
16.2 Warnings for the proper disposal of the product, as required by the European Directive 2006/66/CE - Treatment of dead batteries	pag. 13
17. TECHNICAL FEATURES	PAG.13
17.1 Technical specifications.....	pag. 13
17.2 Dimension RI7xx	pag. 14
18. WARRANTY	PAG.14
18.1 Warranty for rechargeable battery (if provided).....	pag. 14
19. REPAIRING	PAG.15
20. SPARE PARTS	PAG.15
21. EXEMPT CLAUSES	PAG.15

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

- RI700C** - Hydraulic actuator max load 135 Kg, with pedal
- RI701** - Hydraulic actuator max load 180 Kg
- RI701C** - Hydraulic actuator max load 180 Kg
- RI703C** - TiMOTION actuator max load 135 Kg, with pedal
- RI705** - TiMOTION actuator max load 150 Kg
- RI710** - TiMOTION actuator max load 180 Kg
- RI710C** - TiMOTION actuator max load 180 Kg

2. INTRODUCTION

Thank you for choosing a patient lift **ARKIMED by MOPEDIA** range, manufacturer by **MORETTI S.p.A.** Their design and quality are a guarantee of comfort, safety and reliability. **ARKIMED by MOPEDIA lifts by MORETTI S.p.A.** are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the lift you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the patient lift/stand up lift. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

ARKIMED by MOPEDIA patient lifts are intended to lift and move people with motor difficulties. An assistant operates the machine. The lifts cannot be used by only one person.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of PATIENT LIFTS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

To ensure safety for users, Moretti S.P.A. test and respect the EN ISO 10535:2006 specific for lifters, to guarantee greater security.

WARNING!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual, checking with a particular attention at the warnings before use.
- For proper use of the device to consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Attention, should read the instructions

Production dates

Waste Electrical & Electronic Equipment (WEEE)

Protection against dust and water splashes

B Type

7. GENERAL DESCRIPTION

The ARKIMED patient lift helps users to move less self-sufficient. Ideal for wheelchair users. It is the perfect aid to transfers the patient from a sitting or lying down. Facilitates the work of medical staff while making patient transfers from one surface to another much safer, thanks to the breadth and stability of the opening leg. Electric ARKIMED patient lift meets all requirements for a comfortable and safe transfers, due to its unit control: it is an intelligent monitoring system that means a bright code checks the charge level of the battery and the actuator piston conditions.

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




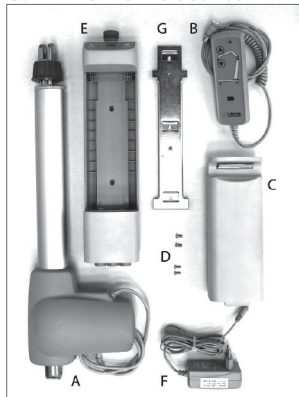
9. FRAME ASSEMBLY INSTRUCTIONS


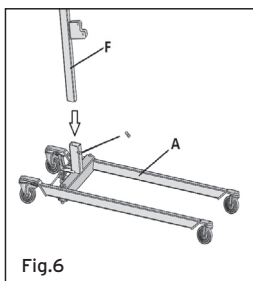
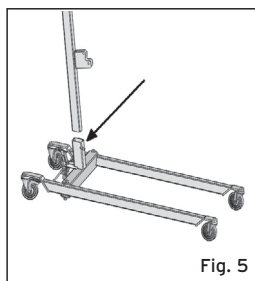
1. Unscrew the screw at the base of the column (Fig.5)
2. Insert the column into the provided cavity of the base. (Fig.6)
3. Insert the screw that blocks the column into the base and screw it well. (Fig.6)

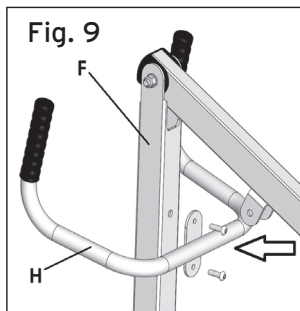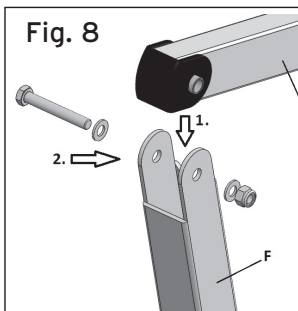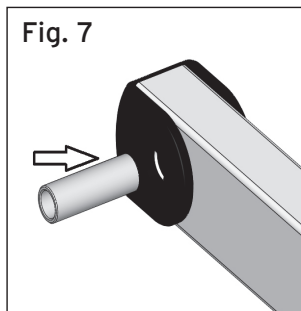
4. Connect the Patient lift arm to the column with the journal box (Fig.7) and provided screw. (Fig.8)
5. Fit the handle on the column and block it with the 2 screws. (Fig.9)

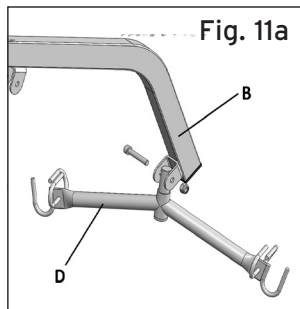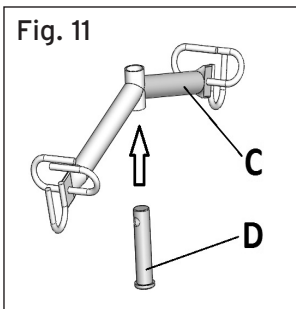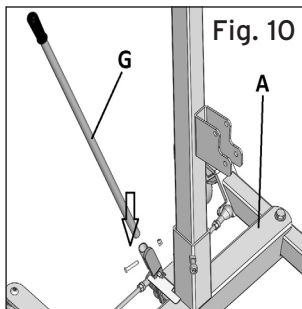6. Insert the lever for opening the legs and block it with its screw (only for RI701-RI701C-RI705-RI710-RI710C). (Fig.10)
7. Place the cradle in the corresponding housing, situated at the end of the patient lift arm. Insert the linchpin and the fixing screw including a nut. (Fig.11)

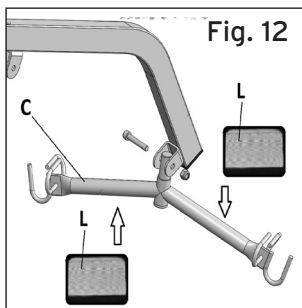8. Put the velcro handles onto the cradle. (Fig.2)
9. The structure is completely assembled (Fig.13). Install the actuator suitable for the chosen patient lift (Fig.13)

*images for illustrative purpose only

Fig. 14

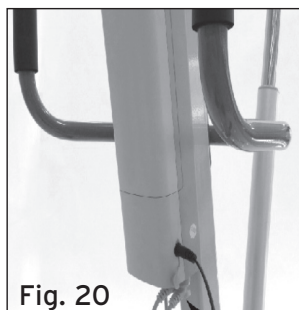
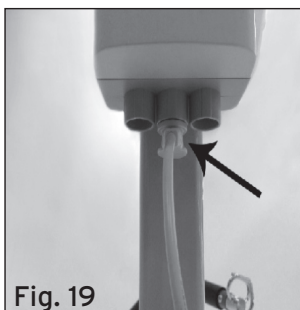
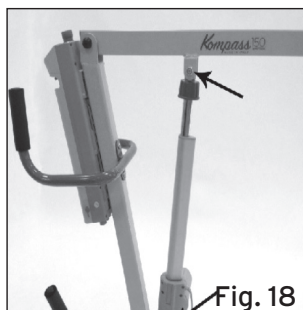
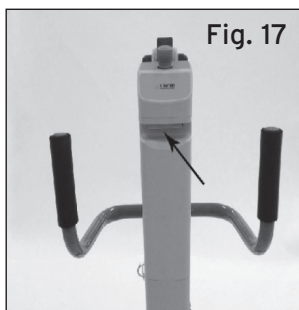
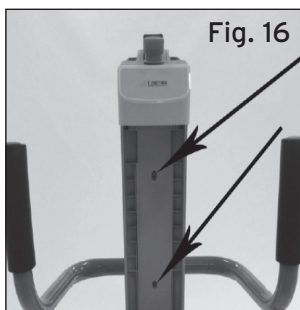
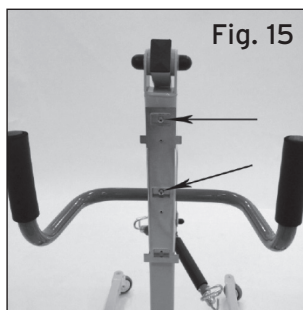
10. ACTUATOR ASSEMBLY ISTRUCTION

10.1 Assembly HYDRAULIC actuator - RI700C, RI701, RI701C

1. Place the hydraulic actuator as in the picture, blocking well its upper and lower part by auto-blocking screws and bolts. (Fig.14)

10.2 Assembly TIMOTION electrical KIT (for RI703C, RI705, RI710, RI710C)

1. Secure the support bracket of the control unit to the mast with the provided screws, using the fitted cavities. (fig.8)
2. Fit the control unit on the bracket, letting it slide from top to bottom and blocking it with the screws. (Fig.9)
3. Fit the battery into the control unit. (Fig.10)
4. Remove the two actuator mounting bolts from the mast boom. Fit the actuator into it and secure it with the two bolts previously removed. (Fig.11)

5. Connect the actuator connector to the central socket of the control unit and the remote control to the right socket. (Fig.19-20)

11. BEFORE USE

- Before lifting a patient, make sure the base legs are opened to the maximum width possible, that the opening lever is secured in the opened position and the rear wheels brakes are blocked.
- While moving the patient, keep an eye out for obstacles that might cause the lift to tip over.

WARNING!

Always check that the sling is in good condition. Do not use slings showing signs of damage or wear.

12. INSTRUCTIONS FOR USE

12.1 Instructions for using HYDRAULIC model RI700C, RI701, RI701C

1. Before lifting a patient, open the base legs to the maximum width possible, gently pulling the opening lever towards you and secure it in the open position.
2. Block the rear wheel brakes.
3. Keep the patient centered between the base legs and always turned towards the assistant.
4. Carefully place the sling on the patient as explain the sling manual.
5. Hang carefully the sling's straps to the cradle (see the sling user manual).
6. Make sure that the valve on the hydraulic pump is completely closed (rotate in the direction "CLOSE").
7. Use the pump lever to start lifting.

ALWAYS CHECK THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!

8. Lift up the patient until the desired height, keeping him facing his assistant.
9. Unlock the rear wheels and move the patient.
10. Before lowering the patient, block the rear wheels again.
11. Start lowering the patient by slowly turning the valve on the hydraulic pump in the direction "OPEN".

Note: To make a slow descent open the valve slightly, while if you want to increase the speed open the valve more always in the direction of "OPEN".

ALWAYS HELP THE PATIENT DURING THE LOWERING PROCESS SO AS TO MAINTAIN THE CORRECT SEATED POSITION! WHILE LOWERING THE PATIENT!

WARNING!

While lowering the patient, please remove any barrier that can impede the downward movement. If the case, lift up the patient again and remove what causes the impediment for lowering.

12.2 Instructions for use ELECTRICAL Timotion model RI703C, RI705, RI710, RI710C

WARNING: BEFORE USING THE ELECTRIC LIFT IS RECOMMENDED TO CHARGE THE BATTERY FOR ABOUT 6/8 HOURS. MAKE SURE THE RED BUTTON EMERGENCY HAS BEEN TRIPPED. THEREFORE MAKE SURE THE BATTERY IS CHARGING BEFORE USING.

1. Before lifting a patient, open the base legs to the maximum width possible, gently pulling the opening lever towards you and secure it in the open position.
2. Block the rear wheel brakes.
3. Keep the patient centered between the base legs and always turned towards the assistant.
4. Carefully place the sling on the patient as explained in the sling manual.
5. Hang carefully the sling's straps to the cradle.
6. Press the up-facing arrow key to lift the patient. (For the model RI710 the led display on the control unit will show the boom moving)

ALWAYS CHECK THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!

7. Lift up the patient until the desired height, keeping him facing his assistant.
8. Unlock the rear wheels and move the patient.
9. Before lowering the patient, block the rear wheels again.
10. To lower the patient, press the down-facing arrow key. (For the model RI710 the led display on the control unit will show the boom moving)

ALWAYS HELP THE PATIENT DURING THE LOWERING PROCESS SO AS TO MAINTAIN THE CORRECT SEATED POSITION! WHILE LOWERING THE PATIENT!

WARNING!

While lowering the patient, please remove any barrier that can impede the downward movement. If the case, lift up the patient again and remove what causes the impediment for lowering.

EMERGENCY STOP

The control unit is equipped with a red “STOP” button which instantly deactivates all controls when pressed. To restore normal operation, turn the button clockwise and pull it outwards.

MECHANICAL LOWERING OF THE ACTUATOR

If using the manual/mechanical lowering, please make sure that it will be done in total safety conditions for the patient. In order to activate the manual/mechanical lowering, pull downwards the red ring placed in the upper part of the actuator and turn it counterclockwise. Once finished the lowering process, take the red ring back to its resting position.

EMERGENCY KEYS

RI710: On the control unit there are two keys (one with an up-facing arrow and the other one with a down-facing arrow) that can be pressed instead of those ones on the remote control to complete all the operations when the remote control fails to work properly.

INDICATION OF THE BATTERY STATE OF CHARGE

There is a 3-Led indicator for the battery state of charge. This indicator is placed on the remote control. 3 Leds on indicate a complete charge of the battery, whereas one just Led on indicates the battery is out of power and needs to be charged.

RECHARGING BATTERIES

WARNING!

Make sure the red emergency button “STOP” is turned off. To do this, turn and pull towards you the RED button


- Connect the adapter to the main socket and then to the socket on the control unit, after removing the protection cap.
- On the charger adapter there is a led that during the charging process becomes orange color. Charge the battery until the led becomes green color.
- Before activating the stand up lift again, make sure to disconnect the battery charger!
- Charge the batteries for at least 6 hours before using the patient lift.

COLOR DISPLAY (Only for the model RI710)

This model is equipped with an electronic control unit with color graphic display.

LEFT KEY: Pressing the left key for more than 0,5 seconds, the boom lifting function can be activated		RIGHT KEY: Pressing the right key for more than 0,5 seconds, the boom lowering function can be activated
---	---	--

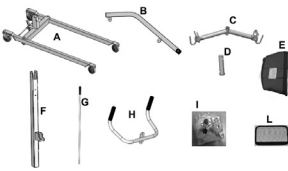If one of the two buttons is pressed for less than 0,5 seconds, one of the following functions will be activated:

		
Activation lifting boom (valid for all the models)	Electrical activation base legs opening (only with an electrical actuator for opening base legs)	Residual battery capacity

MAINTENANCE	MAINTENANCE
	
Months	Cycles
Time of use in MONTHS	Cycles done

NOTICE (Only for the model RI710)

ATTENTION	ATTENTION
 Recharge Or Exchange	 Overloaded
Low battery capacity Recharge or replace the battery	Excess load alarm The load lifted by the patient lift exceeds the maximum level allowed. Remove the load and try again.

13. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

- It is extremely important to check the patient lift before each use. More specifically, please check that all screws and bolts are well tightened.
- Please, check the patient lift can move freely with the wheels unlocked by the brakes.
- Check the rear wheel brakes function correctly.
- It is very important to regularly check the connection pins and stop rings are correctly located (the pins connect the actuator to the upper and lower part of the lift, and the lifting beam where you hook the cradle).
- The mechanical parts must be lubricated with a drop of oil in order to facilitate their sliding and resistance to wear. In particular, pay attention to the mechanical retractor for the legs of the patient lift. Lubricate as indicated above every two or three months.

14. CLEANING AND DISINFECTION

14.1 Cleaning

To clean the cradle and the structure use a damp cloth with solution of alcohol or neutral soap (in equal parts) mixed with water.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacturing company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

14.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

15. SPARE PARTS

For spare parts and accessories, please always refer to the MORETTI master catalogue.

ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

16. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

16.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the product itself bears the barred bin symbol.

16.2 Warnings for the proper disposal of the product, as required by the European Directive 2006/66/CE - Treatment of dead batteries

The symbol on the product indicates that the batteries must not be treated like common household waste. Make sure the batteries are properly disposed; this prevents the possible environmental and health consequences of an improper disposal. Recycling helps to preserve natural resources. Deliver dead batteries to a proper recycling centre. For more detailed information regarding the disposal of dead batteries or of the product itself, please contact the municipality, the local waste disposal centre or the reseller from who you purchased the device from.

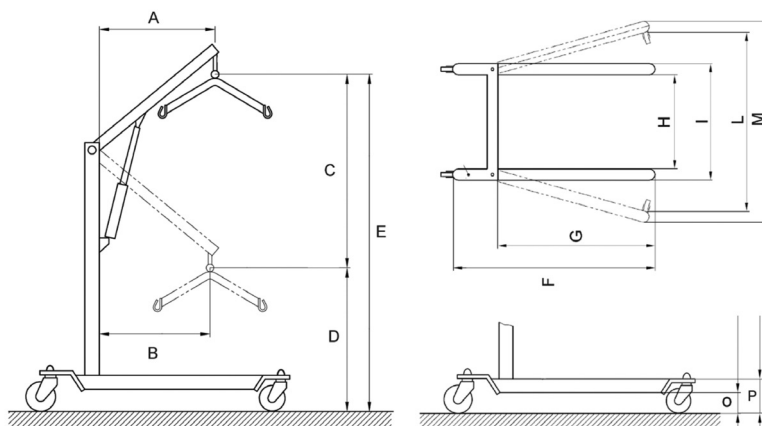
17. TECHNICAL FEATURES

17.1 Technical specifications

- Powder coated steel frame
- Castors with brake and rotational block
- Leg opening lever with safety lock
- ergonomic foam handles for the healthcare professional
- ergonomic polyester grips for the healthcare professional
- Hooks designed specifically to hold the sling in complete safety
- Autonomy: 100 to 200 lifts with fully charged battery
- Total weight: 35 Kg
- Respect of the EN ISO 10535: 2006
- Maximum payload of 135Kg for RI700C and RI703C
- Maximum payload of 150Kg for RI705
- Maximum payload of 180Kg RI701, RI710 and RI710C
- "Emergency STOP" on the control unit
- Lowering the electric actuator by manual emergency
- Electric actuators PUSH ONLY
- The battery charge Viewer 3 levels

17.2 Dimension RI7xx

Dimension and weight

Measures in CM

Cod.	A	B	C	D	E	F	G	H	I	L	M	O	P	MAX loading weight
RI701 RI701C RI705 RI710 RI710C	80	53	91	72	163	126,5	97	54	64	110	120	8	13	RI705: 150 Kg RI701 RI701C RI710 RI710C: 180 Kg
RI700C RI703C	48	48.5	91	69	160	116,5	87	45	54	80	100	8	13	RI700C RI703C: 135 Kg

18. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

18.1 Warranty for rechargeable battery (if provided)

The original batteries and the spare batteries are covered by a (ninety) 90-days warranty as far as their performance is concerned and by a (six) 6-months warranty as far as manufacturing faults are concerned or as required by law. Warranty on fully charged batteries left in an unused state for more than (three) 3 consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than (three) 3 consecutive days is automatically void.

19. REPAIRING

Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the customer, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti customer service. Labour and delivery costs for an out-of-warranty reparation will be fully at the customer's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the customer receives the repaired product.

Non defective device

The customer is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the customer and all the return delivery costs will be at his expense.

20. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

21. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its customers any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period.

Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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