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## INDEX

1. Introduction
3. Shipping
  - Assembling
  - Use specifications
  - Safety information
4. Components SPRING
5. Check before use
  - Aid use
  - Brake - Drum brake
6. Armrests
7. Settings
  - Adjusting the armrest and the side pad
8. Recline the backrest and the seat
9. Adjusting the seat depth
  - Adjusting backrest height
10. Adjusting multipurpose elevating platforms
  - Adjusting calf pad
11. Adjusting footboard height
12. Transport by car
13. Wheelchair maintenance
14. Repairs
15. Measures
16. Optionals
18. General safety rules
19. General safety rules
  - Transfer from the wheelchair
20. Crossing narrow passages
  - Using elevators and lift bridges
  - Transportation by car or other means
21. Declaration of compliance
22. Warranty
23. Plate and EC label
24. Maintenance log
27. Data sheet
28. Section for retailers

## Instructions for use and maintenance

### Introduction

SPRING is the result of twenty years of experience that has seen our company engaged in a process of continuous research to improve and refine techniques for the design and production of products. This has been possible using Technical Orthopedists and advices from experts of Rehabilitation, Education and Support.

This manual provides instructions for the use of these models. Please read the following instructions before using the aid for the first time. The special care taken in the design and manufacture of these products, ensure you a high degree of safety and comfort. Use your wheelchair by following the instructions contained in this manual.

**Model SPRING: multifunctional highchair for dystonic person.**

**Main clinical indications:** indicated for adult / elderly people with mobility absent, and that, by spending a long time in a sitting position, need to be able to vary the posture of the trunk; suitable frame to accommodate complex postural systems.

**Base equipment:** available in three sizes; made with washable and breathable fabrics. It equipped with tilting system from 0 ° to 32 °, reclining from 0 ° to 20 ° with angles, height adjustable backrest, seat adjustable in width and depth. Padding in polyurethane molded and shaped anatomically to accommodate situations of instability and deviation of the trunk-pelvis, basic stabilization of the backrest made of aluminum has lateral containment flaps which favor the action of the laterality of the trunk control. Armrests made of polyurethane material, removable and adjustable in height.

It equipped with absorption system made of different resistance springs, adaptable to the patient's structure characteristics.

**Optionals:** customization possibilities through the choice of accessories to meet your needs for comfort and safety: range of headrests equipped with a new system of regulation, multifunctional platforms in various positions with adjustable fastening system strengthened, tension adjustable backrest, armrest for hemiplegic, tables of various characteristics, pushes for the trunk, retractor, footboard, pelvic girdle 45 ° and push bar for the partner, with adjustable height.

## Introduction

### Symbols

While reading the manual you might come across these symbols of particular importance or interest.

symbolizes danger in general. Particular attention must be paid.

key for nuts or bolts from 6/19 mm.

Allen wrench for 6 mm hollow hex-screws.

The SPRING is equipped with:

- seat and back tilting system
- absorbing spring
- drum brakes
- footrest with adjustable height, tilt and flexion-extension
- height and width adjusting armrests for SPRING

allowing in this way a greater comfort.

Moreover: a turning up backrest, removable footrests and armrests, minimize the space-footprint for easy transport.

## **Delivery**

Every product of Nuova Blandino is checked out in the factory in order to avoid any blemish, then they are packed in special boxes and placed in wooden platforms.

For this reason we ask you to check your wheelchair immediately after receiving it and verify any possible damages happened during its transport. If it is possible do it with the presence of the deliverer.

In case you find any damage caused by the transport, it must be immediately reported in the delivery seal.

Then you need to send it Carriage Free to Nuova Blandino S.r.l - in 8 days after the delivery date – the damaged parts if they are removable, or the whole chair, with a copy of the delivery seal.

The identification of the damage can not happen if you do not follow this process or do not signal the damage after received the wheelchair.

## **Assembling**

The orthopaedic specialized technician will deliver your wheelchair ready for being used and suitable for your personal needs.

## **Using details**


The electronic wheelchair is designed to be used indoor and outdoor. It has the only purpose to transport a person on the seat with the footrest and the side parts assembled.

It does not work as means of traction, transportation or others.

## **Information and security**

Every information for a secure use of this wheelchair are reported in the second part of this booklet.

## Components SPRING

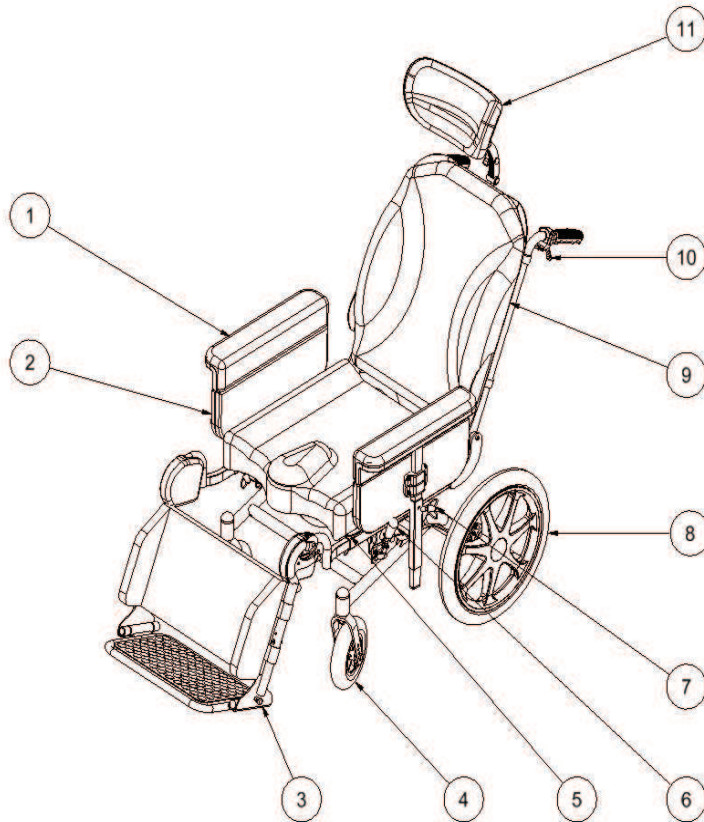1. Adjustable armrest
2. Containment side piece
3. Footboard
4. Front wheel Ø 170
5. Adjustable seat depth
6. Lever for the brake drum
7. Flyer armrest height adjustment
8. Rear wheel Ø 550
9. Back
10. Seat adjustment lever
11. Headrest multifunction

## Check before use

Before using the multifunctional highchair, make sure that all parts are properly assembled and tightened, in particular the armrests and footrests.  
If there are elevating platforms, verify that they allow a comfortable seat with a proper angle of the legs.

Check that the push brakes work properly. While activating the brakes on both wheels, push the chair forward and backward; braked wheels are expected not to rotate.

## Aid use

The aid is ideal for indoor, outdoor leisure pool, as long as the ground support is solid and not slippery.

At the beginning, use the multi-functional high-chair with great caution.  
Read the second part of the manual ("Safety information") carefully.

## Brake - drum brake

Advantages: braking action regardless of the profile of the tire.

Braking is independent of the pressure of the tire.

Braking action even with wet tire.

## Armrests

- the armrests are removable and facilitate the transfer from the wheelchair to the bed, chair, etc. ...
- lateral support of the body
- protective clothing saves against the spokes of the wheels

**Warning:** The wheelchair must only be used when the armrests are mounted.

## Settings

### Adjusting the armrest and the side pad

#### Height adjustment:



- loosen the wing nut (a)
- adjust the height of the armrest
- find the best position
- tighten the wing nut (a)

#### Width adjustment:

- loosen the allen screw (b) (located on the cross under the seat)
- adjusting the width of the support arm
- find the best position
- tighten the allen screw (b)

#### Side pad height adjustment

- loosen the cutting cross screws (c)
- adjust the width of the little edge
- tighten the cutting cross screws (c)

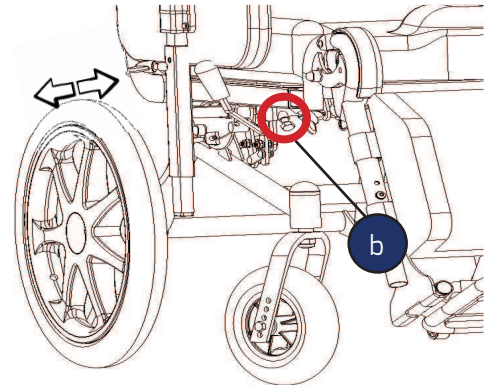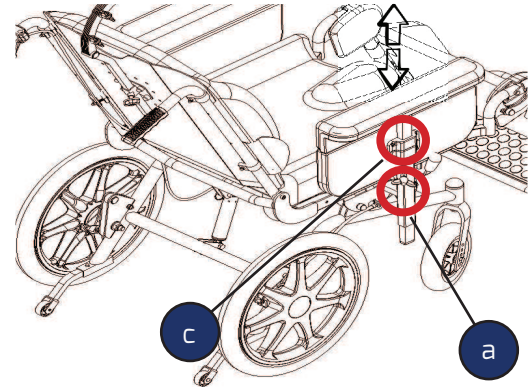
## Recline the backrest and the seat

### Adjusting:


The two pistons allow a soft and gradual adjustment of the backrest and the tilting of the seat.

Left lever  
Reclining seat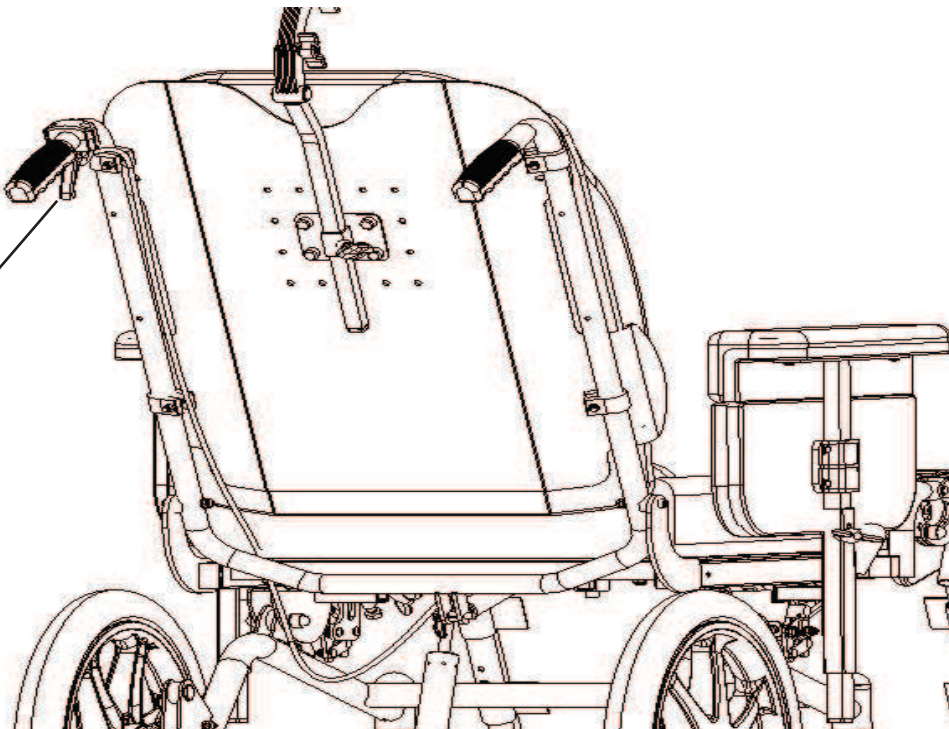

## Adjusting the seat depth


- remove the pillow
- loosen the screws indicated in the figure
- move forward or behind the seat plate and find the correct position
- tighten the screws
- replace the pillow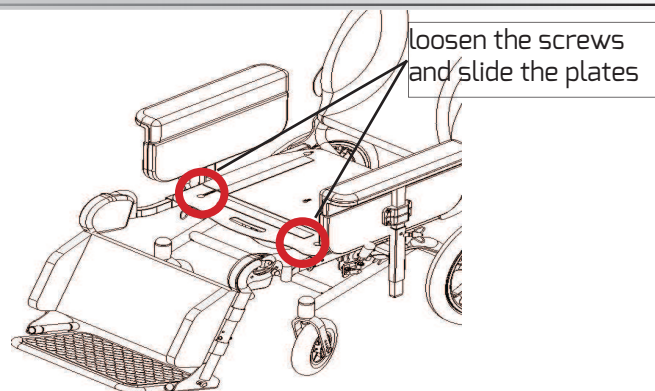

## Adjusting the backrest

### Height adjustment:

- remove the padded backrest
- loosen the nuts and move the seat back to the desired height
- tighten the nuts
- reposition the padded backrest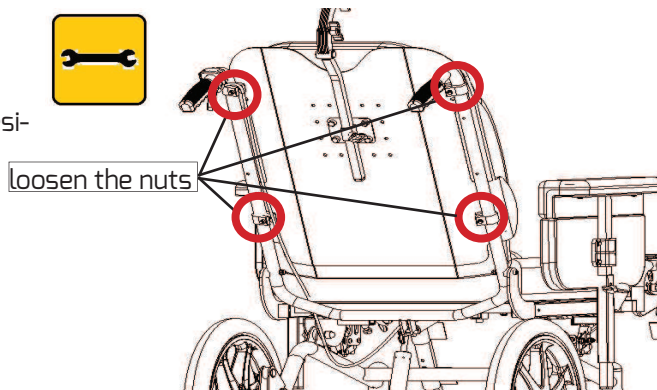


## Functionality of the footboard

The footboard is designed to perform a containment function of sudden spasm that characterize patients with dystonia.

Its padding (a) are designed to wrap the legs in comfortable manner.

The absence of constraints in the joint area of the platform (b), has the aim to allow the free lower limb movement during a spasm, so as to avoid impacts on the joints.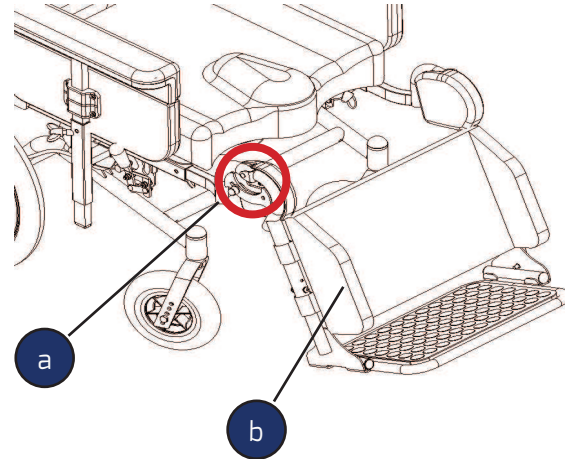

## Footrest depth adjustment


### Footrest depth adjustment:

- loosening knob (a) and adjust the desired depth.
- screw the knob (a) so as to lock the rod of the platform.

### WARNING!

Check that the footrest attachment is inserted at least 5 cm inside the seat frame.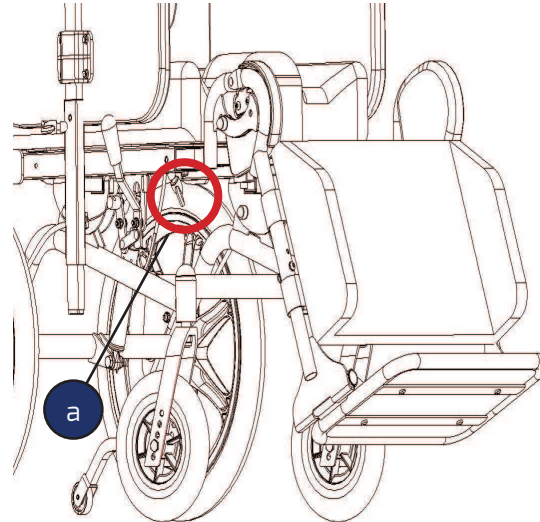


## Transport by car

### Carrying the wheelchair in the car:

- remove the footrests
- pull the armrests
- remove seat and backrest padded
- remove the pin with ring (fixing piston) (a)
- break down the backrest

To reassemble the wheelchair just follow all the steps in reverse order.

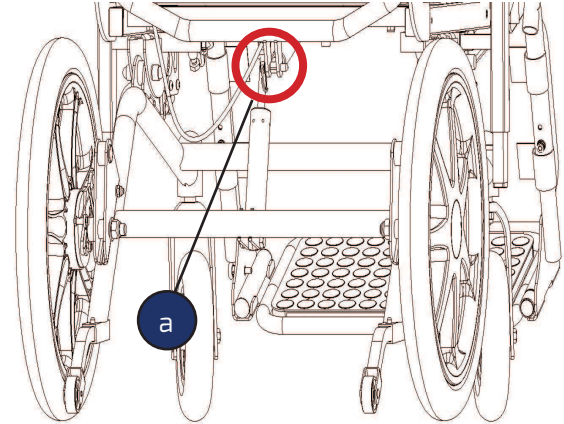## Maintenance of the wheelchair

### Yearly checks

Maintenance should be performed once a year. It is highly recommended to check the state of the bearings, since sand, dust and water may damage them. Lightly lubricate with bicycle resin-free oil.

The frame may be cleaned with hot water and soap (or household detergent) only. Plastic parts should be cleaned with detergents specific for this material.

**Caution!**

Do not use high-pressure cleaning equipment!

The chairs should not be cleaned with harsh detergents.

**Caution!**

All components which come in direct contact with the skin must be replaced if the wheelchair is given to another user.

## Repairs

### **Repairs**

In case any repair is needed, contact your Nuova Blandino S.r.l. dealer, who knows how repairs should be performed according to our specifications and will be able to offer advice, support and provide repair services.


### **Spare parts**

Order through your dealer. In order to ensure the correct identification of the spare parts which must be replaced, indicate the highchair frame registration number. For safety reasons, use spare parts and accessories authorized by Nuova Blandino S.r.l.


### **Disposal**

Disposal must be performed in compliance with specific national legislation.


## Measures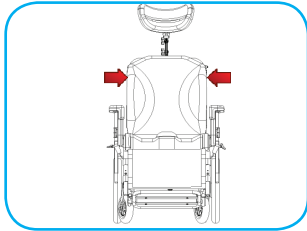


cm 36-40-45

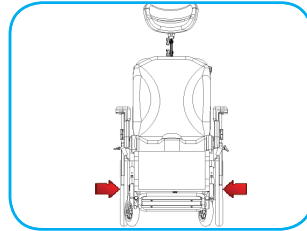
cm 57-61-66

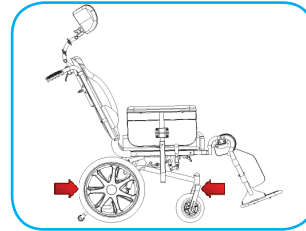
cm 86

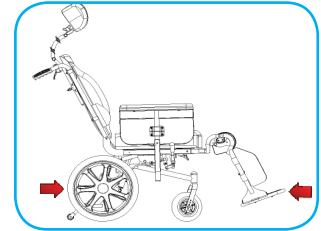
cm 111

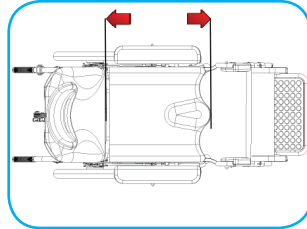
cm da 41 a 56

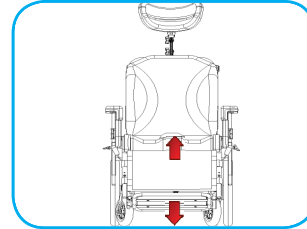
cm 52

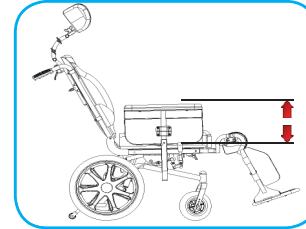cm from 20 to 30

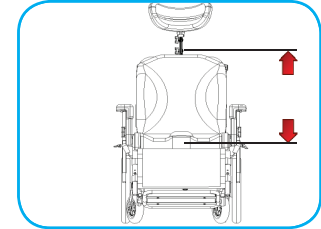cm from 54 to 60

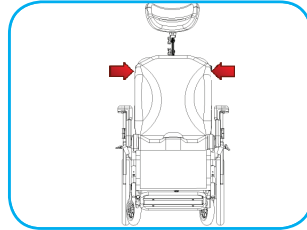optional 50 cm

Kg 29


Max. Load Kg 120


## Optionals

Hemiplegic armrest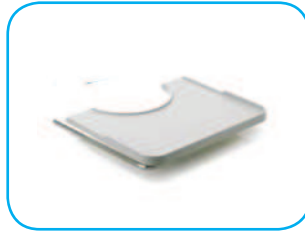

Anatomic tray table with fixing brackets

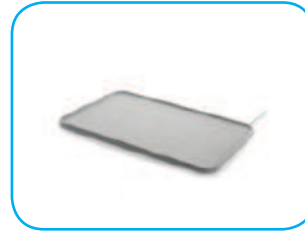Rectangular table with fixing brackets

Padded anatomical table with brackets

Multifunctional headrest

Tension adjustable backrest

Multifunction platform

Footboard

Strap cm 15

Strap cm 25

Abduction

Lateral pad

## Optionals

Single belt

Safety belt

Great pelvic belt

Retractor band

Single harness

"Y" shaped belt

## General safety rules

Use SPRING only as described in the instructions for use. To stop going up or down always choose a leveled and stable ground. Ensure the wheelchair to prevent it from moving by using both brakes.

The cover and the filling of the seats and armrests, leg rests and the handles can reach temperatures of over 41 ° C. If exposed to direct sunlight, if you touch these parts with unprotected skin there is a risk of burns. To avoid the heat of these parts of your wheelchair, park it in the shade.

Your dealer can adjust the SPRING in according to your individual needs, in the utmost respect for your safety.

When carrying objects, make sure that these do not affect or restrict the maneuverability of the wheelchair. Check the state of wear of the tires and check the brakes.

## Portable weight

The maximum permissible load is 120 kg and can be found in the instructions for use and on the type label of the aid.

The aid can transport only one person.

## Transfer from the wheelchair

Disembarking from a wheelchair to move to another place, if done independently, it is dangerous.

Require:

- relevant physical forces
- good coordination skills
- a safe place
- much exercise

knowledge of every possible danger situations.

Let yourself be helped by a man, discuss with him the exercise and keep in mind that there may be unforeseen dangers such as: possible tripping obstacles.

## Crossing narrow passages

When you travel through narrow-tracks, for example, building entrances, doors, etc., Attention to elbows. - **Risk of injury.**

## Use the elevator and lifts

Lifts and elevators are a particular danger for the user.

**Caution!**

Pay attention to the size of the lift and pack of AT to avoid getting stuck.

## Transport by car or other means

The wheelchair should never be used as a seat when you use a car or other means of transport. Provide the disassembly of the wheelchair (as specified in the instructions for use) and store it in the trunk, or at least secure it to prevent accidents in case of dangerous maneuvers.

Occupy a suitable place inside the car or the means of transport and fasten your seatbelts.

## Declaration of conformity

**NUOVA BLANDINO Sr.l.**

registered office: Strada Cebrosa, 24  
10036 Settimo Torinese Fraz. - Torino

Responsible for designing, manufacturing, packaging, and labeling of this product, that falls within the category of  
MULTIFUNCTIONAL CHAIRS

### **DOES GUARANTEE AND DECLARE**

The aforementioned device, launched and marketed as of the date certified in this certificate, fully complies with the provisions stated in annexes I and VII to EEC Directive 2007/47 and L.D. No 37 of 25 January 2010, that confirmed EU provisions at Italian level.

The products listed above are classified as Class 1 non-invasive medical devices as provided in Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product, the complete "EC" declaration of conformity is available on the website [www.nuovablandino.com](http://www.nuovablandino.com)

*The afore-mentioned products:*

- *are not measuring instruments*
- *are marketed in non-sterile packaging*

This is to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

Settimo T.se, 24th March 2016

## WARRANTY

On the aids from our factory, we provide a warranty period of two years from the correct completion of the certificate of purchase (see page 29).

The warranty is valid for products that have defects in workmanship or operation.

The warranty excludes products that exhibit defects due to negligence or improper use.

Through our dealers, you can still get information on estimates for repairs.

For safety reasons, use spare parts and accessories approved by Nuova Blandino S.r.l.

The use of spare parts and accessories not approved by Nuova Blandino S.r.l. are reason of the immediate invalidation of the warranty.

To ensure the efficiency of the electric wheelchair, it must be kept clean, organized and maintained as specified in the instructions for use.

Attenzione:


**Warning:**

**We accept no responsibility for damage caused by the combined use of the electric wheelchair with parts of all kinds from other manufacturers (for example trailers units, electrical devices) that may involve significant dangers - except that a given product has not been expressly authorized in writing by the Nuova Blandino S.r.l.**

*Nuova Blandino S.r.l. declines all liability for damage to persons, property, animals, and any third party due to modifications or improper use of the wheelchair.*

## Plate and EC label

The 'EC label (see picture) on the wheelchair found on the front crossmember under the seat.

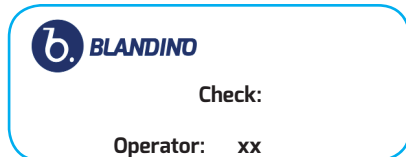## Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

# Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

# Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

## Advisory notice

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive.

- **Intended use**

The device is supplied ready for use by technical orthopedics and adjusted according to the user's needs.

The multifunctional chair is designed for both indoor and outdoor use. It is intended to accommodate one single person on the seat when the legrests and the lateral pads are mounted, and should not be used as a means of traction, transport or otherwise. Carefully read the instruction manual for a correct use of the device.

- **Check before use**

Before using SPRING, make sure that all parts are properly assembled and tightened, especially the platforms. If the product shows visible damage, damaged parts must be replaced to ensure your safety.

- **How to clean**

Do not use high-pressure cleaning equipment!

*The chairs should not be cleaned with harsh detergents.*

- **Lifespan of the device**

The duration of the multifunctional chair of our manufacturing, depends on the intensity of use to which it is subject. Proper use and proper maintenance allows you to maintain and prolong the life of the device, on average, an estimated duration of 5 to 7 years.

- **Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product.

Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 2:00 HOURS AND FROM 13:30 TO 7:00 HOURS. CALL +390112625140.

**RETAILER STAMP**

--

**DATE OF PURCHASE**

--	--	--

**REGISTRATION DATE**

--	--	--

N.B.: THIS MANUAL MUST BE PRODUCED WHENEVER ASSISTANCE IS NEEDED

Nuova Blandino S.r.l.  
Strada Cebrosa, 24 - 10036 Settimo Torinese  
Torino Italy  
Tel. +39 011 2625140 - fax +39 011 2625141  
[www.nuovablandino.com](http://www.nuovablandino.com) - [infonb@nuovablandino.com](mailto:infonb@nuovablandino.com)

Commitment and technology for people with special needs