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COMPONENTS

1. Introduction
3. Shipping
 - Assembling
 - Use specifications
 - Safety information
4. Components
5. Check before use
 - Wheelchair use
 - Brake - push brake
6. Legrest strap
7. Side adjustment
8. Frame size reduction to get on a car
9. Height footrest adjusting - flex-extension
 - Depth adjusting
10. Tilting seat
 - Quick release pins
11. Adjusting the front wheels
12. Adjusting the seat depth
13. Adjusting the caster wheels
14. Wheelchair maintenance
15. Measures
16. Optionals
17. General safety rules
18. Portable weight
 - Transfer from the wheelchair
19. Training
 - Taking objects
20. Sidewalks
 - Up and down the stairs
21. Crossing narrow passages
 - Use on elevators and lift bridges
 - Transport on a car or other means of transport
22. Declaration of compliance
23. Warranty
24. Plate and EC label
25. Maintenance log
28. Data sheet
29. Space for dealer

Introduction

The product described below, "WILD" is the result of our company's twenty years of experience. We have been engaged in a continuous process which seeks to improve and refine techniques for designing and manufacturing of products; everything has been possible thanks to the assistance and support of expert orthopedists, highly qualified in the field of Rehabilitation, Education and Assistance. This manual provides instructions for this particular model.

Please read the following instructions before using the device for the first time.

The special care taken in designing and manufacturing of these products is a guarantee of top-level safety and comfort.

Use your wheelchair according to the instructions provided in this manual.

Model "WILD ":

active wheelchair, lightweight with aluminum rigid frame, reclining backrest and adjustable seat..

Main clinical indications:

suitable for different categories of stable adult neurological patients who need to achieve the maximum degree of self sufficiency.

Basic layout:

Available in different sizes, its technical features may be ergonomically customized to ensure optimum performance:

the backrest may be adjusted to set the trunk-pelvis angle, stretch-adjustable backrest, quick-release rear wheels, clothes protection side pads, single fixed height adjustable footrest, adjustable seat.

Optionals:

it may be customized with accessories for comfort and safety purposes: wheels, tires, forks, caster wheels, anti-tipper and height adjustable push handles (See pag. 16)

Introduction

Symbols

While reading the manual you might come across these symbols of particular importance or interest.

symbolizes danger in general. Particular attention must be paid

6 mm spanner or wrench

Allen wrench for 4-5-6-8 mm hollow hex-screws

“WILD “ is equipped with:

- rigid foldable aluminum frame
- height adjustable push handles and backrest
- tilt adjustable backrest
- clothes protection guards
- caster wheels
- 24"x1 ø 600 rear wheels
- single height, depth and flexion-extension adjustable footrest
- quick release pins
- tilt adjustable seat
- the seat may be pushed forward or backward

Shipping

All Nuova Blandino products are tested in-house to verify compliance with all specifications and are then shipped in extra cardboard boxes.

Therefore, you are invited to immediately check the device after the receipt, and in the presence of the carrier, to verify any damage that might have occurred during transport.

If the purchased product was damaged during transport, the damage must be reported immediately, by attaching an official written claim to the delivery note.

All damaged parts (if removable) or the whole wheelchair must be delivered carriage free to Nuova Blandino S.r.l. - within 8 days from the date of receipt, together with a copy of the receipt.

No damage will be acknowledged unless the aforementioned procedure is followed, or if the damage is reported after the receipt note is issued.

Assembling

The orthopedist will deliver your wheelchair ready for use, adjusted for your personal requirements.

Use specifications

This wheelchair is designed for both indoor and outdoor use.


Its sole purpose is carrying a single person, once the

footrest and the lateral pads are mounted, and not as a means of traction, transportation or otherwise.

Safety informations

All safety information on the wheelchair will be presented in the second part of this manual.

Components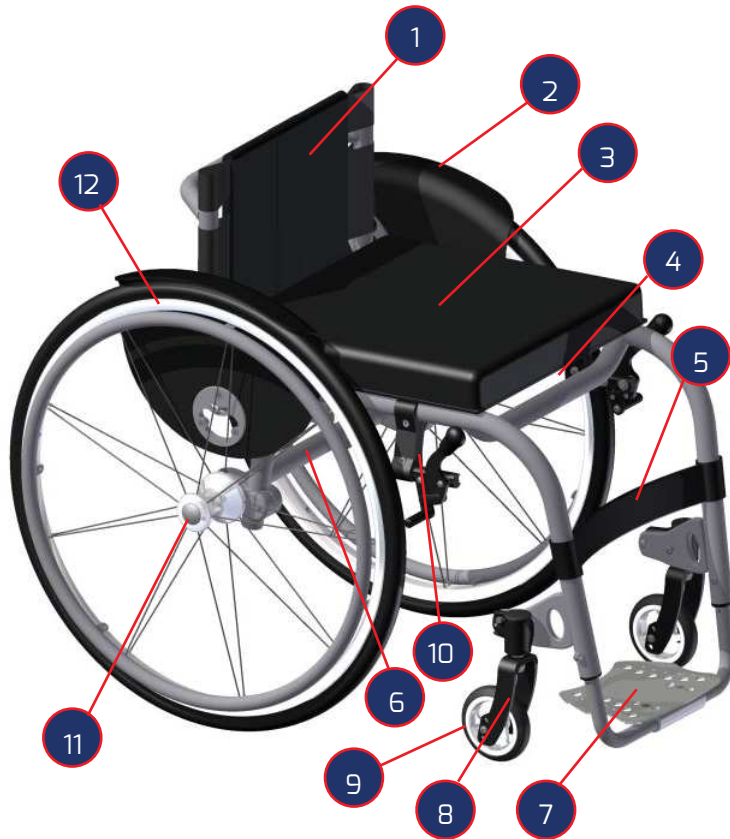

- 1 Backrest
- 2 Clothes protection guards
- 3 Fabric seat + padded cushion
- 4 Pocket under the seat
- 5 Legrest strap
- 6 Axle
- 7 Single footboard
- 8 Fork
- 9 Caster wheel
- 10 Aluminum brake
- 11 Quick-release pin
- 12 Pneumatic rear wheel

Check before use

Before using the wheelchair, check that all its parts have been properly assembled and tightened, paying particular attention to the footrests.

If the footrests are height adjustable, make sure the user is able to sit comfortably and hold his or her legs at the correct angle.

Make sure that the brakes work properly. While activating the brakes on both wheels, push the chair forward and backward; braked wheels are expected to not rotate.

Using the wheelchair

This active wheelchair is designed for people with special needs who carry out mainly outdoor activities. It is ideal for both in-door (i.e. home, workplace) and it may also be used for outdoor leisure activities, provided that the

ground is sturdy and not slippery.

Initially, use the wheelchair with caution.

Read the second part of the manual ("Safety information") carefully.

Brake - Push brake

The parking brake is one of the most important safety devices featured on this wheelchair. It is a push brake that operates directly on the tires.

The brakes must always be fully working. The braking effect of the push brake may be impaired if:

- The sides of the tires are worn out
- The tires are wet
- Tire pressure is low

Check the wheels on a regular basis and before leaving your pressure!!

Recommendation:

Ø 600 light wheel (7.0 bar). Make sure that the wheel spokes are tightened up.

Caution:

Use your wheelchair brakes with great caution when going backwards. Particular caution must be used when going backward on steep terrain. A sudden stop may result in the wheelchair tipping over.

Note: The brakes should not be operated when the wheelchair is in motion.


Legrest strap

Legrest strap prevent the legs from sliding off the boards. It is mounted between the front wheelchair stanchions.

Adjustment

Side adjustment

Adjust the sides according to the position of the wheel. A 1/2 cm distance is required between the tire and the side pads, along its whole radius. Otherwise, loosen the bolts (a) (b) and (c) and slide the side pad inside the slots to adjust to the correct position. Once the position is adjusted, tighten the bolts to secure it.

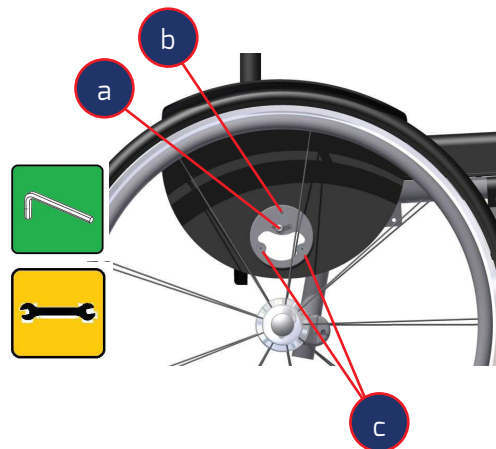Frame size reduction to get on a car

Tilting backrest:

- raise the back pocket of the seat cushion
- pull for the release of the back seat behind the same and fold forward until it stops.
- disengage the rear wheels (see page 10 “Removable Pins”)

Backrest installation:

- recline the backrest until it stops
- insert the rear wheels

Height footrest adjusting - flex-extension - depth adjusting

Height footrest adjusting:

- loosen the bolt (a) and adjust according to the desired height.
- tighten the bolt to lock the footrest staff

Caution:

After adjusting the platform, make sure the bolts are tightened up. Make sure that the footrest staff is inserted at least 5 cm inside the footrest junction. Adjust the footrest staff so that the footrest is about 6 cm from the ground


Flexion-extension adjustment:

- Loosen the bolts (b) and adjust according to the desired angle; tighten the footrest bolts.

Depth adjustment :

- Loosen the bolts (b) and move the footrest in one of the three possible positions. Tighten the footrest bolts.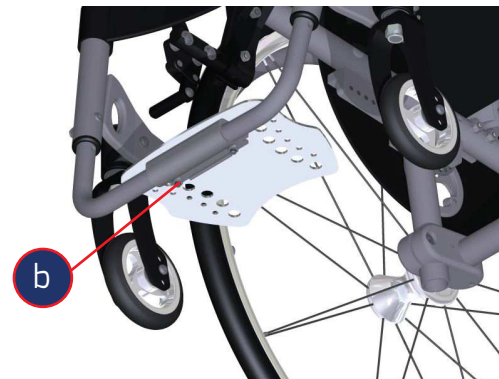

Tilting seat

To tilt the seat, unscrew bolts (a) of both clamps and rear-range the wheelchair into desired position, bearing in mind that the adjustment may vary from -3° to -13° .

After reaching the desired position, loosen the bolts.

By varying the seating position it is consequently necessary to realign the front wheels with the floor (see pag. 11)


Note: The backrest inclination may be adjusted accordingly;

danger of tipping over.

Caution:

Make sure all bolts are tightened before sitting on the wheelchair.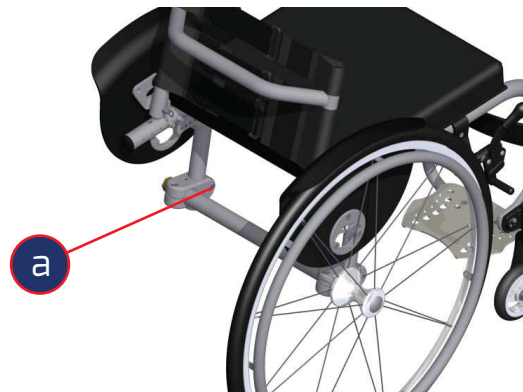

Quick release pins

To remove the push wheel, gently push the pin button. The wheel will be removed instantly, thus reducing the wheelchair width. To mount the wheel back, simply exert the same pressure until the part clicks.

Caution:


Do not perform this action while sitting on the device: risk of falling.

Adjusting the front wheels

After changing the set-up, using the tilt of the seat, it is necessary to adjust the position of the front wheels:

1. Loosen the screw (a)
2. Turn the wheel bearing housing to replace the floor in the front wheel.
3. Tighten the screw (a)
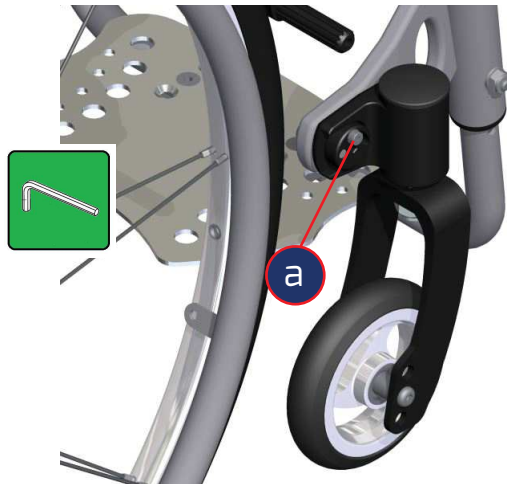


Adjusting the seat depth

The seat depth may be adjusted to five set positions:
37/39/41/43/45 cm.

To change the seat depth follow these instructions.

1. Remove the rear wheels by pressing the removable pins (figure on page 10).
2. Remove the side clothes protection pad by unscrewing the (a) bolts as shown in the figure.
3. Unscrew the (b) bolts and move the backrest to the desired position.
4. Screw all bolts by following the steps in reverse order.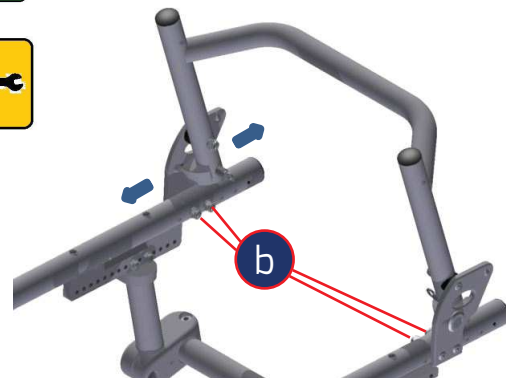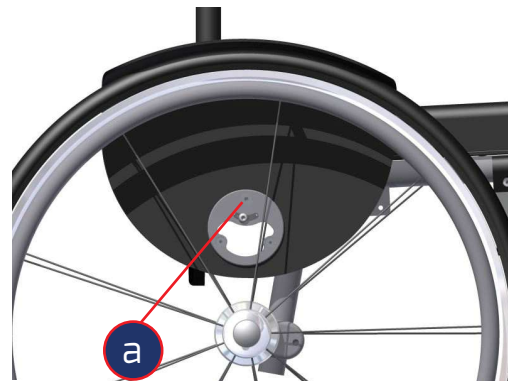

Replacing the caster wheels

Replacing the caster wheels with other models of different diameters:


This is required in order to set the wheelchair at a correct height:

- remove the screws (a), move the wheel so that it matches the holes on the fork
- Insert the screws (a) and tighten firmly

Caution:

Adjusting the front height is necessary when the caster wheel is replaced with wheel of a different size. Whenever the height is changed, its inclination should also be adjusted so as to keep it 90° to the ground.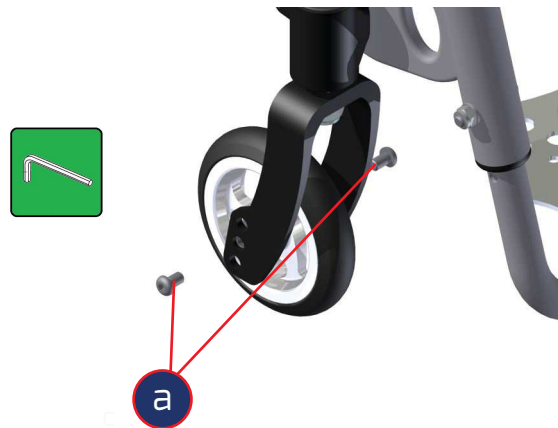

Wheelchair maintenance

Yearly checks

Maintenance should be performed once a year. It is highly recommended to check the state of the bearings, since sand, dust and water may damage them. Lightly lubricate with bicycle resin-free oil.

The frame must be cleaned with hot water and soap (or household detergent) only. Plastic parts should be cleaned with detergents specific for this material.

Caution!

Do not use high pressure cleaning equipment. Mechanical damage may be caused by water jets.

Chairs must not be cleaned with aggressive cleaning products.

Caution!

All components which come in direct contact with the skin must be replaced if the wheelchair is given to another user.

Repairs

In case any repair is needed, contact your Nuova Blandino S.r.l. dealer, who knows how repairs should be performed according to in-house specifications and will be able to offer advice, support and provide repair services.

Spare parts

Order through your dealer. In order to ensure the correct identification of the spare parts which must be replaced, indicate the wheelchair frame registration number. For safety reasons, use spare parts and accessories authorized by Nuova Blandino S.r.l. More information on how to retrieve the frame registration number are provided on page 24.

Disposal

The wheelchair packaging materials may be recycled. The metal parts are collected and recycled in the “old metal” skip.

The fabric and plastic parts may also be recycled. Disposal must be performed in compliance with specific national legislation.

Measures


cm 36 38 40 43 45

cm 52 54 56 59 61

cm da 78 a 90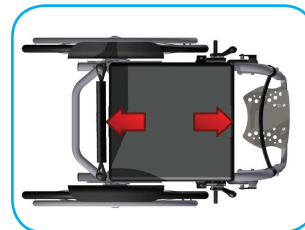

cm 37 39 41 43 45

cm 49

cm 25 27 29 31 33 35

Kg 11

Max. Load Kg 120

OPTIONALS

- Spoke
- Anti-tilt wheel
- 10 cm calf-pad
- Safety pelvic belt
- Single harness
- Lateral support

General safety rules

Use the wheelchair only according to the instructions of this use handbook.

To station, get on and off the wheelchair, make sure the ground is flat and stable and secure the wheelchair to prevent it from moving by activating both brakes.

If exposed to direct sunlight, the covering and the upholstery of the seats and the armrests, the legrests and handles may reach temperatures over 41° C. Direct contact with unprotected skin may cause injury. In order to avoid the over-heating of the said parts, exposing the parked wheelchair to direct sunlight is not recommended.

Please ask your dealer about all the possible driving characteristics, in order to exclude possible dangers and changes, the adjustments and their effect on the operation and the safety of navigation.

The dealer will be happy to adjust your wheelchair to suit your individual needs with the utmost respect for your safety.

When you activate the adjustment devices (e.g. the legrests), take care and do not put your hands on the moving parts. Possible risk of injury.

When objects are carried, make sure that they do not affect or impair the wheelchair control. Check the pressure and the tire wear, and check the brakes.

Portable weight

Do not overload the wheelchair and do not use it for carrying more than one person.

The recommended maximum load is reported in the handbook and on the wheelchair plate.

Transfer from the wheelchair

Getting off the wheelchair to move to another place may be dangerous, if no assistance is available.

It requires:

- relative strength
- good coordination abilities
- an anti-slip back surface (i.e. set the palms of your hands on the seat surface)
- much training
- Being aware of all possible dangers.

Ask for assistance. Agree on how assistance should be provided, and be aware that there may be unforeseen hazards, i.e. stumbling; also make sure that the assistant is aware of the kind of handicap you suffer from.

When the wheelchair does not support any weight, for example when you move from the wheelchair to the bed, the toilet or to the car, do not exert pressure on the footrest.

- Danger of tipping over

Training

In order to use the wheelchair safely, both indoors and outdoors, it is highly recommended to train for different situations, such as: being pushed forward, braking, going forward, turning, going backward on flat surfaces, possibly with the assistance of a helper. Initially, make sure to avoid steep gradients, bumpy roads or other similar situations.

Taking objects

Avoid leaning too much forward, sideways or backward, when trying to grasp or place objects: tipping over or going off-balance may occur, especially if the wheelchair is set on a narrow and high layout (cushion).

Caution: Caution:

- tipping over when leaning forward to grab an object
- loss of balance when leaning sideways to grab an object
- Tipping over may occur when leaning backward to grab an object


Sidewalks

Caution!

Before going off sidewalk alone, please check its height. If it is too high, tipping may occur. Danger of tipping in case.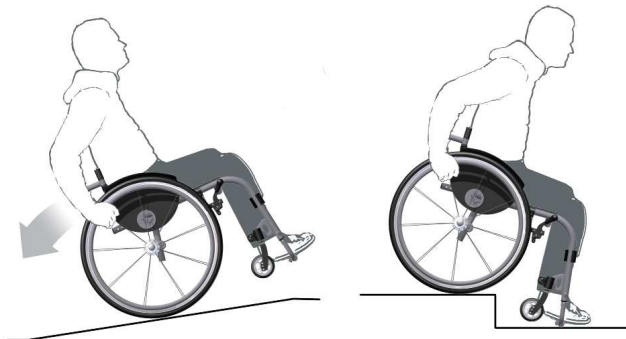

Up and down the stairs

Make sure to lean forward when driving uphill; when going downhill, keep you back to the backrest while holding the brake. For +1% gradients, seeking assistance from a helper is recommended.

Caution!

When going uphill carrying heavy bags or backpacks may be dangerous. Danger of tipping over.

For your own safety, seek the help of at least two people before going up or down a steep staircase. While going down the stairs, the wheelchair must be kept on the large rear wheels and walked down the stairs one step at a time. While climbing the stairs, the helper should turn his or her back to the staircase, and pull the wheelchair one step at a time, keeping it tilted on the rear wheels.

Caution!

If the wheelchair is lifted from the floor, the person sitting on it might lose his or her balance and fall to the ground.

Crossing narrow passages

When crossing narrow passages, such as the entrance of a building, a door, etc., pay attention to your elbows. Danger of injury.

Using elevators and lift bridges

Lifts and elevators are a particular danger to the user.

Caution:

Automated doors may cause injuries for your hands!

Transportation by car or other means of transport

The wheelchair must not be used as a seat when using a car or another means of transport. Demount the wheelchair (as specified in the user's manual) and put it in the luggage compartment, or secure it in order to avoid accidents, in the event of dangerous maneuvers.

Take a proper seat inside the car or the means of transport and fasten your seatbelt.

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Responsible for designing, manufacturing, packaging, and labeling of the products falling within the category
ACTIVE LIGHTWEIGHT WHEELCHAIR

DOES GUARANTEE AND DECLARE

That the afore mentioned device, launched and marketed as of the date certified in this certificate, fully complies with the provisions stated in annexes I and VII to EEC Directive 2007/47 and L.D. No 37 of 25 January 2010, that confirmed EU provisions at Italian level.

The products listed above are classified as non-invasive medical devices in Class I as provided in Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product, the complete "EC" declaration of conformity is available on the website www.nuovablandino.com

These afore-mentioned products:

- *are not measuring devices*
- *are marketed in non-sterile packaging*

This is also to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

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WARRANTY

Our super lightweight wheelchairs are guaranteed for two years as of the date reported on the certificate of purchase (see page 29).

The warranty covers any manufacturing flaw or malfunction on guaranteed products.

Damages caused by carelessness or improper use are excluded from the warranty.

Authorized resellers will provide additional information and quotations if any repair is needed.

For safety reasons, only spare parts or accessories authorized by Nuova Blandino S.r.l. should be used.

The use of spare parts and accessories unauthorized by Nuova Blandino S.r.l. will result in the immediate cancellation of warranty coverage.

In order to guarantee it functions properly, the wheelchair must be kept clean, in order and maintained as described in the use handbook.


Caution:

We shall not be liable for any damage caused by the combined use of our wheelchairs with parts supplied by other manufacturers (eg. trailer units, electrical equipment) that may result in significant risks – unless such products have been expressly authorized in writing by Nuova Blandino S.r.l.

Nuova Blandino S.r.l. will not be liable for any damage to persons, things, animals, and any third party due to modification or misuse of the wheelchair.

Plate and EC label

The EC label (see picture) is printed on the rear axle.

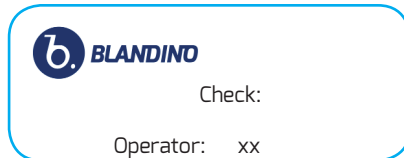Maintenance log

Repair date

Repair date

Replaced or repaired

Replaced or repaired

Stamp or signature of the operator

Stamp or signature of the operator

Stamp or signature of the inspector

Stamp or signature of the inspector

Maintenance log

Repair date	Repair date
Replaced or repaired	Replaced or repaired
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

NOTES

Data sheet

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive

- **Intended use**

The device is supplied ready for use by technical orthopedics and adjusted according to the user's needs.

Your active wheelchair was designed for both indoor and outdoor use; bumpy and sandy terrains should be avoided. It is recommended that only "active" self-sufficient persons use this wheelchair. Carefully read the user's manual to know how to properly use the device.

- **Check before use**

Before using your active wheelchair, check that all the parts have been properly fitted and tightened, with particular attention to the footrests.

For safety reasons, if visible flaws are detected on the product, all damaged parts must be replaced.

- **How to clean the wheelchair**

Do not use high pressure cleaning devices! Mechanical damage may be caused by water jets.

Chairs must not be cleaned with aggressive cleaning products.

- **Lifespan of the device**

The active lightweight wheelchair may be used for a variable period of time, depending on the time and the frequency of use. A correct use and a proper maintenance of the wheelchair may extend its lifespan; on average, it is expected to be fully functional for 5-7 years.

- **Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product.

Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 12:00 HOURS AND FROM 13:30 TO 7:00 HOURS. CALL +390112625140.

RETAILER STAMP

--

DATE OF PURCHASE

--	--	--

REGISTRATION DATE

--	--	--

N.B.: THIS HANDBOOK MUST BE
PRODUCED IF ASSISTANCE IS RE-
QUIRED

Nuova Blandino S.r.l.
Strada Cebrosa, 24 - 10036 Settimo Torinese
Torino Italy
Tel. +39 011 2625140 - fax +39 011 2625141
www.nuovablandino.com - infonb@nuovablandino.com

Commitment and technology for people with special needs