MOPEDIA®

SEDIE COMODE

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INDICE

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

1. CODES

RC325-xx*	Commode chair with adjustable backrest, 125 mm wheels
RC330-xx*	Commode chair with adjustable backrest, 200 mm wheels
RC340-xx*	Commode chair with adjustable backrest, 600 mm rear wheels
RC345-XX*	Commode chair with adjustable backrest, 600 mm front wheels

*xx corresponds with the width of the seat and the relative backrest with dimensions:
(40-43-46-48-50) cm

2. INTRODUCTION

Thank you for choosing a commode chair in the MOPEDIA range by Moretti. The MOPEDIA commode chairs by Moretti have been designed and constructed to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely.

Please read this manual in full before you use the commode chair you have purchased. Should you have any queries, please contact your dealer for suitable advice and assistance.

NB Check to ensure that no parts of the product have been damaged during shipment. Do not use the product if it is damaged and contact the retailer for further instructions.

WARNING!

- Do not use the product for any purpose other than that specified in this manual
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features

3. INTENDED USE

The MOPEDIA commode chairs are intended for the transport and movement of people with mobility impairments in indoor environments: at hospitals and/or healthcare structures and entities or at home.

4. DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products made and sold by MORETTI SpA in the **COMMODE CHAIRS - MOPEDIA** product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES.

To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.

6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

4.1 Applicable regulations and directives

The safety tests required by standard UNI EN 12182 have been performed and passed.

Resistance to ignition: The materials used for the cover and padding have been tested using the methods specified in EN1021-1/2.

5. GENERAL WARNINGS

WARNING!

Take extreme care in the presence of moving parts that could trap limbs and cause injury.

- Please consult this manual carefully for correct use of the device
- Always consult your doctor or therapist for correct use of the device
- Keep the packaged product away from sources of heat, as the packaging is cardboard
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts
- Always take care when children are present
- Never lift the commode chair by gripping the removable parts (armrests, backrest, etc.), which could accidentally detach from it during lifting
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established

5.1 Warnings on use


- Do not exceed the maximum permitted weight under any circumstances
- Lock the wheels of the chair completely whenever the patient is being transferred or when in an elevator
- Before transferring the patient onto/off the commode chair, ensure that the chair is placed on a flat and stable surface and with the wheels locked
- Before inclining the chair forwards or backwards, ensure that the front wheels are locked, to avoid the risk of the chair tipping over
- Avoid leaning over the back of the chair to prevent possible tipping over
- If you need to pick up an object behind the chair, do not lean further than the length of your arm, without changing your position on the chair
- Take care to ensure that clothing does not get caught in the wheels of the commode chair
- Take extreme care to avoid crushing of fingers during assembly and/or adjustment of the footplates, backrest and leg supports during use of the chair
- Keep the chair away from direct sources of heat
- Ensure that the tray has not been shifted or removed;
- Do not rest your weight on the footplates to get onto or off the chair;
- Do not lean out from the commode chair to pick up objects on furniture or other surfaces;
- Avoid sitting on the edge of the commode chair, as it could tip over.

6. SYMBOLS

REF	Product code	UDI	Unique device identification
LOT	Batch Lot		Read the instruction manual
CE	CE marking		Manufacturer
MD	Medical Device		Disposal conditions

7. GENERAL DESCRIPTION

7.1 View and parts list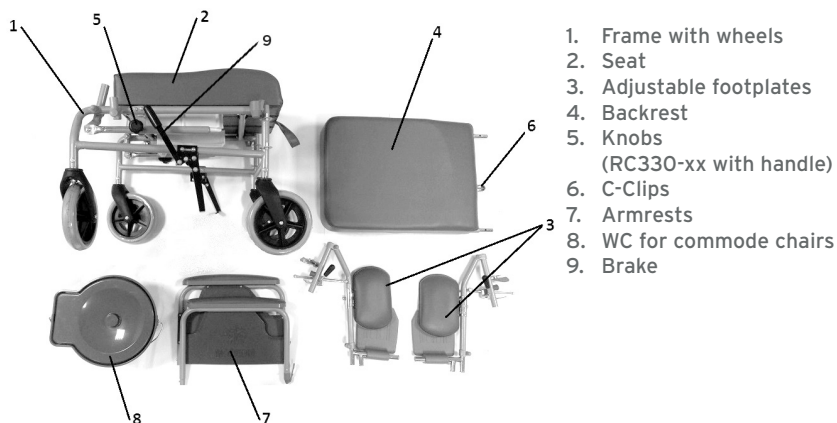

illustrative images

8. ASSEMBLY/ DISASSEMBLY

8.1 Assembly

All MOPEDIA commode chairs are supplied with the backrest, armrests and footplates disassembled.

- For assembly of the backrest, it is sufficient to slide the two ends of the tubes into the specific housings on the back of the seat (FIG. 2).
- Attach the backrest using the tightening wheels (FIG. 3).
- Insert the armrests in the side housings. Ensure that the armrest is attached correctly (FIG. 4).

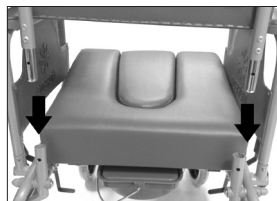
FIG. 2

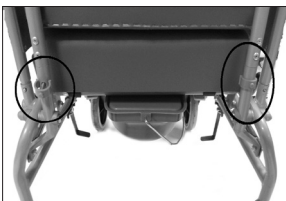
FIG. 3

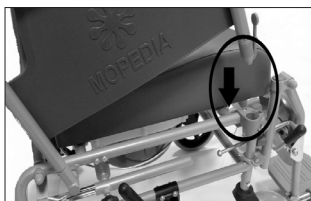
FIG. 4

- The footplates are assembled externally to the structure, by aligning the two overlapping holes (located on the footplate) with the two rounded pins (located on the structure of the commode chair (FIG. 5)).
- Once the footplate is in its housing, rotate it inwards until it reaches the safe locked position (FIG. 6).

**WARNING!**

Correctly identify the footplate to be assembled on the structure. The footrests must always be turned inwards.

FIG. 5

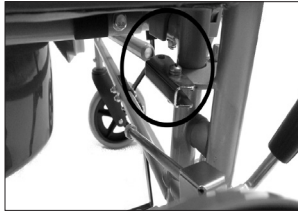
FIG. 6

FIG. 7

- Insert the WC basin from the rear of the chair, taking care to insert it correctly in the guides (FIG. 7).

Illustrative images**9. BEFORE EVERY USE**

- Always check for wear and tear of moving mechanical parts, in order to ensure that the wheelchair can be used in total safety for both people and property.
- Check that the device has been assembled correctly and particularly that the wheels and all components and accessories* (optional) have been inserted properly.
- Check correct functioning of the brake on the wheels.

10. HOW TO USE

Note: Consult the general warnings and user instructions in this manual

**WARNING!**

- Getting into and out of a commode chair on your own can be potentially risky: do so only if you are physically able and with care.
- Do not lean on removable parts, but rather on the seat frame. **DO NOT SHIFT YOUR WEIGHT ONTO THE FOOTPLATE WHEN GETTING INTO OR OUT OF THE COMMODE CHAIR, AS IT COULD TIP OVER!!!**

10.1 Getting into the commode chair

1. Get as close as possible to the chair, in a manner that it is positioned alongside you
2. Lock the brakes on the front wheels. Getting into the commode chair is easier if the armrest is removed on the side where this operation is being performed.
3. Rest your weight on the frame structure using your arms and slowly lower yourself into the chair

10.2 Getting out of the commode chair

1. Get as close as possible to the side of another seat
2. Lock the brakes on the front wheels. Getting out of the commode chair is easier if the armrest is removed on the side where this operation is being performed.
3. Rest your arms on the frame of the seat into which you are moving and raise yourself slowly towards it

10.3 Stairs and height differences (advice on how best to deal with them)

For commode chair RC240-xx, take stairs (from around 1 cm up to 3 cm) in reverse, with rear wheels of \varnothing 600 mm

WARNING!

- These techniques are only suitable for expert users of self-propelled commode chairs. Ask a carer to assist you in all other cases!

WARNING!

- Do not use this technique when the difference in height is over 10 cm!

ASCENDING: Go up to the curb of the pavement or the step you want to ascend. Balance the wheelchair on the rear wheels to lift the front wheels just enough to rest them on the edge of the obstacle. Bend forwards from the waist and firmly push the handrail on the rear wheels forward. Continue pushing until the rear wheels have gone over the obstacle.

DESCENDING: Follow the procedure in reverse to descend. Position the rear side of the wheelchair in front of the obstacle, grip the handrail on the rear wheels firmly and slowly descend the step or the obstacle, then balance the wheelchair on the rear wheels in a manner to raise the front wheels slightly and carefully complete the descent.

10.4 Adjusting the footplates

FIG. 10

FIG. 11

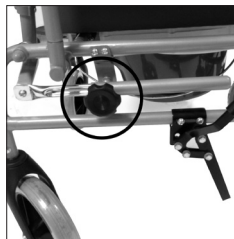
FIG. 12

- The height of the commode chair footplates can be adjusted to offer greater comfort, by loosening the screw (FIG. 10), with a minimum height from the floor of 6.5 cm. Always check that the two footplates are at the same height.
- For adjustment of inclination, pull the footplate outwards from the chair. If you wish to lower it, push the lever on the outside of its junction point and apply pressure on the footplate (FIG. 11).

10.5 Backrest adjustment for models RC325-xx, RC330-xx, RC340-xx, RC345-xx

- To adjust the backrest, loosen the two black knobs on the two sides (FIG. 12), push the backrest until it reaches the desired inclination, then tighten the knobs again, otherwise the backrest will tend to return to its initial position. The models have an adjustable spring backrest that returns to the vertical position.

10.6 Brakes

FIG. 13

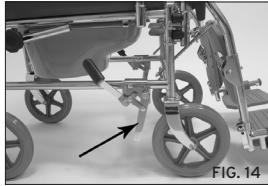
FIG. 14

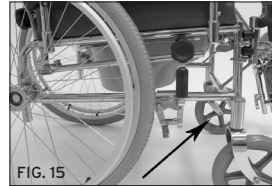
FIG. 15

On model RC325-xx, the brakes are operated by pressing the lever on the rear wheels of the chair (FIG. 13). On all the other models, the brake is activated by pressing on the lever as far as it will go (FIG. 14 and 15).

If adjusted correctly, the brake lever must be firmly gripped around the wheel, i.e. it should penetrate around 5 or 6 mm into the rubber. If it is not, check the tyre pressure. If necessary, adjust the brakes by loosening the locking nut, using the wrench provided, and allow the clamp to slide forwards or backwards, according to need. Tighten the locking nut completely.

10.7 Wheels

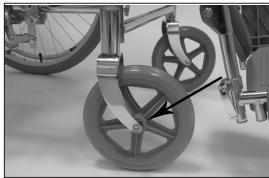
FIG. 16

FIG. 17

Check the wheels periodically and lubricate if necessary. Remove the fork by unscrewing the bolt and lubricate the pin (FIG. 16).

Also remove the wheel from the fork and lubricate the pins. Assemble the wheel and fork again by tightening the screws until there is no clearance, but the wheels rotate freely. On models RC340-xx and RC345-xx, the large wheels with diameter \varnothing 600 mm must be checked periodically and must not wobble. If the wheel wobbles, tighten the pin nut slightly (FIG. 17). To test the wheel, incline the commode chair to one side and turn the wheel. The wheels do not require inflation and are puncture-proof, as they are solid wheels.

10.8 Armrests

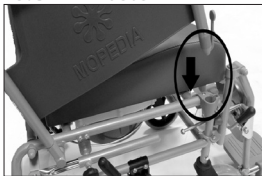
FIG. 18

To remove the armrests, press down on the lever (front clasp) and pull the pin using the special grip (rear clasp), then raise it (FIG. 18).

10.9 WC

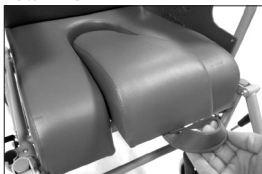
FIG. 19

FIG. 20

Use the WC with the chair fully braked. For use of the WC, extract the removable part of the seat by pulling the handle on the front (FIG. 19). Extract the WC from the back by pulling on the grip (FIG. 20).

10.10 Use

Always and only use the rear push bar to push the chair and never push it using other gripping points. It is recommended always to have a backrest inclination of around 20°. Before moving on a sloping surface, be aware of your own ability and limits in doing so. Ask qualified personnel for advance on moving techniques. Assistance is recommended in all cases for surfaces with a gradient over 10%. Avoid sudden movements if it is necessary to stop on a sloping surface. Do not use the wheel brakes to slow down on a descent. Do not suddenly change direction on a descent. Always ask someone to assist you when going up and down stairs, locking the brakes on the commode chair. If two people are lifting the commode chair, avoid gripping it at points that could be dangerous to the patient, such as the footplates and armrests, and use only the frame and the rear grips.

Use of the large rear wheels

For commode chair RC340-xx, take stairs (from around 1 cm up to 3 cm) in reverse, with rear wheels of \varnothing 600 mm

The \varnothing 600 mm rear wheels are used when the wheelchair is self-propelled. Grip the handrail on the wheel and push it in the desired direction.

WARNING!

- Do not put pressure on the footrest when getting into and out of the chair.

11. MAINTENANCE

The products in the MOPEDIA range by Moretti have been carefully checked and CE marked when placed on sale.

For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original replacement parts and accessories for repairs.

- It is extremely important to check the device before using it for the first time. In particular, check that all the screws and bolts are properly tightened.
- The mechanical parts of the device can move correctly (backrest, leg rest, armrests, wheels, etc.).
- Check that the wheel brakes are working properly.
- Check the mechanical parts (hinges and pins) once a month and lubricate them with a drop of oil to ensure they slide easily and to avoid wear.

The commode chairs in the MOPEDIA range by MORETTI only need a few simple and periodic maintenance operations. The periodic checks to carry out are listed below:

Element	Frequency
Lubrication (pins, couplings, joints, etc.)	Monthly
Braking capacity	Monthly
Cleaning the seat upholstery	As needed
General check by qualified personnel	Every two years

12. CLEANING AND DISINFECTION

12.1 Cleaning

Only use a damp cloth to clean the commode chair and dry it with a clean and dry cloth, or use warm water and mild soap.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based detergents, disinfectants and acetone, as these substances will scratch the plastic parts and cause the metal surfaces to rust.

The manufacture will not be responsible for damage caused by use of materials that could damage the surface of the product or by use of corrosive chemical products during cleaning.

12.2 Cleaning the WC

Clean and disinfect the WC on a regular basis, using a normal bathroom disinfectant. Always clean at least before each use.

12.3 Disinfection

Use a mild disinfectant cleaner if you need to disinfect the device.

WARNING!

Sanitising products are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective goggles and safety shoes. In the case of accidental contact, wash the part concerned with plenty of warm water and seek medical advice immediately.

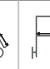
13. CONDITIONS OF DISPOSAL

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

14. TECHNICAL SPECIFICATIONS

14.1 Dimensions and weight

*The measurements may vary by ± 1 cm

Dimensions and weight														
Measurements in cm*														
RC325-40	108	64	48	106	76	55	43-54,5	40	62	55	41,5	21,5 Kg	12	12
RC325-43	108	64	48	106	76	55	43-54,5	43	62	58	44,5	22 Kg	12	12
RC325-46	108	64	48	106	76	55	43-54,5	46	62	61	47	22,5 Kg	12	12
RC325-48	108	64	48	106	76	55	43-54,5	48	62	63	49,5	23 Kg	12	12
RC325-50	108	64	48	106	76	55	43-54,5	50	62	65	51,5	23,5 Kg	12	12
RC330-40	108	83,5	48	106	76	55	43-54,5	40	62	55	41,5	18,5 Kg	20	20
RC330-43	108	83,5	48	106	76	55	43-54,5	43	62	58	44,5	19 Kg	20	20
RC330-46	108	83,5	48	106	76	55	43-54,5	46	62	61	47	19,5 Kg	20	20
RC330-48	108	83,5	48	106	76	55	43-54,5	48	62	63	49,5	20 Kg	20	20
RC330-50	108	83,5	48	106	76	55	43-54,5	50	62	65	51,5	20,5 Kg	20	20
RC340-40	124	97	48	106	76	55	43-54,5	40	62	65	41,5	22,5 Kg	60	20
RC340-43	124	97	48	106	76	55	43-54,5	43	62	68	44,5	23 Kg	60	20
RC340-46	124	97	48	106	76	55	43-54,5	46	62	71	47	23,5 Kg	60	20
RC340-48	124	97	48	106	76	55	43-54,5	48	62	73	49,5	24 Kg	60	20
RC340-50	124	97	48	106	76	55	43-54,5	50	62	75	51,5	24,5 Kg	60	20
RC345-40	124	104	48	106	76	55	37-48,5	40	62	65	41,5	22,5 Kg	20	60
RC345-43	124	104	48	106	76	55	37-48,5	43	62	68	44,5	23 Kg	20	60
RC345-46	124	104	48	106	76	55	37-48,5	46	62	71	47	23,5 Kg	20	60
RC345-48	124	104	48	106	76	55	37-48,5	48	62	73	49,5	24 Kg	20	60
RC345-50	124	104	48	106	76	55	37-48,5	50	62	75	51,5	24,5 Kg	20	60

14.2 Technical specifications

- Painted steel frame;
- Lifting and extractable footplates;
- Padded and adjustable calf rests (several positions);
- Height-adjustable and side-tipping footrests;
- Padded backrest ± 10 cm;
- Backrest adjustable from 0° to 45°, with semi-automatic spring return;
- Padded ergonomic seat ± 10 cm;
- Lifting and extractable armrests in black PU;
- Seat and backrest removable in just a few seconds using attachment clips;
- Backrest frame attached with C-clips for rapid cleaning and assembly;
- Fire-retardant PVC fabric;
- Parking brakes on two wheels (front or rear);
- Maximum permitted weight 135 Kg (*for seats 40-43-46-48);
- Maximum permitted weight 180 Kg (*for seat 50);
- Wheels $\varnothing 125$ (*for RC325-xx);
- Wheels $\varnothing 200$ in PU (*for RC330-xx);
- Rear wheels $\varnothing 600$ mm and front wheels $\varnothing 200$ mm in PU (*for RC340-xx);
- Rear wheels $\varnothing 200$ mm and front wheels $\varnothing 600$ mm in PU (*for RC345-xx);
- Ambient operating temperature -20 C° / + 60 C°;
- Recommended storage: Store in a cool and dry place, away from direct heat sources, such as sunlight (recommended temp. between +10 C° / + 40 C°).
- Resistance to ignition: The materials used for the cover and padding have been tested using the methods specified in EN1021-1/2;
- Conforms with UNI EN 12182

15. ACCESSORIES

- RCA100** Lifting armrests
- RCA310** Height-adjustable headrest
- RCA111** Height-adjustable and depth-adjustable ergonomic headrest
- RCA115** Adjustable leg divider
- RCA316** Depth-adjustable leg divider (LOT 05/2017)
- RCA120** Adjustable drip rod
- RCA125** 90 cm backrest and headrest
- RCA330** Chest padded supports
- RCA135** Oxygen tank holder
- RCA140** Padded ergonomic seat ± 10 cm without hole for WC
- CBR500** Straight table
- CBR505** Shaped table
- CAR235** One-hand drive kit
- RP245** Car type safety belt
- RP247** Safety belt with snap buckle
- RCR150** WC for commode chairs

15.1 Assembly of the accessories




RCA310 Height-adjustable headrest

RCA111 Height-adjustable and depth-adjustable ergonomic headrest

AS shown in the figures, commode chairs with lot number 04/2017 and higher have inserts in the backrest. Place your hand at the centre of the backrest and identify the four inserts.

Use a cutter or a screwdriver to remove the synthetic leather material, insert the headrest attachment plate and fix it in place with the screws provided.


RCA115 Depth-adjustable leg divider (LOT < 05/2017)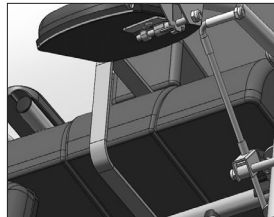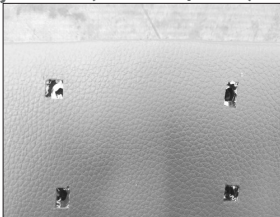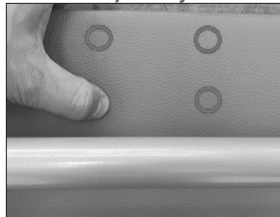

Remove the WC pan from the seat and remove the seat.

Extract the black plugs from the holes in the horizontal tube and insert the specific screws. Insert the saddle washers from the other part of the tube and insert the chrome-plated plate with the clamp. Tighten everything using the nuts. Choose the desired depth and fix in place with the knob.

RCA316 Depth-adjustable leg divider (LOT >05/2017)

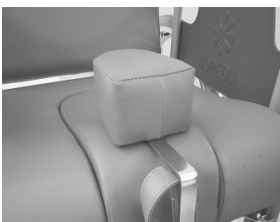Remove the WC pan from the seat and remove the seat.

Extract the black plugs from the holes in the horizontal tube and place the leg divider support on the tubes. Insert the screws provided and insert the saddle washers from the other part of the tube, then tighten everything using the

nuts.

Insert the leg divider and choose the desired depth, then fix in place with the knob.

RCA330 Chest padded supports

Remove the blue backrest and the black plugs on the backrest tube.

Insert the screws into the square holes on the backrest frame. Insert the saddle washers by placing them on the tube.

Insert the chrome-plated plate, complete with the clamp. Tighten everything using the nuts.

Install the left-hand chest padded support as shown in the photo and use the knob to adjust it to the desired height.

Install the chest padded support on the right side as well.

16. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual.

Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual.

Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

17. REPAIRS

17.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

17.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

17.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

18. REPLACEMENT PARTS

Moretti original replacement parts are guaranteed for 6 (six) months from the day of delivery.

19. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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