**MOPEDIA®**

**SEDIE DA DOCCIA**

---

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## INDICE

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## CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

### 1. CODES

**RS939** - Shower chair with four castors

### 2. INTRODUCTION

Thank you for choosing a MOPEDIA Shower chair by Moretti S.p.A. MOPEDIA Shower chair manufactured by Moretti S.p.A. are designed to satisfy all your requirements for a practical, correct and safe use. This manual provides some useful suggestions to correctly use the device you have selected and offers valuable advices for your safety. We recommend carefully reading the entire manual before using the hospital bed you have purchased. Should you have any queries, please contact your retailer for proper advice or assistance.

**Note: Check all parts of the product have not been damaged during shipment. In case of damage do not use the product and contact your dealer for further instructions.**

### 3. INTENDED USE

The MOPEDIA shower and toilet chairs are created for people with movement difficulty and can be used indoors as substitution of toilet or as aid under the shower.

#### CAUTION!

- Do not use the following device for purposes other than those described in this manual.
- Moretti S.p.A. disclaims any liability for damage resulting from improper use of the device or use other than that described in this manual.
- We reserve the right to make changes to the device and this manual without notice in order to improve the characteristics.

### 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SHOWER CHAIRS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

### WARNINGS!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts cannot be repaired and / or replaced.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

## 7. GENERAL DESCRIPTION

**RS939 Shower chair with four small chair** - Please check the components & tell me what we will find in the export carton box

## 8. BEFORE TO USE

- Always check the status of the mechanical parts in order to guarantee they are used in total safety for persons and environment.
- Check the proper assembly of the device with attention to the correct insertion and locking screws and nuts of the backrest and footrest.

## 9. PRECAUTIONS FOR USE

- Do not exceed the maximum load capacity for any reason.
- Assist always the patient during the handling operations.
- Don't skew the chair more than 10% without assistance.
- Block the wheels during patient transfer.
- Wheelchair has to be in a stable surface and wheels locked during user transfer.
- Before leaning or get up, be sure casters are in straight position and rear castors are blocked in order to avoid a turnover.
- Do not lean out of the backrest to avoid a turnover.
- Pay attention to your fingers during assembling, regulations or simply during the use.
- Keep the chair far from fire.
- Use always original accessories and spare parts.

## 10. HOW TO USE

Use only the back handle to move the chair, never use other supports. Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. Assistance is recommended when in slope higher than 10%. If you need to stop in a sloping surface, avoid abrupt movements. During the downhill, do not change suddenly the direction. Lift the chair up if necessary, using the main frame and not armrests or footrests.

**WARNING!** The patient must not put pressure on footrests when go up and down from the chair.

### 10.1 Brakes You can brake the chair pushing the lever on rear wheel.

### 10.2 WC

Use the toilet when the wheelchair is completely blocked. To use the toilet take out the removable part of the seat.

## 11. MAINTENANCE

The MOPEDIA by MORETTI chairs need minimal maintenance and general care, but we suggest you to control regularly:

- Braking system: monthly
- Cleaning parts (seat, backrest, armpad, wheels and so on) if necessary
- Cleaning the upholstery of the seat as required.
- General check by qualified personnel every two years.

It is extremely important that the wheelchair is checked at the first use. In particular, please check that all screws and nuts are tighten enough.

## 12. CLEANING AND SANITISATION

### 12.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

**WARNING!** Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

### 12.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

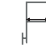
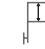
**WARNING!** Sanitising agents are corrosive. Avoid contact with the eyes and skin. Wear protective clothing, gloves, protective glasses and safety shoes. In case of accidental contact, rinse the affected part thoroughly using warm water and consult a doctor immediately.

## 13. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

## 14. TECHNICAL FEATURES

### 14.1 Dimension\* and weight

Dimension* and weight													
Measures in CM													
RS939	74	57	45	93	74	15,5	52	43,5	28	53	8,4	12,5	12,5

\*The dimension could be change  $\pm 0.5\text{cm}$

### 14.2 Technical specifications

- Powder coated aluminum frame
- Net backrest with elastic straps
- PU seat and armrest
- Removable WC bucket
- Max User weight 100 Kg

## 15. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

## 16. REPAIRING

### -Warranty repair-

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable

discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

**-Repair not cover by warranty-**

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

**-Non defective device-**

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

### 17. SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

### 18. EXEMPT CLAUSES

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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