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## TABLE OF CONTENTS

1. Introduction - Sympbology
2. Model: MINERVA
3. Shipping
  - Assembling
  - Use specifications
  - Safety information
4. Components
5. Check before use
  - Highchair use
  - Brake - push brake
6. adjustments
  - Armrests
  - High adjustment - Assembling and remove armrests
7. Footboard height-adjustable (optional)
8. Highchair maintenance
9. Measures
10. Optionals
11. General safety rules
12. Portable weight
  - Transfer from the wheelchair
13. Declaration of compliance
14. Warranty
15. Plate and CE label
16. Maintenance log
19. Data sheet
20. Section for retailers

## Introduction

### Symbols

While reading the manual you might come across these symbols of particular importance or interest.

symbolizes danger in general. Particular attention must be paid

The product described below , “MINERVA” is the result of our company’s twenty years of experience. We have been engaged in a continuous process which seeks to improve and refine techniques for designing and manufacturing of products; everything has been possible thanks to the assistance and support of expert orthopedists, highly qualified in the field of Rehabilitation, Education and Assistance.

This manual provides instructions for this particular model. Please read the following instructions before using the device for the first time.

The special care taken in designing and manufacturing these products is a guarantee of top-level safety and comfort.

Use your wheelchair according to the instructions provided in this manual.

## Introduction - Models

**Model MINERVA:** Highchair with castors

**Prevailing clinical indications:** High chair suitable for children under 18 years who need a personalized session and control during the course of daily activities, educational and recreational activities. Seat and back upholstered and covered with washable material. Wheels with internal braking system. Height adjustable frame (ISO 18.09.18.136). Height-adjustable armrests (18.09.18.145).

**Basic equipment:** available in different sizes, with Ø80mm swivel wheels, parking device, padded back, removable armrests, padded seating plan, footboard height adjustable.

**Optionals:** customization options through the choice of accessories to meet your needs for comfort and safety: padded headrest, footboard height-adjustable, lateral fixed supports padding, retractor, table edge notch, sternum strap, pelvic girdle, footstraps.

## Shipping

All Nuova Blandino products are tested in-house to verify compliance with all specifications and are then shipped in extra cardboard boxes.

Therefore, checking the device immediately after receipt, and in the presence of the carrier, it is recommended to verify any damage that might have occurred during transport.

If the purchased product was damaged during transport, the damage must be reported immediately, by attaching an official written claim to the delivery note.

All damaged parts (if removable) or the whole wheelchair must be delivered carriage free to Nuova Blandino S.r.l - within 8 days from the date of receipt, together with a copy of the receipt.

No damage will be acknowledged unless the afore-mentioned procedure is followed, or if the damage is reported after the receipt note is issued.

## Assembling

The orthopedist will deliver your wheelchair ready for use, adjusted for your personal requirements.

## Use specifications


Your highchair has been designed to be used in a closed environment. It is intended only to transport a person in the seat with the leg and the side parts mounted, and not

used as a means of traction, transport or otherwise.

## Safety informations

All safety information on the highchair will be presented in the second part of this manual.

## Components

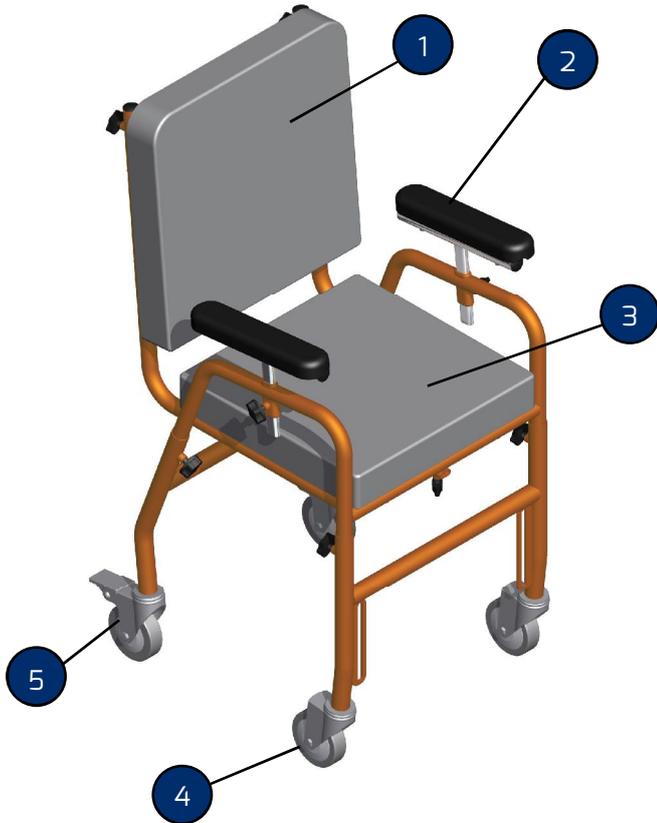1. Padded back
2. Height adjustable arms
1. Padded seat
2. Front wheel castor  $\varnothing$  80
3. Swivel castor with brake  $\varnothing$  80

## Check before use

Before using the highchair, make sure that all parts are properly assembled and tightened, especially the armrests and the height adjusting structures.

Check the correct operation of the brake pressure. Applying the brakes on both wheels, and imparting a slight pressure to the chair forward and backward: the wheels must not rotate.

## Using the wheelchair

The highchair is perfect for indoor MINERVA.

Start with the large chair with great caution.

Please read the second part of the manual "Safety Information".

## Brake - Swivel castor with brake

The castor wheel with brake is activated by the accompanying by a foot pressure directly on the lever of the wheel. Advantages: braking action regardless of the profile of the tire.

Safer use of the companion thanks to the positioning of the brake lever.

## Adjustments

### Armrests

The armrests ensure a correct position of the arms relative to the body, ensuring greater comfort for the user of the chair.

They are adjustable in height.

**Caution: Never lift chair by the armrests. The chair should be used only when the armrests are mounted.**

**If you have removed the armrests to make a transfer, check that you entered the brakes!**

### Height adjustment - Assembly and removal of the armrests

Height adjustment

- FIG.1: Loosen the clamping device (a) and up / down armrest (b). Close the clamping device (a) up to lock the armrest in the desired position.

Assembly and removal:

- FIG.2: Loosen the clamping device (a) and assemble / remove the armrest (b)

FIG.1


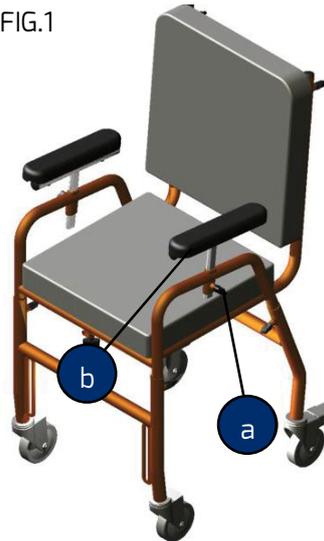
FIG.2
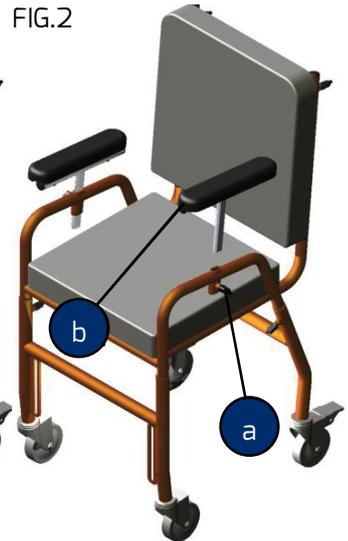
## Footboard height-adjustable (optional)

The footboard (optional) is adjustable in height: to customize the height of the platform, release the leaflets (a) and slide up or down on the footboard (b). Once you have reached the desired position, retighten the knobs (a)

The platform is equipped with cuts that allow the possibility of mounting the footstraps.

## Maintenance of the highchair

### Annual checks:

Maintenance should be performed once a year. To clean the frame, use warm water with a little soap, or a household cleaner. The plastic parts should be cleaned with common detergents specific to this material.

### Attenzione!

Highchairs should not be cleaned with detergents and items for aggressive cleaning.

### Attenzione!

All the components that can be in contact with the skin, if the wheelchair passes from one user to another, must be replaced.

### Repairs:

For repair you should contact your dealer Nuova Blandino S.r.l., which was educated in our factory according to our specifications, and is able to offer advice, support and repair services.

### Replacement Parts:

Order them through your dealer. For the purposes of proper delivery of spare parts is necessary to indicate the number of the frame relative to the chair. For safety reasons, use spare parts and accessories authorized by Nuova Blandino Srl (for the location of the chassis number page 19.)

### Disposal:

The packaging of the highchair can be brought to the recycling materials. The metal parts are collected and recycled in old metal. The fabric parts and plastics must be delivered to a collection center.

Disposal should be in accordance with the relevant rules of national law.

## Measures

cm 28 - 33 - 36


cm 38 - 43 - 46


cm 47/52 - 47/52 - 53/58

cm 61/66 - 61/66 - 67/72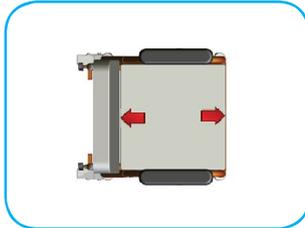

cm 30 - 36 - 40

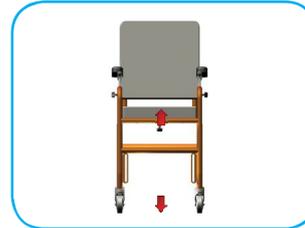cm 48 - 61

cm 35 - 40

cm 17 - 24

Max. load. Kg 80

Weight: Kg 10

## OPTIONALS

Padded headrest

Footplate with adjustable height

Fixed padded side supports

Retractor

Anatomic table edge notch

Sternum strap

Pelvic girdle

Footstraps

## General safety rules

Use the wheelchair only according to the instructions of this use handbook.

To station, get on and off the wheelchair, make sure the ground is flat and stable and secure the wheelchair to prevent it from moving by activating both brakes.

If exposed to direct sunlight, the covering and the upholstery of the seats and the armrests, the legrests and handles may reach temperatures over 41° C. Direct contact with unprotected skin may cause injury. In order to avoid the over-heating of the said parts, exposing the parked wheelchair to direct sunlight is not recommended.

Please ask your dealer about all the possible driving characteristics, in order to exclude possible dangers, and about changes and adjustments and their effect on the operation and safety of navigation.

The dealer will be happy to adjust your wheelchair to suit your individual needs with the utmost respect for your safety.

When you activate the adjustment devices, take care and do not put your hands on the moving parts. Possible risk of injury.

When objects are carried, make sure that they do not affect or impair wheelchair control. Check the pressure and the tire wear, and check the brakes.

## Portable weight

Do not overload the wheelchair and do not use it for carrying more than one person.

The recommended maximum load is reported in the handbook and on the wheelchair plate.

## Transfer from the highchair

Getting off the highchair to move to another place may be dangerous, if no assistance is available.

It requires:

- relative strength
- good coordination abilities
- an anti-slip back surface (i.e. set the palms of your hands on the seat surface)
- much training
- Being aware of all possible dangers.

Ask for assistance. Agree on how assistance should be provided, and be aware that there may be unforeseen hazards, i.e. stumbling; also make sure that the assistant is aware of the kind of handicap you suffer from.

When the wheelchair does not support any weight, for example when you move from the wheelchair to the bed, to the toilet or to the car, do not exert pressure on the footrest.

- Danger of tipping over

## Declaration of compliance

NUOVA BLANDINO S.r.l.  
registered office: via Enrico Fermi, 2/4  
10071 Borgaro Torinese Fraz. Mappano Torino

Responsible for designing, manufacturing, packaging, and labeling of the products falling within the category  
HIGHCHAIR

### **DOES GUARANTEE AND DECLARE**

That the aforementioned device, launched and marketed as of the date certified in this certificate, fully complies with the provisions stated in annexes I and VII to EEC Directive 2007/47 and L.D. No 37 of 25 January 2010, that confirmed EU provisions at Italian level.

The products listed above are classified as non-invasive medical devices in Class I as provided in Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product, the complete "EC" declaration of conformity is available on the website [www.nuovablandino.com](http://www.nuovablandino.com)

*These afore-mentioned products:*

- *are not measuring devices*
- *are marketed in non-sterile packaging*

This is also to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

Settimo, 9th December 2015

## WARRANTY

On highchairs from our factory, we provide a warranty for a period of two years from the correct completion of the certificate of purchase (pag. 19).

The warranty covers any manufacturing flaw or malfunctions on guaranteed products.

Damages caused by carelessness or improper use are excluded from the warranty.

Authorized resellers will provide additional information and quotations if any repair is needed.

For safety reasons, only spare parts or accessories authorized by Nuova Blandino S.r.l. should be used.

The use of spare parts and accessories unauthorized by Nuova Blandino S.r.l. will result in the immediate cancellation of warranty coverage.

In order to guarantee it functions properly, the highchair must be kept clean, in order and maintained as described in the use handbook.


**Caution:**

We shall not be liable for any damage caused by the combined use of our wheelchairs with parts supplied by other manufacturers that may result insignificant risks – unless such products have been expressly authorized in writing by Nuova Blandino S.r.l.

*Nuova Blandino S.r.l. will not be liable for any damage to persons, things, animals, and any third party due to modification or misuse of the wheelchair.*

## Plate and EC label

The EC label (see picture) is printed on the rear axle.

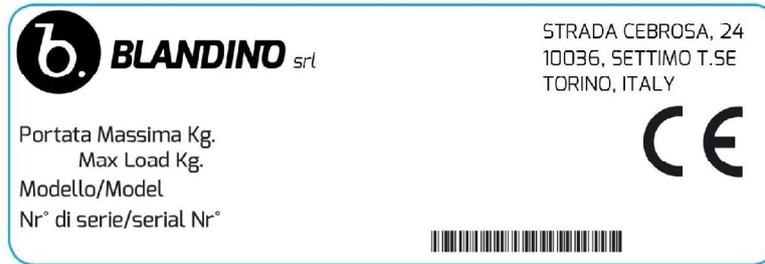## Maintenance log

Repair date	Repair date
Replaced or repaired	Replaced or repaired
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

## Maintenance log

Repair date	Repair date
Replaced or repaired	Replaced or repaired
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive

- **Intended use**

The device is already fitted to the orthoedist and adjusted according to your needs.

The highchairs have been designed to be used in a closed environment. They are exclusively designed to carry a person in the seat with the leg and sides mounted, and not used as a means of traction, transport or otherwise. Carefully read the Instruction Manual for a proper use of the device.

- **Check before use**

Before using your highchair, check that all the part have been properly fitted and tightened, with particular attention to the footrests. For safety reasons, if visible flaws are detected on the product, all damaged parts must be replaced.

- **How to clean the highchair**

*Do not use high pressure cleaning devices! Mechanical damage may be caused by water jets.  
Chairs must not be cleaned with aggressive cleaning products.*

- **Lifespan of the device**

The highchair may be used for a variable period of time, depending on the time and the frequency of use. A correct use and a proper maintenance of the wheelchair may extend its lifespan; on average, it is expected to be fully functional for 5-7 years.

- **Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product.

Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 12:00 HOURS AND FROM 13:30 TO 17:00 HOURS. CALL +390112625140.

**RETAILER STAMP**

--

**DATE OF PURCHASE**

--	--	--

**REGISTRATION DATE**

--	--	--

N.B.: THIS HANDBOOK MUST BE  
PRODUCED IF ASSISTANCE  
IS REQUIRED

Nuova Blandino S.r.l.  
Strada Cebrosa, 24 - 10036 Settimo Torinese  
Torino - Italy  
Tel. +39 011 2625140 - fax +39 011 2625141  
[www.nuovablandino.com](http://www.nuovablandino.com) - [inforb@nuovablandino.com](mailto:inforb@nuovablandino.com)

Commitment and technology for people with special needs