MANUALE D'USO
SEDIA PORTANTINA
COD. 01.03.5L

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USER MANUAL STAIR STRETCHER SKU 01.03.5L

INTRODUCTION

Thank you for choosing Demarta Virginio. The products have been designed and built to meet all your needs for a practical, correct and safe use. This manual contains some small suggestions for a correct use of the device of your choice and some precious advice for your safety. We recommend that you carefully read the entirety of this manual before using the item you purchased. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

The manufacturer reserves the right to make changes to the device and to this manual without prior notice in order to improve its characteristics.

GENERAL WARNINGS

- Use the product in the manner indicated in this manual. If doubts persist, contact DEMARTA VIRGINIO SAS.
- Always evaluate the compatibility of the product with the characteristics of the subject together with the doctor or therapist.
- Always check the integrity of the product before each use, even when new. Do not use if damaged or worn.
- DEMARTA VIRGINIO SAS declines any responsibility for damage to property or persons resulting from improper use, not agreed with a professional figure, not corresponding to what is indicated in the user manual.
- Any accidents, anomalies, breakages must be reported to DEMARTA VIRGINIO SAS in order to monitor the quality and criticalities of the product.

DECLARATION OF CONFORMITY

DEMARTA VIRGINIO SAS declares under its sole responsibility that the product made and traded by DEMARTA VIRGINIO SAS comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

For this purpose, DEMARTA VIRGINIO SAS guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices ARE NOT MEASURING INSTRUMENTS.
- The devices ARE NOT MADE FOR CLINICAL TESTS.
- The devices are packed in NON-STERILE BOX.
- The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation.

INTENDED USE

The Stair Stretcher is used to transport patients in places such as stairs, lifts or narrow areas where normal stretchers do not pass.

DESCRIPTION

The Stair Stretcher consists of a foldable aluminum frame, a seat and a back in waterproof orange nylon. It is equipped with two fixed wheels, two rear handles and four front ones. There are also straps for fixing the patient to the chair itself.

Dimensions	
Seat Dimensions	43 x 50 cm
Seat Height from the Ground	51 cm
Foldable Dimensions	100 x 53 x 20 cm

USE

1. Chair opening:

Fig. 1

Fig. 2

Hold the chair by points A and B, push in the direction of the arrows, until the chair is open, as in figure 2.

2. Position the patient on the chair. Use the three straps supplied at the level of the calves, thighs and back to safely bloc the user. They can be adjusted in length according to the needs.

3. Once the patient is on the chair, the armrests can be lowered as shown in figure 3.

-To bring the rear lifting handles to the horizontal position, raise them upwards.

-To lower the front lifting handles, press until you hear the click of the hook.

The front lifting handles can be extended by pressing the button and pulling the handle towards you, figure 4.

Fig. 3

Fig. 4

4. Locking system:

Fig. 5

To lock the front lifting handle in the safety position, lower the bracket until the crook is hooked into the lower pin.

5. Closure:

Return the handles to the starting position and perform the reverse operation described in figure 1.

CLEANING

Washing can be done with warm water and neutral soap
--

Disinfection can take place with simple disinfectants of a domestic use

GENERAL DISPOSAL CONDITIONS

When disposing of the device, never use housing solid waste collection systems. It is necessary to dispose of the device through the common ecological islands common for the recycling of used materials.

WARRANTY

DEMARTA products are guaranteed from material or manufacturing faults for 2 years from the purchasing date.

DEMARTA does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance and failure to adhere to the guidelines and instructions for use.