Di Blasi®

Mod. R30

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GENERAL SAFETY WARNINGS

- Before operating the Di Blasi scooter, carefully read this manual in order to avoid any personal injury or damage to the scooter as a result of improper use. In particular, keep in mind when using the scooter the limitations arising due to its technical characteristics, which are specified in section 7 of this manual.
- The scooter is intended as a mobility aid for people who are both physically and mentally capable of safely riding it. Therefore before using this scooter consult your doctor to check if your health conditions may exempt you from using it.
- The scooter is built to carry only its rider. The maximum weight capacity of the scooter is 100kg (up to 16 stone).
- Do not modify the scooter in any way, as this can limit the safety of the product and invalidates any warranty you may have.
- In order to avoid the electronic components getting wet or damaged, do not drive the Di Blasi scooter in rain. Avoid prolonged exposure to excessive heat or cold conditions.
- The maximum range of the scooter, as specified in section 7, is measured when the scooter is driven by a rider weighing 75kg, on a flat and plain road, without wind and with the scooter containing a new battery. The range is reduced when these factors are altered.

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
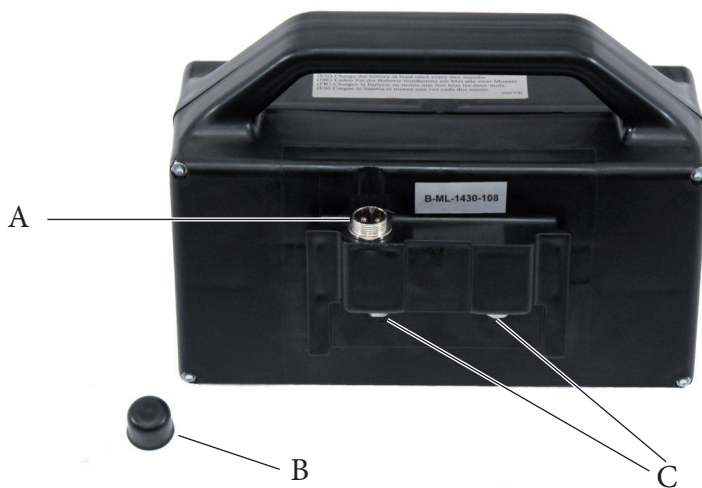
F.40

- When shipping or carrying the scooter on other vehicles, remove the battery and protect the terminals [marked (C) in (F.40)] against short circuits. To clarify any procedures or restrictions for shipping the lithium-ion battery, contact the airline/travel provider or courier concerned.
- Do not allow children to ride the scooter.
- The manufacturer shall not be held liable for personal injuries and/or damage to the scooter that is a result of improper use.

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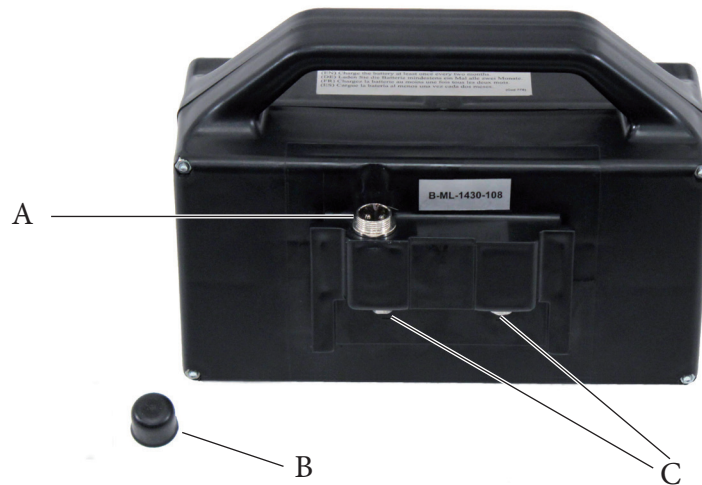DRIVING SAFETY WARNINGS

- Obey all traffic rules and drive with caution, paying attention to other vehicles and pedestrians.
- Reduce speed as much as possible when driving downhill or on uneven roads. To avoid the risk of the scooter tipping, never drive backwards or at an angle when travelling downhill and proceed with caution when driving up hill with a gradient up to 6 degrees. On long journeys ensure to take regular stops to avoid the breaks overheating, which may reduce their efficiency.
- Avoid making any sharp turns and always reduce speed when turning, especially when riding on sloping roads.
- Do not use the scooter on uneven paths. Do not ride the scooter on potentially hazardous surfaces, such as snow, ice, gravel roads, etc.
- Do not allow your clothing or any other object to drag behind you or hang outside of the seat whilst driving: they could be caught by the wheels and cause injury to you.

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SUGGESTIONS

- Save the packaging/box in case it is required for any future shipments of the scooter, for instance if it is required to be serviced or repaired. Otherwise you may be charged.
- If communicating with the seller regarding a concern about the scooter, specify the frame number marked, as shown in (F.01)

NOTICES

- Photos, illustrations and descriptions in this manual are not binding: the manufacturer reserves the right to introduce any modifications or updates to the scooter at any time, without obligation to update.

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INDEX

Automatic brake	4.3 (g)
Battery & charger	4.5
Battery & charger description	4.5.1
Battery: cautions	4.5.3
Battery: charging	4.5.2
Charger: cautions	4.5.4
Checks & adjustments	5
Emergency brake	4.3 (h)
First riding	4.1
Folding the scooter	3
Freewheel mode	4.3 (o)
Horn	4.3 (k)
Lighting	4.3 (l)
Maintenance & repairs	6
Monthly control checks	6.1
Operating instructions	4
Reversing	4.3 (i)
Riding instructions	4.3
Scooter controls	1
Specifications	7
Standby mode	4.3 (n)
Tyre pressure	5.1
Unfolding the scooter	2
Wiring harness	6.2

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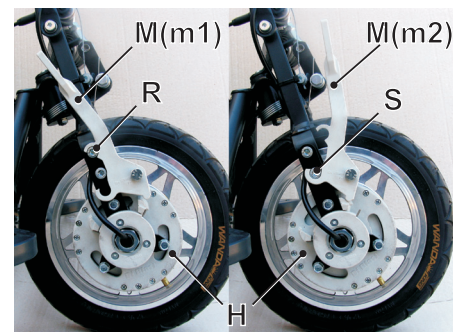
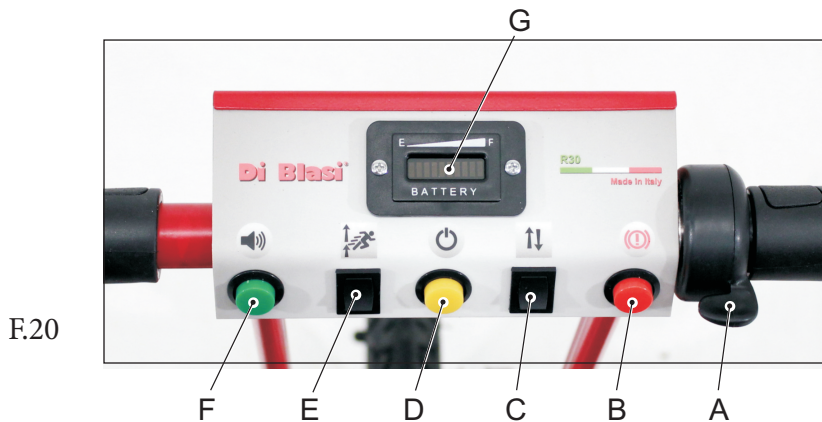
1. DEVICES FOR CONTROLLING THE SCOOTER

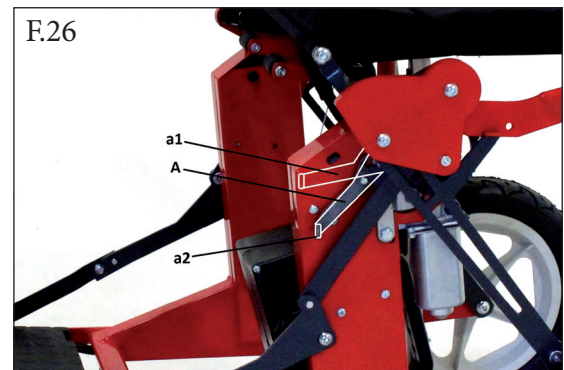
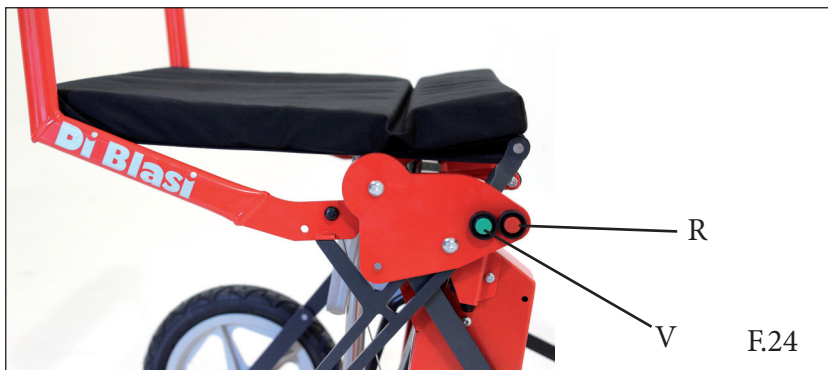
- Handlebar devices (F.20)
 - * (A) Speed control lever
 - * (B) Button for activating the emergency electric brake
 - * (C) Rocker switch for running forward and backward
 - * (D) Main switch button for activating or deactivating the scooter.
 - * (E) Rocker device for reducing the maximum speed
 - * (F) Horn button
 - * (G) Battery charge indicator
- Devices for folding and unfolding the scooter (F.24)
 - * (R) Red button folds the scooter
 - * (V) Green button unfolds the scooter
- Device for disengaging the automatic brake (F.22)
 - * (H) Mechanical automatic main brake Lever for disengaging the automatic brake (H): push all the way forward to disengage.
- Device for disconnecting the automatic folding mechanism (F.26)
 - * (A) Lever for disconnecting the automatic folding mechanism.

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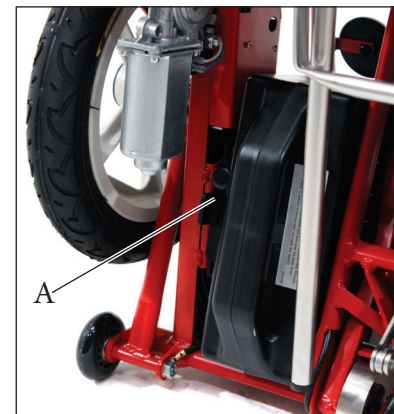
F.03

2. UNFOLDING THE SCOOTER AUTOMATICALLY

- Check that the handle for pulling the scooter is fully lowered (F.02).
- Insert the battery and make sure that it remains locked by the hook (A) (F.03). A short tone (beep) indicates that the battery is correctly inserted.
- Press the green button on the right side of the seat until you hear a short tone (beep) indicating that the scooter is fully unfolded (F.05).

WARNING

- * **When pressing the button ensure your fingers and the other hand are kept away from the surrounding area to prevent them from being pinched/injured in the folding mechanism.**
- * **Do not allow loose clothing e.g ties/scarves or long hair to hang near the mechanism.**
- Check that the cable (V) (F.07) which operate the handlebar folding is placed in the centre of the guide (W) (F.07)

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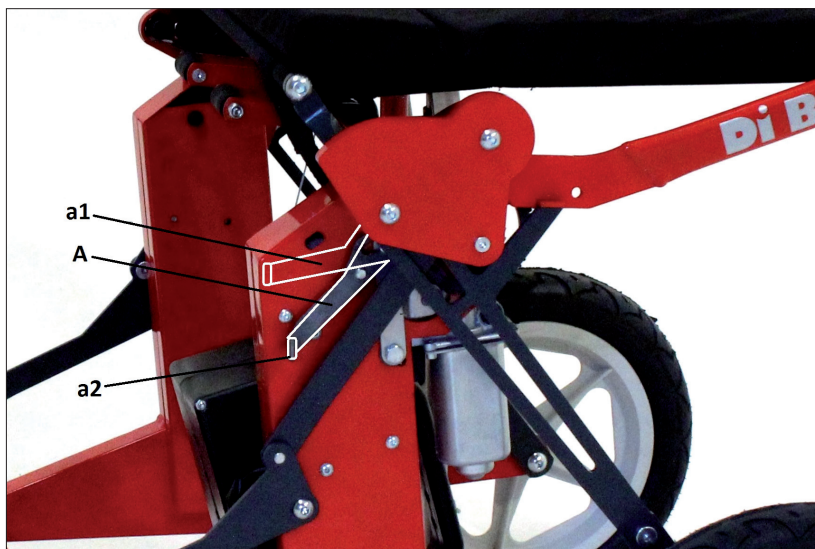
F.26

Unfolding the scooter manually

If the automatic folding/unfolding mechanism does not work properly, fold or unfold the scooter manually, as follows (F.26):

- Disconnect the automatic mechanism by pushing the lever (A) upwards to the position (a1).
- Manually fold or unfold the scooter by pulling the back rest..

To reconnect the automatic mechanism, move the lever (A) downwards to the position (a2)

If necessary to make this process easier, move the seat up and down slightly whilst moving the lever (A) downwards.

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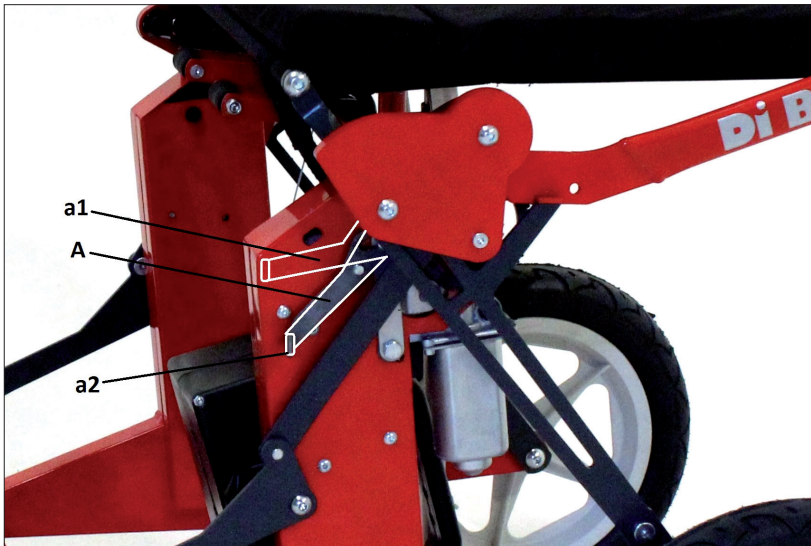
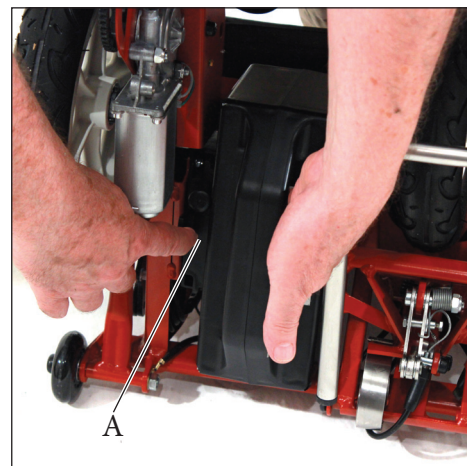
F12

3. FOLDING THE SCOOTER AUTOMATICALLY

- Ensure the handle bar is straight.
 - Press the red button on the right side of the seat until you hear a short tone (beep) indicating that the scooter is fully folded (F10).
- WARNING**
- * **When pressing the button ensure your fingers and the other hand are kept away from the surrounding area to prevent them from being pinched/injured in the folding mechanism.**
 - * **Do not allow loose clothing e.g ties/scarves or long hair to hang near the mechanism.**
 - To remove the battery from the scooter, first release its box from the hook (A) (F.12).
 - To pull the scooter on the small service wheels, completely raise the rear handle (F.14).

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F.16

WARNING

- Once folded, the scooter can be lifted by those physically capable of holding its weight. In order to reduce this weight, remove the battery.
- The weight of the scooter and battery are specified in Section 7 of this manual (Specifications).
- Consult a health professional regarding the correct positioning of your body when lifting the scooter.
- Lift the scooter by the seat frame F.16.

Important notices

- Do not press the red button for folding when somebody is seated on the scooter.

To fold manually the scooter

- See Sec. 2 – Par. “To unfold manually the scooter”

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4. OPERATING INSTRUCTIONS

4.1 First riding

Before taking your first ride in an area where there are vehicles and pedestrians, become familiar with the feel and handling of the scooter. Ensure you are aware of the controls and **in particular learn how to use the emergency brake** [see Section 4.3 (h)].

4.3 Riding the scooter

a) To sit on the scooter:

- * Place your hands on the handlebar grips (F.30);
- * Put one foot in the central section of the foot board (F.30);
- * Sit down with both feet now placed on the foot board (F.32).

IMPORTANT NOTICE

Do not mount by putting your foot on the side of the foot board, as this may cause the scooter to overturn (F.34).

b) Push the handlebar forward with a moderate force to check that it is locked in riding position. Once locked a small amount of forward movement may still occur.

IMPORTANT NOTICE

Never drive the scooter if the handlebar is not locked in riding position.

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
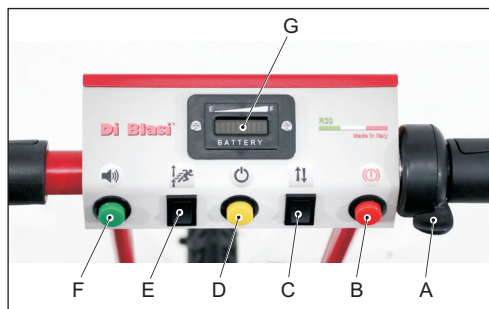
F.20

- c) Check that the rocker switch (C) (F.20) is pushed upwards so that the scooter is set for running forward.
An intermittent beep will sound if the rocker switch is pushed backwards (towards the seat) i.e. if the scooter is set for running rearward.
- d) Check that the lever (M) of the brake (H) (F.22) is locked in the position (m1) by the hook (R).
The scooter will run only if the lever (M) is locked in this position (m1).
- e) To activate the scooter, press button D (F.20). The scooter will be activated when any LED light shows on the battery charge indicator (G).
If the lever (M) of the brake (H) is not locked in position (m1), the scooter cannot be activated. Three short tones (beep) will sound to indicate that lever (M) is not in the correct position.
- f) To start and regulate the speed, push the speed control lever (A) (F.20) down gradually. To slow down or to stop the scooter release the lever (A) (F.20).
- g) When pushing the speed control lever (A) (F.20) downwards, the brake (H) (F.22) is automatically released to allow the drive motor to run.
When the speed control lever (A) (F.20) is released, the brake (H) (F.22) automatically engages and slowly stops the scooter.
The brake (H) also works as a parking brake.

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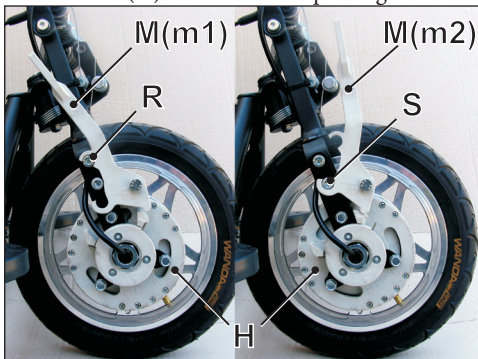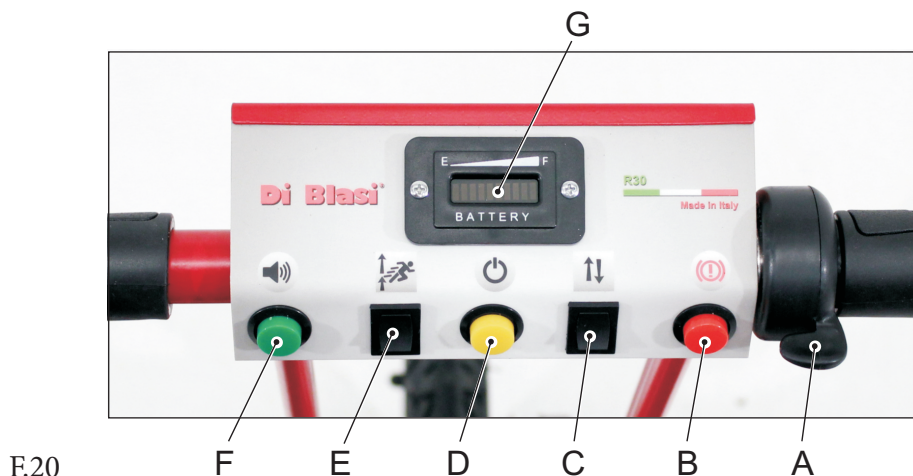
A

h) Emergency brake

In the case of automatic brake failure (H) (F.22) press the red button (B) (F.20), which will activate an emergency electric brake.

- i) To reverse, wait until the scooter is stationary and then turn the rocker switch (C) (F.20) backwards (towards the seat). Operate the speed control lever (A) in the same way. An intermittent beep will sound indicating that the scooter is in reverse mode.
When reversing, drive carefully and limit the speed as much as possible. Only reverse for very short distances and never use this mode on descent.
- j) To reduce the maximum speed, turn the rocker switch (E) (F.20) downwards.
- k) To activate the horn, push button (F) (F.20).
- l) The front light on the scooter will automatically switch on when the environment lighting is poor/reduced.

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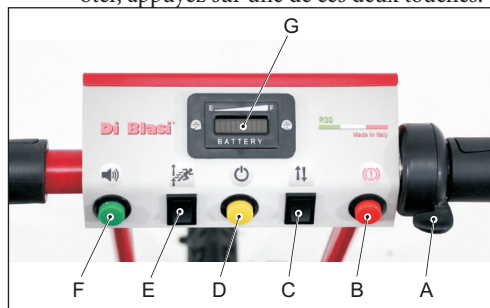
F.20

m) The level of battery charge is shown by the position of the lit light (LED) in the charge indicator (G) (F.20).

The battery is fully charged when the tenth bar is lit.

When the first bar is lit, the battery is very low/empty and will require charging (it is not advisable to let it get this low).

This indication however is an approximation, as it is based on standard discharging parameters, which can differ significantly from normal discharging. For instance while driving on a slope the battery charge indication may be incorrect.

To get a more accurate indication stop the scooter for about one minute.

It is advisable to keep track of the distance travelled on the scooter since last charged to give a supporting indication to the remaining battery charge.

n) Standby mode

* In order to save energy when the scooter is in running mode, all functions are automatically deactivated when the scooter is not in use for more than two minutes. In this case press button (D) (F.20) to reactivate the scooter.

* In order to save energy when the scooter is partially or completely folded, the functions are deactivated if either the folding red button (R) (F.22) or the unfolding green button (V) (F.22) are not pressed for more than 10 seconds. In this case press either the red or green button to reactivate the scooter

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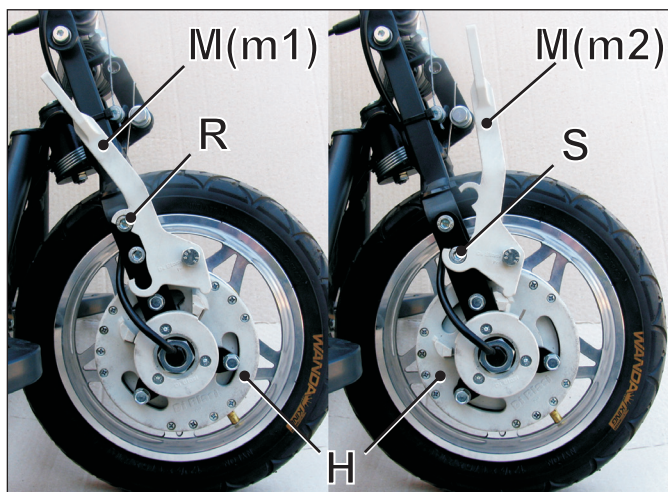
o) Freewheel mode

To push the scooter by hand, disengage the automatic brake (H) (F.22) by pushing the lever (M) back until it is locked in position (m2) by the hook (S).

To put the scooter back in drive mode push the lever (M) forward into position (m1), where it must remain locked by the hook (R).

WARNING

- If the automatic brake (H) (F.22) is not engaged, it is not possible to activate the drive motor : this is signalled by three short tones (beep).
- Never attempt to drive the scooter by motor if the automatic brake is not engaged.
- To prevent that the scooter can roll on its own uncontrollably, and therefore dangerously, never disconnect the automatic brake (H) (F.22) if the scooter is on an incline or decline.

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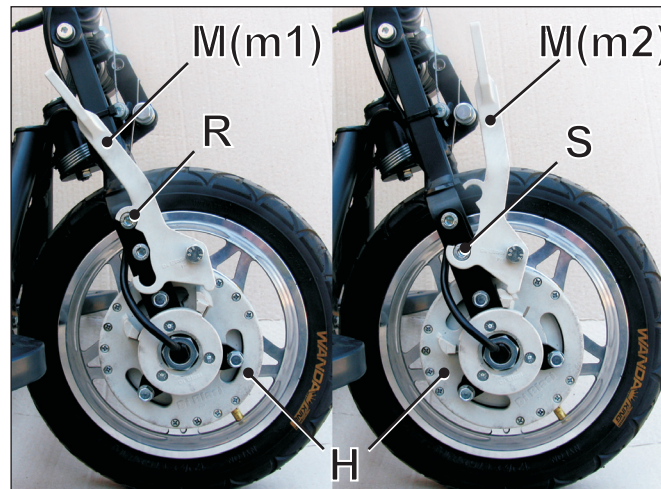
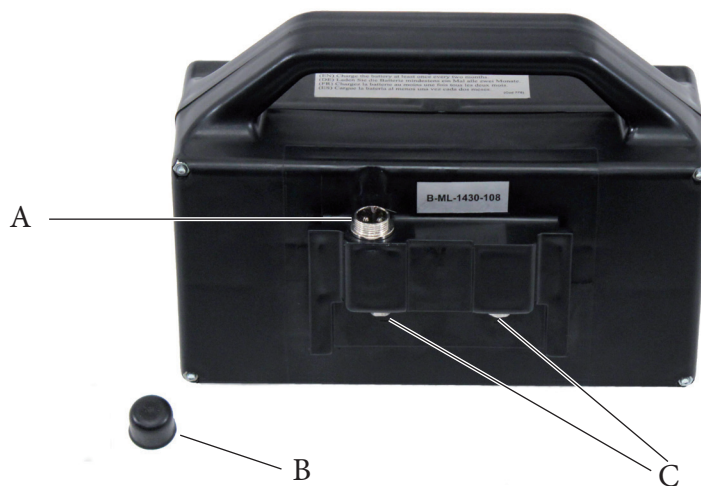
4.5 Battery & Charger

4.5.1 Description

- Battery box (F.40)
 - * (A) Charging plug
 - * (B) Plug protection cap
 - * (C) Terminals (+) and (-) of the battery
- Charger (F.42)
 - * (F) Battery connection socket
 - * (G) Mains power supply plug
 - * (H) Lighted indicators (LED1 – LED2) showing the charging status

4.5.2 Charging of the battery

- The charger can be connected to a 110V or 220V mains supply without setting any switch.
- Remove the cap (B) from the plug (A) of the battery.
- Connect the charger plug (G) to a mains power socket: LED1 will light up red, LED2 will light up green.
- Connect the charger socket (F) to the plug (A) of the battery: LED2 will then light up red. The battery is fully charged when LED2 lights up green again constantly.
- **Charging time is approximately 5 hours**

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4.5.3 Cautions concerning the battery

- The battery is stored in the box shown in F.40. Never open this box or tamper with the battery.
- Only charge the battery with the correct, provided charger.
- Only charge or use the battery if the ambient temperature is between 5° C and 40° C. Do not charge the battery immediately after it has been used.
- Keep the battery dry, as rain and humidity can damage the electronic circuit protection.
- Do not allow the battery to come into contact with fire or sources of heat.
- Do not use the battery to operate any equipment other than the scooter.
- Do not short circuit the battery. Short circuiting occurs when a metallic object is placed in direct connection with the (+) and (-) terminals (C) of the battery (F.40). A short-circuit can damage the battery and/or the connecting object.
- Never dispose of batteries with domestic waste. They must be taken to an authorised disposal point. Never throw the batteries in the fire, as they can explode very dangerously.
- If you do not expect to use the scooter for a long period of time:
 - * Remove the battery from the scooter
 - * Store the battery in a cool and dry location
 - * **Charge the battery at least every two months**

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4.5.4 Cautions concerning the use of the charger

- Do not use the charger to charge batteries other than those fitted in the scooter.
- To avoid overheating, do not cover the charger whilst it is charging.
- The charger is active as soon as the plug (G) is connected to a mains supply socket. Use a socket which is easily accessible so that the plug can be immediately disconnected in case of need.
- Disconnect the plug (G) from the mains supply socket when the charger is not in use. To avoid damage to the plug, do not pull the plug out by the cable.
- Do not use the charger if the plug (G), socket (F) or cables are damaged. Do not use the charger if it has suffered from a violent shock (for instance falling from a significant height).
- Never dismantle or tamper with the charger.
- Do not remove the protective cover from the charger.
- Any necessary repairs should be carried out by qualified personnel (contact your retailer).

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• Carga máxima admisible: 100 kg

7. SPECIFICATIONS

- **Dimensions**
 - * Unfolded scooter (F.71)
 - Wheelbase: 55 cm (21.7 in)
 - Maximum width: 61,5 cm (24.2 in)
 - Track: 77,5 cm (30.5 in)
 - * Folded scooter (F.72)
 - Length: 61,5 cm (24.2 in)
 - Width: 38,5 cm (15.2 in)
 - Height: 49,5 cm (19.5 in)
 - * Seat (F.74 - F.75)
 - Width: 47 cm (18.5 in)
 - Depth: 40 cm (15.7 in)
 - Height from the footboard: 47cm (18.5 in)
 - Height from the ground: 61cm (24.0 in)
 - * Seat back rest (F.74 - F.75)
 - Height from the seat: 29 cm (11.4 in)
 - Height from the ground: 90 cm (35.4 in)
 - Distance from the handlebar: 66 cm (26 in)
 - * Ground frame minimum clearance: 12 cm (4,7 in) (F.75)
 - * Turning minimum radius: 71,5 cm (28.1 in) (F.76)
- **Masses**
 - * With battery: 26,5 kg (58.5 lbs)
 - * Without battery 23,6 kg (52.0 lbs)
- **Motor:**
 - * Rated output: 200 W
 - * Voltage: 24V
- **Battery**
 - * Type: LiFePO4
 - * Voltage: 24V
 - * Capacity: 9Ah
 - * Masse: 2,9 kg (6.4 lbs)
- **Wheels (front and rear)**
 - * Rim: 8"
 - * Tyres (pneumatic): 12.5x 2.25
- **Brakes**
 - * An automatic mechanical brake
 - * A manually operated electric brake
- **Maximum speed** : 6 km/h (3.7 mph)
- **Maximum safe slope**: 6° (10,5%)
- **Maximum obstacle height that can be climbed and descended**: 50 mm (2 in)
- **Range**: 24 km (15 Miles) approximately
- **Seating capacity**: 1
- **Max load**: 100 kg (16 stones)

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