MA ST708\_xx\_ST709\_xx\_ST712\_xx\_ST713\_xx\_ST716\_xx\_ST719\_xx 01 A\_ITA\_10 2020

## CUSCINI

---

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## INDICE

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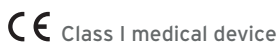
REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 on medical devices

### 1. CODES

ST708/xx	Doughnut cushion in memory foam
ST709/xx	Compound anatomical cushion with rigid base
ST712/xx	Cushion in fluid silicone gel
ST713	Doughnut cushion in fluid silicone gel
ST716/xx	Cushion in compact memory foam
ST719/xx	Cushion with self-modelling fluid

### 2. INTRODUCTION

Thank you for choosing a cushion in the LEVITAS range by Moretti. Moretti cushions have been designed and manufactured to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this entire manual carefully before you use the device. Should you have any queries, please contact your dealer for suitable advice and assistance.

### 3. INTENDED USE

Device to be used to provide relief to the patient when seated after surgery and in the case of bedsores, haemorrhoids and a congested prostate.

#### WARNING!

- Do not use the product for any purpose other than that specified in this manual.
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual.
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features.

### 4. EU DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products manufactured and sold by MORETTI SpA in the CUSHIONS product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES.

To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT INSTRUMENTS.
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATIONS.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

**Note:** The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

## 5. GENERAL PRECAUTIONS

- Please consult this manual for correct use of the device.
- Check that the product delivered to you is undamaged and keep it packaged away from sources of heat.
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts.
- Always take care when children are present.
- Always consult your doctor or therapist for correct use of the device.
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

## 6. SYMBOLS USED

Product code

Unique device identification

CE marking

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Disposal conditions

Do not bleach

Hand wash at a maximum temperature of 30°C

Do not dry clean

Do not tumble dry

Do not iron

## 7. GENERAL DESCRIPTION

### 7.1 Cushions in fluid silicone gel

COD. ST712/40 - ST712/42 - ST712/45

The cushions are manufactured in fluid silicone gel, with a removable zip cover. The interior is formed of a fluid silicone gel and a layer of polyurethane foam enclosed in a 400 micron PVC protective membrane that allows washing and prevents the migration of mineral oils. The top of the cover that comes into contact with the user is manufactured in Hightex, a 100% polyester coupled microfibre fabric, and the bottom that comes into contact with the wheelchair in an anti-slip fabric.

### COD. ST713 WITH CENTRAL HOLE

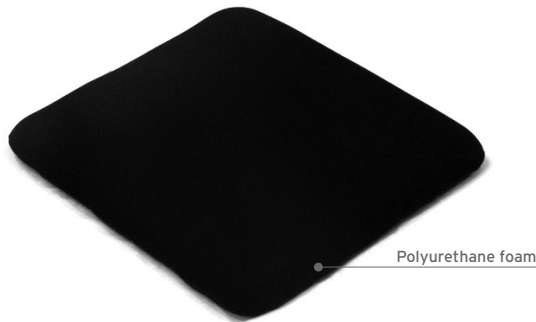
The cushions are formed of a polyurethane foam base and a layer of memory foam. Hermetically sealed with cover, does not include handles. The top of the cover that comes into contact with the user is manufactured in Hightex, a 100% polyester coupled microfibre fabric, and the bottom that comes into contact with the wheelchair in an anti-slip fabric.

### 7.2 Cushions in memory foam

COD. ST716/40 - ST716/43 - ST716/46


ENGLISH

The cushions are formed of a polyurethane foam base and a layer of memory foam. Hermetically sealed with cover, does not include handles. The top of the cover that comes into contact with the user is manufactured in Hightex, a 100% polyester coupled microfibre fabric, and the bottom that comes into contact with the wheelchair in an anti-slip fabric.

### 7.3 Doughnut cushions and weight-bearing base in memory foam

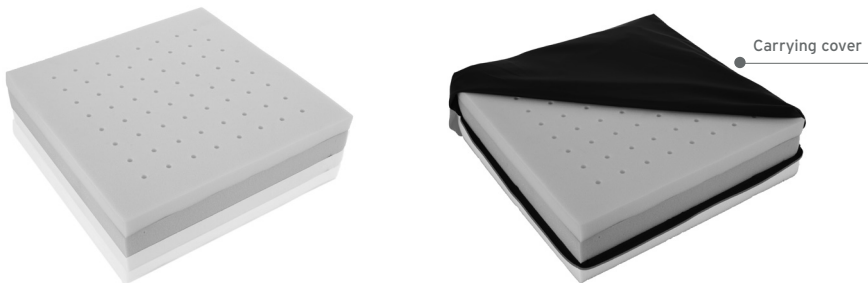
COD. ST708/40 - ST708/42 - ST708/44 - ST708/46 - ST708/48 - ST708/50

The cushions are manufactured in memory foam and a polyurethane foam weight-bearing base, with a removable zip cover. The interior is formed of a 4 cm weight-bearing base in polyurethane foam with a density of  $35\text{kg/m}^3$  and a 3 cm upper layer in memory foam with a density of  $55\text{kg/m}^3$ , perforated with 6mm pass-through channels


The top of the cover that comes into contact with the user is manufactured in a polyurethane coating that is impermeable to liquids and permeable to vapours, bi-elastic, antifungal and antibacterial, and the bottom that comes into contact with the wheelchair in an anti-slip fabric.

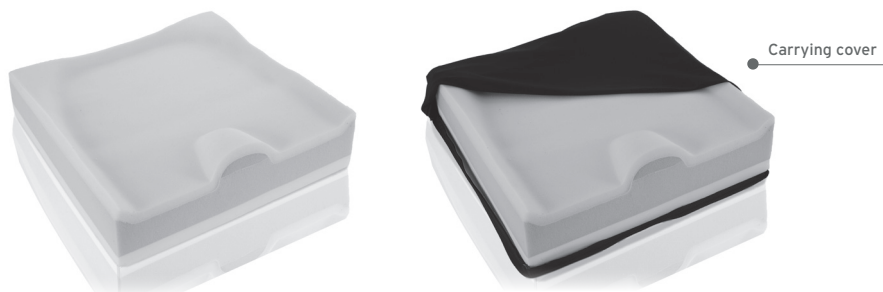**7.4 Compound anatomical cushions with rigid base in polyurethane foam**  
**COD. ST709/40 - ST709/42 - ST709/44 - ST709/46 - ST709/48 - ST709/50**

The compound cushions are manufactured in polyurethane foam of differing density, with a removable zip cover. The interior is formed of a rigid base in polyurethane foam; an anatomically shaped layer in polyurethane foam with a density of 50kg/m<sup>3</sup> glued onto the base; an upper layer in memory foam with a density of 55kg/m<sup>3</sup> glued onto the previous one. The top of the cover that comes into contact with the user is manufactured in a breathable and bi-elastic microfibre fabric and the bottom that comes into contact with the wheelchair in an anti-slip fabric.

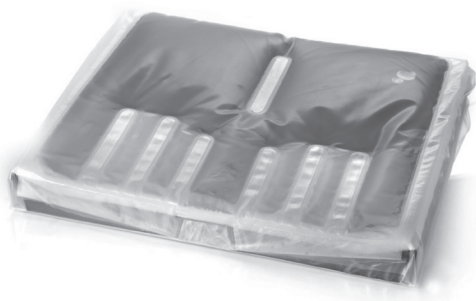**7.5 Cushions with self-modelling fluid**  
**COD. ST719/1 - ST719/2 - ST719/3 - ST719/4 - ST719/5 - ST719/6 - ST719/7 - ST719/8 - ST719/9**

The compound cushions have a rigid, preformed base and self-modelling fluids. The cover is removable. The interior is formed of a rigid, preformed based in LD2 polyurethane foam, high at the front and low at the back, onto which a 2 cm layer of low-resilience memory foam is glued. The bag containing a 200 micron polyester film, the self-modelling fluid, is positioned above the seat.

The two parts (base and bag) are enclosed in a second membrane, a 300 micron PU film on the lower part and a 200 micron one on the upper part.

The top of the cover that comes into contact with the user is manufactured in a breathable and bi-elastic microfibre fabric and the bottom that comes into contact with the wheelchair in an anti-slip fabric.

## 8. BEFORE EVERY USE

- Always check for wear and tear of parts, in order to ensure that the device can be used in total safety for both people and property.
- Position the device with the zip turned towards the back of the seat and the anti-slip part (rubberised) towards the bottom.

## 9. WARNINGS ON USE

- Avoid leaving the device outside and in contact with water for lengthy periods.
- Contact the retailer for any doubts over use or maintenance of the product.
- To avoid any risks, keep the device out of the reach of children.

## 10. MAINTENANCE

LEVITAS devices manufactured by Moretti are CE marked and carefully checked before being commercialised.

## 11. CLEANING AND DISINFECTION

### 11.1 Cleaning

Washing instructions for the cushion and the cover (where present):

**COD. ST708/40 - ST708/42 - ST708/44 - ST708/46 - ST708/48 - ST708/50**

**COD. ST709/40 - ST709/42 - ST709/44 - ST709/46 - ST709/48 - ST709/50**

**Interior:** wash with water and mild detergents. Wring with either a mangle or on a low-speed spin cycle. Dry in a room, away from sunlight, in air currents or with a forced-air heater (maximum temperature of 70°C)

**Exterior-Cover:** wash the cover by hand using mild soaps, at a maximum temperature of 60°C, or machine-wash on a delicate cycle.

**COD. ST712/40 - ST712/42 - ST712/45 - ST713**

Wash the cover using mild soap, at a maximum temperature of 30°C, or machine-wash on a delicate cycle. The membrane containing the fluid gel can be cleaned with alcohol.

**COD. ST716/40 - ST716/43 - ST716/46**

Hand-washable or machine-washable with mild soaps, at a maximum temperature of 60° C. Tumble dry. Do not iron.

**COD. ST719/1 - ST719/2 - ST719/3 - ST719/4 - ST719/5 - ST719/6 - ST719/7 - ST719/8 - ST719/9**

Wash the cover by hand using mild soap, at a maximum temperature of 40°C, or machine-wash on a delicate cycle. The membrane containing the fluids can be cleaned with alcohol.

Do not bleach

Hand wash at a maximum temperature of 40°C

Do not dry clean

Do not tumble dry

Do not iron

**WARNING!**

The product cannot be washed. Only the cover can be washed, following the instructions provided above. Never use acid, alkali or solvents (acetone or diluent) to clean it.

**12. DISPOSAL CONDITIONS**

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

**13. TECHNICAL SPECIFICATIONS**

CODE	MATERIAL	COVER	MEASUREMENTS	WEIGHT
ST708/40	Memory foam	Polyurethane coating	40x40x7 cm	2.5 Kg
ST708/42	Memory foam	Polyurethane coating	42x42x7 cm	3 Kg
ST708/44	Memory foam	Polyurethane coating	44x44x7 cm	3.5 Kg
ST708/46	Memory foam	Polyurethane coating	46x46x7 cm	3.5 Kg
ST708/48	Memory foam	Polyurethane coating	48x48x7 cm	4 Kg
ST708/50	Memory foam	Polyurethane coating	50x50x7 cm	4.5 Kg
ST709/40	Polyurethane foam	Microfibre	40x40x7.5-9 cm	0.7 Kg
ST709/42	Polyurethane foam	Microfibre	42x42x7.5-9 cm	0.8 Kg
ST709/44	Polyurethane foam	Microfibre	44x44x7.5-9 cm	0.9 Kg
ST709/46	Polyurethane foam	Microfibre	46x46x7.5-9 cm	1 Kg
ST709/48	Polyurethane foam	Microfibre	48x48x7.5-9 cm	1.05 Kg
ST709/50	Polyurethane foam	Microfibre	50x50x7.5-9 cm	1.1 Kg
ST712/40	Fluid silicone gel	Hightex fabric	40X40X4 cm	4 Kg
ST712/45	Fluid silicone gel	Hightex fabric	45X45X4 cm	4.7 Kg
ST713	Fluid silicone gel	Hightex fabric	42X42X4 cm	3.5 Kg
ST716/40	Compact memory foam	Hightex fabric	40X40x4.5 cm	1.8 Kg
ST716/43	Compact memory foam	Hightex fabric	43x43x4.5 cm	1.8 Kg
ST716/46	Compact memory foam	Hightex fabric	46x46x4.5 cm	1.8 Kg
ST719/1	Self-modelling fluid	Microfibre	38x38x6.5 cm	0.9 Kg
ST719/2	Self-modelling fluid	Microfibre	40x40x6.5 cm	0.9 Kg
ST719/3	Self-modelling fluid	Microfibre	40x45x6.5 cm	0.9 Kg
ST719/4	Self-modelling fluid	Microfibre	42x42x6.5 cm	0.9 Kg
ST719/5	Self-modelling fluid	Microfibre	44x42x6.5 cm	0.9 Kg
ST719/6	Self-modelling fluid	Microfibre	45x40x6.5 cm	0.9 Kg
ST719/7	Self-modelling fluid	Microfibre	45x45x6.5 cm	0.9 Kg
ST719/8	Self-modelling fluid	Microfibre	45x50x6.5 cm	0.9 Kg
ST719/9	Self-modelling fluid	Microfibre	50x45x6.5 cm	0.9 Kg

**Note:** All products are resistant to ignition according to standard EN1021-1, but maximum care around naked flames is recommended, as the products could catch fire.

## 14. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations. The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual. Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual. Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

### 14.1 Non-Liability clause

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

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