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LOT 04/2014

2019-04

Besmed Health Business Corp. Taipei, Taiwan

Mdi Europa GmbH

Wittekamp 30, D-30163 Hannover, Germany

0434

Distrib: Moretti S.p.A. - www.morettispa.com

EN READ CAREFULLY THIS INSTRUCTION MANUAL BEFORE USE**INTRODUCTION**

Thank you for purchasing a mask for oxygen therapy **Easyred** by **MORETTI S.p.A.** Their design and qualities guarantee comfort, safety and reliability to the user.

Masks for oxygen therapy **Easyred** by **MORETTI S.p.A.** have been designed to a practical and correct use.

This instruction manual contains main instructions but we recommend the use of the device by qualified personnel only. Read carefully this instruction manual before use, in case of doubts, contact your retailer. **MORETTI S.p.A.** declines all responsibilities for an incorrect use of the device or for a use not indicated in this manual.

INTENDED USE

The product is created for oxygen therapy. **MORETTI S.p.A.** declines all responsibilities for an incorrect use of the device or for a use not indicated in this manual.

INSTRUCTIONS

1. Connect the tube to the oxygen source
2. Connect the other part of the tube to the mask inlet
3. Adjust the oxygen flow at the level prescribed by the physician
4. Place the mask on patient face, covering both mouth and nose neck passing the elastic strap over patient's head and ears
6. Adjust soft metal nose piece to patient's face

WARRANTY

The device is guaranteed for manufacturing fault for 24 months as from the date of purchase. The warranty shall not apply in the event of damages caused by improper use or alterations of the product.

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