MOPEDIA®

MA RH780-RH830-RH840 02 A_09-2020

SEDIE DOCCIA

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INDICE

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REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 concerning medical devices

1. CODES

- RH780** 4-in-1 commode chair
RH830 Bath bench with backrest
RH840 Self-propelled WC and shower chair

2. INTRODUCTION

Thank you for purchasing a MOPEDIA bath products by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

3. INTENDED USE

The bath chair by Moretti is intended for disable patient assistance in bath tub.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SEAT AND SUPPORT FOR WC (RH780) - SHOWER CHAIRS (RH830-RH840) complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur.
- Keep the packed bath chair away from heat sources.
- The product use limit is defined by the wear of the parts.
- ALWAYS keep hands and fingers clear of moving parts to avoid injury.
- DO NOT allow children to play on or operate the bath chair.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

	Product code		. Batch Lot
	Unique Device Identification		Medical Device...
	Read the instruction manual		CE mark
	Conditions of disposal		Manufacturer

7. GENERAL DESCRIPTION

7.1 RH780

Steel structure with seat and backrest in HDPE. Flip-up and PE armrest. Non-slip rubber tips. Adjustable in height. Usable in 4 functions: raised toilet seat, support for toilet, commode chair and shower bench.

7.2 RH830

Shower chair with backrest. Aluminum frame reinforced with the addition of a cruise. Seat and backrest in HDPE and non-slip rubber tips. Adjustable in height for greater adaptability.

7.3 RH840

Self-propelled shower chair. Demountable steel frame, seat and backrest in PU. Flip-up armrests, flip-up and removable footrests.

8. ASSEMBLING

8.1 RH780

The product is delivered already assembled. The only elements that need assembly are the legs. To install the legs, press the locking button on each leg and insert the legs. Fix the position by locking the button in one of the holes present.

8.2 RH830

The seat and legs of the product are delivered already assembled.

To install the backrest:

- Turn the seat upside down;
- Press the lock buttons on the backrest structure;
- Insert the backrest tubes inside the guides under the seat up to the lock buttons do not reach the holes present.

8.3 RH840

The product is delivered partially assembled and no tool is required to complete the assembly. The shower chair consists of the following parts:

- Left side of the frame;
- Right side of the frame;
- Backrest;
- Left and right footrest;
- Seat and seat cover;
- Bucket;
- 4 x C-clip with safety lock;
- 2 x fixing knobs;
- 2 x connecting pipes.

To complete the assembly carry out the following operations:

- Insert the fixing tubes on the left or right side frame, align the holes and insert the C-clips as shown in Fig.1;
- Connect the newly assembled part to the other side of the frame by using the same two fixing tubes and two additional C-clips as described in the previous point (Fig.2);
- Insert the backrest inside the holes on the wheelchair frame and fix the backrest using the two fixing knobs (Fig.3);
- Place the seat on the wheelchair frame and press down to fix its position, then add the cover (Fig.4);
- Insert the footrests from the side and rotate them to the front until fastening into the holder;
- Insert the bucket from the rear side of the shower chair by sliding it into the slots (Fig.5).

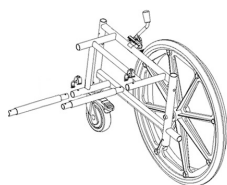
Fig.1

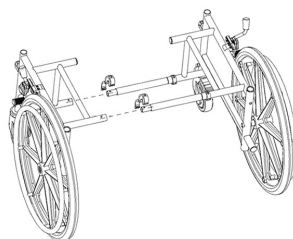
Fig.2

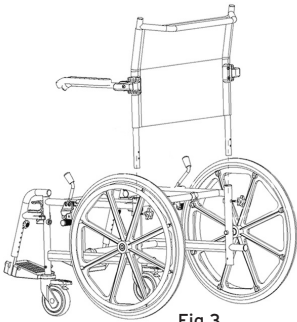
Fig.3

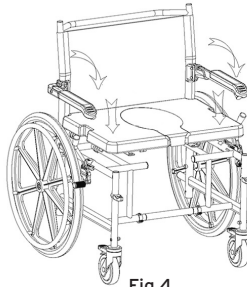
Fig.4

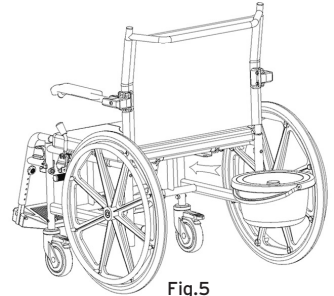
Fig.5

9. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things.
- Check the correct assembly of the device.

10. CAUTIONS FOR USE

The manufacturer and distributor are not responsible for any responsibility for the consequences of improper use of the product.

- Always help the patient with walking difficulties to position themselves on the seat.
- Do not use with patients who exceed the maximum weight shown on the label placed on the product frame.
- Avoid sudden movements during positioning.
- Be careful on wet floors, on wet or slippery surfaces, it could make difficult the patient positioning.
- Do not store the product outside. Periodically check the stability of the product and the correct fastening screws and the status of protections rubber.
- In case of doubt for the use, the adjustment or the maintenance of the product, please contact your dealer.

WARNING!

Do not exceed the maximum load capacity for any reason!

11. HOW TO USE

11.1 RH780

- Setting out to sit with your weight on the seat assembly. Do not sit by pushing on one side only;
- Leaning always on both arms to get up and keep your balance;
- In case of use of the chair by people with movement difficulties, help them always in the sitting or in getting up from the chair.

HEIGHT ADJUSTMENT

To adjust the height:

- Press the fixing button and move the barrel to the desired height;
- Fix the position by locking the button in one of the holes present.

11.2 RH830

- Setting out to sit with your weight on the seat assembly. Do not sit by pushing on one side only;
- Leaning always on both arms to get up and keep your balance;
- In case of use of the chair by people with movement difficulties, help them always in the sitting or in getting up from the chair.

HEIGHT ADJUSTMENT

To adjust the seat height:

- Removes the clip placed on the leg;
- Move leg up or down to the desired height;
- Insert the clip into the hole to fix the leg.

11.3 RH840

Use only the back handle to move the chair, never use other supports. Know your own capabilities and limitations in terms of strength and endurance before attempting to go up a ramp. Assistance is recommended when in slope higher than 10%. If you need to stop in a sloping surface, avoid abrupt movements. During the downhill, do not change suddenly the direction. Lift the chair up if necessary, using the main frame and not armrests or footrests.

WARNING!

The patient must not put pressure on footrests when go up and down from the chair.

BRAKES

You can brake the chair pushing the lever on each wheel. If you need to adjust the brake lever, you can unscrew the nut, move the clamp at the required position and then tighten again.

WC

Use the toilet when the wheelchair is completely blocked. To use the toilet take out the removable part of the seat using the handle on the front part. Take out the toilet using the handle.

HEIGHT ADJUSTMENT

The backrest and the armrests can be adjusted together on two different heights. Unscrew and remove the knobs that lock the backrest to the seat and raise or lower the backrest as necessary. Finally insert and tighten the knobs again.

FOOTREST ADJUSTMENT

The footrests of shower chair can be adjusted according to your needs:

- Unfasten the screw;
- Adjust the footrest height;
- Fasten the screw again. Check that both footrests are adjusted at the same height.

12. MAINTENANCE

The ONDA HD by MORETTI S.p.A. chairs are carefully checked before marketing and are provided with the CE trademark. For patient safety we suggest you to control regularly the following parts:

- The main frame of the product and connecting parts;
- Assembly and operation of the wheels / legs;
- All welding points;
- Wear of the tips where present.

13. CLEANING AND DISINFECTION

13.1 Cleaning

To clean the aid and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of metal surfaces.

The manufacture company cannot be held responsible for the damage that is caused by using materials that could damage the product surface or corrosive chemicals during cleaning.

13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent. For the seat and the backrest use a damp sponge and neutral soap (for RH840). The WC must be regularly cleaned and disinfected using a normal bathroom disinfectant (for RH780 and RH840).

14. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

15. TECHNICAL FEATURES

RH780

*Dimensions:

 L: 71 cm  W: 51,5 cm  H: 74,5÷85,5 cm

Seat:

59 x 47 x 5,5 cm

Seat height:

49÷60 cm

Armrest height:

67÷78 cm

 Max loading weight:

290 kg

RH830

*Dimensions:

 L: 46 cm  W: 40,5 cm  H: 76÷87 cm

Seat:

51 x 31 x 8,1 cm

Seat height:

41÷52 cm

 Max loading weight:

227 kg

RH840

*Dimensions:

 L: 83 cm  W: 100 cm  H: 97 cm

Seat:

61 x 40 cm

Seat height:

50,5 cm

Armrest height:

73÷75 cm

 Max loading weight:

200 kg

* The measurements refer to the medium dimensions

16. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use.

Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

17. REPAIR

17.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

17.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

17.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

18. SPARE SPARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date. For spare parts and accessories, refer only to Moretti general catalog.

19. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE**

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Sold to _____

Street _____ Place _____

 **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com
MADE IN P.R.C.

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