**SKEMA®FORM**

MA M070x-M071x-M072x 02 A\_ITA\_09 2020

**LETTINI PROFESSIONALI IN LEGNO**

---

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ITALIANO

## INDICE

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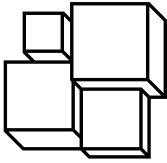
## CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

### 1. CODES

<b>MO700</b>	Antares wooden couch for therapy and examination - Cream
<b>MO700BLU</b>	Antares wooden couch for therapy and examination - Blue
<b>MO705</b>	Antares wooden couch with face hole, paper holder and shelf
<b>MO711</b>	Deneb wooden couch for therapy and examination with face hole
<b>MO712</b>	Deneb wooden couch for therapy and examination with face hole and width 90cm
<b>MO714</b>	Deneb wooden couch with face hole and 3 joints
<b>MO715</b>	Deneb wooden couch for cardiology and examination
<b>MO722</b>	Diadema wooden couch for Bobath therapy
<b>MO723</b>	Diadema wooden couch for Bobath therapy with backrest

### 2. INTRODUCTION

Thank you for choosing an examination couch from the SKEMA range by Moretti. Moretti examination couches have been designed and manufactured to meet all your needs. They are practical and safe to use. The ideal solution for examination and therapy. This manual contains instructions to help you get the most from the medical device you have chosen and also valuable safety advice. Please read this manual before you use the examination couch. If you require any further information or advice, contact your reseller. They will be glad to help.

### 3. INTENDED USE

Moretti examination couches have been designed for short physiotherapy and examination sessions.

#### CAUTION

- Using this device for purposes other than those defined in this manual is strictly forbidden.
- Moretti S.p.A. declines all liability for damage or injury caused by improper use of the device or for uses other than those specified in this manual.
- The manufacturer operates a policy of continuous improvement and reserves the right to make changes to the device and this manual without prior notice.

### 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of EXAMINATION COUCHES - SKEMA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation

6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

## 5. GENERAL WARNINGS

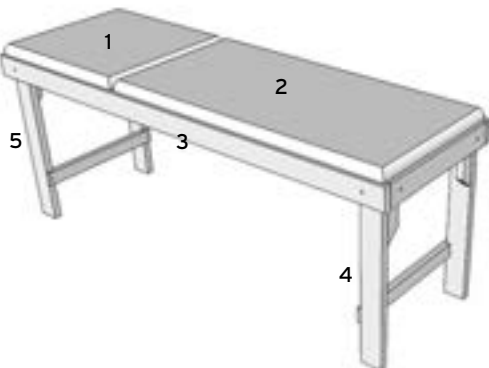
Please read this manual. It contains important information and instructions on how to use and maintain this device correctly.

- Keep the packaged product away from sources of heat. The packaging is made of cardboard and is flammable.
- The useful life of the device depends on the wear of its non-replaceable and non-repairable parts and is in any case no longer than ten years.
- Pay particular attention to moving parts. Limbs may become trapped in moving parts resulting in personal injury.
- Take particular care where children are present.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

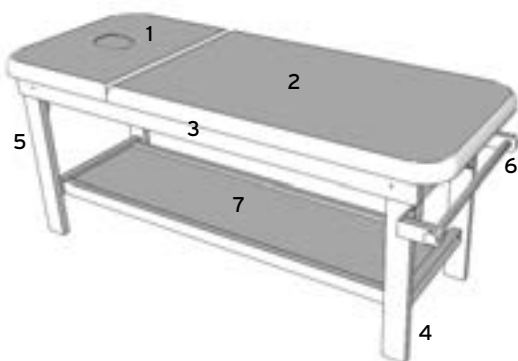
## 6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal

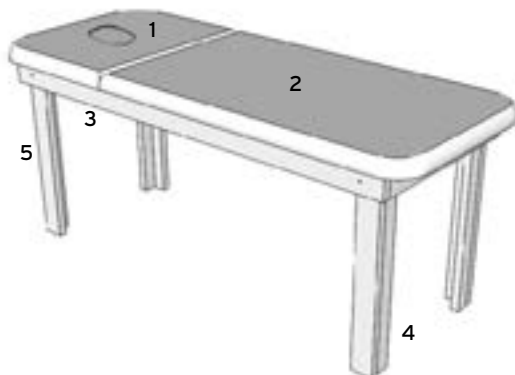
**7. GENERAL DESCRIPTION**
**7.1 Drawing and description of M0700 and M0700BLU parts**

**List of parts**

1. Padded backrest
2. Padded couch
3. Wooden frame
4. Leg (foot)
5. Leg (head)

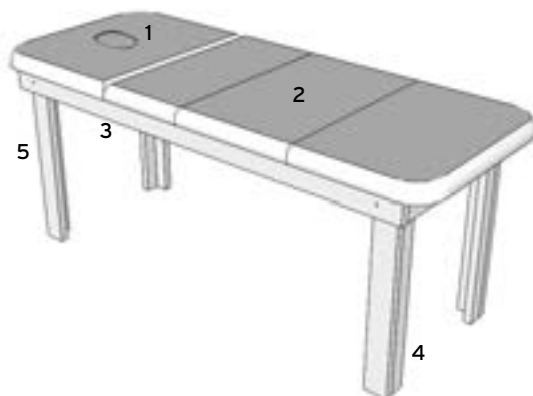
**7.2 Drawing and description of M0705 parts**

**List of parts**

1. Padded backrest with face hole
2. Padded couch
3. Wooden frame
4. Leg (foot)
5. Leg (head)
6. Paper holder
7. Shelf

**7.3 Drawing and description of M0711-M0712 parts**

**List of parts**

1. Padded backrest with face hole
2. Padded couch
3. Wooden frame
4. Leg (foot)
5. Leg (head)

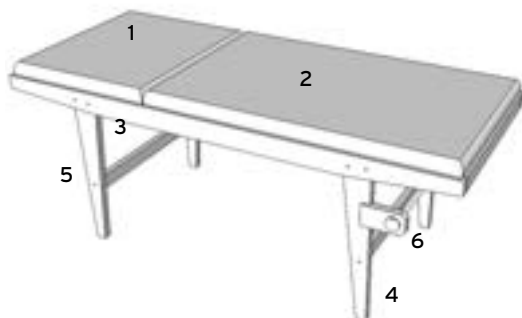
#### 7.4 Drawing and description of M0714 parts

##### List of parts

1. Padded backrest with face hole
2. Padded couch
3. Wooden frame
4. Leg (foot)
5. Leg (head)

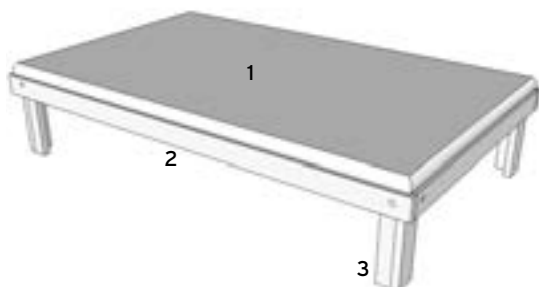
#### 7.5 Drawing and description of M0715 parts

##### List of parts

1. Padded backrest
2. Padded couch
3. Wooden frame
4. Leg (foot)
5. Leg (head)
6. Paper holder

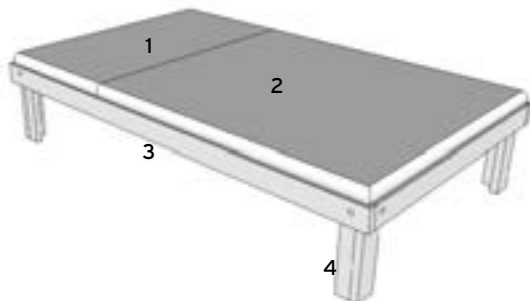
#### 7.6 Drawing and description of M0722 parts

##### List of parts

1. Padded couch
2. Wooden frame
3. Legs

### 7.7 Drawing and description of M0723 parts

#### List of parts

1. Padded backrest
2. Padded couch
3. Wooden frame
4. Legs

## 8. ASSEMBLY

#### CAUTION

Perform these operations with the help of another person. Take precautions to prevent injury or damage when moving the couch.

### 8.1 Assembling the M0700 and M0700BLU

1. Open the couch packing box and check the contents. Check that the contents correspond to the model you ordered. Check for any shipping damage. If the contents are not as ordered or are damaged, contact your reseller immediately.

2. Contents of the box:

- A. Wooden couch frame with padding
- B. Four legs and two cross-pieces
- C. Hardware

Bolt 8 x 60  
8 pcs

Washer  
10 x 30  
16 pcs

Nut M8  
8 pcs

Screw 7 x 50  
4 pcs

3. Open the Pluriball packing. Unscrew and remove the nuts and washers from the bolts. Turn the couch upside down. Insert the legs in their seats using the numbering on the legs as a guide. Insert the bolts as shown in Fig.1. Fit and tighten the nuts.

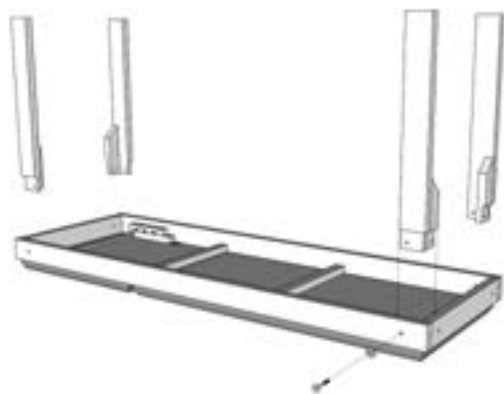
Fig. 1

4. Using the 7x50 screws provided, screw the two cross-pieces to the legs of the couch. With the help of another person, place the couch upright on a flat surface.

Fig. 2

#### 8.1.1 Fitting the M0729-1 paper holder (optional)

Using the four screws provided, fix the M0729-1 paper holder (optional) to the couch frame.

Fig. 3

YOUR KIT IS NOW FULLY ASSEMBLED AND READY FOR USE.

## 8.2 Assembling the M0705

1. Open the couch packing box and check the contents. Check that the contents correspond to the model you ordered. Check for any shipping damage. If the contents are not as ordered or are damaged, contact your reseller immediately.

2. Contents of the box:

- A. Wooden couch frame with padding
- B. Legs
- C. Shelf
- D. Paper holder
- E. Hardware

**Bolt 8 x 60**  
**8 pcs**

**Washer**  
**10 x 30**  
**8 pcs**

**Nut M10**  
**8 pcs**


**Screw 5 x 35**  
**4 pcs**

3. Open the Pluriball packing. Join the shelf to the couch legs as shown in **Fig.4**.

**Fig. 4**

4. With the help of another person, position the shelf and the couch legs to the padded frame.

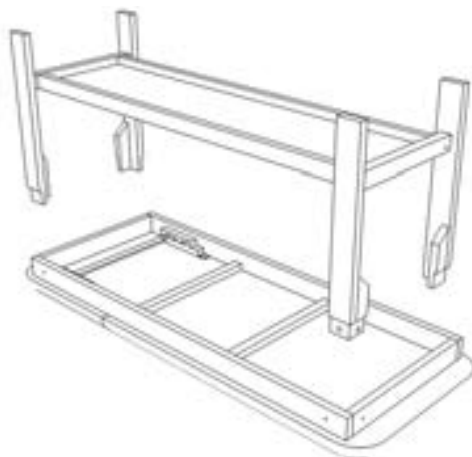**Fig. 5**

5. Position the couch upright. Using the four screws provided, fix the paper holder to the couch frame.

Fig. 6

YOUR KIT IS NOW FULLY ASSEMBLED AND READY FOR USE.

### 8.3 Assembling the M0711, M0712 and M0714

1. Open the couch packing box and check the contents. Check that the contents correspond to the model you ordered. Check for any shipping damage. If the contents are not as ordered or are damaged, contact your reseller immediately.

2. Contents of the box:

- A. Wooden couch frame with padding
- B. Four legs
- C. Hardware

Bolt 8 x 60  
8 pcs

Washer  
10 x 30  
16 pcs

Nut M8  
8 pcs

3. Open the Pluriball packing. Unscrew and remove the nuts and washers from the bolts. Turn the couch upside down. Insert the legs in their seats using the numbering on the legs as a guide. Insert the bolts as shown in Fig.7. Fit and tighten the nuts.

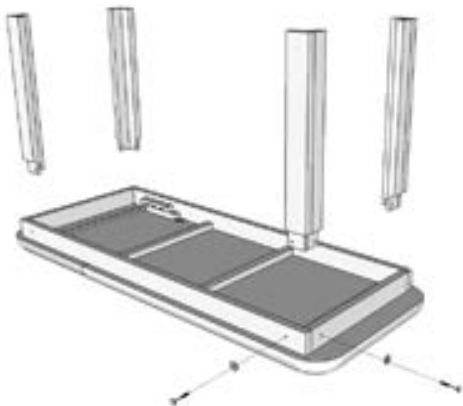
Fig. 7

### 8.3.1 Fitting the M0729-2 paper holder (optional)

With the help of another person, place the couch upright on a flat surface. Using the four screws provided, fix the M0729-2 paper holder (optional) to the couch legs.

Fig. 8

YOUR KIT IS NOW FULLY ASSEMBLED AND READY FOR USE.

### 8.4 Assembling the M0715

1. Open the couch packing box and check the contents. Check that the contents correspond to the model you ordered. Check for any shipping damage. If the contents are not as ordered or are damaged, contact your reseller immediately.

2. Contents of the box:

- A. Wooden couch frame with padding
- B. Legs
- C. Hardware

Bolt 8 x 60  
8 pcs

Washer  
10 x 30  
16 pcs

Nut M8  
8 pcs

3. Open the Pluriball packing. Unscrew and remove the nuts and washers from the bolts. Turn the couch upside down. Insert the legs in their seats using the numbering on the legs as a guide. Insert the bolts as shown in Fig.9. Fit and tighten the nuts.

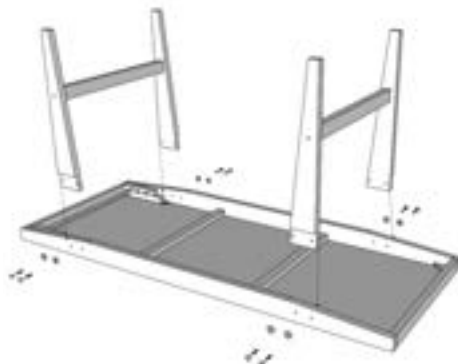
Fig. 9

#### 8.4.1 Fitting the M0728 paper holder (optional)

With the help of another person, place the couch upright on a flat surface. Using the four screws provided, fix the M0728 paper holder (optional) to the couch legs.

Fig. 10

YOUR KIT IS NOW FULLY ASSEMBLED AND READY FOR USE.

#### 8.5 Assembling the M0722 and M0723

1. Open the couch packing box and check the contents. Check that the contents correspond to the model you ordered. Check for any shipping damage. If the contents are not as ordered or are damaged, contact your reseller immediately.

2. Contents of the box:

- A. Couch frame with padding
- B. Legs
- C. Hardware

Bolt 8 x 60  
8 pcs

Washer  
10 x 30  
16 pcs

Nut M8  
8 pcs

3. Open the Pluriball packing. Unscrew and remove the nuts and washers from the bolts. Turn the couch upside down. Insert the legs in their seats using the numbering on the legs as a guide. Insert the bolts. Fit and tighten the nuts.

Fig. 16

4. With the help of another person, place the couch upright on a flat surface.

#### 9. BEFORE EACH USE

- Check the mechanical parts for wear. Check that the device can be used in perfect safety.
- Check that the device has been correctly assembled. Pay particular attention to the pins and stops locking the backrest and the mechanical joints in general.

## 10. PRELIMINARY CHECKS

Before you make any adjustments to the couch, ensure that it is in a stable position on its feet.

## 11. INSTRUCTIONS FOR USE

The couch backrest can be adjusted to angles from 0° to 45°. For models with manual adjustment, raise the backrest and fix it at the required angle using the adjuster rack on the frame. For models with a metallic adjuster stay, raise the backrest, move it to the required height and fix it there with the adjuster stay.

### CAUTION

Before you position a patient on the couch, check that the backrest is safely locked in position.

## 12. MAINTENANCE

The SKEMA series devices by Moretti are thoroughly checked before they leave our production facility and have the CE mark. For the safety of the patient and physician we recommend that you check the suitability of your product for its intended use at least once a year. Regularly inspect the following:

- Load bearing structure of the couch.

For repairs, only use original spare parts and accessories.

## 13. CLEANING AND DISINFECTION

### 13.1 Cleaning

Use a damp cloth and neutral soap only. Dry thoroughly before use. Use water with a temperature not exceeding 30 °C.

### 13.2 Disinfection

Where necessary, disinfect the product with a standard household disinfectant.

**N.B.** Do not use acid or alkaline detergents. Do not use solvents (e.g. acetone, diluent).

## 14. CONDITIONS OF DISPOSAL

### 14.1 Disposal regulations

When disposing of the device at the end of its life cycle, do not use the usual systems for the disposal of solid urban refuse. Dispose of the device at a municipal waste separation and recycling centre.

## 15. OPTIONALS WITH SURCHARGE

### 15.1 Frame colour

Dark walnut	Product code M0790-01
Cherry	Product code M0790-02
Wenghè	Product code M0790-03
Light walnut	Product code M0790-04

### 15.2 Width variation

Cod. **MO793-L** Variation width up to 15 cm

**MO700:** up to 77 cm padding and frame (not considering the paper holder)

**MO705:** up to 80 cm only padding

**MO711:** up to 89 cm only padding

**MO712:** up to 105 cm only padding

**MO714:** up to 89 cm only padding

**MO715:** up to 91 cm padding and frame

**MO722:** up to 130 cm padding and frame (only in blue or cream colour)

**MO723:** up to 130 cm padding and frame (only in blue or cream colour)

Cod. **MO793-XL** Variation width more than 15 cm

**MO705:** NOT AVAILABLE

**MO711:** more than 15 cm only padding

**MO712:** more than 15 cm only padding

**MO714:** NOT AVAILABLE

**MO715:** more than 15 cm padding and frame

**MO722:** more than 15 cm padding and frame (only nautical fabric in blue or cream colour)

**MO723:** more than 15 cm padding and frame (only nautical fabric in blue or cream colour)

### 15.3 Height variation

Cod. **MO793-H**

**MO700:** 10 cm

**MO705:** 10 cm

**MO711:** 10 cm

**MO712:** 10 cm

**MO714:** 10 cm

**MO715:** 10 cm

**MO722:** 10 cm

**MO723:** 10 cm

## 16. TECHNICAL SPECIFICATIONS

### 16.1 Material specifications for all models

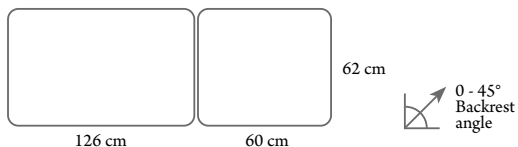
Frame material: **Wood**

Shelf material: **Chipboard**

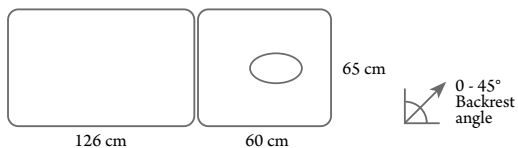
Padding: **Polyurethane density 25**

Upholstery: **Washable, class 11M fire retardant leatherette**

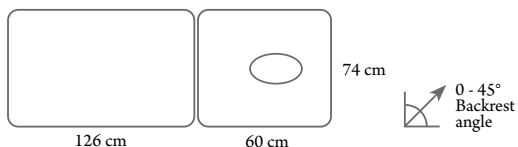
Painting: **Water-based paints**

**16.2 Dimensions and weight of M0700 and M0700BLU**

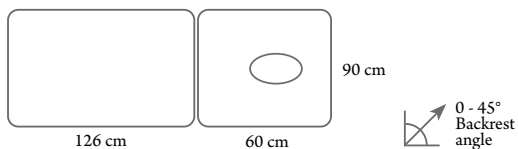
Dimensions: **L: 1890 W: 620 H: 780 mm**  
 Couch weight: **27 Kg**  
 Maximum load bearing capacity: **200 Kg**

**16.3 Dimensions and weight of M0705**

Dimensions: **L: 1890 W: 650 H: 780 mm**  
 Couch weight: **35 Kg**  
 Maximum load bearing capacity: **200 Kg**

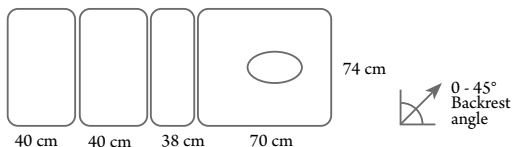
**16.4 Dimensions and weight of M0711**

Dimensions: **L: 1890 W: 740 H: 780 mm**  
 Couch weight: **36 Kg**  
 Maximum load bearing capacity: **200 Kg**

**16.5 Dimensions and weight of M0712**

Dimensions: **L: 1890 W: 900 H: 780 mm**  
 Couch weight: **36 Kg**  
 Maximum load bearing capacity: **200 Kg**

### 16.6 Dimensions and weight of M0714

Dimensions:

Couch weight:

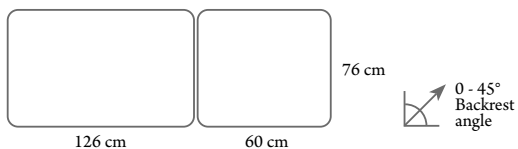
Maximum load bearing capacity:

**L: 1890 W: 740 H: 780 mm**

**36 Kg**

**200 Kg**

### 16.7 Dimensions and weight of M0715

Dimensions:

Couch weight:

Maximum load bearing capacity:

**L: 1890 W: 760 H: 750 mm**

**27 Kg**

**150 Kg**

### 16.8 Dimensions and weight of M0722

Dimensions:

Couch weight:

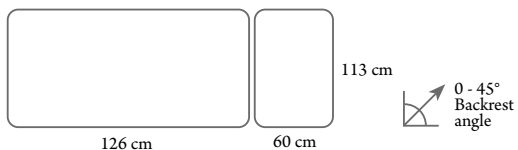
Maximum load bearing capacity:

**L: 1890 W: 1150 H: 500 mm**

**45 Kg**

**200 Kg**

### 16.9 Dimensions and weight of M0723

Dimensions:

Couch weight:

Maximum load bearing capacity:

**L: 1890 W: 1150 H: 500 mm**

**45 Kg**

**200 Kg**

## 17. WARRANTY

All Moretti products are guaranteed against material and manufacturing defects for a period of two (2) years from the date of sale of the product, subject to any exclusions or limitations specified below. This guarantee becomes invalid in the event of improper use, misuse or modification of the product or failure to follow the use instructions in this manual. The intended use of this product is specified in the instruction manual. Moretti declines all liability for damage or personal injury caused by failure to follow the instructions in the installation, assembly and use manuals. Moretti does not guarantee Moretti products against damage or defects in the following circumstances: natural disasters; unauthorised maintenance or repairs; damage deriving from electrical power supply problems (where present); use of parts or components not supplied by Moretti; failure to follow guidelines and operating instructions; unauthorised modifications; shipping damage (other than original Moretti shipping); failure to follow the maintenance instructions in this manual. This guarantee does not cover wear components if the damage can be attributed to normal use of the product.

## 18. REPAIRS

### 18.1 Repairs under guarantee

In the event that a Moretti product appears to have material or manufacturing defects during the guarantee period, Moretti will assess together with the customer if the product defect is covered under guarantee. Moretti can decide, at its sole discretion, to substitute or repair the item under guarantee at the premises of a specified Moretti reseller or at its own premises. Labour costs for the product repair can be charged to Moretti if the repair comes within the terms of the guarantee conditions. Repairs or substitutions do not renew or extend the guarantee.

### 18.2 Repair of a product no longer covered by guarantee

You may return a product no longer under guarantee for repair to Moretti provided that you obtain an authorisation to do so from Moretti Customer Assistance Service beforehand. Labour and shipping costs for repairs not covered by guarantee will be invoiced in full to the customer or reseller concerned. Repairs to products not covered by guarantee are guaranteed for six (6) months from the day on which the repaired product was received.

### 18.3 Non-defective products

The customer will be duly informed if Moretti, after examining and testing a returned product, finds that the product is not defective. The product will be returned to the customer and the customer will be billed for the costs of the return.

## 19. SPARE PARTS

Moretti original spare parts are guaranteed for six (6) from the day on which the spare part was received.

## 20. DISCLAIMER CLAUSES

Unless otherwise expressly specified in this guarantee and within the limits of the law, Moretti offers no further statements, warranties or guarantees, expressed or implied, including any statements regarding warranties and conditions of merchantability, fitness for particular use, non violations and non interference. Moretti does not guarantee that the use of a Moretti product will be uninterrupted or error-free.

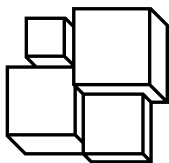
The duration of any implied guarantees which may be implied by law is limited to the guarantee period within the limits of the law. Some states or countries do not allow limitation on how long an implied warranty lasts and some do not allow exclusions or limitations of incidental or consequential damages in relation to consumer products. In some states and countries, some exclusions and limitations to this guarantee do not apply to the user. This guarantee is subject to variations without prior notice.

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