MA ST600/xx_ST601/xx_ST602_ST605_ST606_ST607_ST690_ST692_ST694 02 C_ITA_07 2021

ANTIDECUBITO IN FIBRA CAVA

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INDICE

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**MORETTI S.P.A.**

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN ITALY**

[illegible]

MORETTI S.p.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com

ANTI-DECUBITUS DEVICES IN HOLLOW FIBRE

INSTRUCTION MANUAL

INDEX

1. CODES.....	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL PRECAUTIONS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.5
7.1 Anti-decubitus devices with 3D lining	pag. 5
7.2 Anti-decubitus devices with cotton lining.....	pag. 5
8. HOW TO USE	PAG.6
9. CLEANING SILICONE HOLLOW FIBRE DEVICES.....	PAG.7
10. STORAGE	PAG.7
11. MAINTENANCE.....	PAG.7
12. CONDITIONS OF DISPOSAL	PAG.7
13. TECHNICAL SPECIFICATIONS.....	PAG.7
13.1 Anti-decubitus devices with 3D lining	pag. 7
13.2 Anti-decubitus devices with cotton lining.....	pag. 8
14. WARRANTY	PAG.9
15. REPAIRS	PAG.9
15.1 Repairs under warranty.....	pag. 9
15.2 Repairing a product not covered by the warranty	pag. 9
15.3 Non-defective products.....	pag. 9
16. SPARE PARTS	PAG.9
17. NON-LIABILITY CLAUSE	PAG.9

CE Class I medical device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 on medical devices

1. CODES

ST601/xx -	Pillow with removable 3D lining
ST606 -	Back support in 3D lining
ST602 -	Pillow with hole
ST607 -	Set of four elements for wheelchairs
ST600/xx -	Pillow
ST605 -	Back support
ST690 -	Heel pad in silicone hollow fibre
ST692 -	Elbow pad in silicone hollow fibre
ST694 -	Knee pad in silicone hollow fibre

2. INTRODUCTION

Thank you for choosing an anti-decubitus device in the LEVITAS range by Moretti. Moretti anti-decubitus devices have been designed and manufactured to satisfy all your needs for practical, correct and safe use. This manual contains useful suggestions for using your device properly and safely. Please read this entire manual carefully before you use the device. Should you have any queries, please contact your dealer for suitable advice and assistance.

Note: indicate several recommendations.

Caution: indicate the correct operating or maintenance procedures, in order to avoid damage to the device, the equipment or other items.

Warning: draw attention to a potential hazard that requires correct procedures or practices to avoid injury.

3. INTENDED USE

The LEVITAS anti-decubitus pillows by Moretti are intended to prevent the bedsores that can occur in any situation requiring lengthy hospitalisation and/or being laid up at home.

WARNING!

- Do not use the product for any purpose other than that specified in this manual
- The device must be installed by personnel qualified to perform general nursing procedures and who have received appropriate training in the prevention and treatment of bedsores
- Moretti S.p.A. declines any and all liability for damages resulting from improper use of the device or any use other than that specified in this manual
- The manufacturer reserves the right to modify the device and its relevant manual without prior notice, in order to improve the product's features

4. DECLARATION OF CONFORMITY

MORETTI SpA declares, under its own exclusive responsibility, that the products made and sold by Moretti SpA in the HOLLOW FIBRE Anti-DECUBITUS product family conform with the applicable provisions of Regulation (EU) 2017/745 of 5 April 2017 on MEDICAL DEVICES. To this end, MORETTI SpA guarantees and declares as follows, under its own exclusive responsibility:

1. The devices in question satisfy the general safety and performance requirements set out in Annex I to Regulation (EU) 2017/745, as prescribed by Annex IV to said regulation.
2. The devices in question ARE NOT MEASUREMENT TOOLS.

3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION.
4. The devices in question are sold in NON STERILE packing.
5. The devices in question should be considered as belonging to class I, in accordance with the rules set out in Annex VIII to said regulation.
6. MORETTI SpA maintains and provides to the competent authorities, for at least ten years from the date of manufacture of the last production lot, the technical documentation proving conformity with Regulation (EU) 2017/745.

Note: The complete product codes, the manufacturer's single registration number (SRN), the basic UDI-DI code and any references to standards used are indicated in the EU Declaration of Conformity that MORETTI SPA issues and provides through its own channels.

5. GENERAL PRECAUTIONS

- Please consult this manual carefully for correct use of the device
- Always consult your doctor or therapist for correct use of the device
- Use by one user only is recommended
- Use and store in the horizontal position
- Always sanitise the device before any reuse
- The devices are extremely sensitive to air and light.
- The lifetime of the device depends on wear and tear of non-repairable and/or non-replaceable parts
- Store and use the device away from open flames and sources of heat
- Always take care when children are present
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

6. SYMBOLS

	Product code
	Unique device identification
	CE marking
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Disposal terms
	Maximum washing temperature 60°C; very mild process
	Do not bleach
	Do not iron

Tumble drying possible; low temperature

Line drying

Do not dry clean

Flat drying

7. GENERAL DESCRIPTION

7.1 Anti-decubitus devices with 3D lining

ST601/xx - Pillow with removable 3D lining

ST606 - Back support in 3D lining

Manufactured in silicone hollow fibre, contained in a polyester fabric. Removable lining through a zip on the side only for ST601/xx. The main property of silicone hollow fibre is its resilience and thus its ability to return to the original shape even after heavy stress or pressure, thereby maintaining its softness and elasticity over time. It insulates against heat and cold, as hollow fibre has a high power of thermal insulation.

In addition to being highly breathable, it allows uniform distribution of weight and thus helps to avoid the vein occlusions that are the main cause of bedsores. The quantity of hollow fibre allows an excellent load response, due to the high denier, allowing resting of areas of the body with bony prominences and thus avoiding bottom out.

They are attached with polyester straps on the corners of the lining.

Note: Make sure the pillow is properly anchored to the seat before positioning the patient

ST601/xx

ST606

7.2 Anti-decubitus devices with cotton lining

ST600/xx - Pillow

ST602 - Pillow with hole

ST605 - Back support

ST607 - Set of four elements for wheelchairs (supplied already assembled)

The set comprises four elements: a back support, a seat cushion and two armrests. Each element has a detachable half of the zip that is reattached by zipping the backrest onto the back of the seat cushion and the two armrests onto the sides. Make sure the seat and armrests are properly anchored to the wheelchair.

Note: Make sure the pillow is properly anchored to the seat before positioning the patient

ST602

ST607

ST600/xx

ST605

ST690 - Heel pad in silicone hollow fibre
 ST692 - Elbow pad in silicone hollow fibre
 ST694 - Knee pad in silicone hollow fibre

ST690

ST692

ST694

Manufactured in hollow fibre contained in a cotton cover. The main property of silicone hollow fibre is its resilience and thus its ability to return to the original shape even after heavy stress or pressure, thereby maintaining its softness and elasticity over time. It insulates against heat and cold, as hollow fibre has a high power of thermal insulation. In addition to being highly breathable, it allows uniform distribution of weight and thus helps to avoid the vein occlusions that are the main cause of bedsores. The quantity of hollow fibre allows an excellent load response, due to the high denier, allowing resting of areas of the body with bony prominences and thus avoiding bottom out.

8. HOW TO USE

Position the device on the part of the body concerned or on the seat or backrest; attach with the straps or Velcro strips (if available). It is recommended to place a further lining between the device and the user (except for ST690, ST692 and ST694).

9. CLEANING SILICONE HOLLOW FIBRE DEVICES

Maximum washing temperature 60°C; very mild process

Do not bleach

Do not iron

Tumble drying possible; low temperature

Line drying

Do not dry clean

Flat drying

10. STORAGE

Keep in a cool, dry place, away from damp environments.

11. MAINTENANCE

LEVITAS devices manufactured by Moretti are CE marked and carefully checked before being commercialised.

For patient and doctor safety, have the product checked at least every two years by the manufacturer or by an authorised laboratory to verify its suitability for use. Use only original spare parts and accessories for repairs.

12. CONDITIONS OF DISPOSAL

Never dispose of the product as normal domestic waste. Dispose of the product at a sorted waste collection centre for recycling.

13. TECHNICAL SPECIFICATIONS

13.1 Anti-decubitus devices with 3D lining

ST601/xx - Pillow with removable 3D lining

ST606 - Back support in 3D lining

Dimensions and weight

Product codes	ST601/38	ST601/40	ST601/43	ST601/45	ST601/48	ST601/50	ST606
 L:	380 mm	400 mm	430 mm	450 mm	480 mm	500 mm	600 mm
 D:	380 mm	400 mm	430 mm	450 mm	480 mm	500 mm	440 mm
 H:	120 mm						
 *	470 g	570 g	660 g	750 g	830 g	950 g	560 g

* The weight refers to the product complete with lining

- External lining removable through a zip on the side 100% polypropylene
- 100% polyester lining
- 100% polyester hollow fibre - 17 denier

13.2 Anti-decubitus devices with cotton lining

ST602 - Pillow with hole

ST607 - Set of four elements for wheelchairs (it is possible to replace the elements individually)

Dimensions and weight

Product codes	ST607 seat	ST607 backrest	ST602
 L:	430 mm	450 mm	430 mm
 D:	430 mm	600 mm	430 mm
 H:	-	-	110 mm
	640 g	600 g	540 g

* The weight refers to the product complete with lining

ST600/x - Pillow

ST605 - Back support

Dimensions and weight

Product codes	ST600/38	ST600/40	ST600/43	ST600/45	ST600/48	ST600/50	ST605
 L:	380 mm	400 mm	430 mm	450 mm	480 mm	500 mm	600 mm
 D:	380 mm	400 mm	430 mm	450 mm	480 mm	500 mm	440 mm
 H:	100 mm						
	460 g	480 g	520 g	590 g	720 g	790 g	620 g

* The weight refers to the product complete with lining

ST690 - Heel pad in silicone hollow fibre

ST692 - Elbow pad in silicone hollow fibre

ST694 - Knee pad in silicone hollow fibre

Dimensions and weight

Product codes	 L:	 D:	 H:	minimum circumference	maximum circumference	
ST690	310 mm	350 mm	100 mm	-	-	130 g
ST692	310 mm	320 mm	40 mm	320 mm	450 mm	120 g
ST694	380 mm	400 mm	60 mm	470 mm	800 mm	270 g

* The weight refers to the product complete with lining

- 100% cotton lining
- 100% polyester hollow fibre - 17 denier

14. WARRANTY

Moretti products are guaranteed for 2 (two) years from the date of sale against material and manufacturing defects, subject the following limitations.

The warranty is voided by improper use, abuse, modifications to the product and failure to follow the instructions. The intended use of the product is given in the user manual.

Moretti is not liable for damage, injury or any other consequences resulting from installation or use which are not scrupulously conforming with the instructions given in the installation, assembly and user manual.

Moretti does not guarantee its products against damage or defects in the following circumstances: natural disasters, unauthorised repair or maintenance, improper electric power supply (as applicable), use of parts or components not supplied by Moretti, failure to follow the guidelines and instructions for use, tampering, shipping damage (other than the original shipping by Moretti), or failure to perform maintenance as indicated in the manual. Components subject to wear and tear are not covered by this warranty if the damage is caused by normal use of the product.

15. REPAIRS

15.1 Repairs under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

15.2 Repairing a product not covered by the warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

15.3 Non-defective products

The client will be notified if Moretti concludes that the product is not defective after having received and examined it. The product will be returned to the client at his expense.

16. SPARE PARTS

Moretti original spare parts are guaranteed for 6 (six) months from the day of delivery.

17. NON-LIABILITY CLAUSE

Unless otherwise expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, guarantee or condition, express or implicit, including any future declaration, guarantee or condition of sale, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of its product will be uninterrupted and problem-free. The duration of any implicit guarantee under the law is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations on the duration of an implicit guarantee or the exclusion of limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not apply to the user. This warranty is subject to modification without notice.

**WARRANTY CERTIFICATE**

Product _____

Date of purchase _____

Reseller _____

Street _____ Town _____

Sold to _____

Street _____ Town _____

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) - ITALY - Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN ITALY**

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