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Materasso a Bolle d'Aria BP2.

Destinazione d'uso

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Bubble Air Mattress BP2.

Use

The mattress BP2 has to be combined with: Vircare pump (code 05.03.6600A1), Butterfly pump (code 05.03.6600A2) and Prestige pump (code 05.03.6600A4). The combination of mattress and pump creates a medical device for the prevention and relief of pressure ulcers in patients who have to stay in bed for a long time. It is suitable to use both at home or in a hospital. The alternation of the contact points between the skin and the mattress makes a light massage; in this way it is possible to stimulate the blood flow and a good oxygenation of the tissues.

Features	
Code	05.03.BP2
Bubbles Material	Non-Toxic PVC
Number of Bubbles	130
PVC Thickness	300 Microns
Dimensions Inflated	200 x 85 x 8 cm
Dimensions Deflated	320 x 90 x 1 cm
Therapeutic Capacity	120 Kg
Air Mode	Two Ways Dynamic (A-b-A-b)
Flaps 40 cm long at the ends for attachment to the bed	
Warnings	
The mattresses can be used only with pumps recommended by the manufacturer	
Check the correct assembly of the device with particular attention to the connection of the tubes between the pump and the mattress	
Maintenance and Cleaning	
Clean the mattress with a damp cloth and mild soap	
The mattress has to be dry before use, do not expose it to direct sunlight	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	03.33.06.018
CND Code	Y033306
RDM code	9768/R