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TABLE OF CONTENTS

1. Introduction
 - 17. General safety rules
 - 18. Carry weight
 - Transfer from the wheelchair
 - 19. Practice
 - Take objects from ground
 - 20. Pavements
 - 21. Crossing narrow passages
 - Usage in the elevator and on lifting platforms
 - Transportation by car or other means
 - 22. Declaration of compliance
 - 23. Warranty
 - 24. Plate and EC label
 - 25. Maintenance register
 - 27. Advisory notice
 - 28. Section for retailers
2. Shipping
 - Assembling
3. Wheelchair list
 - Use specifications
 - Safety information
4. Components
5. Check before use
 - Use of wheelchair
 - Brake
6. Pressure brake
 - Drum brake
7. Calf pad
 - Armrests
 - Desk armrest / Height adjustment
8. Settings
9. Opening and closing of the wheelchair
10. Backrest
11. Footrest adjustment
12. Footrest height adjustment
13. Quick release rear axles for narrow passages
14. Wheelchair maintenance
15. Measures
16. Optionals

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Congratulations on your purchase. This manual provides instructions for the use of folding wheelchairs. Please read the following instructions before using the wheelchair for the first time.

The special care in the design and manufacture of these products, will ensure a high degree of safety and comfort.

Use your wheelchair by following the instructions in this manual.

Note: The images and illustrations in this manual are purely indicative; We may change the product without prior notice and without providing justification..

WARNING! DO NOT USE THE WHEELCHAIR FOR WHICH IS NOT SHOWN IN THIS GUIDE

These instructions are valid for all standard folding wheelchairs, in its equipment versions and accessories. Therefore, some chapters may not be related to the model purchased.

Simboli

It indicates to pay particular attention to risk in general

Allen key

Wrench

Shipping

All products are verified in Nuova Blandino for faults and are then packed in special boxes.

Therefore, please check the aid immediately after receipt, check for any damage incurred during transport, and if possible in the presence of the carrier.

In the event that you were to point out that the aid was damaged in transit, the damage must be reported immediately, making reservations in writing on withdrawal note.

Assembling

The orthopedic technician specialist will hand you your wheelchair ready for use, adapted to your personal needs.

Attention should be paid to the sending in Nuova Blandino Srl - within 8 days of receipt - of damaged parts, if these are removable, or the entire chair, together with a copy of the withdrawal note.

The recognition of the damage on our part can not occur in the case of failure to follow this procedure, or reporting the damage after the receipt.

Wheelchair list

These user guide are valid for the following models:

GR 101
GR104F
GR 105
GR 106

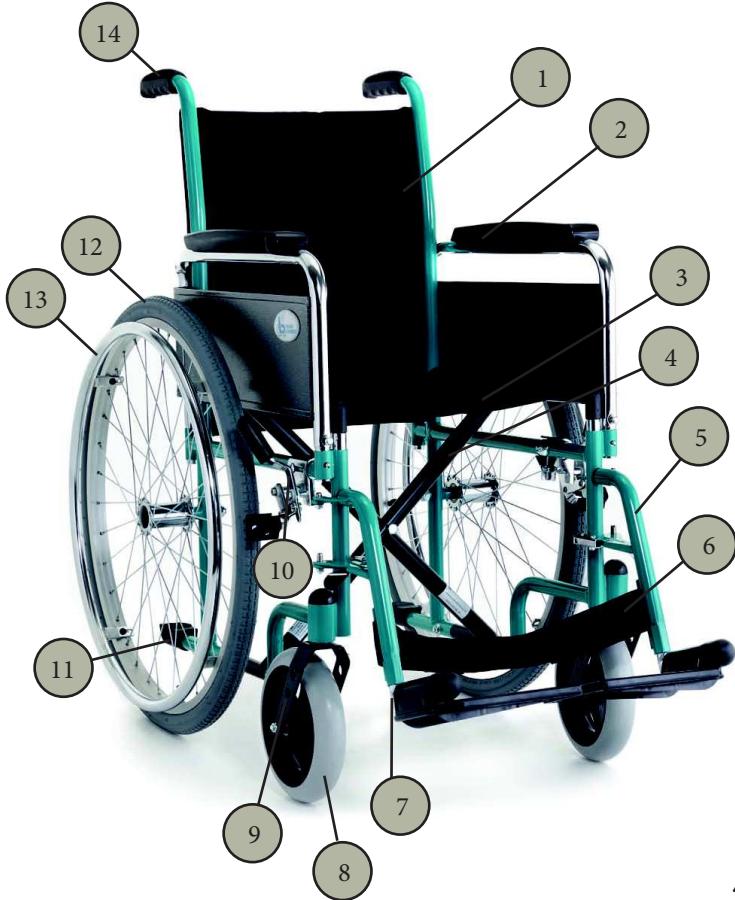
Use specifications

The wheelchair has been designed specifically to be used in a closed environment. Its sole purpose of transporting a person in the seat with the leg support and the side parts mounted, and not used as a means of traction, transport or otherwise.

Safety informations

All information for safe use of the wheelchair are given in second part of this manual.

Components

- 1 Backrest
- 2 Removable armrest
- 3 Seat
- 4 Crossbar
- 5 Swivel removable footrests
- 6 Calf pad
- 7 Footrests rod
- 8 Front wheel
- 9 Fork
- 10 Brake
- 11 PVC tip
- 12 Rear wheel
- 13 Handrim
- 14 Pushhandle

Check before use

Before using the multifunctional highchair, make sure that all parts are properly assembled and tightened, in particular the armrests and footrests.

If there are elevating platforms, verify that they allow a comfortable seat with a proper angle of the legs.

Check that the push brakes work properly. While activating the brakes on both wheels, push the chair forward and backward; braked wheels are expected not to rotate..

Use of wheelchair

Wheelchairs “Serie 100” are ideal for interior (home, workplace), for the outdoors free time, as long as the ground support is solid and not slippery.

Initially used the wheelchair with very carefully. Read and understand the second part of the manual “Safety Information”.

Brake

The parking brake is one of the main safety of a wheelchair element and is available as a pressure brake or drum brake.

The wheelchair is equipped, depending on the model, of a pressure brake which acts directly on the tires or drum brakes, identifiable by the levers and the wheel hubs.

Freno a pressione

The brakes must always be kept fully efficient: the braking effect of the brake pressure can be reduced or not be guaranteed if:

- The profile of the tires is worn
- The tires are wet
- The pressure of the tire is low

Regularly check and before every ride the pressure of your wheels !!

Recommendations: standard wheel Ø 600 3.0 bar pressure

Wheel light Ø 600 7.0 bar pressure.

Check that the spokes of the wheels are well pulled.

Warning:

Braking your wheelchair with great caution when you are going back.

This caution should be taken especially when you go back into the slope. Brake sharply to the hand rims or using the brake levers can cause a reversal of the wheelchair.

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The drum brake is activated by the accompanying through the brake lever mounted on the push handle.

Benefits: braking action regardless of the profile of the tire.

Calf pad

The calf pad prevents shifting of the legs from the footrests. It is placed in special attacks or outside of engagement footrest based on the type of model.

The calf pad is not included in the model with elevating footrests and is replaced by a upholstered legrests.

Armrests

The armrests are detachable (depending on model) and equipped with padded armrest and a side skirt.

- The removable armrests make it easy to transfer from a wheelchair to the bed, chair, etc.
- Lateral support of the body
- Protective clothing saves against the spokes.

Warning:

Never lift the chair by the armrests!

The wheelchair should only be used when the arms are mounted

Desk armrests / Height adjustment

The desk armrest allows to have shortest upholstered armrests necessary for specific cases of application.

The wheelchair can slide more easily under tables, desks etc ...

The armrest height adjustment allows the user greater adaptability

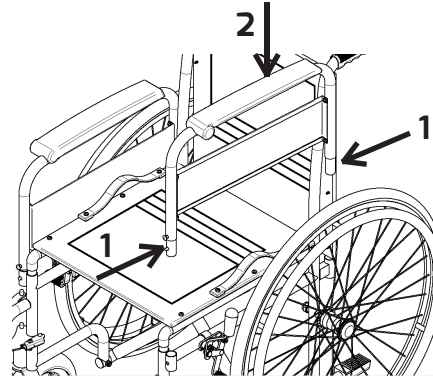
If using a cushion you can adjust the height of the armrests to the new level of the seat.

Settings

Assembly and removal of the armrest



Assembly:

- Simultaneously connect the front and rear in the attacks (1).
- Push the armrest until you hear the “click” of the button inside the front bracket (2).

Removal

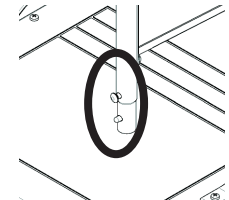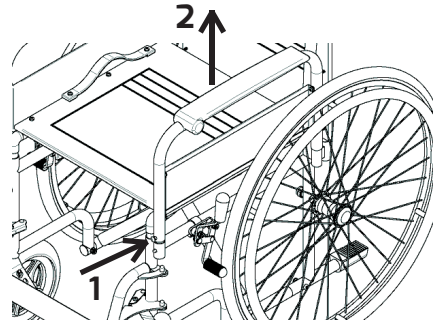- Press the spring (1).
- Slowly push the arm out from the attacks of the wheelchair at the same time pulling the front and the rear part (2).

ARM RELEASE
PIN DETAIL

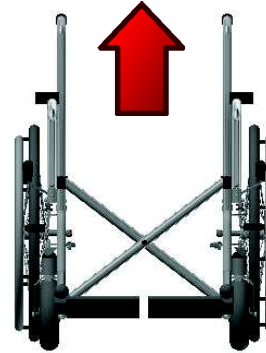
Opening and closing of the wheelchair

The wheelchair can be closed with ease, without the use of tools, performing a few manual operations.

Closing:

- remove the cushion, if present
- lift platforms plastic or disassemble the docking footrest
- raise the central part of the cloth seat using the handles

Caution: for GR 104F, unlock the rod back and turn the leg protections outward.

OPENED WHEELCHAIR

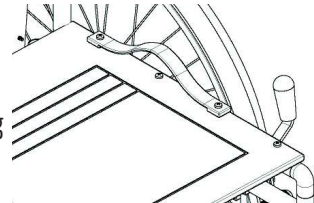
CLOSED WHEELCHAIR


Opening:

- push down on the top of the crossbar until it stops.
- lowering the footrests or plastic mount docking footrest
- place the pillow if alive.

Caution: for GR 104F, lock the rod back and turn the leg protections inward

HANDLE DETAILS

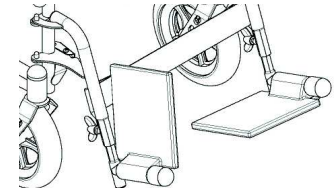
FOOTREST DETAIL

WARNING! NEVER CLOSE THE WHEELCHAIR USING THE ARMRESTS

Backrest

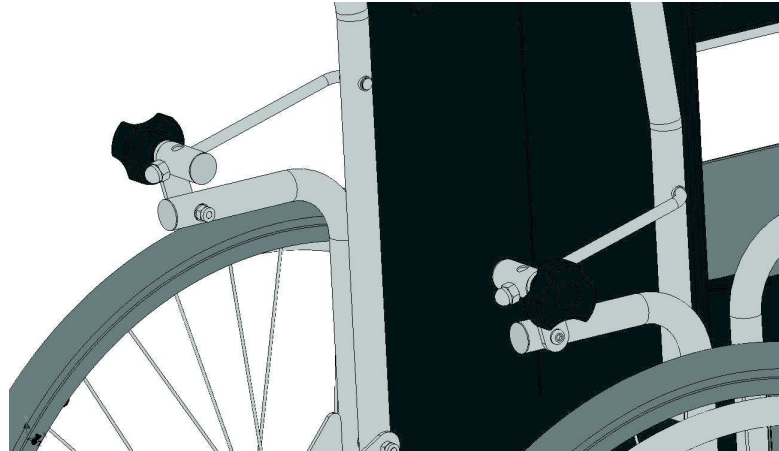In addition to the standard backrest into the range of products we find, for the model of wheelchair GR 104F, manually reclinable backrest.



The reclination is done by unscrewing the two knobs placed behind the backrest.

Footrest adjustment


The footrests are removable. The footrest can be folded down in order to facilitate the approach of the person on the wheelchair.

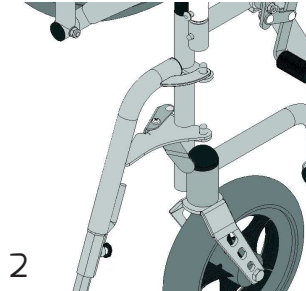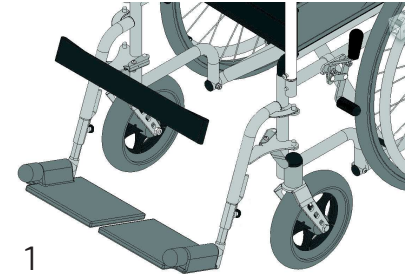1. Release footrest:
2. Remove the calf pad
3. Press the release lever footrest
4. Turning outwards and slide upwards

Fitting footrest:

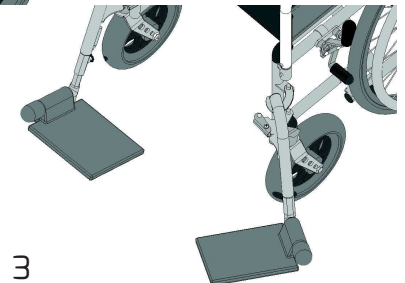1. Insert in the appropriate external attacks the footrest
2. Turning towards the inside of the wheelchair, until it is fully engaged to the frame
3. Replace the calf pad

Footrest height adjustment

For all folding wheelchair there is the height footrest adjustment.

Adjusting the footrest height:

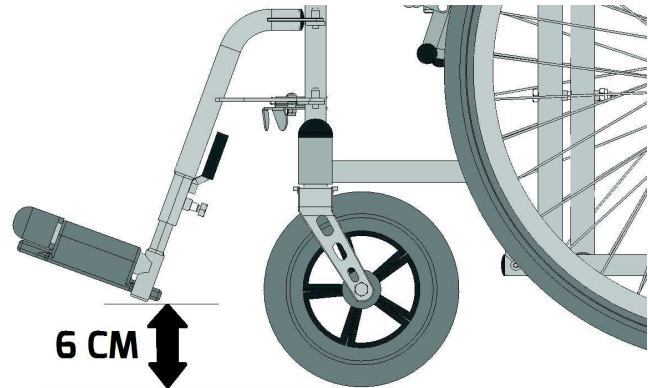
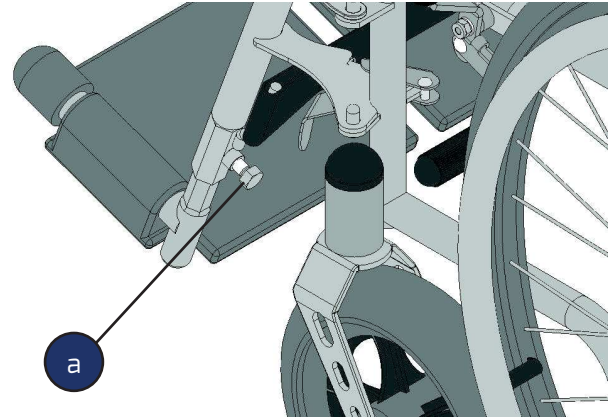
1. Loosen the bolt (a) and adjusted to the desired height
2. Tighten the bolt (a) to lock the rod of the platform

Warning:

verify that the footrest rod is inserted at least 5 cm in the footrest support.

Regulate the footrest rod in order to have always a gap between the plastic platform and the ground about 6 cm

Quick release rear axles for narrow passages

Light pressure on the pin allows you to pull out the rear wheel quickly reducing the width of the wheelchair. To replace the wheel, just applying pressure on the button until you hear a click.

The wheels for narrow passages are used to pass through small doors after extracting the pushing wheels.

Quick release rear axle

Wheelchair maintenance

Annual checks

Maintenance should be performed once a year. We recommend careful monitoring of bearing condition: sand, water and dust damage them. Lubricate lightly with bicycle free-resins oil.

To clean the frame use hot water with a little soap, or a household cleaner. The plastic parts must be cleaned with common detergents specific for this material.

Warning!

Do not use high-pressure cleaning equipment! Risk of damage to the mechanical part caused by water jets.

The wheelchairs should not be cleaned with detergents and items for aggressive cleaning.

Warning!

All the components that can be in contact with the skin, when the wheelchair passes from one user to another, need to be replaced.

Repairs

To repair trust your dealer Nuova Blandino, which has been instructed in our factory according to our specifications, and is able to offer advice, support and repair services.

Spare parts

Sort them through your dealer. For the purposes of proper delivery of spare parts you are made to the number of frame on the wheelchair. For reasons of safety, use spare parts and accessories authorized by Nuova Blandino Srl indications of where to find the chassis number. (p. 27)

Disposal


The packaging of the wheelchair can be brought to the recycling materials. The metal parts are collected and recycled in the old metal. The fabric parts and plastics are recycled.

Dispose of it according to their national laws.


MISURE

Model	Seat width (cm)	Seat depth (cm)	Backrest height (cm)	Seat height (cm)	Armrest height (cm)	Total width (cm)	Total depth (cm)	Weight	Optional width (cm)	Maximum load	Package (cm)
GR101	40 - 43	42	41	50	22	50 - 61	106	18 kg	36 - 38 - 45 48 -53	120 kg	26x81x92
GR104F	40 - 43	42	41	50	22	61 - 64	118	23.5 kg	36 - 38 - 45 48 -53	120 kg	31x89x98
GR105	40 - 43	42	41	50	22	58 - 61	84	15 kg	36 - 38 - 45 48 -53	120 kg	26x81x92
GR106	40	42	38	55	22	50	101	18 kg	43 - 45 48	120 kg	31x89x98

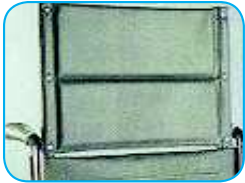OPTIONALS

Headrest

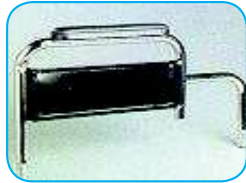Desk armrest

Height adjustable
armrest

Abduction

Double crossbar

Elevating footrest

Quick release axle

Backing

Drum brake

Ruota a timone

Ruotino da transito con
puntale

Sedile intero

Sedile wc

Spinte tronco

Tavolino bordo incavo

General safety rules

Use wheelchairs only as described in the instructions for use.

To stand, climb or descend always choose a level and stable ground. Secure the wheelchair to prevent it from moving. If exposed directly to sunlight, the coating and the upholstery of the seats and armrests, leg supports and handles can reach temperatures above 41 ° C. Touching these parts with unprotected skin there is a risk of injury. To avoid the heat of these parts you park your wheelchair in the shade.

Inform yourself at your authorized dealer of all possible handling characteristics, in order to exclude possible hazards and the possibility of changes and adjustments and their effect on the operation and safety of the move. The authorized dealer will be happy to adjust the wheelchair according to the user's individual needs, with maximum respect for safety.

When you turn the adjusters (eg. Those of the back or the leg), take care and never put your hands in the orientation of the components. There is a risk of injury.

When transporting objects, make sure these do not affect or restrict the maneuverability of the wheelchair.

Check the condition of the tires and check the brakes

Carry weight

Do not overload the wheelchair with excessive weight and never use it to carry more than one person.

The maximum permissible load is shown in instructions for use and on the plate of vehicle.

Transfer from the wheelchair

Step off the wheelchair to go elsewhere, if made independently, it is dangerous.

It Requires:

- relevant physical forces
- good coordination skills
- a non-slip surface (eg. with the palms of your hands on the seat surface)
- much exercise

the knowledge of all the possible situations of danger

Let you be then helped by somebody. Discuss with the exercise and to do that there may be unexpected dangers such as. any stumbling points, or explained the type of your disability.

When the wheelchair is unloaded, for example, when you move from the wheelchair to the bed, the toilet or car, never doing strenght on the platform.

- Danger of tipping!

Warning:

The footrests are in the area of the legs, so you can stumble. For this reason, the footrests are folded up so that there is free space in the foot area.

Practice

For safe use of the wheelchair, both indoors and outdoors, you need a workout to gain confidence in the different situations, such as pushing, driving, braking the wheelchair, driving forward, cornering, in reverse on flat surfaces (possibly with the aid of an attendant).

Initially avoid steep inclines, rough roads or other similar situations

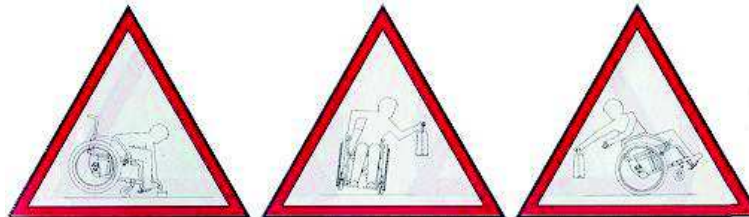
Take object from ground

Avoid to bow down in extreme ways, forward, sideways or backward, to collect or lay objects. There is a risk of rollover or loss of balance of the wheelchair if it has a seat width and reduced height elevated (pillow).

Warning:

- tilting in case of gripping objects forward
- loss of balance in the case where an object is taken from the side
- tilting in case you collect an object placed behind the seat back.


Pavements

Warning!

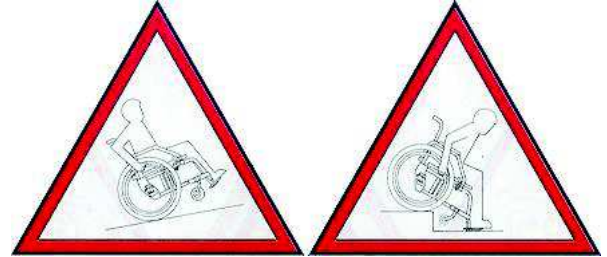Before you get off the pavement alone, weigh up the height. If it is too high, there is a risk of tipping over. Danger of overturning on steep slopes.

Crossing narrow passages

When you ride on narrow passages such building entrances, gates etc., Attention to elbows.

Risk of injury.

Usage in the elevator and on lifting platform

Elevators and lifts are a particular danger to the user.

Warning:

The automatically close doors involves the risk of causing injury to your hand!

Transportation by car or other means

The wheelchair must never be used as a seat when using a car or other means of transport. Provide the disassembly the wheelchair (as specified in the instructions for use) and place it in the luggage compartment, or otherwise secure it to prevent accidents in case of dangerous maneuvers.

Occupied a proper place inside the car or the means of transport and fasten your seatbelts.

Declaration of compliance

NUOVA BLANDINO Sr.l.

registered office: Strada Cebrosa, 24
10036 Settimo Torinese Fraz. - Torino

Responsible for designing, manufacturing, packaging, and labeling of this product, that falls within the category of
MANUAL WHEELCHAIR

DOES GUARANTEE AND DECLARE

The aforementioned device, launched and marketed as of the date certified in this certificate, fully complies with the provisions stated in annexes I and VII to EEC Directive 2007/47 and L.D. No 37 of 25 January 2010, that confirmed EU provisions at Italian level.

The products listed above are classified as Class 1 non-invasive medical devices as provided in Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product, the complete "EC" declaration of conformity is available on the website www.nuovablandino.com

The afore-mentioned products:

- *are not measuring instruments*
- *are marketed in non-sterile packaging*

This is to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

Settimo, 18th January 2016

WARRANTY

On the aids from our factory, we provide a warranty period of two years from the correct completion of the certificate of purchase (see page 29).

The warranty is valid for products that have defects in workmanship or operation.

The warranty excludes products that exhibit defects due to negligence or improper use.

Through our dealers, you can still get information on estimates for repairs.

For safety reasons, use spare parts and accessories approved by Nuova Blandino S.r.l.

The use of spare parts and accessories not approved by Nuova Blandino S.r.l. are reason of the immediate invalidation of the warranty.

To ensure the efficiency of the electric wheelchair, it must be kept clean, organized and maintained as specified in the instructions for use.

Attenzione:

Warning:

We accept no responsibility for damage caused by the combined use of the electric wheelchair with parts of all kinds from other manufacturers (for example trailers units, electrical devices) that may involve significant dangers - except that a given product has not been expressly authorized in writing by the Nuova Blandino S.r.l.

Nuova Blandino S.r.l. declines all liability for damage to persons, property, animals, and any third party due to modifications or improper use of the wheelchair.

PLATE AND EC LABEL

The EC labels (see picture) of the wheelchair are on the front crossbar element under the seat.

MAINTENANCE REGISTER

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

MAINTENANCE REGISTER

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

Advisory notice

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive.

- **Intended use**

The device is supplied ready for use by technical orthopedics and adjusted according to the user's needs.

The multifunctional chair is designed for both indoor and outdoor use. It is intended to accommodate one single person on the seat when the legrests and the lateral pads are mounted, and should not be used as a means of traction, transport or otherwise. Carefully read the instruction manual for a correct use of the device.

- **Check before use**

Before using OPTIMA, make sure that all parts are properly assembled and tightened, especially the platforms. If the product shows visible damage, damaged parts must be replaced to ensure your safety.

- **How to clean**

Do not use high-pressure cleaning equipment!

The chairs should not be cleaned with harsh detergents.

- **Lifespan of the device**

The duration of the multifunctional chair of our manufacturing, depends on the intensity of use to which it is subject. Proper use and proper maintenance allows you to maintain and prolong the life of the device, on average, an estimated duration of 5 to 7 years.

- **Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product.

Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 2:00 HOURS AND FROM 13:30 TO 7:00 HOURS. CALL +390112625140.

RETAILER STAMP

--

DATE OF PURCHASE

--	--	--

REGISTRATION DATE

--	--	--

N.B.: THIS MANUAL MUST BE PRODUCED WHENEVER ASSISTANCE IS NEEDED

Nuova Blandino S.r.l.
Strada Cebrosa, 24 - 10036 Settimo Torinese
Torino Italy
Tel. +39 011 2625140 - fax +39 011 2625141
www.nuovablandino.com - infonb@nuovablandino.com

Commitment and technology for people with special needs