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## INDEX

1. Introduction
3. Shipping
  - Assembling
  - Use specifications
  - Safety information
4. Components GR117
5. Check before use
  - wheelchair use
  - Brake - Push brake
6. Drum brake (if available)
7. Armrests
8. Adjusting the backrest tension (if available)
  - push handle
9. Opening and closing of the wheelchair
10. Complete removing of the platform
11. Adjusting the footrest height
12. Adjusting the rear wheel
13. Adjusting the fork and front wheel
14. Quick release pins and wheels for narrow passages
15. Transport by car
16. Wheelchair maintenance
17. Measures
18. Optionals
19. General safety rules
20. Portable weight
  - Transfer from the wheelchair
21. Training
  - Take the objects
22. Sidewalks
  - Ascent and descent - Stairs
23. Crossing narrow passages
  - Using elevators and lift bridges
  - Transportation by car or other means of transport
24. Declaration of compliance
25. Warranty
26. Plate and EC label
27. Maintenance log
32. Data sheet
33. Section for retailers

## Instructions for use and maintenance

### Introduction

We have been working for more than 20 years to guarantee maximum comfort, convenience and functionality to those who need wheelchairs and walking devices. This commitment of ours made us leaders in the field of research and technological innovation.

Our range of products is constantly evolving and some models have been studied to improve their features and performance levels.

For this reason, the GR117 model was designed.

This manual provides instructions for this particular models. Please read carefully the following instructions before using the device for the first time.

The special care taken in design and manufacture of these products is a guarantee of top-level safety and comfort. Use your wheelchair according to the instructions provided in this manual.

**Model GR117:** lightweight folding wheelchair in a low frame reinforced with double cross.

**Main clinical indications:** indicated for patients with chronic neurological disease or progressive able to achieve independently, routes and transfers.

**Basic equipment:** available in different sizes and heights, it presents the design characteristics that allow for the customization of the ergonomic structure, in order to optimize its functionality: variable positioning of the pins of the rear wheels, quick release wheels, the possibility of adjusting the front wheel, removable desk armrests height-adjustable swivel pegs, removable and height adjustable wheels to transit passages.

**Optionals:** customization options through the choice of accessories to meet your needs for comfort and safety for the hemiplegic arm, tension adjustable backrest, headrest multipurpose, footboard and stops walking.


## Introduction

### Symbols

While reading the manual you might come across these symbols of particular importance or interest.

symbolizes danger in general. Particular attention must be paid.

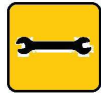6 mm spanner or wrench.

Allen wrench for 6mm hollow hex-screws.

### GR 117 is equipped with::

- Folding frame
- Height adjustable backrest
- Sides pad
- Variable positioning in multiple locations of the pins of the big wheels
- Adjusting angle of the forks
- Rear wheels 22"x1
- Front wheels Ø 125 mm hub 54 mm
- Height adjustable footrest
- Quick release pin
- Wheels for narrow passages

## Shipping

All Nuova Blandino products are tested in-house to verify compliance with all specifications and are then shipped in extra cardboard boxes.

Therefore, we recommend to check the device immediately after receipt, and in the presence of the carrier, to verify any damage that might have occurred during transport.

If the purchased product was damaged during transport, the damage must be reported immediately, by attaching an official written claim to the delivery note.

All damaged parts (if removable) or the whole wheelchair must be delivered carriage free to Nuova Blandino S.r.l - within 8 days from the date of receipt, together with a copy of the receipt.

No damage will be acknowledged unless the aforementioned procedure is followed, or if the damage is reported after the receipt note is issued.

## Assembling

The orthopedist will deliver your wheelchair ready for use, adjusted for your personal requirements.

## Use specifications

This wheelchair is designed for both indoor and outdoor use. Its sole purpose is carrying a single person, once the

footrest and the lateral pads are mounted, and not as a means of traction, transportation or otherwise.

## Safety informations

All safety information on the wheelchair will be presented in the second part of this manual.

## Components GR117

1. Cloth back
2. Adjustable armrest
3. Cloth seat + cushion padded
4. Crossbar
5. Quick release pin
6. Fork
7. Front wheel
8. Stop heels bands
9. Swivel removable footrests
10. Wheel lift with toe
11. Aluminum brake
12. Rear wheel

## Check before use

Before using the wheelchair, check that all its parts have been properly assembled and tightened, paying particular attention to the footrests.

If the footrests are height adjustable, make sure the user is able to sit comfortably and hold his or her legs at the correct angle.

Make sure that the brakes work properly. While activating the brakes on both wheels, push the chair forward and backward; braked wheels are expected to not rotate.

## Using the wheelchair

This active wheelchair is designed for people with special needs who carry out mainly outdoor activities. It is ideal for both in-door (i.e. home, school) and it may also be used for outdoor leisure activities, provided that the ground is sturdy and not slippery.

Initially, use the wheelchair with caution. Read the second part of the manual (“Safety information”) carefully.

## Brake - Push brake

The parking brake is one of the most important safety devices featured on this wheelchair. It is a push brake that operates directly on the tires.

The brakes must always be fully functional. The braking effect of the push brake may be impaired if:

- the sides of the tires are worn out
- the tires are wet
- the tire pressure is low

Check the wheels on a regular basis and before leaving your pressure!!

Reccomandations:

Ø 600 light wheel (7.0 bar). Make sure that the wheel spokes are tightened up.

**Caution:**

Restrain your wheelchair with great caution when you are going backwards.

Particular caution must be used when going backward on steep terrain. A sudden stop may result in the wheelchair tipping over.

Note: The brakes should not be operated when the wheelchair is in motion.

### **Drum brake (if available)**

The drum brake is activated by the accompanying by the brake lever mounted on the push handle.

**Advantages:** braking action regardless of the profile of the tire.

Braking is independent of the pressure of the tire. Safer use part of the companion due to the positioning of the brake levers on the two handles.

## Armrest

The armrests are removable and adjustable in height:

- removable, facilitate the transfer from the wheelchair to the bed, chair, etc ...
- are a lateral support of the body
- protect clothes against the spokes of the wheels
- adjustable in height, allowing the user a greater adaptability
- if you use a pillow you can adapt the seat height to the new level.

The armrest desk allows for shorter padded armrests for specific use cases. The wheelchair can slide more easily under tables, desks, etc. ..

**Caution:** Never lift the wheelchair by the armrests.


## Settings

### Adjusting the backrest tension (when available) - push handles

The tension of the backrest is adjustable to facilitate the containment of the trunk.


#### Adjusting the backrest tension:

- remove the cushion cover
- loosen the velcro straps (a)
- placing the person in a manner as to allow a greater winding trunk
- close bands
- reposition the backrest cushion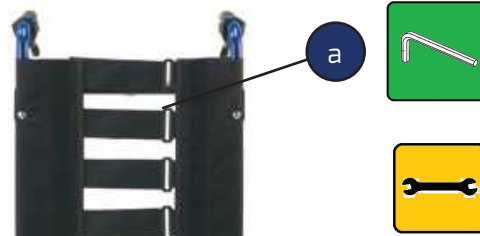

#### Adjusting the height of the push handles:

- unscrew the screw and nut (b)
- pick up or turn down the handle in according to the desired height
- tighten the screw and nut (b)

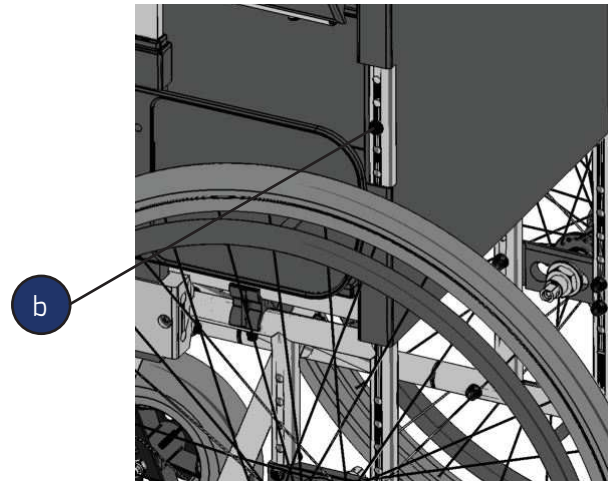## Opening and closing of the wheelchair

The wheelchair can be folded easily without the use of tools, running a few manual operations

### Opening:

- push down on the top part (d) of the cruise until it stops
- fold down the plastic footrests (d)
- place the pillow (if available)

### Closure:

- remove the pillow (if available)
- fold up the plastic footrests
- raise the central part of the cloth seat (c)

## Complete removing of the platform

The footrest can be folded up to facilitate the approach of the person on the wheelchair.

### Removing the platform:

- press the release lever platform (e)
- turn outwards and pull upwards

### Installation of the platform:

- enter the platform outside in the special attacks
- turn it towards the inner part of the wheelchair, till it is totally connected to the frame

## Adjusting the footrest height

### Adjusting the footrest height:

- loosen the bolt and adjust it to the desired height
- tighten the bolt in order to lock the rod of the platform

**Caution:**

After adjusting the platform, make sure the bolts are tightened up. Make sure that the footrest staff is inserted at least 5 cm inside the footrest junction.

Adjust the footrest staff so that the footrest is about 6 cm from the ground.

## Adjusting the rear wheel

Adjusting important for choosing the right set of wheelchair and can act both on the height and depth.

Height adjustment: the support plate of the rear wheel can be positioned at different heights:

- remove the rear wheel (see page 14) “Quick release pins and wheels for narrow passages”.
- remove the fixing screws
- move the plate up or down (step 2 cm)
- replace the screws and tighten the nuts.

### Depth adjustment:

the plate allows to fix the bushing (f) in different positions

- remove the rear wheel (see page 14) “Quick release pins and wheels for narrow passages”
- remove the bushing and the respective nut and washer
- find the right set-insert the screws in the appropriate locations and tighten all parts.

The adjustment of the rear wheel can lead to a variation of the angle of the seat, and in this case it is absolutely necessary to adjust the height of the front wheel (see page 13) “Adjusting the front wheels.”

### Caution:

the movement of the bushing in the forward position makes the wheelchair with an attitude “Active”, this will cause the wheelchair rears up with greater ease.

The displacement of the bushing in the retracted position, makes the wheelchair with a trim “Prudent”.

## Adjusting the fork and front wheel

### Height adjustment of the front wheels:

is required in order to obtain the correct height at the front of the wheelchair.

- remove the screws (g), move the wheel at the holes on the fork
- insert the screws and tighten firmly

**Caution:**

The front height adjustment is necessary when you go to replace the front wheel with a wheel of a different size or when the rear wheel is adjusted or simply if you want to change the height of the front of the wheelchair.

## Quick release pins and wheels for narrow passages

A slight push on the button of the pin (h) allows you to extract the drive wheel quickly by reducing the width of the wheelchair. To replace the wheel it is sufficient to exert the same pressure until it clicks into place to lock.

The wheels for narrow passages are used to pass through small doors after removing the drive wheels.

## Transport by car

### To carrying the wheelchair in the car:

- remove padding seat
- raise the footrests
- pull the quick release pins (page 14)
- raise the central part of the fabric seat and close the wheelchair (Open / Close wheelchair page 9.)

To reassemble the wheelchair just follow all the steps in reverse order.

## Wheelchair maintenance

### Yearly checks

Maintenance should be performed once a year. It is highly recommended to check the state of the bearings, since sand, dust and water may damage them. Lightly lubricate with bicycle resin-free oil.

The frame may be cleaned with hot water and soap (or household detergent) only. Plastic parts should be cleaned with detergents specific for this material.

#### Caution!

Do not use high pressure cleaning equipment. Mechanical damage may be caused by water jets.

Chairs must not be cleaned with aggressive cleaning products.

#### Caution!

All components which come in direct contact with the skin must be replaced if the wheelchair is given to another user.

### Repairs

In case any repair is needed, contact your Nuova Blandino S.r.l. dealer, who knows how repairs should be performed according to in-house specifications and will be able to offer advice, support and provide repair services.

### Spare parts

Order through your dealer. In order to ensure the correct identification of the spare parts which must be replaced, indicate the wheelchair frame registration number. For safety reasons, use spare parts and accessories authorized by Nuova Blandino S.r.l. More information on how to retrieve the frame registration number are provided on page 26.

### Disposal

The wheelchair packaging materials may be recycled. The metal parts are collected and recycled in the “old metal” skip. The fabric and plastic parts may also be recycled.

Disposal must be performed in compliance with specific national legislation.

## Measures

cm 36 38 40 43 45 48


cm 54 56 58 61 63 66


cm 76

cm 102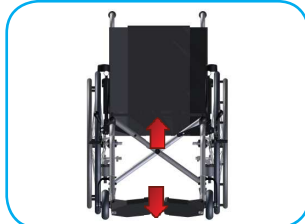


cm 51

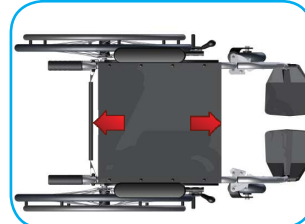cm 41

cm 41

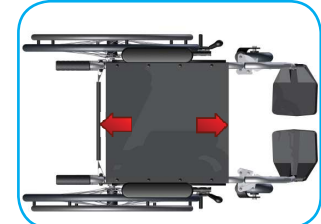optional cm 45

Kg 14,5


Max. Load Kg 120

## Optionals

Long armrest

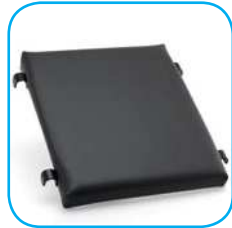Seat without hole

Seat with WC hole

headrest

Lateral support pad

Anatomic tray table

Rectangular table

Hemiplegic armrest

Strap cm 25

Elevating footrest

Simple belt

Big pelvic belt

Anti-tipping wheels

Tension adjustable backrest

Monorail system

## General safety rules

Use the highchair only according to the instructions of this use handbook.

To station, get on and off the wheelchair, make sure the ground is flat and stable and secure the wheelchair to prevent it from moving by activating both brakes.

If exposed to direct sunlight, the covering and the upholstery of the seats, the armrests, the legrests and the handles may reach temperatures over 41° C. Direct contact with unprotected skin may cause injury. In order to avoid the over-heating of said parts, exposing the parked wheelchair to direct sunlight is not recommended.

Please ask your dealer about all available setups, in order to avoid any danger or any change and adjustment which may affect wheelchair operation and safety of navigation.

The dealer will be happy to adjust your multifunctional highchair to suit your individual needs, by maximum respect for your safety.

When you turn the adjusters (eg. Footrest), be careful and do not ever put your hands in the orientation of the components.

There is a risk of injury.

When carrying objects, make sure that these do not affect or restrict the handling of highchair. Check the pressure and the wear of the tires and check the brakes.

## Portable weight

Do not overload the wheelchair and do not use it for carrying more than one person.

The recommended maximum load is reported in the handbook and on the highchair plate.

## Transfer from wheelchair

Getting off the highchair may be dangerous, if no assistance is available.

It requires:

- considerable strength
- good coordination abilities
- an anti-slip back surface (i.e. set the palms of your hands on the seat surface)
- much training

The knowledge of all of the possible dangers.

Let yourself be helped by a man then. Discuss with him the exercise and to do that may lead to dangers unexpected, for example any stumbling points, and explain the type of your handicap.

When the wheelchair is not charged, for example, when you move from the wheelchair to the bed, on the WC or car, do not ever force on the platform.

**- Danger of tipping!**

## Training

To ensure safe use of wheelchairs, both indoors and outdoors, you need a workout to gain confidence in the different main situations, such as pushing, braking, driving on the curve, reversing on flat surfaces possibly with the help of an attendant. Initially avoid steep hills, uneven roads or other similar situations.


## Take the objects

Do not bow down to an extreme, forward, sideways or backward to collect or place objects: you run the risk of tipping or loss of balance, especially in the case of the chair in which the seat is equipped with a narrow width and height high (pillow).

**Caution:**

- tipping in case of gripping objects forward
- loss of balance in the case where an object is taken from the side
- tipping in case you collect an object placed behind the back.

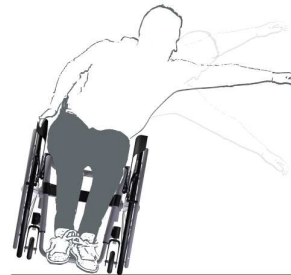
## Sidewalks

**Caution!**

Before you get off the pavement alone, weigh up the height. If it is too high, there is a risk of tipping over.

**Danger from tipping over in the event of a severe slope.**

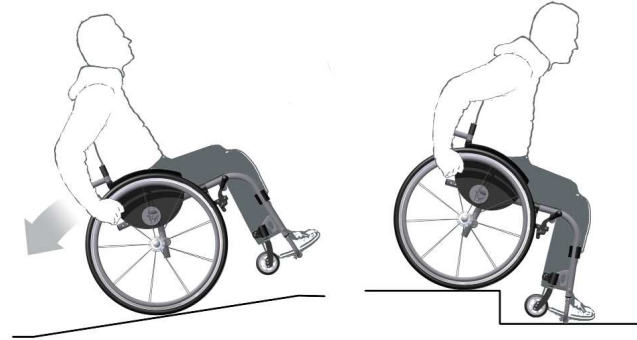## Ascent and descent - Stairs

To walk the climbs, tilt the trunk forward, for the descents, however, stay firmly on the back braking the wheelchair. In the event of a climb gradient greater than 1% is recommended to be pushed by a second person.

**Caution!**

In case of climbing, bringing bags and heavy bags at the back is dangerous. Danger of tipping.

We recommend, for your safety, you take advantage of the aid of two people to go up and down difficult stairs. To get off the wheelchair should be overturned on the large rear wheels and accompanied step by step. To go up, the person who helps you, turning his back to the ladder, must pull towards you the wheelchair from step to step by holding it upside down on the rear wheels.

**Caution!**

If the wheelchair is lifted weight there is the danger of overreach and fall.

## Crossing the narrow passages

When crossing narrow passages, such as the entrance of a building, a door, etc., pay attention to your elbows.

**Danger of injury!**

## Using elevators and lift bridges

Lifts and elevators pose a particular danger to the user.

**Caution!**

Automated doors may entail a risk of injury for your hands!

## Transportation by car or other means of transport

The wheelchair should never be used as a seat when you use a car or other means of transport. Provide the disassembly of the wheelchair (as specified in the instructions for use) and store it in the car, or at least secure it in ways to avoid accidents in case of dangerous maneuvers.

Take a proper seat inside the car or the means of transport and fasten your seatbelt.

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Responsible for designing, manufacturing, packaging, and labeling of this product, that falls within the category of  
**FOLDED LIGHTWEIGHT WHEELCHAIRS**

### **DOES GUARANTEE AND DECLARE**

The aforementioned device, launched and marketed as of the date certified in this certificate, fully complies with the provisions stated in annexes I and VII to EEC Directive 2007/47 and L.D. No 37 of 25 January 2010, that confirmed EU provisions at Italian level.

The products listed above are classified as Class 1 non-invasive medical devices as provided in Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product, the complete "EC" declaration of conformity is available on the website [www.nuovablandino.com](http://www.nuovablandino.com)

*The afore-mentioned products:*

- *are not measuring instruments*
- *are marketed in non-sterile packaging*

This is to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

Settimo, 2nd December 2015

## WARRANTY

Our folding lightweight wheelchairs are guaranteed for two years as of the date reported on the certificate of purchase (page 33).

The warranty covers any manufacturing flaw or malfunction on guaranteed products.

Damages caused by carelessness or improper use are excluded from the warranty. Authorized resellers will provide additional information and quotations if any repair is needed. For safety reasons, only spare parts or accessories authorized by Nuova Blandino S.r.l. should be used. The use of unauthorized accessories is a reason of immediate decline of warranty.

In order to guarantee it works properly, the highchair must be kept clean, in order and maintained as described in the use handbook.


**Warning:**

The company will not be liable for damages caused by the combined use of the highchair with parts supplied by other manufacturers (eg. trailer units, electrical equipment) that may result in significant danger - unless such parts have been expressly authorized in writing by Nuova Blandino S.r.l.

Nuova Blandino S.r.l. will not be liable for any damage to persons, things, animals, and any third party due to modification or misuse of the wheelchair.

## Plate and EC label

The 'EC label (see picture) on the wheelchair found in the central part of the cruise.

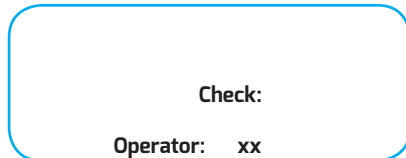## Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

## Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

## Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

## NOTES

---

---

---

---

---

---

---

---

---

---

---

---

---

---

**NOTES**

---

---

---

---

---

---

---

---

---

---

---

---

---

## Data sheet

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive.

- **Intended use**

The device is supplied ready for use by technical orthopedics and adjusted according to the user's needs.

The folding lightweight wheelchair is designed for both indoor and outdoor use. It is intended to accommodate one single person on the seat when the legrests and the lateral pads are mounted, and should not be used as a means of traction, transport or otherwise. Carefully read the instruction manual for a correct use of the device.

- **Check before use**

Before using this folding lightweight wheelchair, make sure that all parts are properly assembled and tightened, in particular the footrests.

For safety reasons, if visible flaws are detected on the product, all damaged parts must be replaced.

- **How to clean**

*Do not use high pressure cleaning devices! Mechanical damage may be caused by water jets.*

*Chairs must not be cleaned with aggressive cleaning products.*

- **Lifespan of the device**

This folding lightweight wheelchair may be used for a variable period of time, depending on the time and the frequency of use. A correct use and a proper maintenance of the wheelchair may extend its lifespan; on average, it is expected to be fully functional for 5-7 years.

- **-Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product. Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 2:00 HOURS AND FROM 13:30 TO 7:00 HOURS. CALL +390112625140.

**RETAILER STAMP**

--

**DATE OF PURCHASE**

--	--	--

**REGISTRATION DATE**

--	--	--

N.B.: THIS MANUAL MUST BE PRODUCED WHENEVER ASSISTANCE IS NEEDED

Nuova Blandino S.r.l.  
Strada Cebrosa, 24 - 10036 Settimo Torinese  
Torino Italy  
Tel. +39 011 2625140 - fax +39 011 2625141  
[www.nuovablandino.com](http://www.nuovablandino.com) - [inforb@nuovablandino.com](mailto:inforb@nuovablandino.com)

Commitment and technology for people with special needs