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INDEX

1. Introduction
2. Symbols
3. Components
4. Shipping
 - Assembling
 - Use specifications
 - Safety information
5. Electronic remote control
6. Buttons and indicators on the control panel
7. Reports by the command
8. Locking and unlocking the command
9. Checks before starting off
10. Reports of failure of the control
11. Charge display batteries
12. Driving
 - Run command
 - The braking
13. Maximum drive speed adjusting
14. Command position adjusting
15. Armrest adjusting
16. Backrest adjusting
17. Footrests adjusting
18. Electrical system
19. Batteries
20. Batteries replacement
21. Transport by car
22. Maintenance of the wheelchair
24. Wheelchair technical data
25. Measures
26. Accessories
27. General safety rules
 - High frequency radiation
28. Push mode
 - Tutorials guide
29. Start
 - Portable weight
 - Brakes
 - Emergency braking
30. Tires
31. Height adjustable footrest
 - Backrest reclining
 - Headrest
 - Transfer from the wheelchair
32. Crossing narrow passages
 - Driving uphill
33. Driving downhill
 - Overcoming obstacles
 - Use the elevator and lifts
34. Transport by car or other means
 - Notes on the electrical system
35. Batteries
36. Maintenance operations
37. Notes
38. Declaration of conformity
39. Warranty
40. Plate and CE label
41. Maintenance Log
43. Warranty certificate
44. Information note
45. Section for retailers

Instructions for use and maintenance

Introduction

The product here introduced, GR558, is the result of a twenty years study of our company, that wants to improve our design and production techniques. All this has been possible thanks to the help of orthopaedists and advises by experts of rehabilitation, education and assistance. This manual gives you the instruction for the use of this model. Please read it carefully before using the product for the first time.

The unique attention paid for the design and production of these products guarantees high security and comfort. Please use your wheelchair following the instructions of this manual.

Model GR558: electronic wheelchair with rigid frame for indoor and outdoor: indicated for patients with congenital or contracted pathologies, chronic or progressive, that reduce their movement ability, even though they are able to manage the electronic element for the driving actions in order to achieve the highest degree of autonomy and independence. This wheelchair is also for patients that spend a long time seated and for this reason need to change the trunk posture. It is also indicated for very disjointed grounds.

Base equipment: LED direction indicator; available in several measures and colours, equipped with electric tilting, mechanically reclining backrest and adaptable to make an easy transport; adjustable seat in depth; removable side panels and armrests with adjustable height and width, removable arm for joystick, swivel footrests, removable, adjustable in height and depth, stopping heel belt and security belt.

Optionals: : you can personalise your wheelchair with accessories that fulfil comfort and security needs: selection of headrests, aluminium foldable or firm backrest, ideal for ergonomic backrests and multifunctional footrest even in entire version.

Motorization: two magnet motors 24Volt for 180Watt, electromagnetic security brake, speed regulator, two batteries for 12Volt, 42Ampere singularly removable.

Performances:

- maximum speed of the device 15 km / h ; **please note that the device can be used (according to the Highway Code) on the sidewalks with equal speed and no greater than 6 km / h ;**
- maximum slope 20%;
- maximum capacity 120kg.

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Simbologia

Often, when reading the user manual, you may encounter these symbols of particular importance or interest.

It indicates to pay particular attention to a danger in general

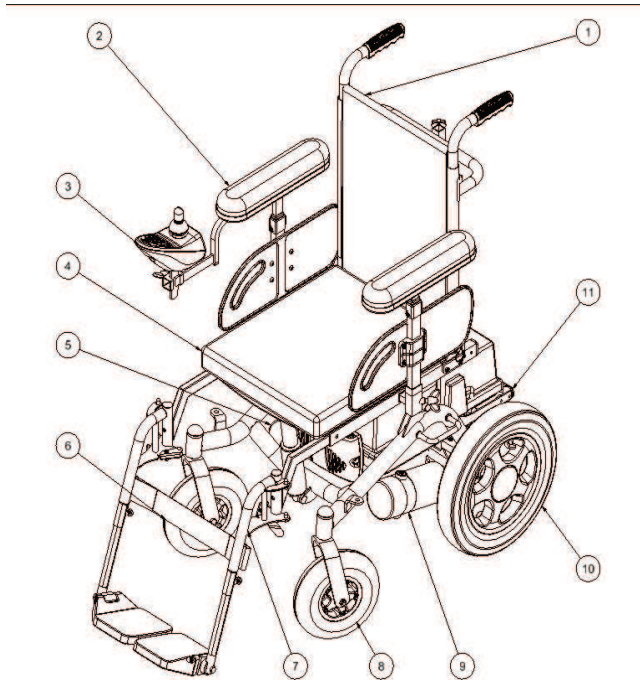
Wrench for bolts 6 / 19mm.

Allen key for screws with hexagon socket
6

Components

1. Back cloth
2. Removable and adjustable armrest height
3. Remote control
4. Seat cloth
5. Left cruise
6. Right engine
7. Leg-end
8. Removable rotating platform
9. Front wheel Ø 200x50
10. Right cruise
11. Left engine
12. Battery compartment door
13. Rear wheel Ø 300x55

Delivery

Every product of Nuova Blandino is checked out in the factory in order to avoid any blemish, then they are packed in special boxes and placed in wooden platforms.

For this reason we ask you to check your wheelchair immediately after receiving it and verify any possible damages happened during its transport. If it is possible do it with the presence of the deliverer.

In case you find any damage caused by the transport, it must be immediately reported in the delivery seal.

Then you need to send it Carriage Free to Nuova Blandino S.r.l - in 8 days after the delivery date – the damaged parts if they are removable, or the whole chair, with a copy of the delivery seal.

The identification of the damage can not happen if you do not follow this process or do not signal the damage after receiving the wheelchair.

Assembling

The orthopaedic specialised technician will deliver your wheelchair ready for being used and suitable for your personal needs.

Using details

The electronic wheelchair is designed to be used indoor and outdoor. It has the only purpose to transport a person on the seat with the footrest and the side parts assembled. It does not work as means of traction,

transportation or others.

Information and security

Every information for a secure use of this wheelchair are reported in the second part of this handbook.

Electronic remote control

The Shark dk-remd has been designed with the aim to guarantee the security of its user. It has got a lot of self-controlling sophisticated features, that look for possible problems with a frequency of 100 times per second. If the control system finds a problem, in its circuit or in the electronic system, it can stop the chair according to the gravity of the problem.

The remote control is designed to optimize the security of its user. In every normal condition. Although its sophisticated features, the Shark Dk-remd does not consider situations that put the chair or its controller out of the specified operative conditions. For this reason it is important that the user follows these instructions.

Never drive the wheelchair:

- a) Beyond its indicated limits, for example maximum slope, height of the sidewalk ecc.
- b) On ground where it is possible to lose the control of the wheels, for example on grassy or wet slopes.
- c) If the controller or other components are waiting to be repaired.

If the chair moves in an unexpected way, leave the joystick.

These actions will stop the chair in any circumstances.

Even though the remote control is designed and produced to be extremely reliable and every component is severely tested, it always exists the possibility of a malfunctioning system (even if it is very rare).

In some conditions of the malfunctioning system, its controller must immediately stop the chair (for security reasons).

If the physics obstacle of the user are such that a sudden braking could cause a fall from the chair, it is compulsory to install some containment system, like a security belt, that must be used every time the chair is moving.

In difficult driving conditions it is possible that the temperature of the under part and the extremity of the Shark Dk-remd goes over 41°C. the user must not touch these parts in these situations.

Buttons and indicators on the control panel

- **ON/OFF Button:**

- This button turns on or off the SHARK DK-REMD. Do not use this key to stop the chair, except in case of emergency.

- **Battery charge indication:** This display indicates whether the SHARK DK-REMD is turned on and shows the state of charge of the batteries. In addition, means any failure of the electronic system of the chair.

- **Speedometer:** This display provides guidance on the speed that you are holding.

- **Attendant control LED:** This LED remains on when you insert the attendant command

- **Horn:** This button is used to activate the buzzer.

- **Lights:** This button is used to switch on the lights.

- **Left Indicator LED:** This button is used to actuate the left Indicator LED.

- **Right Indicator LED:** This button is used to actuate the right Indicator LED.

- **Emergency:** This button is used to activate the emergency lights.

- **Speed control:** These two buttons allow you to adjust your speed.

- **Tilting:** This button is used to operate the tilting.
• Select the 1 button and move the joystick lever to tilt the seat back, forward to returning to the horizontal position.
• Once you have found the best position, press the 2 button to return to the drive mode.

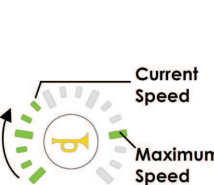
Joystick lever

- This component controls the speed and direction of the chair. Press the joystick in the direction in which you want to go. The more you press the lever, the higher the speed is.
• Releasing the joystick the chair will stop and the brakes are activated automatically.

Reports command SHARK

Display	Description	It means that..	Note
	All lights are off	SHARK is off	If the on / off button is pressed while the chair is in motion, the chair will perform an emergency stop.
	All the lights are on fixed	SHARK is on	The lights will turn on one by one.
	After switching on only a few lights will stay on	Indicates the level of battery charge	Green light: Full charge Yellow light: Weak charge Red light: Low battery
	Lights up to the mark of the maximum speed.	Indicates the maximum speed set with the cruise control buttons	It can be set with the buttons to adjust the speed
	Only a few lights on and notch the top speed	Indicates the current speed and the maximum speed that can be set to achieve	If the green LED on the bottom left is flashing, the SHARK is in speed limit. It happens when the chair is in an unstable position and a driving too fast could be dangerous, for example when the seat is raised or tilted.

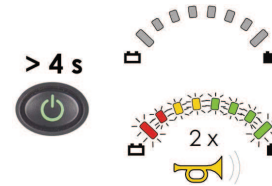
Locking and unlocking of the command SHARK

If the Lock Enable parameter is set to 'Yes', the SHARK can be locked by pressing the on/off button for more than 4 seconds. Lock the SHARK to prevent unauthorized persons from operating the chair.

To lock the SHARK

The SHARK turns off immediately when the on/off button is pushed.


After 4 seconds all LEDs of the battery gauge will flash twice and the horn will give two short beeps, to indicate that the SHARK is now locked.

To unlock the SHARK

1. Press the on/off button, The SHARK will turn on, and the LEDs of the battery gauge will slowly flash from right to left to indicate that the SHARK is locked.
2. Press the horn button twice within 10 seconds to unlock.

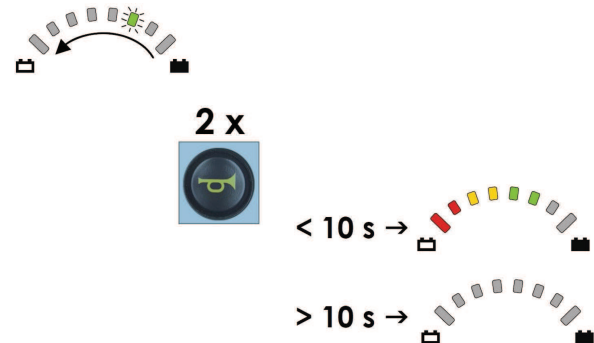NB: If the horn button is not pressed within 10 seconds, the SHARK will turn off again.

Check before starting

After switching on the electronic control SHARK (press the ON / OFF button) starts a self-control system.

After this self-analysis none of the 10 segments of the charge level of the batteries should be flashing.

If any segment is flashing, this indicates a possible failure, which will be marked by a series of short flashes followed by a pause.

Please refer to page 9 for a description of the type of fault reported.

If the type of fault is a serious electronic control Shark prevents the guidance of the chair.

For any type of failure not described on the next page contact support.

Battery charge condition

After switching on the segments of the battery gauge showing the charging status of the same.

Refer to page 10 to get the correct interpretation of the light signal that expresses battery charge level.

It is recommended to test at least once the autonomy of the wheelchair, or the batteries.

Adjust the maximum drive speed

The maximum final speed can be selected using the appropriate buttons on the control panel of the SHARK.

It is adjusted according to the characteristics of the path that must be addressed, so as to ensure maximum safety during driving.

To set the maximum final speed read carefully the instructions on page 12.

Caution: In case of uphill or downhill, the speed must be adapted and reduced according to the slope.

Caution: Danger of accidents in case of wrong selection of the final maximum speed!

You should choose a reduced maximum speed in case of situations that generate insecurity (for example when driving in extremely cramped, starting uphill).

The Fault Indicator - Flash Codes

Flash Code	Fault source	Meaning
1	Route / Temperature	<p>The motor current has been at the maximum value for too long:</p> <ul style="list-style-type: none"> • The motors may not be strong enough for the chosen route. Turn off the SHARK, let it cool down, then turn it back on again and choose another route. • The wheels may be rubbing on the frame. Make sure that the wheels can turn freely. • The motors may be faulty. Have the motor(s) checked by a service technician. • The SHARK Power Module is too hot; Wait a few minutes and try again. If this happens often, contact an authorized technician.
2	Battery	<p>The battery voltage is too low or too high:</p> <ul style="list-style-type: none"> • Check the batteries and the cables. • Batteries may be empty: charge the batteries. • Batteries may be overcharged: if driving downhill, slow down or turn on the lights, if fitted. • Batteries may be damaged: contact your dealer. <p>If this fault occurs during battery charging, the battery charger is defective or not adjusted correctly.</p>
3	Motor 1 (usually left)	<p>The motor is not connected to the SHARK, or there is a short-circuit in the motor connection:</p> <ul style="list-style-type: none"> • The motor brushes may have lost connection. Turn the wheels of the chair to reconnect the motor brushes, and then turn the SHARK off and on. If this happens often, the motors may be faulty. • Check that the motor cables are not loose or damaged. • Contact your dealer.
4	Motor 2 (usually right)	
5	Parkbrake 1 (usually left)	<p>The parkbrake has been released manually.</p> <ul style="list-style-type: none"> • Enable the parkbrake, and then turn the SHARK off and on. <p>The parkbrake is not connected to the SHARK, or there is a short-circuit in the parkbrake connection.</p> <ul style="list-style-type: none"> • Check that the motor cables are not loose or damaged. • Contact your dealer.
6	Parkbrake 2 Right	
9	Missing Power Module	<p>A communication error between the Remote and the Power Module.</p> <ul style="list-style-type: none"> • Batteries may be completely empty: charge the batteries. • Batteries may be damaged: contact your dealer. • Check that the SHARK Bus cable is not loose or damaged.

Charge display batteries

Battery Gauge	Meaning
	<p>Flash Code 2 fault: battery too full. Slow down if you are driving down a slope. Turn on the lights, if fitted.</p>
	<p>Battery full.</p>
	<p>Battery half full. Start the return journey.</p>
	<p>Battery low. Recharge soon.</p>
	<p>Battery almost empty. Recharge now.</p>
	<p>Flash Code 2 fault: battery completely empty. Battery damage will occur if you keep on driving. Recharge immediately.</p>

Driving

Use particular caution during the first movement. Select the maximum final speed.

Caution: Observe the safety instructions. While driving, decide for yourself the speed and direction of travel by activating the joystick lever, you can also determine the final set maximum speed of your wheelchair.

Run command

The command is given by the joystick lever and it is simple and logical. You will in a short time able to maneuver safely in your own wheelchair.

The control essential and most important is the joystick on the control group.

It serves to impart the run command, or directional guidance and, if necessary, to braking.

The electronic system comprises a delay circuit which serves as a limiter of acceleration, always guaranteeing a starting extremely cautious.

Even if the joystick is moved quickly in the desired direction of travel, the wheelchair will start in a very gradual mode.

If the joystick is left free it immediately returns to the neutral position and excludes the motor current.

The breaking

If the joystick comes back to the neutral position the braking resistors ensure a gradual stop.

How to overcome the obstacles: Before you go up or down from one edge of a sidewalk, make sure that the height is not more than 6 cm.

Caution: sidewalks must always be addressed in a straight line and never at an angle. Overcoming an obstacle incorrectly implies the danger of tipping over sideways.

To get on the sidewalk: as close as possible with the front wheels, then increase the speed and reduce only when the rear wheels have passed the obstacle entirely.

To get off the sidewalk: reduce speed and keep it so until the rear wheels have not passed the obstacle entirely.

Maximum drive speed adjusting

Using the Shark control you can set the maximum speed.

This control option is very useful and in some cases essential when dealing with rough terrain that can endanger your safety (eg. Steep slopes, roads with slippery or uneven).

Warning: weigh up the path you are about to face, and set the maximum speed from your wheelchair accordingly.

Button	Description	Light gauge
 Slower	<ul style="list-style-type: none">• Press and release the 'slower' button to decrease the maximum speed of the chair by 20 %.• Press and hold the 'slower' button to decrease the maximum speed of the chair in fine steps*.	
 Faster	<ul style="list-style-type: none">• Press and release the 'faster' button to increase the maximum speed of the chair by 20 %.• Press and hold the 'faster' button to increase the maximum speed of the chair in fine steps*.	

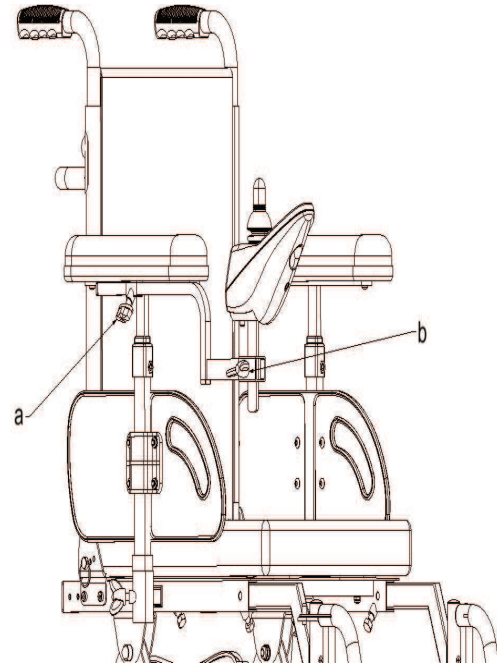
Settings

Command adjusting

- Loosen the handwheel (a).
- Move forward or backward the rod support.
- Find the best position for the hand.
- Tighten the handwheel (a).

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
- Loosen the handwheel (b).
- Move up or down the rod support.
- Find the best position for the hand.
- Tighten the handwheel (b).

Armrest adjusting


Height adjusting

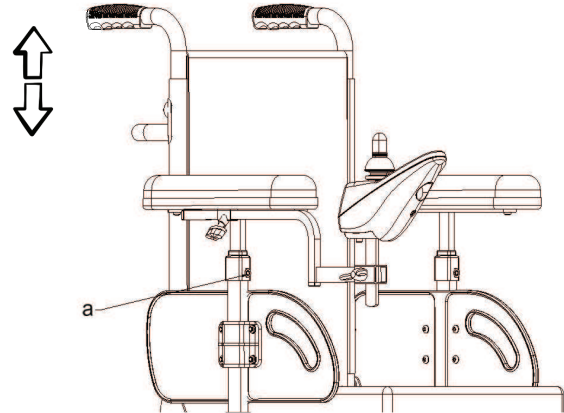- Loosen the handwheel (a)
- Adjust the height of the armrest
- Find the best position
- Tighten the handwheel (a)

Width adjusting

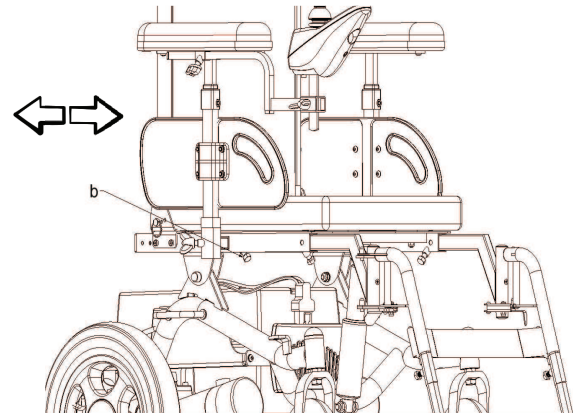- Loosen the bolt (hex head screw) (b)
- Adjusting the width of the arm support
- Find the best position
- Tighten the bolt (hex head screw) (b)

Backrest adjusting

The backrest of the wheelchair can be adjusted forward and backward, relative to the seat, in order to find the best position posture.

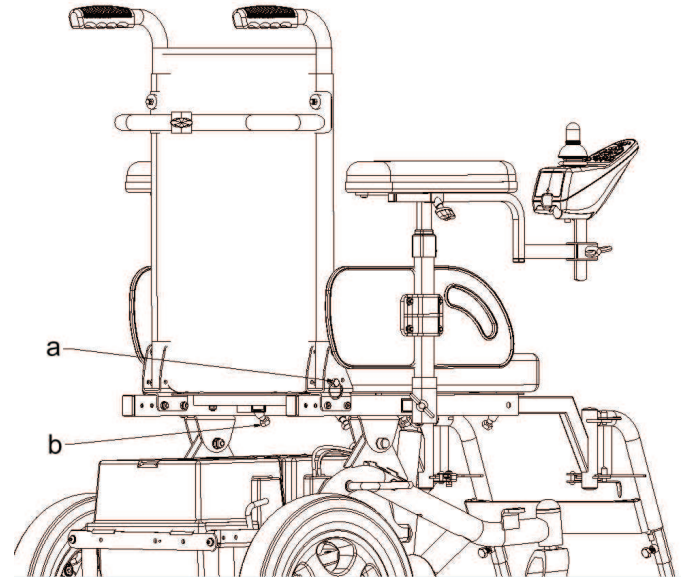
The backrest can be folded forward by reducing the foot-print for easier transport.

Tilt adjustment

- remove the pin (a) from both sides
- tilt the backrest to the desired position
- insert pin (a) on both sides

Rollover back

- unscrew bolts and screws (b) from both sides
- remove bolts and screw (b) from both sides
- move forward or backward the backrest into the desired position
- insert bolts and screws (b) from both sides
- tighten bolts and screw (b) from both sides

Footrest adjusting

The footrests are removable and can be adjusted in height

The footrest can be folded down to facilitate the person approach to the wheelchair.

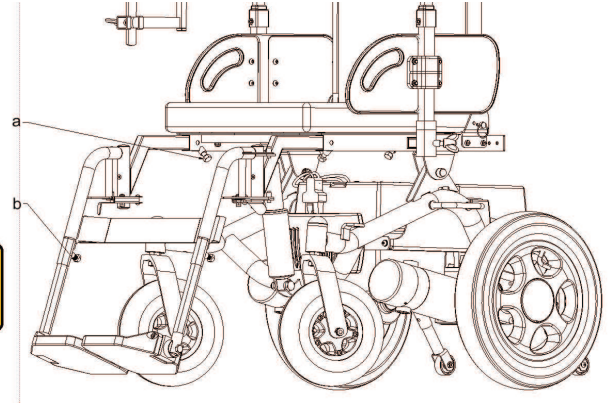
Depth adjustment

- Loosen the screw (a) and adjusted according to the desired depth.
- Tighten the screw (a)

Adjusting the footrest height

- Unscrew and remove the bolt (b)
- Adjust the rod to the desired length by using one of the holes on the dipstick platform.
- Replace the bolt (b) in the corresponding hole and tighten securely

Caution: Make sure that the pin of the platform is inserted at least 5 cm inside the support platform. Adjust the fore-end platform in order to always have a gap between the plastic platform and the ground about 6 cm.

Disassembly / assembly platform

- Press the lock lever shown in FIG.1
- Turn out the platform and pull upward to release it completely FIG. 2
- To replace the platforms perform the above operations in reverse order.


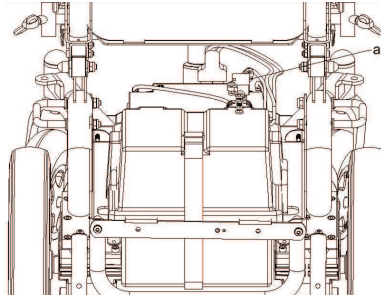
FIG.1

FIG.2

Electrical system

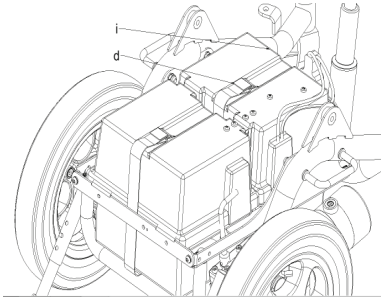Fuse

The electrical system of the power wheelchair is protected against power surges by a fuse type BF1 60 A, which is located inside the battery compartment (a).

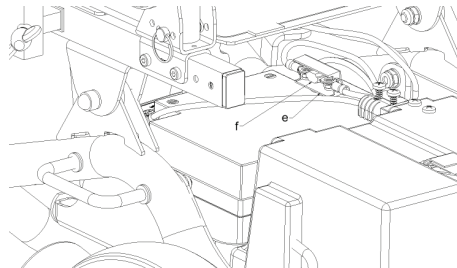
It is advisable to replace the fuse of the dealer or by qualified personnel from Nuova Blandino.

If you are not able to replace the fuse, proceed as follows:

- Remove the seat bottom frame by unscrewing the connecting screws / joint (b) and the screw on the actuator (c)
- Loosen the belt latch of the battery compartment (d)
- Lift the top cover of the battery compartment
- Open the fuse (s)
- Replace the fuse (f)
- Close the fuse holder

Then proceeds in reverse order from step 3 to step 1.

Batteries

To make the batteries charging, insert the charging plug into the socket; the device battery charger must be connected to the mains.

The poles of the batteries must be kept clean in order to avoid the danger of oxidation that would limit the electrical conductivity.

The batteries supplied by us are maintenance.

It 'should always put the batteries in office during the period of extended downtime. Subjecting them to a cycle of charging at least every seven days.

Always check the charge of the control panel display, charge the batteries when the display shows a state of charge is low.

How to charge the batteries

- Plug the charger into the socket under the electronic control
- Plug the charger into the outlet.

Completed the charging:

- Unplug the charger plug into the socket
- Disconnect the plug from the electronic control.

Replace the old batteries

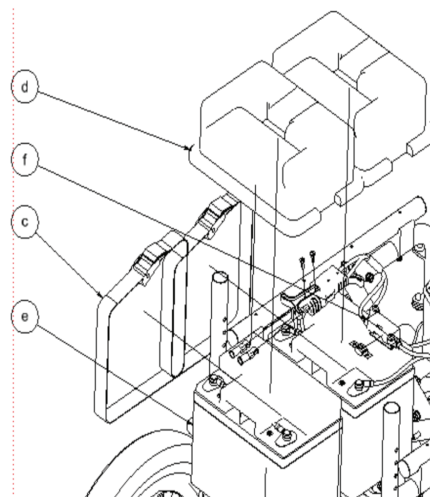
It is advisable to have the battery replaced by a specialized dealer or by qualified personnel from Nuova Blandino.

If you are not able to replace the battery, do the following:

1. Remove the seat bottom frame by unscrewing the connecting screws / joint and the screw on the actuator.
2. Loosen the belt latch of the battery compartment (c).
3. Lift the top cover of the battery compartment (d).
4. Unscrew the cable clamps on the negative pole of both batteries (e).
5. Unscrew the cable clamps positive pole of both batteries (e).
6. Raise the bridge that supports the connectors and fuse holder (f).
7. Pull the battery from the battery case.

Install new batteries by following the above steps in reverse.

N.B. Use only batteries approved or comply with those mounted by the manufacturer of your wheelchair! Any damage caused by not complying batteries will not be imputed to the company.


Transport by car

To transport the wheelchair by car:

1. Remove the seat cushion.
2. Remove the footrest (see p.17).
3. Remove the electronic remote control (see p. 14).
4. Pull off the arm (see p. 15).
5. Breaking down the back (see p. 16).
6. Transfer the wheelchair in the car
7. Make sure that the engines are placed to block the wheels turning the lever (a)
8. Secure the wheelchair with the belts in the front of the frame.

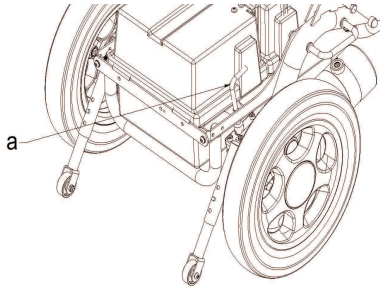To remount the wheelchair must follow all the steps in reverse order.

Maintenance of the wheelchair

Avoid shocks to the system command Shark Dk-Remd, especially the joystick. During the transportation of the chair, make sure that the control system SHARK DK-REMD is well protected. To prolong the life of the control system SHARK DK-REMD, expose as little as possible in extreme conditions.

Always clean the system command SHARK DK-REMD when it is soiled by food or drink. Use a damp cloth. Do not use abrasive cleaners or alcohol-based.

Daily checks

Joystick: with the control system SHARK DK-REMD off, check the joystick is not bent or damaged and that it returns to the center position when released.

If there is a problem not using the chair and contact the service center.

Weekly checks

Electromagnetic brakes: This check should be done on a level surface with at least one meter of free space around the chair.

Turn the system control SHARK DK-REMD. After a second check if the battery level is stable or the lights flash slowly. Move the joystick forward until you hear a click indicating that the operation of the electronic brakes. Caution: the chair could begin to move. Immediately release the joystick; you should hear each brake operate within a few seconds.

Repeat this test three times, pushing the joystick respectively backward, left and right.

Connectors: verify that all connectors are properly connected and free from any damage.

Cables: check the condition of the cables, which are devoid of any damage.

Bellows joystick: make sure that the fine rubber boot around the base of the joystick is not damaged or cracked. Check visually.

Assembly: make sure that the control system SHARK DK-REMD is securely fastened to the chair. Do not over-tighten the screws.

Semiannual checks

Every six months, the wheelchair must be inspected by an authorized dealer, which will provide a general assessment of the state of the wheelchair and, if necessary, send it to the Nuova Blandino S.r.l. for any repairs or tune-ups.

Removal of faults

When used properly and treated with care, the electric wheelchair will always work perfectly. However, if you notice any faults you can keep these tips that we expose below and which will serve to identify the cause of the fault and to bring it to possibly remedy.

CONNECTING THE MOTOR MUST BE PERFECT

Make sure that the plug connection on the back of the command SHARK DK-REMD is properly inserted.

ON THE GROUND FLOOR THE CHAIR IS INTENDED TO BENDING TO ONE SIDE

In the event that the wheelchair along a flat terrain and with the "joystick" in the straight ahead position, showed a tendency to curve to the right or to the left, make sure that the pressure in all four tires have the prescribed value of 2.5 bar 3.5 bar front and rear.

Warning: failure to observe the obligation of the half-yearly checks during the guarantee period of the wheelchair will void the warranty.

NOTE:

In case of failure to control SHARK DK-REMD with the engines or the unit to charge the batteries, we offer our customers, as part of our service, to repair the faulty part that will perform after receiving through our retailer of the defective unit.


We ask our customers not to try to carry out repairs to the groups mentioned mechanical and electrical; the strict control of the operation of them can only be carried out by our specialists with the help of the special instrument.

The undue opening of the closures of sealed electronic groups leads to loss of warranty on them.


Technical specifications of the wheelchair

Electrical system	24 V DC
Motorization	2 permanent magnet motors by 180 Watt
Control system	integral remote control
Batteries	2 batteries by 12V to 55 Ah
Viable slope.....	Up to 12% and short stretches up to 20%
Speed... ..	15 Km/h max
Front tires	Ø200x50
Rear tires	Ø300x55
Tire pressure	2,5 BAR (Front) · 3,5 BAR (Rear)
Max load.....	120 Kg

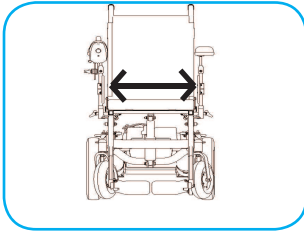
Measures


cm 40 - 45

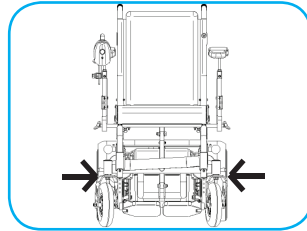
cm 59 - 64

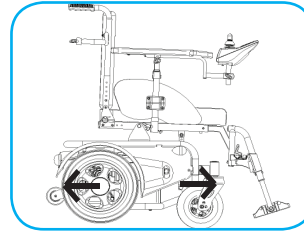
cm 70

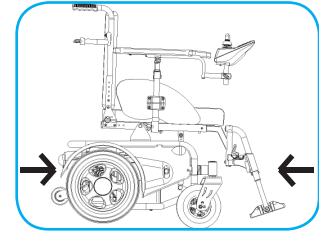
cm 106

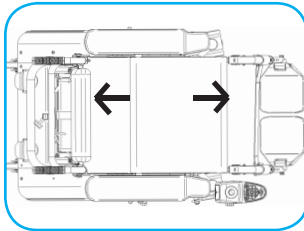
cm 41 ÷ 56

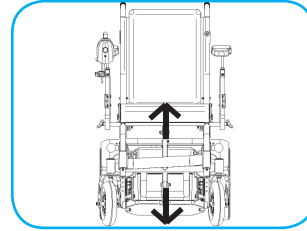cm 52

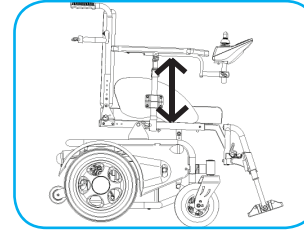
cm 18 ÷ 28

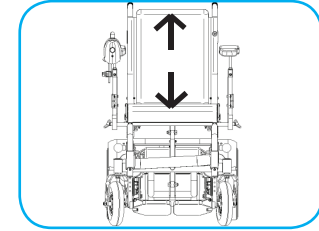cm 41

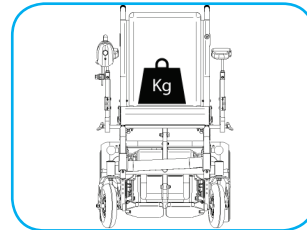kg 105

Max load 120kg

OPTIONALS

headrest

“y” belt

safety belt

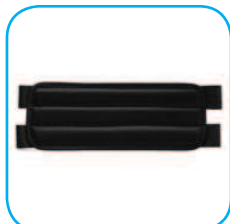
large pelvic belt

simple belt

belt abductor

big belt legrests

single footboard

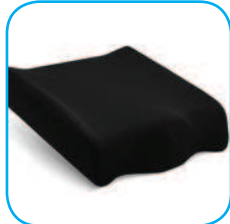
single bib

shaped backrest

rigid aluminum backrest

shaped seat

lateral support pad
26

table

General safety rules

Use the wheelchair only as described in the instructions for use.

To stop going up or down always choose a level and stable ground. Secure the wheelchair to prevent it from moving. If you leave the wheelchair unattended, you should make sure that it can not be used without permission.

Do not switch off the wheelchair while driving otherwise it will stop immediately.

If exposed to direct sunlight, the coating and the filling of the seats and armrests, leg rests and the handles can reach temperatures of over 41 ° C. Touching these parts with unprotected skin there is a risk of burns. To avoid the heat of these parts of your wheelchair, park it in the shade.

Please inquire at your authorized dealer of all possible driving characteristics of the vehicle specification, in order to rule out possible dangers and possibilities of variation and control of the vehicle and their effect on the performance and driving safety. The dealer will be happy to adjust the wheelchair according to the individual needs of the user, the utmost respect for safety.

When you turn the adjusters (eg. those of the back or leg), take care and do not ever put your hands in the orientation of the components. There is a risk of injury.

When carrying objects, make sure that these do not affect or restrict the maneuverability of the wheelchair.

Check the condition and pressure of your tires and check the brakes.

High frequency radiation

Our electric wheelchairs comply with the requirements of Directive 2007/47 EEC for electromagnetic compatibility. It is not, however, exclude the possibility of interference due to high-frequency radiation of other electronic devices. In proximity of radar equipment and radio transmission of each type and cell phones may present high frequency radiation. In the event of a failure, if the wheelchair reacts in an uncontrolled manner or if disturb other electronic devices, immediately stop the wheelchair and turn off. When using the electric wheelchair mobile phones should be switched off. A cell phone, in fact, also transmits when it is in "stand by". And do not make a phone call. Turn off the electric wheelchair before operating the phone.

In principle, do not operate our electric wheelchairs in the immediate proximity of medical equipment with high potential risk and / or with vital functions and diagnostic equipment.

Over fields of strong electrical noise for example from welding equipment or radio or television transmitters, you may see some disturbance of function in spite of the electrical components of the vehicle have been sufficiently shielded.

These disorders result in abnormal behavior. In such situations of disorder stop the wheelchair and turn it off.

Push mode

Use the wheelchair by push with the engine disconnected only on level ground and with the help of an attendant.

Warning: at the end of the push mode do not forget to put the lever in the operating position of travel.

Tutorials guide

A safely use of the wheelchair is needed for training the main driving situations, such as:

- start
- drive
- brake

For the first exercises of driving, use a low speed.

Familiarize yourself with the behavior of your wheelchair with electric drive and try gradually to know your limits

In certain driving situations for example uphill, downhill, or when circumvented an obstacle always take advantage of the aid of another person that you will be of help.

In the face of cliffs, stairs and other obstacles always keep a safe distance that you have enough space to brake, react and turn the wheelchair.

On wet, slippery or sloped funds the distance for braking increases.

Avoid sudden movements while driving when you are in steps of a climb or descent and a horizontal level, or when circumvented the obstacles or curves.

Always slow down before making a turn. The more close the curve will be the more reduced the speed must be. Do not lean out over the curve.

Start

In all electric wheelchairs you can pre-select the maximum speed in the command SHARK. Take heed to this preset to not be caught by surprise when, while the speed to maximum, the wheelchair starts quickly without meaning to.

When you push the lever joystick switches off the magnetic brake safety; wheelchair starts to move only after this operation.

Among the brake release and the movement of the vehicle can be, according to the set maximum speed, a phase of "idling", in which the wheelchair can climb back you are in or are overcoming some obstacle. Therefore, quickly push the lever steering and driving forward to overcome this phase.

Portable weight

Do not overload the wheelchair with heavy weights and never use it to carry more than one person.

The maximum permissible load is given in the instructions for use and on the type of the vehicle.

Brakes

Our models are equipped with electromagnetic brake suitable for the intended use. It would be appropriate to always check the operation before setting in motion: the wheelchair is stationary.

Insert the push mode and try to push the wheelchair: the brake works correctly if it is impossible to move the wheelchair.

Emergency braking

Emergency braking is done by releasing the joystick lever. The sudden arrest of the electric wheelchair caused by emergency braking can push forward the user and make him fall out of the vehicle, depending on the clinical picture.

You can prevent it putting a seat belt.

Warning: If you suffer for example of the osteogenesis imperfecta or a similar disease, the risk of injury during an emergency braking is high. Therefore proceed very cautiously to avoid such a situation of risk.

Tires

The tires are made with a mixture of rubber which, according to the type of floor, can leave permanent traces or otherwise difficult to remove (eg On floors of plastic, of wood, boxes, rugs or carpets).

Inflation pressure of tires:

Regularly check the inflation pressure of your tires.

The tire pressure is shown on the tires themselves and in the section "Technical data" of the Instructions for Use.

A lower tire pressure increases tire wear and reduces the performance for each battery charge, ride comfort, driving safety and the life of the tires.

If the pressure is too high, the front wheels tend to oscillate.

Practical tip:

If there is not a suitable pressure gauge or there is no technical possibility to use it, check the inflation pressure of the tire by pressing your thumb against it. In this case, the tire must yield very little.

Another method to evaluate the tire pressure is to sit in a wheelchair and observe the crushing of the tires at the point of ground contact / rubber.

If the camber of the tire is strong, you have to inflate the tire. If the camber is only just visible it can be assumed that the inflation pressure is sufficient.

Condition of tires:

The tire wear is a natural process. A tread very worn and damaged tires affect driving safety; in these cases, therefore, it is necessary to replace the tire as soon as possible by an authorized workshop.

Warning:

Replace the tires always in pairs, because tires with a different degree of wear do not permit a straight line travel of the vehicle.

Height adjustable footrest

In wheelchairs equipped with height adjustable leg rest you need to consider that when the leg is adjusted in a horizontal position, it produces a forward displacement of the center of gravity of the body; it affects the stability and the ability to brake and steer the wheelchair.

Warning: bring down the leg operated with caution the seat to prevent abrupt lowering of leg weight load. Risk of injury.

Keep the first leg and then operated lever click. Take down the leg slowly.

Reclining backrest

A backrest tilted back very significantly reduces the stability of the wheelchair.

Warning: before facing an uphill before climbing an obstacle and you always put the seat back in the upright position

Danger of tipping!

Headrest

In wheelchairs with adjustable headrest, the upper edge of the head restraint should always be roughly at eye level. Do not place it at the nape.

Transfer from the wheelchair

Disembarking from a wheelchair to move to another place, if done independently, it is dangerous.

require:

- relevant physical forces
- good coordination skills

- a non-slip surface (eg With the palms of your hands on the seat surface)
 - much exercise
 - knowledge of every possible danger situations
- Let yourself be helped by a man then discuss with the exercise and keep in mind that there may be unforeseen hazards such as:

any stumbling points, or explain the type of your handicap.

To get on and off by the wheelchair does not rely ever to command SHARK. Unintentional operation of control elements may cause an accident.

When the wheelchair is not charged, for example, when you move by the wheelchair to the bed, on the toilet or a car, do not ever force on the platform. - Danger of tipping!

Crossing narrow passages

When you travel with traits such as narrow-track, for example, building entrances, doors, etc., Attention to elbows. - Risk of injury.

Driving uphill

When you approach a hill, the vehicle stability is greatly reduced.

Warning: in the uphill or downhill does not lead never release lever brakes in place of manual operation (by push). The wheelchair would start to slide out of control in the direction of the slope.

Observe the maximum permissible slope for wheelchair included in the Instructions for use.

Warning:

The footrests are in the area of the legs, so you can stumble. For this reason, the pegs are to be folded up so that there is free space in the foot area.

Traveling downhill

Never drive faster than walking speed. It is necessary to maintain control of the wheelchair at all times of the descent.

Warning: Do not ever bring the release lever brakes in place of manual operation (by push).

Overcoming obstacles

Overcoming obstacles always involves a risk. Using ramps can reduce this risk.

Overcoming barriers is a particularly dangerous situation which must be respected in a combination of safety information already recorded in the chapters on driving uphill and downhill.

We expressly declare that the ability to overcome the obstacles of this wheelchair is limited.

Before you go up or down from one edge of a sidewalk, make sure that the height is not more than 6 cm.

Warning: the sidewalks need to be addressed in a straight line forever and never at an angle. Overcoming an obstacle incorrectly implies the danger of tipping over sideways.

Use the elevator and lifts

Lifts and elevators are a particular danger for the user.

Warning: the doors that close automatically involve the risk of causing injury to your hands! If there is a risk of a fall, take the elevator to the wheelchair push, with the help of an attendant.

See that the pegs do not touch the ground when the wheelchair has reached the end of the slope and that the immediate crash of the wheelchair does not put you in danger.

To get on the sidewalk, come as close as possible with the front wheels, then increase the speed and reduce only when the rear wheels have passed the obstacle entirely.

To step off the curb, reduce your speed and maintain it until the rear wheels did not pass the obstacle entirely.

Warning: in case of a collision with an obstacle, check all visible cables and check whether they were also eventually went for damage to your wheelchair.

Secure the wheelchair so that it will not move even if unintentional actuation of the lever joystick per the user.

Transport by car or other means

The wheelchair should never be used as a seat when you use a car or other means of transport.

Provide the disassembly of the wheelchair (as specified in the instructions for use) and store it in the boot, or at least secure it to prevent accidents in case of dangerous

maneuvers. Occupied a suitable place inside the car or the means of transport and fasten your seatbelts.

Information for the electrical system

If the wheelchair reacts unwanted or uncontrolled performs maneuvers, immediately take the joystick lever in central position and immediately turn off the SHARK.

The changing of the driving behavior of a vehicle electrical system must be carried out only with the proper tools and qualified personnel.

After each change of the driving behavior it is necessary to verify the safety of the wheelchair in all driving conditions and braking maneuvers.

Caution: Incorrectly editing and / or not appropriate to the driving behavior may affect the user's safety.

We accept no responsibility for any accidents caused by incorrect changes or not appropriate behavior of running.

Do not make any modifications to the vehicle control SHARK.

Any modifications, including the use of unauthorized spare parts by Nuova Blandino S.r.l., may affect the safety of the command SHARK and the wheelchair Nuova Blandino S.r.l. does not provide any warranty for products modified or liable for any damage and / or injury resulting from unauthorized modifications.

Warning: we do not assume any responsibility for damages due to the combination of our wheelchairs with third party products of any type (eg Trailers, electrical and electronic equipment), which under certain circumstances can be extremely dangerous, except in the case where the product has been expressly authorized by Nuova Blandino S.r.l.

Batteries

Over time, the batteries discharge even when not used. For this reason always an indication of mind control battery charger before starting the march. Never stand in motion with weak batteries (control via the level indicator). This increases the risks of emergency situations.

Charge the battery:

The charge of the batteries is simple and easy to perform. No need for tools and no disassembly required.

Follow warnings in the instructions on the use of electric wheelchair battery charger and the device.

Since the device battery charger is under voltage, uncharged battery packs in damp places. - Danger of short circuit and death!

Take care during loading to a good ventilation of the unit charge batteries (not cover it) to dissipate the heat that is being formed - Risk of Fire!

Warning: Our models utilize absorbed acid batteries, which do not require maintenance to charge these batteries in any circumstances - you must use special charger!

Recharging with battery charger “normal” (suitable for standard batteries) will irreparably damage absorbed acid batteries, because the voltages are too high.

Danger of explosion!

Replacing the batteries

When you replace the batteries use only those conforming to specifications.

When working near the battery, do not touch the battery terminals with tools, cable ends or other metal objects. Danger of short circuit and death!

Never bring any batteries in flames and avoid sparks. There is a danger of explosion!

Do not throw used batteries in the trash. Battery acid causes serious damage to the environment and lead metal is a raw material that would be lost. Used batteries can be delivered:

- the battery manufacturer.
- the dealer.
- to a central collection and recycling.

Maintenance operations

Before you start moving subject the wheelchair to the normal controls and a visual inspection to eliminate any damage that might affect the safety of the wheelchair and user.

Any damage must be made as soon as possible by authorized technicians.

For safety reasons, use spare parts and accessories authorized by Nuova Blandino S.r.l.

To ensure the efficiency of your wheelchair, it must be kept clean, organized and maintained as specified in the Instructions for use.

Warning: we do not assume any liability for damage caused by the combined use of our wheelchairs with parts of all kinds from other manufacturers (for example trailers units, electrical devices) that may involve significant dangers - except that a given product have been expressly authorized in writing by the Nuova Blandino S.r.l.

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Responsible for designing, manufacturing, packaging, and labeling of this product, that falls within the category of
ELECTRICAL
DOES GUARANTEE AND DECLARE

The aforementioned device, launched and marketed as of the date certified in this certificate, fully complies with the provisions stated in annexes I and VII to EEC Directive 2007/47 and L.D. No 37 of 25 January 2010, that confirmed EU provisions at Italian level.

The products listed above are classified as Class 1 non-invasive medical devices as provided in Annex IX and X of Directive 2007/47 EEC.

Concerning the remaining guidelines and standards applicable to the product, the complete "EC" declaration of conformity is available on the website www.nuovablandino.com

The afore-mentioned products:

- *are not measuring instruments*
- *are marketed in non-sterile packaging*

This is to certify that the above products were manufactured in compliance with the technical provisions set by the company and that all checks and trials were carried out before commercialization.

Settimo T.se, 2nd December 2015

WARRANTY

On electric wheelchair from our factory, we provide a warranty period of two years from the correct completion of the certificate of purchase (pag. 41).

The batteries have a 6 month warranty.

The warranty applying to the phone battery charge and command void in case of undue manomissionne of the equipment or the above mentioned groups.

The warranty is valid for products that have defects in workmanship or operation.

The warranty excludes products that exhibit defects due to negligence or improper use.

Through our dealers, you can still get information on estimates for repairs.

For safety reasons, use spare parts and accessories approved by Nuova Blandino S.r.l.

The use of spare parts and accessories not approved by Nuova Blandino S.r.l. are reason of the immediate invalidation of the warranty.

To ensure the efficiency of the electric wheelchair, it must be kept clean, organized and maintained as specified in the instructions for use.


Warning:

We accept no responsibility for damage caused by the combined use of the electric wheelchair with parts of all kinds from other manufacturers (for example trailers units, electrical devices) that may involve significant dangers - except that a given product has not been expressly authorized in writing by the Nuova Blandino S.r.l.

Nuova Blandino S.r.l. declines all liability for damage to persons, property, animals, and any third party due to modifications or improper use of the wheelchair.

Plate and EC label

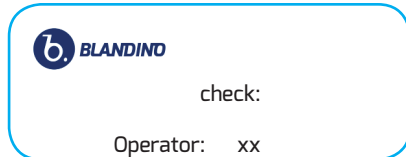The 'EC label (see picture) on the wheelchair found on the front crossmember under the seat.

Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

Maintenance log

Repair date	Repair date
Replaced or repaired parts	Replaced or repaired parts
Stamp or signature of the operator	Stamp or signature of the operator
Stamp or signature of the inspector	Stamp or signature of the inspector

WARRANTY

Model	<p>Certificate of Warranty</p> <p></p> <p>Warning! The shipping damage must be reported IMMEDIATELY (by reserves also written on the bill of withdrawal), and in this way it will be possible to make claims against the courier.</p> <p>(copy for the dealer)</p>
Date	
Chassis No.	
Pilot No.	
Unit	
Battery Charger No.	
Dx Series Engine No.	
Sx Series Engine No.	
Retailer	
Street / No.	
Zip code Town	
Nation	

Information Notes

The device described in this manual is EC labeled and complies with the regulations laid down in EEC Directive 2007/47 and L.D. 37 of 25 January 2010, for medical devices. It is classified as a Class 1 non-invasive medical device, in compliance with annex IX and X of the above-mentioned Directive.

- **Intended use**

The device is supplied ready for use by technical orthopedics and adjusted according to the user's needs.

The electric wheelchair is designed for both indoor and outdoor use. It is intended to accommodate one single person on the seat when the legrests and the lateral pads are mounted, and should not be used as a means of traction, transport or otherwise. Carefully read the instruction manual for a correct use of the device.

- **Check before use**

Before using the wheelchair, make sure that all parts are properly assembled and tightened, especially the platforms. If the product shows visible damage, damaged parts must be replaced to ensure your safety.

- **How to clean**

Do not use high-pressure cleaning equipment! Danger of damage to the mechanical parts caused by water jets.

The chairs should not be cleaned with detergent and items for aggressive cleaning.

- **Lifespan of the device**

This *electric wheelchair* may be used for a variable period of time, depending on the time and the frequency of use. A correct use and a proper maintenance of the wheelchair may extend its lifespan; on average, it is expected to be fully functional for 5-7 years.

- **Annex VII-Declaration of compliance-**

Nuova Blandino S.r.l. performs and updates a systematic post-production evaluation of the devices, on the basis of the provisions referred to in Annex X. It also provides support for the implementation of any corrective measure that may be necessary to carry out, bearing in mind the nature and the risks that may affect the product.

Malfunctions or any lack of information which has resulted in accidents or other type of damage on the user must be promptly communicated to Nuova Blandino S.r.l. by the user or anyone who acts in his or her stead.

FOR TECHNICAL OR USE INFORMATION, OUR EXPERT IS AVAILABLE MONDAY TO FRIDAY, FROM 08:30 TO 12:00 HOURS AND FROM 13:30 TO 7:00 HOURS. CALL +390112625140.

RETAILER STAMP

--

DATE OF PURCHASE

--	--	--

REGISTRATION DATE

--	--	--

N.B.: THIS MANUAL MUST BE PRODUCED WHENEVER ASSISTANCE IS NEEDED

Nuova Blandino S.r.l.
Strada Cebrosa, 24 - 10036 Settimo Torinese
Torino Italy
Tel. +39 011 2625140 - fax +39 011 2625141
www.nuovablandino.com - infonb@nuovablandino.com

Commitment and technology for people with special needs