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Pressoterapia Beauty Line Digitale.

Destinazione d'uso

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Product Name

Pressure Therapy System Beauty Line Digital.

Use

The products of the Beauty Line series are devices for lymphatic drainage; a treatment against water retention. The pressure starts from the distal section and moves in the proximal section of the leg cuff. In this way, the flow of body fluids is stimulated, facilitating their elimination. The pressure therapy is a medical device, do not use without a physician opinion in case of health problems.

Pump Features	
Code	07.01.HF3008
Dimensions	24 x 18 x 11 cm
Display	Digital
Pressure Range	20-250 mm Hg
Cycle Time	From 10 to 30 min.
Power	220/240 V - 50 Hz
Consumption	30 Watt
Noise Level	≤ 35 Dba
Massage Modes	A • From Distal to Proximal, B • Progressive, C • Alternate
Remote Control	Included
Leg Cuff Features • Included	
Code	07.01.LEG11072
Dimensions	110 x 72 cm
Material	Nylon • PU
Overlapping model: the cells that make up the leg cuff are overlapped. This feature allows for a more gradual and effective compression	
Warnings	
Pressure therapy is contraindicated for people suffering from the following pathologies: cardiopathy, suspected or ascertained thrombosis, arteriosclerosis or ischemia, dermatitis, gangrene, local infections	
Check the correct assembly of the device with particular attention to the connection of the tubes between the pump and the leg cuff	
Maintenance and Cleaning	
Clean the leg cuff with a neutral soap; the drying must not take place in direct sunlight	
Do not immerse the tubes or the pump in water	
Check the connection tubes are not obstructed by dust. In this case use a soft swab to clean them	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Pump, 1 Leg Cuff Set • 2 Pcs
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
CND Code	Z120607
RDM Code	1745140/R