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Materasso Ibrido Sadevo Active C.F.A.

Destinazione d'uso

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Hybrid Mattress Sadevo Active C.F.A.

Use

Device designed for the prevention of bedsores in subjects who have to maintain the lying position for a long time.

Features

Code	05.04.H1002CFA
Number of Cells	7
Cell Features	Expandable air cells SADCEL adapt automatically to the shape of the patient according to the weight and the body movement of the latter
Dimensions	195 x 85 x 16 cm, available in different sizes on request
Therapeutic Capacity	160 Kg
The cells are divided in two different compartments, therefore it is possible to connect a pump	
Two ways STATIC and DYNAMIC mode (A-b-A-b)	
Totally fireproof, class 1IM	
Pressure reduction antidecubitus system with pump (Supercare P08, code 05.03.P08)	
The OUT-PUT and IN-PUT air flow are regulated by self-adjusting valves placed inside the system	
AMF viscoelastic foam contact surface (High Density Memory Foam)	
The external cover in 100% Igne Dahlia (breathable, waterproof, fireproof, two way stretchable and Sanitized fabric) with zip on three sides	
Ventilated Critical Foot Area (C.F.A.): system allows the heels to sink, reducing the pressure and the friction. This part of the mattress consists of expanding independent cylinders with ultra soft viscoelastic foam. In this way, the risk of creating a decubitus lesion is cleared.	

Cover Maintenance and Cleaning

Water temperature not above 95° C	Do not iron
-----------------------------------	-------------

Sterilization is possible in autoclave up to 134° C, rubber cycle

Mattress Maintenance and Cleaning

Clean the mattress with a damp cloth and mild soap

Disinfection is possible with 5% diluted disinfectants even with alcohol base

Storage and Disposal

Single-user Product

Keep away from heat, sparks, open flame or other sources of ignition

Dispose of the device in compliance with current regulations on environmental protection and separate waste collection

Package and Nr. Items

Box	1 Piece
-----	---------

Warranty

European	24 months for manufacturing defects with the exception of consumable parts
----------	--

Product References

Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	04.33.06.006
CND Code	Y033306
RDM code	1384101/R