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 **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

INDEX

1. Codes	pag.15
2. Introduction	pag.15
3. Intended use	pag.15
4. EC Declaration of conformity	pag.15
5. General warning	pag.16
6. Assembling/Dsiassembling	pag.16
6.1 Opening of CP71x and CP870-35	pag.16
6.2 Opening of CP850-37	pag.17
6.3 Opening of CP860-30.....	pag.17
6.4 Closing of CP71x e CP870-35	pag.17
6.5 Closing of CP850-37.....	pag.18
6.6 Closing of CP860-30	pag.18
6.7 Install/remove the lateral and elevating footrest.....	pag.19
6.8 Adjust the lateral footrest	pag.19
6.9 Adjust the elevating footrest	pag.19
7. Before use	pag.19
8. Using warnings	pag.19
9. How to use	pag.20
9.1 Get in the wheelchair	pag.20
9.2 Get out of the wheelchair.....	pag.20
9.3 Stairs and height difference (how to face them)	pag.20
9.4 Brakes	pag.21
9.5 Wheels.....	pag.21
9.6 Carrying	pag.21
10. Maintenance	pag. 21
11. Cleaning and disinfection	pag. 21
11.1 Cover and stuffing.....	pag. 21
11.2 Frame and metal parts.....	pag. 21
11.3 Brakes and wheels	pag. 21
11.4 Disinfection	pag.21
12. Condition of disposal	pag.22
13. Spare parts / accessories	pag.22
14. Technical features	pag.22
14.1 Dimensions	pag.22
14.2 Technical specification CP71x.....	pag.22
14.3 Technical specification CP870-35	pag.22
14.4 Technical specification CP850-37-860	pag.23
15.Warranty	pag.24
15.1 Repairing	pag.24
15.2 Spare parts	pag.24
15.3 Exempte clauses.....	pag.24

I Class Medical Device
according to Directive 93/42/EEC and
further modifications

1. Codes

CP710-40	Wheelchair LIGHT 1 seat 40 cm - BC - PL
CP713-40	Wheelchair LIGHT 1 seat 40 cm - BC - PE
CP710-45	Wheelchair LIGHT 1 seat 45 cm - BC - PL
CP713-45	Wheelchair LIGHT 1 seat 45 cm - BC - PE
CP870-35	Wheelchair KIDS seat 35 cm - BC - PL
CP850-37	Wheelchair TRAVEL seat 37 cm
CP860-30	Wheelchair FOLDY seat 30 cm

(**BC** = desk type armrest; **PL** = lateral footrest; **PE** = elevating footrest)

2. Introduction

Thank you for purchasing a **ARDEA ONE** wheelchair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advise you properly.

Note: check all parts for shipping damage and test before using. In case of damage, do not use the device. Contact your dealer for further instruction.

3. Intended use

The wheelchairs are intended for people with movement difficulties.

The wheelchairs can be for self-propulsion (with 60cm casters) or can be wheeled

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without notice

4. EC Declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the wheelchair category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the said directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES.**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
6. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC and further modifications to the proper authorities

5. General warnings

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before using this equipment otherwise injury or damage may occur.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the wheelchair
- Keep the packed wheelchair away from heat sources
- SERVICE LIFE- the wheelchair use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the wheelchair
- Close the wheelchair before lifting it.
- DO NOT attempt to lift the wheelchair by any removable parts

6. Assembling / Disassembling

Note: Make reference to safety cautions in this manual.

6.1 Opening CP71x and CP870-35

Get the handles and pull outward, then press the seat until the wheelchair is completely open.

6.2 Opening CP850-37

Lift up the backrest until levers block (fig. 1); pull the footrest outward (fig. 2); push the seat downward until it is completely opened (fig. 3); push-down the armrests until 90° degrees.

(fig. 1)

(fig. 2)

(fig. 3)

6.3 Opening CP860-30

Remove the wheelchair from the bag (fig. 4); push on the seat (fig. 5); lift up the backrest and lock the levers (fig. 6); push down the armrest until 90° degrees.

(fig. 4)

(fig. 5)

(fig. 6)

6.4 Closing CP71x e CP870-35

Lift powerfully the seat and close the wheelchair. Push the handles close

6.5 Closing CP850-37

Lift up the armrests (fig. 7); Press backrest levers and push it downward (fig. 8); pull up seat and footrest until the wheelchair is totally folded (fig. 9); pull up the footrest (fig. 10);

(FIG.7)

(FIG.8)

(FIG.9)

(FIG.10)

6.6 Closing CP860-30


Unlock backrest levers and push it downward holding it from handles (fig. 11); Pull seat upward until the wheelchair is totally folded (fig. 12); Pull the footrest upward near to the foot belt (fig. 13);

(FIG.11)

(FIG.12)

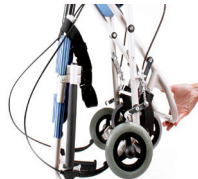(FIG.13)

6.7 Install/remove the lateral or elevating footrest (only for CP71x)

INSTALL: Assembly the lateral or elevating footrest by install the hinge plates on the footrest onto the hinge pins on the wheelchair frame. Push the footrest toward inside of the wheelchair until it locks into place

REMOVE: Remove the footrest by pushing the footrest release lever inward, rotate outward, lift the footrest off the hinge pins.

6.8 Adjust the lateral footrest

We have elevating and swing-away footrests, both of which can be adjusted in height. Use the equipment key to unscrew fixing nut and adjust footplate for a height lower not than 6,5 cm. from the floor.

6.9 Adjust the elevating footrest

Turn leg support outside and choose agreed position. Take it back paying attention to proper stop. To elevate footrest, lift it on. Push outside lever to make footrest lower.

7. Before use

- Inspect the frame for any damage. Check the tubing for cracks and rust, in order to guarantee the safe for people and things
- Check the correct assembly, in order to guarantee a safe use
- Check the seat is completely open. If not push the seat down from the lateral frame.
- Check the brake correctly works

8. Using warnings

- Check the automotive parts usage, in order to guarantee a safe use of the device for people and things
- Assure that the seat is completely open and locked
- Assure that all parts are correctly assembled. Checking carefully the correct assembling of the wheels
- Assure the brakes correctly works

9. How to use

Note: make reference to “GENERAL WARNINGS” chapter

WARNING!

- Getting in and out of the wheelchair may be performed independently, provided you have an adequate mobility and upper body strength
- Do not stand on removable parts

DO NOT STAND ON THE FOOTRESTS WHEN GETTING IN OR OUT OF THE WHEELCHAIR, OVERTURNING RISK!!!

9.1 Get in the wheelchair

- Position the wheelchair as close as possible to you
- Engage wheel locks. We suggest to remove the armrest to ease the patient getting in the wheelchair
- Lean on the frame with arms and slowly go down towards the seat

9.2 Get out of the wheelchair

- Position the wheelchair as close as possible along side the seat to which you are transferring
- Engage wheel locks. We suggest to remove the armrest to ease the patient getting out of the wheelchair
- Shift body weight into seat with transfer

9.3 Stairs and height difference

WARNING!

The following method must be used only from experienced patients. If you are not experienced, let your help from an assistant!

Before moving on an inclined surface make sure of your ability and limits. Refer to the movement techniques with qualified personnel. An assistance for surfaces with a slope greater than 10% is recommended. If you need to stop on an inclined surface, avoid sudden movements. To slow in a stop do not use the wheel brakes. Not suddenly change the downward direction.

- Get in

Position the wheelchair at the curb to be crossed. Balance the wheelchair on the rear wheels, in order to lift the front wheels, enough to lean them on the obstacle. Bend your trunk forward and vigorously push the handrail. Continue pushing until the rear wheels are climbing over the obstacle.

- Get off

To get off the stair follow the reverse procedure, positioning the rear side of the wheelchair at the curb, keep the handrails vigorously and get off the obstacle slowly; then balance the wheelchair on the rear wheels in order to complete the descent


**WARNING!**

It is recommended not to use this technique in the presence of a vertical drop more than 10cm

9.4 Brakes

Stop wheels before getting in/out, stretching out or getting on lift. Pull completely lever up to stop chair. If correctly done, brakes press 5/6 mm against the rubber. Pls. check tyre pressure in case of inflate wheels. If needed, use the equipment key to unscrew nut while clamping bit forwards and backwards. Fix correctly nut. to adjust brakes.

9.5 Wheels

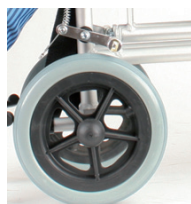The wheels must be periodically checked and lubricated if necessary. Remove the fork taking out the screw and lubricate the peg. Remove the wheel from the fork as well and lubricate the pegs. Assembly again the wheel and the peg and tighten the screws well. The front wheels are height adjustable in three positions.

9.6 Carrying

The wheelchair has been designed to be easily carried.

Once the armrest and the footrest are removed, can be closed and easily positioned in the car boot, for example. If this can't be possible, position the wheelchair in front of the car rear seat so that it is stable and cannot slip or slide. Lock the wheelchair with the seatbelt.

10. Maintenance

The ARDEA ONE devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair use only original spare parts. Before using follow the "BEFORE USE" chapter.

11. Cleaning and disinfection

11.1 Cover and padding

The cover and padding are in Nylon. Clean with warm water and mild soap.

11.2 Frame and metallic parts

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting.

11.3 Brakes and wheels

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous use the wheelchair with wet wheels.

11.4 Disinfection

if you need to disinfect the device use a common disinfectant cleaner

Note: Do not use solvent and abrasive product

12. Condition of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycle.

13. Spare parts and accessories

For spare parts only advert to MORETTI master catalogue

14. Technical features

14.1 Dimensions

MEASUREMENT CM												Kg			Kg
	CODES												Front	Back	
CP710-40 CP713-40	103 VARIABLE	87 VARIABLE	40	93 VARIABLE	70 VARIABLE	50 VARIABLE	43/51	40	40	60	34	14,5	15	60	100
CP710-45 CP713-45	103 VARIABLE	87 VARIABLE	40	93 VARIABLE	70 VARIABLE	50 VARIABLE	43/51	45	40	65	34	14,5	15	60	100
CP870-35	97	84	36	87	65	45	23/31	35	43	52	33	12	15	57	100
CP850-37	85	58	38	89	65	43	33	37	41	44	29	7	13	20	75
CP860-30	69	47	34	90	67	42	35	30	35	51	50/65	7	13	15	75

14.2 Technical specifications CP71x

- Aluminium frame
- Max weight **100 kg**
- PU rear casters, 15 cm diameter
- Quick release rear wheels with pneumatic tires
- Nylon coating
- Resistance to ignition: the materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2
- Reference regulation: EN12183
- Environmental temperature: -20°C/+ 40°C
- Storage: store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)

14.3 Technical specifications CP870-35

- Steel frame
- Max weight **100 kg**
- PU rear casters, 15 cm diameter
- Rear wheels 57cm diameter, with pneumatic tires
- Nylon coating
- Resistance to ignition: the materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2
- Reference regulation: EN12183
- Environmental temperature: -20°C/+ 40°C
- Storage: store in a cool place, away from heat sources, such as sunlight

(suggested temperature between +10°C and +40°C)

14.4 Technical specificatios CP850-37/CP860-30

- Aluminium frame
- Max weight **75 kg**
- PU rear casters, 13 cm diameter
- Rear casters:
 - CP850-37 diameter 20cm, solid tires
 - CP860-30 diameter 15cm, solid tires
- Nylon coating
- Resistance to ignition: the materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2
- Reference regulation: EN12183
- Environmental temperature: -20°C/+ 40°C
- Storage: store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)

15. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15.1 Repairing

Warranty repair If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not covered by warranty A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

Non defective device The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

15.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

15.3 Exempt clauses

Moretti does not offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchasing date _____

Authorized dealer _____

Street _____ Place _____

Sold to _____

Street _____ Place _____

MORETTI S.P.A.

Via Bruxelles, 3 - Melegnano 20022 Cavriaglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Distribuito da / Distributed by:

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200

www.morettispa.com

info@morettispa.com