Sherpa / Montascale a cingoli

Sherpa / tracked stair climber

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KSP

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A completely renewed electronic at the service of safety and simplicity of use

The display placed on the control panel allows the stable monitoring of the stair climber ensuring safety during its operations. Messages on the display guide the operator and assist him to guarantee a proper use of the machine ensuring safety and simplicity. Acoustic and descriptive messages on the interactive display alert the operator, assisting him during each stage of the operations reporting any manoeuvring errors. The machine is constantly monitored and any problems or malfunctions are promptly reported on the display.

Visual messages and acoustic alarms which ensure an extremely safe use:

- *Low battery alarm, motion inhibited.*
- *Visual and acoustic alarm for low battery.*
- *Expressive load signal.*
- *On excessive slopes, acoustic signal.*
- *On negative inclination, non reversal acoustic signal.*
- *Check of excessive engine absorption; if the machine collides with an obstacle the device stops the movement in that direction and takes only commands of reverse motion.*
- *Display of message: "obstacle, reverse the direction of motion."*
- *Inhibited functions with inserted battery charger.*
- *Active acoustic alarm during the manoeuvres with additional wheels down.*
- *Stand by after 5 minutes of inactivity and acoustic alarm every 30 seconds.*

The new Sherpa has also undergone some mechanical changes to further improve the safety of the machine:

- *Protection of the battery poles.*
- *Protection of the internal use.*
- *Protection of the bottom of the machine.*
- *Removal of side openings.*
- *Protection of the tiller's contacts.*

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Powered tilting system of the tiller.

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Electrical moving system on the landing.

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Tracked stair climber with ramps

New SHERPA tracked stair climber series version with ramps by KSP Italia. Adaptable and efficient; born for the electric wheelchairs transport, it allows to carry many electric wheelchair models, also with very low crossbars, and multipurpose chair.

Technical specifications:

- Load 150 kg.
- Reinforced tiller thickness.
- Hooking bar for wheelchair adjustable also in depth.
- Folding zinc-coated steel support ramps coated in non-slip aluminium with resealable extremities.
- New electronic making the machine even more reliable with better performances.
- High driving comfort also thanks to soft breaks and departures.
- New easy and intuitive control panel.
- Multifunctional display positioned on tiller control panel, it supplies to the customer a self-diagnosis system integrated in the machine and stable information for a machine assisted-guide.
- Emergency stop button.
- Safety belt.
- Release and safety controls of the tiller placed on the tiller upper part itself that facilitate access and use.
- Adjustable and removable headrest.
- Height and width adjustable Wheelchair hooking bar with adjustable clamps.
- External battery-charge (standard version).
- Auto-balanced manoeuvre wheels for an immediate flat surface use.

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Tracked stair climber with ramps equipped with two additional motorized wheels. Load 150 kg.

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In addition to the N959 features, this version is equipped with 2 additional motorized wheels that further facilitate the landing manoeuvringwheels that can be activated during the landing manoeuvring.

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Tracked stair climber with ramps. Full-optional version. Load 150 kg.

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In addition to the N959 features, this version is equipped with:

- 2 additional motorized wheels that can be activated during the landing manoeuvring.
- Powered tilting system of the tiller.

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Sherpa tracked stair climbers are indicated for disabled persons who need to navigate stairs in buildings without suitable equipment; for this reason, they are suitable for overcoming architectural barriers. The stair climber consists of a metal structure that moves on rubber tracks and a tiller, which allows steering and anchoring a wide range of wheelchairs. The control and emergency buttons are ergonomically arranged on the tiller. Movement is provided by an electric motor managed by an electronic controller for soft starts and stops along with a high level of comfort. The unit is battery powered and the residual charge is displayed on the control panel. The track provides forward and backward movement while changes in direction are made by a pair of manoeuvring wheels that extend automatically. The machine is easy to transport because unhooking the steering column reduces its overall dimensions and weight, making it easy to manoeuvre. The body of the machine is activated by a button located on the rear control panel. The structure's loading-bearing frame is made of electrically welded steel treated with a cathodisation process, while the steering mechanism consists of a chrome-plated steel framework. The thermoformed high-impact polystyrene chassis gives the product a refined, modern look. The machine's extremely modest height makes it simple to attach a wide variety of wheelchairs."

...reliability from a far-away land... The name chosen for this line of stair climbers comes from the mountains of Nepal, the home of the Sherpa, a people whose culture is founded on respect for human dignity. Their great resistance to fatigue and sociable character, together with the congenital ability to adapt to great heights, has made them the perfect "high-altitude bearers" and essential companions for the first mountain-climbing expeditions, capable of carrying the heaviest loads and making trails safe so that climbers could reach the peaks... and these characteristics are the ones that distinguish Sherpa stair climbers, whose goal is to become a faithful and reliable travelling companion.

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Internal battery charger (Optionally only for N902):

- Electronic recharge control.
- Recharge indicator.
- Reverse polarity and short circuit protection.
- Power cable detection device.

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Lower control panel:

- Forward / Reverse switch.
- Tiller safety lock (Only N902).
- Tiller detachment lever (Only N902).
- Charge connector.

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Motorized manoeuvre additional wheels system (only for N958 / N959TM).

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Manual device for the machine's movement in case of breakdown (except N956 / N959 / N959R / N959TM).

Carica batterie elettronico esterno (Standard).

External electronic battery charger (Standard).

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Internal electronic battery charger on demand (only for N 902).

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N9020

Coppia pedane per carrozzine standard. (Solo N959 / N959R / N959TM)

Pair of plates for standard wheelchair. (Only N959 / N959R / N959TM)

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N9012

Prolunga per carrozzina XL.
Extension for XL wheelchairs.

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- *Powered by irreversible gear motor and electronic control with soft start and stop operation.*
- *Automatic self-braking system in case of emergency or power failure.*
- *Maximum gradient: 70% equal to 35°.*
- *Power input: 24 VD, electronically controlled.*
- *Silent movement: elimination of "clamp" noises at start-up and stopping (no electromagnetic switches).*
- *Driving comfort: soft starts and stops, no vibrations.*
- *Safety: electronic and mechanical features ensure a high level of safety.*
- *Maintenance: the parts subject to wear are easy to replace thanks to easy access and the electronic connection system.*
- *Design: modern and functional.*
- *Quality: the control standards applied to the production process ensure a top quality product.*

Special safety features include:

- *Motor with "irreversible" mechanical transmission.*
- *Moving parts are thoroughly protected to prevent debris from entering the device.*
- *Power cord check with device to inhibit operation.*
- *Tiller attachment system complete with safety lock.*
- *Reduction of electromagnetic emissions.*
- *Emergency button for total inhibition of functions.*
- *Manual device for the movement of the machine in case of breakdown (except N956 / N959 / N959R / N959TM)*
- *No opening structure for water and sand protection as requested by the current regulation.*

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