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Bracciale Beauty Line.

**Destinazione d'uso**

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Arm Cuff Beauty Line.

**Use**

The arm cuffs of the Beauty Line series are combinable with 07.01.HF3006 and 07.01.HF3008 pumps. They are devices for lymphatic drainage; a treatment against water retention. The pressure starts from the distal section and moves in the proximal section of the arm cuff. In this way, the flow of body fluids is stimulated, facilitating their elimination. The pressure therapy is a medical device, do not use without a physician opinion in case of health problems.

Features	
Code	07.01.ARM8850 *
Dimensions	88 x 50 cm
Material	Nylon • PU
Nr. Sections	4
Code	07.01.ARM9060 *
Dimensions	90 x 60 cm
Material	Nylon • PU
Nr. Sections	4
Overlapping model *: the cells that make up the arm cuff are overlapped. This feature allows for a more gradual and effective compression	
Warnings	
Pressure therapy is contraindicated for people suffering from the following pathologies: cardiopathy, suspected or ascertained thrombosis, arteriosclerosis or ischemia, dermatitis, gangrene, local infections	
Check the correct assembly of the device with particular attention to the connection of the tubes between the pump and the arm cuff	
Maintenance and Cleaning	
Clean the arm cuff with a neutral soap; the drying must not take place in direct sunlight	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr. Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
CND Code	Z120607
RDM Code	1745140/R