**Description**

CliniSilk is a natural non-absorbable suture. Silk is a protein filament produced by the silk worm. CliniSilk sutures are dyed black for better visibility and are coated with silicone to make it smoother and to prevent capillarity.

**Indications**

Areas where non-absorbable sutures are indicated without the need for long term wound support. The most popular indication is for skin closure.

**Actions**

Silk sutures exhibit moderate tissue reaction.

**Warning**

Users should be familiar with surgical procedures and techniques before employing CliniSilk sutures. Because it is a braided suture it should not be used in the presence of infected or contaminated incisions.

**Precautions**

In handling this or any other suture material, care should be taken to avoid damage. Avoid crushing or crimping damage due to the application of surgical instruments, such as forceps or needle holders.

**Sterility**

CliniSilk sutures are sterilized by ethylene oxide gas. Do not use if package is opened or damaged. Discard opened unused sutures.

**Storage**

Recommended storage conditions: 15°C to 30°C, away from direct heat. Do not use after expiry date.

**How supplied**

CliniSilk Sutures are available in sizes 6/0 through 2. The suture is supplied in non-needled standard lengths or attached to various needle types in boxes of one dozen.

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