TERMOMETRO DIGITALE

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DIGITAL THERMOMETER

Carefully read the following instructions before use. The digital thermometer is a device for body temperature measurement suitable both for hospital and home use. Compared to the traditional glass and mercury models, it offers the advantage of simple reading, quick and accurate measurement, memory and alarms, etc. In particular it is safer for people and environment because of the absence of mercury. The thermometer has been calibrated by the manufacturer and further calibrations are not necessary. For any doubt on the device accuracy, please get in contact with your retailer.

Technical features

Measuring range:
25560 (°C): 32-42.9 °C

Accuracy:
25560 (°C): 32-35 °C ± 0.2 °C - 35.1-42.9 °C ± 0.1 °C

Battery life: ca. 100 hours of uninterrupted use
Dimensions: 129x22x14 mm (case included)
Weight: ca. 11 g
Measurement time: ca. 60 seconds

Use

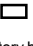
- Before the use, disinfect the probe with alcohol or place a cover on the probe.
- Press the power button; a "beep" sound is issued.
- After 2 seconds the display visualizes the last measurement taken.
- After 2 seconds the letter °C or °F starts blinking, now it is possible to start measuring.
- During the measurement, remain in the same place/position until the end.
- The measurement lasts about 60 sec. and when it is complete, the thermometer will sound an acoustic signal. The detected temperature is shown on the display.
- The thermometer will automatically turn off after about 10 minutes. However to prolong the life of the battery, it is best to press the power button soon after the use.

- For measurements under the tongue, open your mouth for about 5 minutes before starting measuring. Place the probe under the tongue in a stable position; do not open your mouth until measuring has been concluded.
- For measurements under the armpit, before starting, keep your arm close to your body so the armpit remains close for about 5 minutes. Place the probe in the centre of the armpit in a stable position; do not move your arm until the measurement end. **Add 0.5 °C / 1 °F to the value indicated by the display to get the correct measurement.**
- For measurement in rectum, delicately insert the probe into the rectum. Do not move until the measurement is completed. **Deduct 0.5 °C / 1 °F from the value indicated by the display to get the correct measurement.**

Disinfect the probe only with alcohol or similar, DO NOT boil it! DO NOT sterilize in autoclave! Do not dip it in water or liquids! Models 25562 and 25563 (water-resistant) can be washed under running water, however DOT NOT DIP the product! Avoid exposing the unit to direct sunlight or atmospheric agents. Do not measure except body temperature!

Battery replacement

- The battery must be replaced when the symbol  or  blinks on the display.
- Delicately withdraw the back cover and remove the old battery by means of a toothpick or a needle tool.
- Insert the new 1,5 V battery (LR/SR-41).
- Replace the back cover

Do not disassemble the unit except for battery replacement! Do not leave the batteries or the back cover within children's reach! The product or battery disposal must be in compliance with the standards in force in your country.

Comply with the laws in the country of use to properly dispose of the product.

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