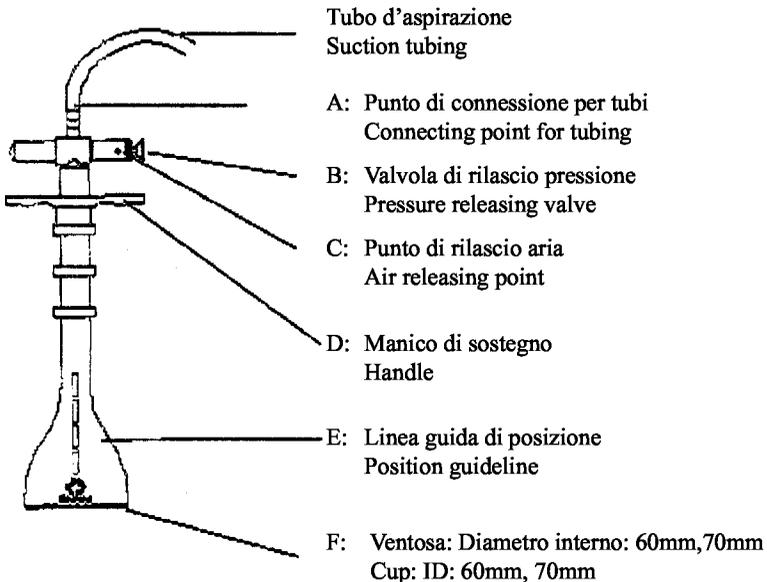
# GIMA

## VENTOSA FETALE VACUUM FETAL VACUUM CUP

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**ATTENTION:** *The operators must carefully read and completely understand the present manual before using the product.*

**GIMA Spa - Via Marconi, 1 - 20060 Gessate (MI) - Italia**

**ITALIA: [gima@gimaitaly.com](mailto:gima@gimaitaly.com) - [www.gimaitaly.com](http://www.gimaitaly.com)**

**INTERNATIONAL: [export@gimaitaly.com](mailto:export@gimaitaly.com) - [www.gimaitaly.com](http://www.gimaitaly.com)**

Besmed Health Business Corp. No.5, Lane 116, Wu-Kong 2nd rd.,  
Wu-Ku Industrial Park, Taipei, Taiwan

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## DESCRIPTION

Fetal Vacuum Cup is for birth assistance. Available for two sizes, 60 mm, and 70 mm ID. Made of medical grade silicone grade rubber.

## OPERATION INSTRUCTIONS

1. Connect the tubing to both the suction point of the suction machine and connecting point of the suction cup (A).
2. Turn on the suction pump.
3. Attach the cup (F) to the fetal head. The fetal head will be gripped once been attached.
4. Press pressure releasing valve (B) to release fetal head.

## CAUTIONS

### **CAUTION:**

1. *U.S. Federal law restricts this device to sale by or on the order of a physician (USA only).*
2. *Don't apply this device for a long time after gripping the target. It may cause bruise of fetal.*
3. *Don't apply it on vulnerable area, such as eyes, mouths, and temples. Only apply it on area with bones.*
4. *After gripping the fetal head, make sure the guideline is straight when pulling it. Don't twist the cup.*
5. *This product can be treated into a sterilizer at 134°C up to 100 times.*

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Mdi Europa GmbH Langenhagener Str. 71, 30855 Hannover-Langenhagen, Germany